

Firmware Update Guide



NR series recorders


NR-1207
NR-1207-E
NR-1207-3
NR-302
NR-314
NR-314-T


*Specifications are subject to changes without notice.

Cautions

- Make sure you are using completely charged alkaline battery during the firmware update
- DO NOT remove battery from the recorder before the updating is completed
- Firmware update file for the incompatible model will be ignored by the recorder
- If you fail to update the firmware after multiple attempts, contact Norav Medical support

A. How to check the current Firmware Version of NR recorder

1. Insert Battery into NR recorder battery compartment.
2. Connect ECG Cable.
3. Turn ON the recorder by pressing on its “ENTER”  Button
4. The recorder will Power Up and show Main screen.
5. Press on LEFT Button, the recorder will show Setup screen.
6. Navigate to “About” menu item using UP and/or DOWN buttons.
7. Press on ENTER button to see the recorder information screen.
8. Check the recorder current Firmware Version.

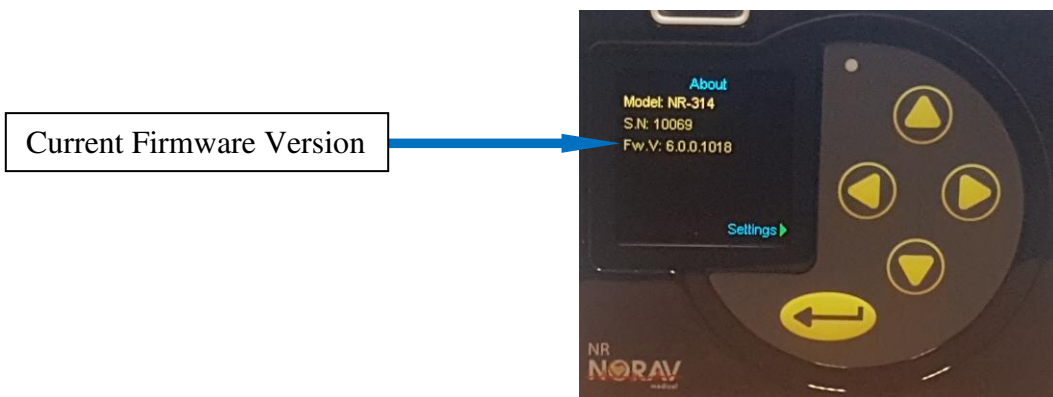


*Specifications are subject to changes without notice.

B. Preparation for updating

1. Get from Norav Medical support firmware update file *nrupdate.nru*
2. Copy *nrupdate.nru* file to the root directory of the SD card
3. Insert the card into SD card slot on the NR recorder (*inside battery compartment*).

C. Firmware update

1. Insert **NEW ALKALINE** battery into NR recorder battery compartment.
2. Connect ECG Cable.
3. Turn ON the recorder by pressing on its “ENTER”  Button
4. The recorder will Power Up and if update file is detected correctly will start updating procedure.
5. Wait recorder finishing the update and turns off.
6. Check Firmware Version as described in step A.

WAIT Firmware Update is complete!!

