



Welcome to your quick-start guide for Norav Medical's **PC-ECG 1200** – an advanced PC-based electrocardiogram solution. This solution enables real-time ECG acquisition and analysis with high accuracy and clinical reliability, tailored for medical professionals in hospitals and clinics.

This guide provides a quick overview of the software installation process and the contents of the installation package. For complete setup instructions, refer to the Instructions for Use (IFU).


1 Identify the Installation USB Stick

Two USB sticks may be included in your package:



Installation USB

Contains the software installer and documentation. **Software name and version clearly labeled.**

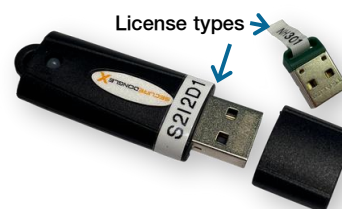

License Key USB (Dongle)

Required for software activation and continued use. **License type labels included.**

Important Notes:

- Connect the Installation USB first to begin setup
- Connect the License Key USB only after installation has completed

2 Contents of the Installation USB

After connecting the Installation USB, open Windows File Explorer to view the following contents:

Setup.exe – launches the PC-ECG 1200 software installer

GettingStarted.pdf – overview of the software package and installation process

ReleaseNote.pdf – details of version updates and changes

PC-ECG.ICO – application icon file

AUTORUN.INF – autorun configuration file

Documentation/ – includes relevant Instructions for Use (IFU) in PDF format

Hardware Drivers/ – required drivers for specific hardware configurations

Remote View/ – optional remote image viewer tool

3 Launch Installation

- 1 If setup doesn't auto-start, open **File Explorer**, go to the **Installation USB**, and run **Setup.exe**.
- 2 Click **Next** on the installation welcome screen.
- 3 Select a **destination folder** (default location is recommended).
- 4 Choose whether to create **backup files** (recommended: Yes).
- 5 Select your **language**.
- 6 Choose whether to install **demo data files** (optional).
- 7 Select the software **access level**:

Basic Security

No user password required

Elevated Security

Login password required

- 8 Click **Next** to begin installation.
- 9 Wait while the files are copied.
- 10 When prompted, click **Finish** to complete.
- Restart your PC after installation is complete.

Manufacturer and Regulatory Information

Manufacturer: Norav Medical GmbH, Christof-Ruthof-Weg 10, 55252 Mainz-Kastel, Germany
UK Responsible Person: Physiological Measurements Ltd., The Old Malt House, Shropshire, SY11 1AJ, UK
CH-Rep: Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Switzerland

This product is intended for installation on equipment that complies with IEC 62368-1.

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare provider.





4 Technical Support

For technical support or additional assistance, contact Norav Medical or your local distributor. Troubleshooting and configuration information is available in the **Instructions for Use (IFU)**.

5 Product Registration

According to Regulations (EU) 2017/745 (MDR) and ISO 13485, manufacturers of medical devices intended to be placed on the market are required to register all product installations. Please help us at Norav Medical fulfill this obligation by returning the completed questionnaire either by email (info@norav.com) or by fax **+49 (0) 6134 567983-14**.

Installation Registration Form

Product Registration (e.g., PC-ECG 1200 Series)

Model Name (on device back, e.g., 1200M)

Serial Number (on device back)

License Key ID (from software's About)

Share Your Thoughts

Software Name and Version (from Installation USB label, e.g., PC-ECG 1200 Rev. 5.986)

Who Performed the Installation? (name and job title)

Was the Installation Successful? (yes / no)

Date of Installation



Email

info@norav.com



Fax

+49 (0) 6134 567983-14

Manufacturer and Regulatory Information

Manufacturer

Norav Medical GmbH
Christof-Ruthof-Weg 10
55252 Mainz-Kastel, Germany

UK Responsible Person

Physiological Measurements Ltd.
The Old Malt House
Shropshire, SY11 1AJ, UK

CH-Rep

Arazy Group Swiss GmbH
Bruderholzallee 53
4059 Basel, Switzerland

For Norav PC-ECG 1200 Software (Basic UDI-DI: 426049856DE55252NMKEYVQ) | D/N: NV-100.000.060 Revision: 01

This product is intended for installation on equipment that complies with IEC 62368-1.

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare provider.