

Holter NH-301 INSTRUCTIONS FOR USE



VERSION: 4.0.0.10

DATE: 07.08.2025

C € 2797

www.norav.com



info@norav.com

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 2 of 393

Contents

CON	TENTS	2
1. GE	ENERAL INFORMATION	4
2. IN	TRODUCTION	6
2.1.	Document Conventions	6
2.2.	Safety Measures	6
2.3.	Intended Use	7
2.4.	Contraindications for Use and Adverse Effects	7
2.5.	General Specifications and Sample Rates	
2.6.	Supported Holter Recorders	9
3. S O	OFTWARE INSTALLATION AND SETUP	10
3.1.	Recommended PC Specifications	10
3.2.	Installation Process	10
3.3.	Data Structure after Automatic Installation	11
3.4.	Setup Process after Installation	11
4. IN	TERFACE CORE ELEMENTS OVERVIEW	40
4.1.	Menu Bar	40
4.2.	Title Bar Text	50
4.3.	Window Control Buttons	51
4.4.	View Area	52
4.5.	Views - Views Sidebar	53
5. PR	REPARING FOR HOLTER RECORDING	324
5.1.	Preparing Patient	324
5.2.	Preparing Recorder	332
5.3.	Entering Patient Information	349
5.4.	Adjusting Record Settings	350

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 3 of 393

5.5.	Adjusting Record Settings: NR-314-P Model Workflow	354
5.6.	Initiating Recording	359
5.7.	Verifying Recorder Data	360
5.8.	Verifying Recorder Data: NR-314-P	361
5.9.	Starting Recording	362
6. DO	WNLOADING HOLTER RECORDING DATA	364
7. RE	COMMENDED WORKFLOW OPTIONS	367
7.1.	Standard Workflow Overview	367
7.2.	Perfectionist Workflow Overview	
7.3.	Workflow Clarifications and Explanations	381
8. AP	PENDIX A: OVERVIEW AND UTILIZATION OF RR TREND VISUALIZATIONS	382
8.1.	RR Trend Examples	384
9. AP	PENDIX B: HOLTER MONITORING PATIENT DIARY FORM	391
40.	PPENDIX C: PATIENT-RELATED INSTRUCTIONS	000
10. A	PPENDIX C: PATIENT-RELATED INSTRUCTIONS	392
11. S'	TORAGE, CLEANING AND DISPOSAL OF LICENSE PHYSICAL KEY	393

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 4 of 393

1. General Information

Holter NH-301 analysis system. Instructions for Use

For software version: 4.0.0.10

For device models: NR-302, NR-314, NR-1207, NR-1207-3, NR-314-P, DL800/900/1200

Document number (D/N): NV-200.40010.028 Revision: 04 Date of document release: 07.08.2025

Item(s)	Basic UDI-DI
Software: NH-301 v4.0.0.10	426049856DE55252NMKEYVQ
Devices: Norav NR Series	426049856DE55252NMAMBTG
Devices: NR-1207-3	426049856DE55252NMNRSX2

Copyright Information

This document and the software it describes are proprietary to Norav Medical and protected under copyright law. They may not be copied, reproduced, transmitted, or translated, in whole or in part, without prior written consent from Norav Medical. Unauthorized modifications to this document or the software may void regulatory compliance and manufacturer liability.

The information in this document is for guidance only and is provided solely for the proper and safe use, maintenance, or servicing of the software described herein. This information is subject to change without notice and should not be construed as a commitment by Norav Medical. Norav Medical assumes no liability for errors or inaccuracies that may appear in this document. Norav Medical also assumes no responsibility for improper or illegal use of the software or for failure to follow the instructions, warnings, or intended use guidelines provided herein.

The Windows® name is a registered trademark of Microsoft Corporation in the United States and other countries.

All other trademarks mentioned are the property of their respective owners.

The most recent version of this document can be downloaded from our website: https://www.noravmedical.com/ ©2025, Norav Medical. All rights reserved.

Manufacturer and Contact Information



Manufactured by:

Norav Medical GmbH Christof-Ruthof-Weg 10 55252 Mainz-Kastel Germany

Phone: +49 6134-567983-0 E-mail: <u>info@norav.com</u>

Responsible Person in UK

Physiological Measurements Ltd. The Old Malt House Willow Street Shropshire, SY11 1AJ United Kingdom

Phone: +44 1691-676-496 E-mail: <u>info@pml.tel</u>

Representative in Switzerland

Arazy Group Swiss GmbH Bruderholzallee 53 4059 Basel Switzerland

Phone: (+41) 33533 2267 E-mail: swiss.ar@arazygroup.com

Compliance Information

This product (software) complies with the applicable requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices, as well as the UK Medical Devices Regulations 2002 (Statutory Instruments 2002 No. 618 Consumer Protection), as amended.



This product (software) is intended for installation on equipment that meets the applicable edition of IEC 62368-1. Medical devices used in conjunction with this product must comply with the relevant IEC 60601 series standards, as appropriate. In addition, any electromagnetic interference generated by devices in this configuration must conform to Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 5 of 393

∆Caution: Federal Law restricts this device to sale by or on the order of a licensed physician or healthcare provider.

Disclaimer

This system is intended as a decision support system for persons who have received appropriate medical training and should not be used as the sole basis for making clinical decisions pertaining to patient diagnosis, care, or management. Any application of medical information from the program, other than the original design or intended use thereof, is not advised and is considered a misuse of the software product.

Important Usage Notice

Like all Holter systems, noise and artifacts may produce false positive ECG events. Therefore, patient data MUST be reviewed and edited by a qualified technician or physician. Norav Medical and its staff cannot be held liable for patient data edited by a non-qualified person or for data edited by a qualified person.

Norav Medical Warranty Limitations

Norav products are warranted to be free from manufacturing and material defects for a period of two (2) years from the date of shipment from Norav or the dealer to the original purchaser.

Excluded from this warranty are expendable supply items including, but not limited to, electrodes, lead wires, patient cables, and batteries. This warranty does not apply to any product that Norav determines to have been modified or damaged by the customer.

Except for the express warranties stated above, Norav disclaims all warranties, including implied warranties of merchantability and fitness. The stated express warranties are in lieu of all obligations or liabilities on the part of Norav for damages, including but not limited to, special, indirect, or consequential, arising from or in connection with the use or performance of Norav products. Any action for breach of warranty shall be commenced within one (1) year of said breach or be forever barred. Any repairs made to the product that are not covered by the warranty shall be billed to the customer.

For service or technical support, contact your local supplier or Norav Medical.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 6 of 393

2. Introduction

These instructions for use explain in detail how to operate the Norav Medical NH-301 Holter analysis system and will guide you through the features of the software and their relevant controls.

2.1. Document Conventions

Before working with the NH-301 Holter analysis system, review this section to familiarize yourself with the recommended standards, labels, and practices used throughout this manual. These conventions are designed to ensure clarity, accuracy, and safety for both medical personnel and patients.

2.1.1. Warning, Caution, and Note Labels in this Instruction

Pay particular attention to specific items in this instruction when you see one of the following labels:



Warning:

Warning labels highlight potential hazards that might cause damage or injury to individuals.



CAUTION:

Cautions indicate practices to prevent equipment damage or loss. Always follow these instructions closely.



Note:

Notes highlight key information for optimal software performance or important steps or procedures requiring careful attention.

2.1.2. Abbreviations and Acronyms

Acronym	Description	
AHA	American Heart Association	
ECG	Electrocardiogram	
EMC	Electro Magnetic Compatibility	
HRV	Heart Rate Variability	
ID	Patient Identifier	
IEC	International Electrotechnical Commission	
МІ	Myocardial Infarction	
NH	Norav Holter	
NR	Norav Recorder	
Record	ECG test	
ST	ST segment: a time period from the end of the S wave and the start of the T wave on an ECG readout	
USB	Universal Serial Bus	

2.2. Safety Measures

Before operating the NH-301 Holter analysis system, read key safety guidelines to ensure optimal device performance and the safety of both professionals and patients:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 7 of 393

- The Instructions for Use are integral to the NH-301 Holter analysis system. Always keep them stored near the device for reference.
- The NH-301 Holter analysis system is designed for use under clinical supervision. Only personnel with proper healthcare training should operate and interpret its results. This system isn't designed for critical-care applications, e.g. emergency units. Medical institutions using this system must ensure its use is restricted to qualified personnel and suitable patients. Patients should not access or handle the software, or any computer where it's installed.
- While this software is designed using advanced technology, errors are possible. Before prescribing therapeutic measures based on NH-301 or its modules' analysis results, consult an expert and verify with other diagnostic methods.
- Using non-approved third-party software with the NH-301 analysis system might result in conflicts. If such software
 doesn't state its compatibility with NH-301 beyond any doubt, consult the software manufacturer or a relevant
 expert.
- For patient safety, optimal equipment performance, maintenance, and interference immunity use only accessories and consumables recommended by Norav Medical.
- Prior to use, always check the equipment's functionality and condition.
- Devices emitting magnetic and electrical fields can affect instrument functionality. Ensure that all adjacent non-Norav Medical devices meet EMC standards (regulations for Electro Magnetic Compatibility). Notably, X-Ray and Tomographs, due to higher allowed electromagnetic emissions, may interfere with other equipment.



CAUTION: Failing to follow the operation instructions in this manual may result in improper analysis of the data. The manufacturer accepts no liability for damages resulting from improper use.

2.3. Intended Use

The NH-301 Holter analysis system is intended for patients requiring ambulatory Holter cardiac monitoring from 1 to 336 hours (14 days), which is most frequently used for the following indications:

- Evaluation of symptoms suggesting arrhythmia or myocardial ischemia.
- Evaluation of ECG documenting therapeutic interventions in individual patients or patient groups.
- Evaluation of patients for ST segment changes.
- Evaluation of a patient's response after resuming occupational or recreational activities, e.g., after a Myocardial Infarction (MI) or cardiac surgery.
- Clinical and epidemiological research studies.



Note: The NH-301 Holter analysis system contains Heart Rate Variability (HRV) measurements. The clinical significance of HRV measurements should be determined by a physician.

2.3.1. Evaluation of Patients with Pacemaker

The NH-301 Holter analysis system works in conjunction with a compatible ECG recorder to record and analyze pacemaker activities. Please refer to the Supported Holter Recorders (on page 9) section to obtain detailed information on which models of Holter Recorders are compatible with this analysis system.

2.4. Contraindications for Use and Adverse Effects

There are no known contraindications or adverse effects associated with using Holter monitoring equipment.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025	

2.5. General Specifications and Sample Rates

The NH-301 Holter analysis system reads and analyzes data recorded at sample rates of 128, 250, 500, and 1000 (or 128, 256, 512 and 1024 for NR-314-P device) samples per second, with recording durations ranging from 1 hour to 336 hours (14 days).

Patient data, including raw ECG and other relevant information, are stored under identifiers such as Patient ID, Last Name, First Name, among others, for future review, editing, or reanalysis.

The system offers comprehensive arrhythmia detection based on high-precision beat analysis data. It also provides Pacemaker, Respiration, and Accelerometer analysis, ST analysis, and HRV analysis. Standard event types extracted and reported by the system include:

Arrhythmia Event Types Notations
Pause
Bradycardia
Tachycardia
Minimum Heart Rate
Maximum Heart Rate
Minimum RR
Maximum RR
VPB (Ventricular Premature Beats or Premature Ventricular Contraction)
VPB Couplet (Ventricular Couplet)
VPB Triplet (Ventricular Triplet)
VPB Tachycardia (Ventricular Tachycardia)
VPB Flutter
VPB Bigeminy (Ventricular Bigeminy)
VPB Trigeminy (Ventricular Trigeminy)
SVE (Supraventricular Ectopic Beat)
SVE Couplet (Supraventricular Couplet)
SVE Triplet (Supraventricular Triplet)
SVE Tachycardia (Supraventricular Tachycardia)
SVE Flutter
SVE Bigeminy (Supraventricular Bigeminy)
SVE Trigeminy (Supraventricular Trigeminy)
Atrial Fibrillation
Atrial Flutter
ST (ST Elevation or Depression)

Pacemaker Event Types Notations		
Paced Beats (Total Paced Beats)		
Atrial Paced (Atrial Paced Beats)		

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 9 of 393

Ventricular Paced (Ventricular Paced Beats)
AV Paced (AV Paced Beats)
Capture Failures
Sense Failures
Inhibitions

Respiration Event Types Notations
Sleep Apnea

In addition to the standard list of Event Types that the NH-301 system can extract and analyze, there are also user-generated events:

- **Custom Events**: Users can add events of this type based on their specific needs.
- **Unconducted Pacemaker Event**: It's recommended to utilize this event type when pacemaker data in a record appears to be evidently incorrect.

It's crucial to emphasize that these specific events are not automatically detected and extracted by the system. Users must create them manually.

Another unique Event Type is Diary events. These can be entered by the patient directly into the Recorder, provided the recorder offers this function. In such instances, the analysis system will automatically detect them during the ECG analysis. If the events were tracked using an offline diary for the Holter monitoring, medical personnel would need to add Diary events manually.

2.6. Supported Holter Recorders

The NH-301 Holter analysis system is compatible with the following Norav Medical Recorders:

· · · · · ·
Recorder Model
Norav NR-302/NR-314/NR-1207/NR-1207-3/NR-314-P Holter Recorders
Norav DL800 Holter Recorder
Norav DL900 Holter Recorder
Norav DL1200 Holter Recorder

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 10 of 393

3. Software Installation and Setup

This chapter provides an overview of the essential steps required to install and set up the Norav Medical NH-301 software. Additionally, you'll find valuable tips on maintaining both the software and data.



CAUTION:

Before beginning the installation, carefully read the following instructions to prevent potential data loss or permanent damage to the computer and its peripherals.

3.1. Recommended PC Specifications

To ensure optimal performance, the hardware on which NH-301 operates should meet or exceed the specifications listed below. Using hardware with characteristics lower than recommended specifications may compromise the software's efficiency, leading to slower operations or potential instability. Note that if a laptop meets the specifications listed below, it can also support the NH-301 software.

Component	Recommended Configuration	Minimum Configuration	
Operating System	Windows [®] 10/11 Pro 64-bit		
СРИ	Intel [®] Core [™] i9		
RAM	2 x 16 GB 3200 MHz	1 x 16 GB	
Data Storage	1 TB Secondary SSD	250 GB Secondary SSD	
Graphics	Discrete video card GeForce 2600 or equivalent		
Display Resolution	1920 x 1200, HD+ or higher 1600 x 900		
USB Ports	4 x USB A slots for: Software license key Memory card reader for downloading recorder data Preparing recorders for a new study Local printer to printout analysis results, reports, etc.		
Memory Card Reader	SD card reader for NR-series devices (excluding the NR-314-P model) or DL900 Norav recorder; CF card reader for DL800 and DL1200 Norav recorders		
Network Capability	TCP/IP network interface		
Printer	A4 or Letter format standard color printer, Laser or InkJet		

3.2. Installation Process

The installation process for the Norav Medical NH-301 software is designed to run automatically and doesn't require any specific skills to complete. Note that during the installation, certain third-party software, as listed below, will be installed on your PC to ensure the proper functioning of the Norav Medical NH-301.

To install the Norav Medical NH-301 analysis system software, it's advised to close all active applications and programs. This will prevent potential errors during the installation and setup. Follow these steps for a smooth installation:

- 1. Either click the provided link to download the NH-301 installation pack or insert a data storage device with the pack into an available USB slot on your computer.
- 2. If you're using a USB, the installation process should start automatically. If it doesn't, or you downloaded the installation package via the link, do the following:

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 11 of 393

- 1. Open a Windows Explorer window.
- 2. Navigate to the respective drive and directory (e.g. D:\, F:\) or access the location where the installation package (often in ZIP format) was downloaded.
- 3. **(Optional)** If you downloaded the installation package in a .zip file, double-click to access its contents. A new Windows Explorer window showcasing the archive's contents will appear.
- 4. Double click on the **NoravHolterSetup.exe** file within the Windows Explorer window to launch the installation process.
- 3. Choose the language when the installer prompts you. The language you select will be set as the default for the installed software.
- 4. Adhere to the on-screen instructions until the installation is complete and a success message appears.



Note: During the installation the following third-party software will be installed on your computer:

- Relevant version of Matlab Runtime, if not already on your PC.
- HASP security token driver to ensure the NH-301 analysis system's security.
- Required version of C++ Prerequisites.
- Relevant version of .NET, including support for legacy .NET versions.
- 5. The installation is complete, please restart your computer.

3.3. Data Structure after Automatic Installation

NH-301 Holter Software Path:

Default Location: C:\Program Files\Norav\Holter

NH-301 Holter Software Settings:

 Configurations such as layout, setup, and department name are stored by default in: C:\Program Files\Norav \Holter

Raw Data and Analysis Results Storage Path:

- Default Location: C:\Norav\Holter\Data
- All ECG raw data files (*.nrr) and analysis results (*.hl5) are stored here by default.

3.4. Setup Process after Installation

Once the installation is complete, you have the option to customize the NH-301 software settings according to your requirements.

To access the NH-301 software settings, follow these steps:

1. Double-click the **Norav Holter** icon on your desktop to launch the NH-301 analysis system.


Figure 1. Norav Holter Icon



2. On the initial screen, navigate to the menu bar located at the top left, select **File**, and then choose **Settings** from the drop-down list.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 12 of 393

Figure 2. Selecting Settings from the File menu

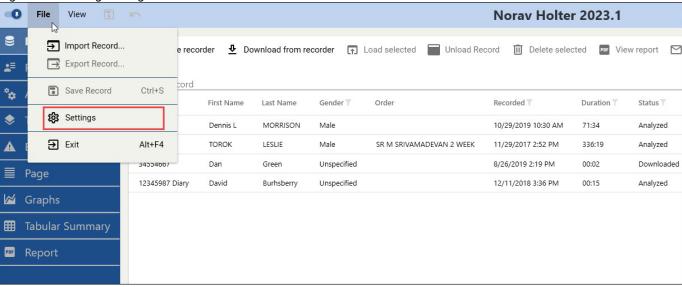

- 3. A **Settings** dialog box will open.
- 4. Use the tabs in the left pane to access and either check or modify specific settings.

Figure 3. Settings Dialog Box - Defaults Tab

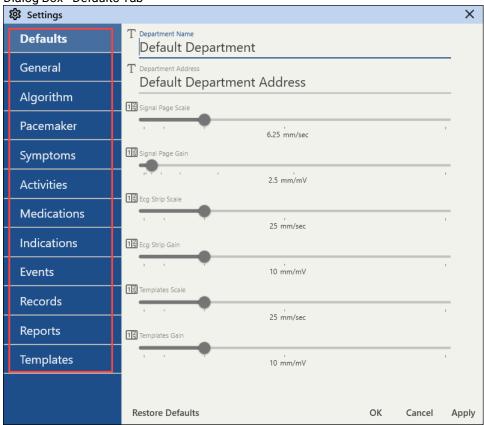

3.4.1. Defaults Tab

The Defaults Tab contains a list of global default settings for tailoring various NH-301 analysis system **Views** to your preferences.


This tab presents two primary groups of settings:

Department Settings

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 13 of 393

- **Department Name**: Enter your Department Name. This name will serve as a default parameter in relevant fields and forms across the analysis system, such as reports.
- **Department Address**: Enter your Department Address. This address will serve as a default parameter in relevant fields and forms across the analysis system, such as reports.

Figure 4. Defaults Tab - Settings Groups

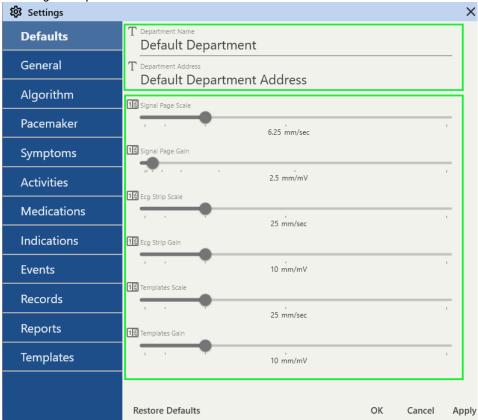


Scale and Gain Settings

- Signal Page Scale: Drag the slider to adjust the scale of the ECG waveform in the Signal Page.
- Signal Page Gain: Drag the slider to modify the gain of the ECG waveform in the Signal Page.
- **ECG Strip Scale**: Drag the slider to adjust the scale of the ECG Strip.
- ECG Strip Gain: Drag the slider to modify the gain of the ECG Strip.
- **Templates Scale**: Drag the slider to adjust the scale in the Templates View.
- Templates Gain: Drag the slider to modify the gain within the Templates View.

Scale and gain are two important settings that control the size and amplitude of the ECG waveform across diverse NH-301 analysis system **Views** and **View** components, including **Analysis**, **Templates**, and **Events Views**. Ensure you adjust the scale and gain settings appropriately to obtain an optimized ECG waveform display.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 14 of 393

Figure 5. Analysis, Templates, Events Views

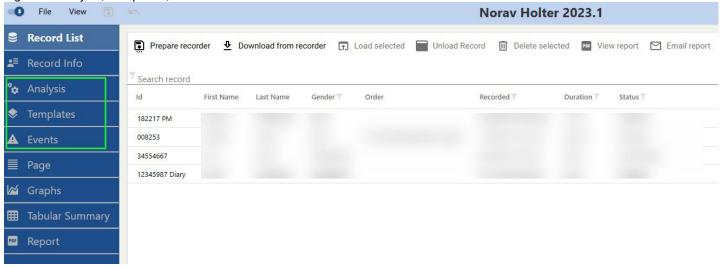



Note: If the ECG waveform is too small, the scale can be increased. If the ECG waveform is too noisy, the gain can be decreased. The scale and gain settings should be adjusted carefully to ensure the ECG waveform is displayed accurately. Incorrect settings can make interpreting the ECG waveform and diagnosing heart conditions difficult.

3.4.1.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 6. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click Yes to restore default settings and exit the Settings dialog box.
 - Click No to abort default settings restoration and return to the Settings dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Figure 7. Restore Default Settings Box




3.4.2. General Tab

The General Tab contains a list of general settings affecting NH-301 analysis system behavior and output parameters.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 15 of 393

Figure 8. Settings Dialog Box - General Tab

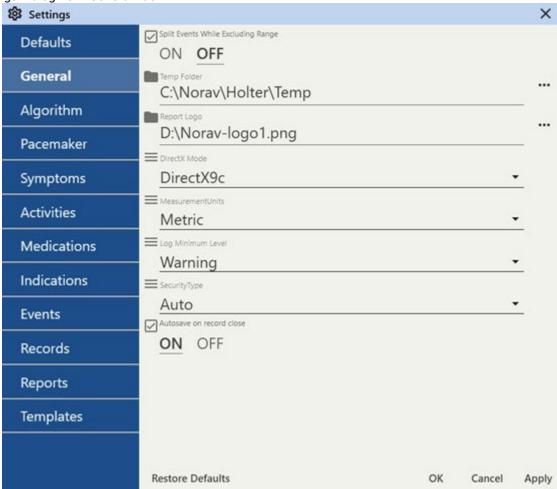


Split Events While Excluding Range

Select **ON** to divide each continuous event that contains one or more "Exclude from analysis" areas into an equivalent number of separate events and standalone "Exclude from analysis" areas.

Select **OFF** to maintain each continuous event, even if it has one or more "Exclude from analysis" areas. Note that in this mode, event parameters such as duration, HRV, etc., will consider these "Exclude from analysis" areas. The event will not be fragmented into multiple events but will remain a single event.

Figure 9. General Tab - Split Events

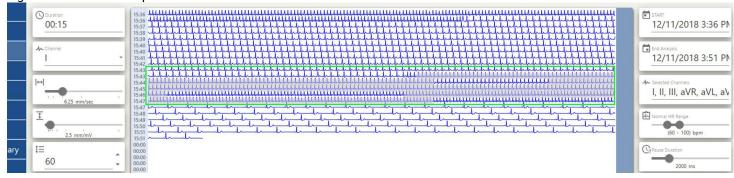



Note: This function applies to both existing events and new events you overlay on an "Exclude from analysis" area.

Document Title:	NH	301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 16 of 393

Set the temporary folder for the NH-301 analysis system using one of two methods:

Figure 10. General Tab - Temp Folder Settings



- Type in a new **Temp Folder** path in the text field.
- Click the ellipsis button on the right to open a Windows Explorer Window, and then:
 - 1. Navigate to the desired location on your PC.
 - 2. Select or double-click the folder you want to use as a temporary folder for the NH-301.
 - 3. Click the **Select Folder** button in the bottom right corner of the Windows Explorer Window.

Once done, you will see the new **Temp Folder** path displayed in this Tab.

Report Logo

Set the path for the **Report Logo** which will appear in the report output, using one of two methods:

Figure 11. General Tab - Report Logo Settings



- Type in a new path to the logo in the text field.
- Click the ellipsis button on the right to open a Windows Explorer Window, and then:
 - 1. Navigate to the desired folder on your PC.
 - 2. Select a .png file you want to use as a report logo.
 - 3. Click the **Open** button in the bottom right corner of the Windows Explorer Window.

Once done, you will see the new **Report Logo** path displayed in this tab.

DirectX Mode

Select an appropriate mode from the drop-down list depending on your PC configuration:

Figure 12. General Tab - DirectX Mode Settings



- None
- AutoDetect this option detects the suitable mode automatically.
- DirectX9c

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 17 of 393

DirectX11c

Measurement Units

Select your preferred measurement units from the drop-down list:

- · Metric for metric units.
- Imperial for US units.

Log Minimum Level

Select the **Log Minimum Level** from the drop-down list to set the log sensitivity grade:

- Verbose all types of system notifications are logged.
- Debug debug-level and all higher-level notifications are logged.
- Information information-level and all higher-level notifications are logged.
- Warning only warning notifications and errors are logged.
- Error errors and fatal errors are logged.
- Fatal only fatal system errors are logged.

Security Type

Select the NH-301 analysis system **Security Type** from the drop-down list:

- Auto autodetects the security type in use.
- Hasp select this option if using a HASP security token.
- SecureX select this option if using a SecureX security token.

Autosave on Record Close

Select **ON** to save a patient record every time you close it.

Select **OFF** if you prefer to close patient records without saving them.

3.4.2.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 13. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click Yes to restore default settings and exit the Settings dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 18 of 393


Figure 14. Restore Default Settings Box



3.4.3. Algorithm Tab

The Algorithm Tab contains a list of algorithmic settings directly affecting the analysis process of the patients' ECG records.

Figure 15. Settings Dialog Box - Algorithm Tab

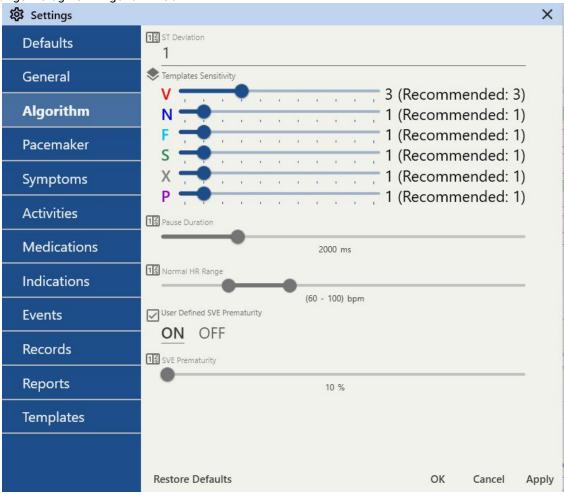

ST Deviation

Input the minimum deviation value for detecting ST events (either elevation or depression) during the analysis.

Templates Sensitivity

Drag relevant sliders to set the analysis sensitivity level for each beat template type as recommended or set it to 0 to disable certain template types. Higher sensitivity levels will extract more distinct beat templates of the relevant type during analysis.



Note: By disabling certain template types you can speed up the Analysis step, weigh the importance of analysis accuracy against optimization.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 19 of 393

Drag the slider to set the detection threshold for an ECG pause, ranging from 1000 ms to 6000 ms.

Normal HR Range

Drag double slider controls to set the minimum and maximum heart rate values. Any significant deviations from this range will be classified as Tachycardia or Bradycardia.

User Defined SVE Prematurity

Select **ON** to set new user conditions for detecting supraventricular premature beats. The **SVE Prematurity** value then serves as a deviation benchmark for comparing the current RR interval with the moving average of the three previous normal RR intervals.

Select **OFF** to use default settings for detecting supraventricular premature beats.

SVE Prematurity

Drag a slider to set a new deviation value for detecting supraventricular premature beats, with options ranging from 10% to 40%.

If the deviation between current RR interval and the moving average of the three last normal RR intervals is smaller than the set value, the system will reclassify relevant events SVE event (S) as a Normal event (N).

For instance, if the SVE Prematurity is set to 20%, and the current RR interval deviation is 15%, the corresponding SVE event (S) will be reclassified as Normal (N).

3.4.3.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 16. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click OK to save and exit the Settings dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click Yes to restore default settings and exit the Settings dialog box.
 - Click No to abort default settings restoration and return to the Settings dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Figure 17. Restore Default Settings Box




3.4.4. Pacemaker Tab

The Pacemaker Tab contains a list of settings for configuring pacemaker beats analysis.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 20 of 393

Figure 18. Settings Dialog Box - Pacemaker Tab

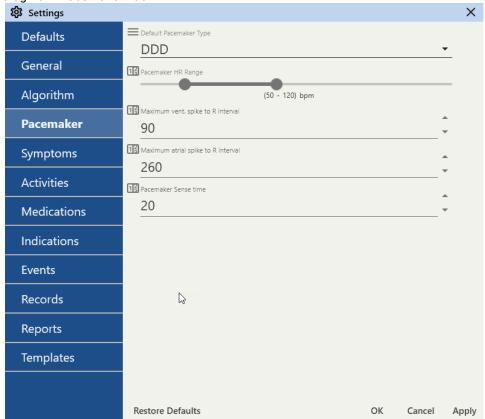



Note: When pacemaker detection is activated by setting it to **ON** in the **Recorder** menu, these parameters become available both on the "Download from recorder" page and on the main page of the Analysis View (on page 83). On these pages, you can also adjust the pacemaker parameters to match the actual configuration of the patient's implanted device.

Figure 19. Recorder - Pacer Detector is ON




Default Pacemaker Type

Select one of the available pacemaker types from the drop-down list to consider pacemaker beats during analysis:

Figure 20. Pacemaker Tab - Default Pacemaker Type Settings

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 21 of 393

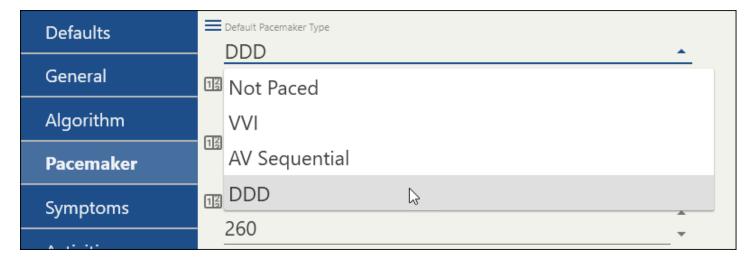

- Not Paced
- WI
- AV Sequential
- DDD

Pacemaker HR Range

Drag double slider controls to set the minimum and maximum heart rate values initiated by the pacemaker.

Maximum vent. spike to R interval

Set this parameter by using the picker arrows on the right. It represents the longest time a pacemaker can wait between delivering a ventricular pacing spike and the occurrence of the following R wave.

Maximum atrial spike to R interval

Set this parameter by using the picker arrows on the right. It represents the longest time that a pacemaker can wait between delivering an atrial pacing spike and the following ventricular pacing spike.

Pacemaker Sense Time

Set this parameter using the picker arrows on the right. It signifies the amount of time a pacemaker waits after detecting a heartbeat before administering another pacing stimulus.

3.4.4.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 21. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click OK to save and exit the Settings dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 22 of 393

- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click Yes to restore default settings and exit the Settings dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Figure 22. Restore Default Settings Box



3.4.5. Symptoms Tab

The **Symptoms Tab** contains a list of predefined symptoms that can be used with the **Patient Diary** (on page 388) feature. You can modify or delete existing predefined items, or add new ones. If the full list of symptoms doesn't fit on your PC screen, use the scroll function to access the complete list.

Figure 23. Settings Dialog Box - Symptoms Tab




Editing Existing Symptoms Items

To modify an existing symptom entry in the **Symptoms Tab**, follow these steps:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 23 of 393

Figure 24. Symptoms Tab - Editing Item

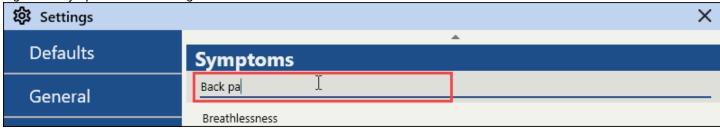


- 1. Click the line containing the symptom you want to edit.
- 2. Update the line's content as required.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Symptoms Tab** after editing the necessary entries.

After you finish **Step 3**, you should see the updated name of the symptom that you just modified.

Removing Existing Symptoms Items

To remove an existing symptom entry in the **Symptoms Tab**, follow these steps:

Figure 25. Symptoms Tab - Removing Item

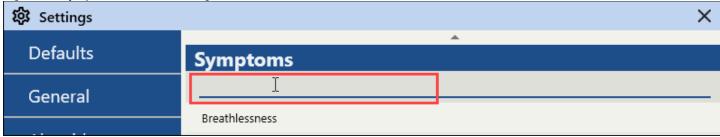

- 1. Click the line containing the symptom you want to remove.
- 2. Delete the line's content by selecting the text you want to remove and pressing Backspace or Delete on your keyboard.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Symptoms Tab** after editing the necessary entries.

After you finish **Step 3**, you should see that the item is removed from the list.

Adding New Symptoms Item

To add a new symptom entry in the **Symptoms Tab**, follow these steps:

Figure 26. Symptoms Tab - Adding Item



- 1. Scroll down to the very bottom of the symptoms list until you see the "Add New" line.
- 2. Click the "Add New" line.
- 3. Enter the new symptom you want to add to the symptoms list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 24 of 393

- 4. Press Enter on your keyboard to confirm your changes.
- 5. Remember to save the changes in the **Symptoms Tab** after editing the necessary entries.

After you finish **Step 4**, you should see a new line with the symptom you just typed in.

3.4.5.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 27. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click **Restore Defaults** to restore default settings. The **Restore Default Settings** dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Figure 28. Restore Default Settings Box




3.4.6. Activities Tab

The **Activities Tab** contains a list of predefined daily activities for patients for use with the **Patient Diary** (on page 388) feature. You can modify or delete existing predefined items, or add new ones. If the full list of activities doesn't fit on your PC screen, use the scroll function to access the complete list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 25 of 393


Figure 29. Settings Dialog Box - Activities Tab

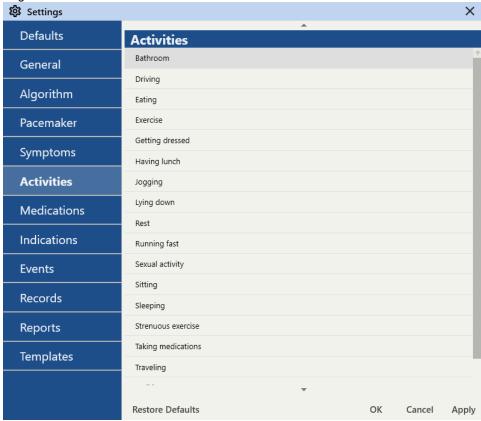

Editing Existing Activities Items

To modify an existing activity entry in the **Activities Tab**, follow these steps:

Figure 30. Activities Tab - Editing Item

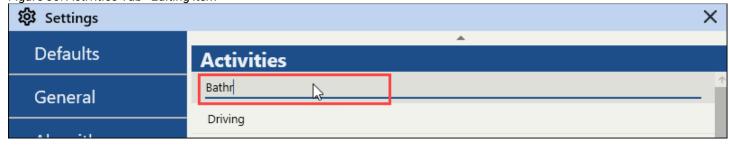

- 1. Click the line containing the activity you want to edit.
- 2. Update the line's content as required.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Activities Tab** after editing the necessary entries.


After you finish **Step 3**, you should see the updated name of the activity that you just modified.

Removing Existing Activities Items

To remove an existing activity entry in the **Activities Tab**, follow these steps:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 26 of 393

Figure 31. Activities Tab - Removing Item

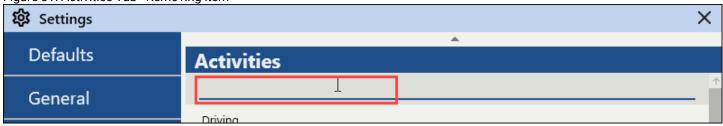

- 1. Click the line containing the activity you want to remove.
- 2. Delete the line's content by selecting the text you want to remove and pressing Backspace or Delete on your keyboard.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Activities Tab** after editing the necessary entries.

After you finish **Step 3**, you should see that the item is removed from the list.

Adding New Activities Item

To add a new activity entry in the **Activities Tab**, follow these steps:

Figure 32. Activities Tab - Adding Item



- 1. Scroll down to the very bottom of the activities list until you see the "Add New" line.
- 2. Click the "Add New" line.
- 3. Enter the new activity you want to add to the activities list.
- 4. Press Enter on your keyboard to confirm your changes.
- 5. Remember to save the changes in the **Activities Tab** after editing the necessary entries.

After you finish **Step 4**, you should see a new line with the activity you just typed in.

3.4.6.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 33. Saving or Discarding Changes



- Click **Apply** to save the changes and continue working within the **Settings** dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click **Restore Defaults** to restore default settings. The **Restore Default Settings** dialog box will appear:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 27 of 393

- Click **Yes** to restore default settings and exit the **Settings** dialog box.
- Click No to abort default settings restoration and return to the Settings dialog box.
- Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.


Figure 34. Restore Default Settings Box



3.4.7. Medications Tab

The **Medications Tab** contains a list of medications taken by the patient on a regular basis. You can modify or delete existing predefined items, or add new ones. If the full list of medications doesn't fit on your PC screen, use the scroll function to access the complete list.

Figure 35. Settings Dialog Box - Medications Tab

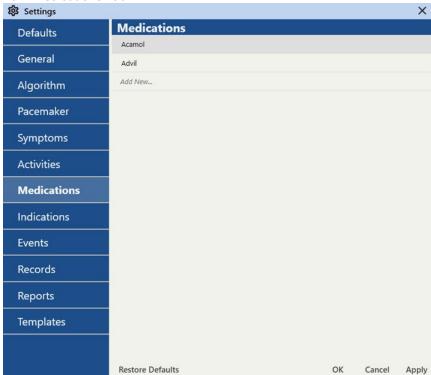

Editing Existing Medications Items

To modify an existing medication entry in the **Medications Tab**, follow these steps:

Figure 36. Medications Tab - Editing Item



- 1. Click the line containing the medication you want to edit.
- 2. Update the line's content as required.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 28 of 393

- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Medications Tab** after editing the necessary entries.

After you finish Step 3, you should see the updated name of the medication that you just modified.

Removing Existing Medications Items

To remove an existing medication entry in the **Medications Tab**, follow these steps:

Figure 37. Medications Tab - Removing Item



- 1. Click the line containing the medication you want to remove.
- 2. Delete the line's content by selecting the text you want to remove and pressing Backspace or Delete on your keyboard.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Medications Tab** after editing the necessary entries.

After you finish **Step 3**, you should see that the item is removed from the list.

Adding New Medications Item

To add a new medication entry in the **Medications Tab**, follow these steps:

Figure 38. Medications Tab - Adding Item



- 1. Scroll down to the very bottom of the medications list until you see the "Add New" line.
- 2. Click the "Add New" line.
- 3. Enter the new medication you want to add to the medications list.
- 4. Press Enter on your keyboard to confirm your changes.
- 5. Remember to save the changes in the **Medications Tab** after editing the necessary entries.

After you finish **Step 4**, you should see a new line with the medication you just typed in.

3.4.7.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 29 of 393

Figure 39. Saving or Discarding Changes



- Click **Apply** to save the changes and continue working within the **Settings** dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click **Cancel** to abort changes and exit the **Settings** dialog box.
- Click **Restore Defaults** to restore default settings. The **Restore Default Settings** dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.

Figure 40. Restore Default Settings Box




3.4.8. Indications Tab

The Indications Tab contains a list of predefined indications for patients. You can modify or delete existing predefined items, or add new ones. If the full list of indications doesn't fit on your PC screen, use the scroll function to access the complete list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 30 of 393


Figure 41. Settings Dialog Box - Indications Tab

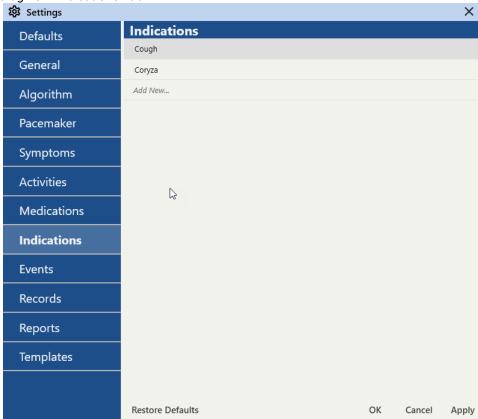

Editing Existing Indications Items

To modify an existing indication entry in the **Indications Tab**, follow these steps:

Figure 42. Indications Tab - Editing Item

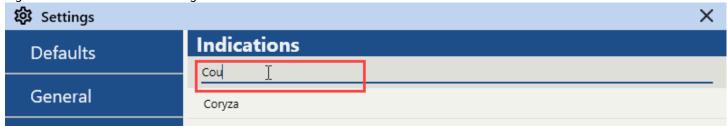

- 1. Click the line containing the indication you want to edit.
- 2. Update the line's content as required.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Indications Tab** after editing the necessary entries.


After you finish **Step 3**, you should see the updated name of the indication that you just modified.

Removing Existing Indications Items

To remove an existing indication entry in the **Indications Tab**, follow these steps:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 31 of 393

Figure 43. Indications Tab - Removing Item

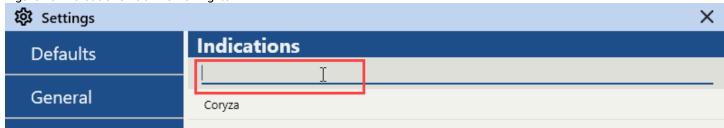


- 1. Click the line containing the indication you want to remove.
- 2. Delete the line's content by selecting the text you want to remove and pressing Backspace or Delete on your keyboard.
- 3. Press Enter on your keyboard to confirm your changes.
- 4. Remember to save the changes in the **Indications Tab** after editing the necessary entries.

After you finish **Step 3**, you should see that the item is removed from the list.

Adding New Indications Item

To add a new indication entry in the **Indications Tab**, follow these steps:

Figure 44. Indications Tab - Adding Item

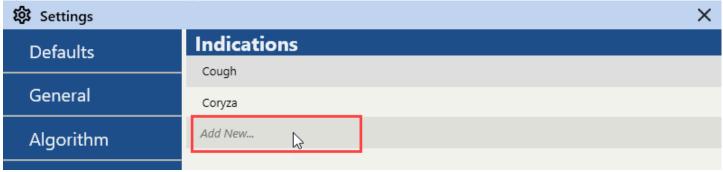

- 1. Scroll down to the very bottom of the indications list until you see the "Add New" line.
- 2. Click the "Add New" line.
- 3. Enter the new indication you want to add to the indications list.
- 4. Press Enter on your keyboard to confirm your changes.
- 5. Remember to save the changes in the **Indications Tab** after editing the necessary entries.

After you finish **Step 4**, you should see a new line with the indication you just typed in.

3.4.8.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 45. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click OK to save and exit the Settings dialog box.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 32 of 393

- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - \circ Click No to abort default settings restoration and return to the Settings dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.


Figure 46. Restore Default Settings Box



3.4.9. Events Tab

The **Events Tab** contains sliders to configure the layout of the **Events Panel** in the **Events View**. These controls allow you to adjust how the events blocks within the **Events Panel** are displayed.

Figure 47. Settings Dialog Box - Events Tab

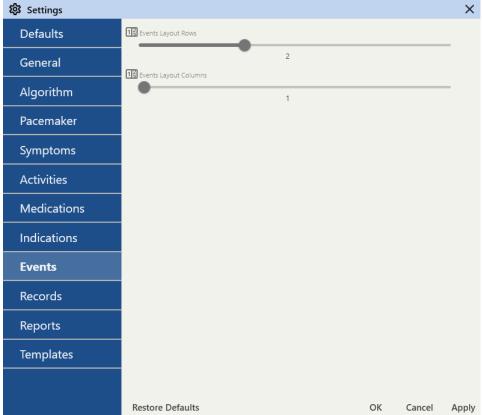

Events Layout Rows

Drag this slider to set the number of rows for the **Events Panel** layout, ranging from 1 to 4. This adjustment affects how many rows of events blocks are visible in the **Events Panel**.

Events Layout Columns

Drag this slider to set the number of columns for the **Events Panel** layout, ranging from 1 to 10. This adjustment affects how many columns of events blocks are visible in the **Events Panel**.



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 33 of 393



The combination of these settings allows you to change the default layout, determining the total number of event blocks that you can see at the same time, ranging from 1 to 40. Customize this default setting to suit your preferences and viewing needs.

For example, if you set the **Events Layout Rows** value to 2 and the **Events Layout Columns** value to 5, you will see a total of 10 event blocks in the **Events Panel**.

Figure 48. Events View - Events Panel



3.4.9.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 49. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click OK to save and exit the Settings dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click Cancel to abort default settings restoration and exit the Settings dialog box.

Figure 50. Restore Default Settings Box




3.4.10. Records Tab

The **Records Tab** contains the **Base Data Folder** path, where the patients' records are stored on your PC. In this Tab, you have the option to modify the default path to a location of your choice.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 34 of 393


Figure 51. Settings Dialog Box - Records Tab

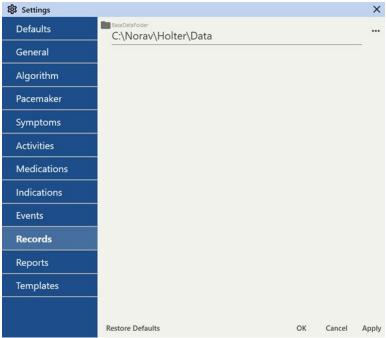

Base Data Folder

To set the default **Base Data Folder** path, where the NH-301 analysis system will store patients' records on your PC, you can use one of two methods:

Figure 52. Records Tab - Base Data Folder Settings

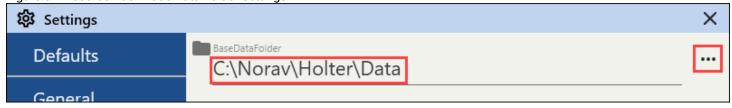

- Type in a new Base Data Folder path in the text field.
- Click the ellipsis button on the right to open a Windows Explorer Window, and then:
 - 1. Navigate to the desired location on your PC.
 - 2. Select or double-click the folder you want to use as a **Base Data Folder** for the NH-301.
 - 3. Click the **Select Folder** button in the bottom right corner of the Windows Explorer Window.

Once done, you will see the new **Base Data Folder** path displayed in this Tab.

3.4.10.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 53. Saving or Discarding Changes



Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 35 of 393

- Click OK to save and exit the Settings dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click **Restore Defaults** to restore default settings. The **Restore Default Settings** dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - Click **No** to abort default settings restoration and return to the **Settings** dialog box.
 - Click **Cancel** to abort default settings restoration and exit the **Settings** dialog box.


Figure 54. Restore Default Settings Box



3.4.11. Reports Tab

The **Reports Tab** contains the default settings that influence the report output of the NH-301 analysis system.

Figure 55. Settings Dialog Box - Reports Tab

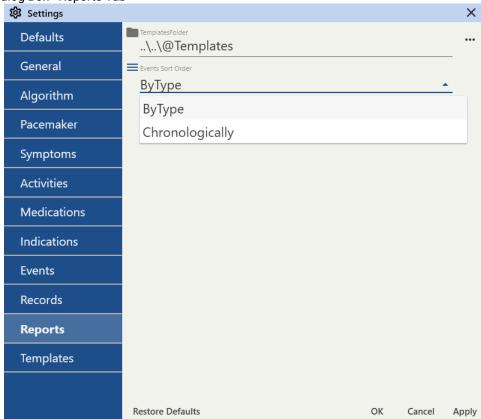


Templates Folder

To set the default **Templates Folder** path, where the NH-301 analysis system will store report templates on your PC, you can use one of two methods:

Figure 56. Report Tab - Templates Folder Settings

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 36 of 393

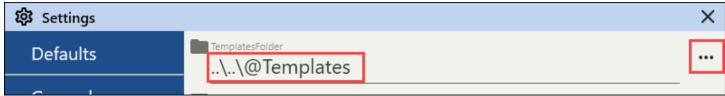

Type in a new **Templates Folder** path in the text field.


- Click the ellipsis button on the right to open a Windows Explorer Window, and then:
 - 1. Navigate to the desired location on your PC.
 - 2. Select or double-click the folder you want to use as a **Templates Folder** for the NH-301.
 - 3. Click the **Select Folder** button in the bottom right corner of the Windows Explorer Window.

Once done, you will see the new **Templates Folder** path displayed in this Tab.

Events Sort Order

Choose the default method by which the NH-301 analysis system will present events in the report output, using the drop-down list:

Figure 57. Report Tab - Events Sort Order Settings

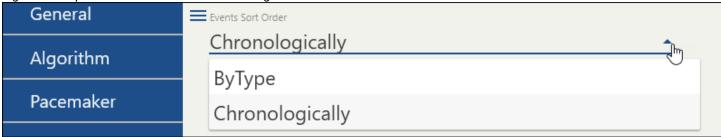

- By Type
- Chronologically



Note: You can also modify the **Events Sort Order** directly in the **Report View** (on page 309), just before selecting the **Generate Report** button.

3.4.11.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 58. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.
- Click Restore Defaults to restore default settings. The Restore Default Settings dialog box will appear:
 - Click Yes to restore default settings and exit the Settings dialog box.
 - Click No to abort default settings restoration and return to the Settings dialog box.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 37 of 393

• Click Cancel to abort default settings restoration and exit the Settings dialog box.


Figure 59. Restore Default Settings Box



3.4.12. Templates Tab

The **Templates Tab** contains options to configure the process of how you work with beat templates in the **Templates View**.

Figure 60. Settings Dialog Box - Templates Tab

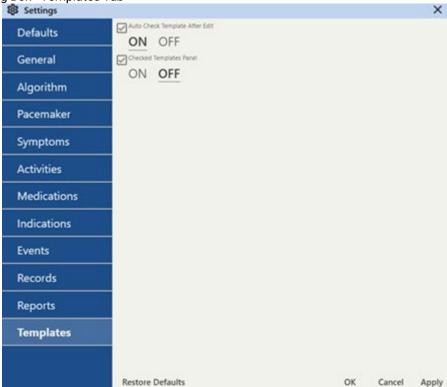

Auto Check Template After Edit

Select **ON** to automatically assign the "Checked" status to beat templates after you complete editing them. This option can help you speed up the templates review process.

Select **OFF** if you prefer to assign the "Checked" status to beat templates manually. In this case, after you complete editing a template, it won't be marked as a "Checked" one.

Checked Templates Panel

Select **ON** to enable a dedicated panel for checked templates in the top part of the **Templates View**. When it's set to ON, you will also see additional control buttons in the top action bar, allowing you to disable or enable the the **Checked Templates Panel** itself and a display of the checked templates in the panel below the **Checked Templates Panel** (i.e., in the **Templates Panel**).


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 38 of 393	

Figure 61. Templates View - Checked Templates Panel ON



Select **OFF** if you prefer to have this panel off the screen when you are in the **Templates View**.

Figure 62. Templates View - Checked Templates Panel OFF

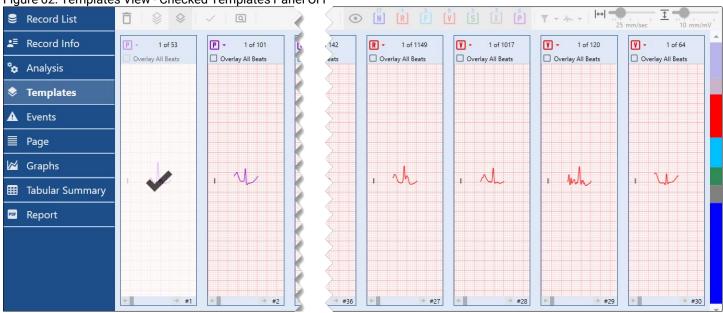

3.4.12.1. Saving or Discarding Settings Changes

After making adjustments, you have several options located in the bottom part of the box:

Figure 63. Saving or Discarding Changes



- Click Apply to save the changes and continue working within the Settings dialog box (like navigating to other Tabs);
- Click **OK** to save and exit the **Settings** dialog box.
- Click Cancel to abort changes and exit the Settings dialog box.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 39 of 393	

- Click **Restore Defaults** to restore default settings. The **Restore Default Settings** dialog box will appear:
 - Click **Yes** to restore default settings and exit the **Settings** dialog box.
 - \circ Click No to abort default settings restoration and return to the Settings dialog box.
 - \circ Click Cancel to abort default settings restoration and exit the Settings dialog box.

Figure 64. Restore Default Settings Box



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 40 of 393	


4. Interface Core Elements Overview

In this chapter, an overview of the basic interface elements of the Holter NH-301 analysis system is provided. You will learn about the functionality of the following user interface (UI) elements: the top menu bar, the top title bar, the window control buttons, and the left side bar with the **Views** menu (including Record List, Analysis, Graphs Views, etc.).

Here, a detailed description of the basic interface elements is provided, along with guidance on how to use them. To enhance clarity, the elements in the figures are distinguished by color:

- Actionable elements, such as buttons and menus, are marked in red.
- Non-actionable elements, including informative elements or specific screen areas, are marked in green.

Figure 65. User Interface Basic Elements

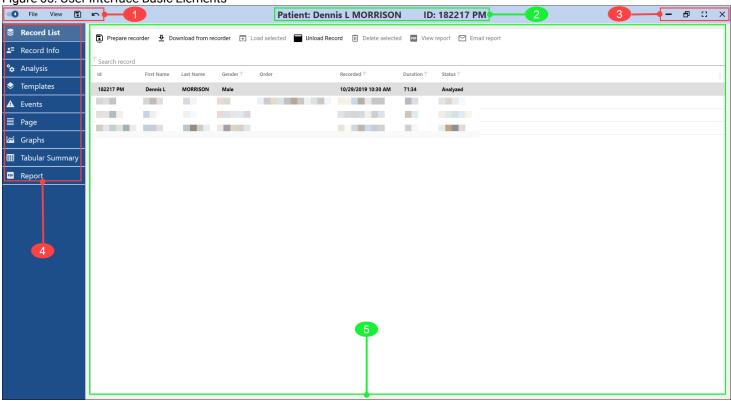

The elements depicted in the figure above are consistently available across all screens and views within the NH-301 analysis system:

- 1. **Menu Bar**: contains menus and buttons that control functions such as records import/export, saving, settings, views, etc.
- 2. **Title Bar Text**: includes the patient's name and patient ID for easy reference.
- 3. Window Control Buttons: features a standard set of controls for window states.
- 4. Views Sidebar: a designated sidebar that allows switching between different Views available within the system.
- 5. **View Area**: unlike other interface elements, which remain largely consistent across screens (except being active or inactive under certain circumstances), this area houses distinct content, data, and controls for each **View** and **View mode**.

Please refer to the relevant sections in this Chapter for detailed descriptions of each element.

4.1. Menu Bar

The **Menu Bar** contains menus and buttons that control various general functions of the NH-301 analysis system, such as importing/exporting records, saving, adjusting settings, switching views, etc. Please note that some buttons or menu items may be inactive until a patient's **Record** is loaded (e.g., the Save Record and Undo icons). A detailed description of

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

the Menu Bar components can be found below.

The **Menu Bar** can be in one of three different states:

• **Default State (No Records Loaded):** When no patient records are loaded in the system, the Save Record, Undo, and Views switch functions and buttons are inactive.

Figure 66. Menu Bar - No Records Loaded



• **Record Loaded (No Actions Made):** When a patient record is loaded in the system but no actions have been made, only the Undo button and Export Record function are inactive.

Figure 67. Menu Bar - Record Loaded, No Actions



• **Record Loaded (Actions Made):** When a patient record is loaded in the system and actions have been taken (e.g., templates edited and checked), only the Export Record function is inactive.

Figure 68. Menu Bar - Record Loaded, Undo Action Active



Sidebar Control Button

The **Sidebar Control Button** allows you to toggle the **Views Sidebar** (Record List, Analysis, Events, Report, etc.), located in the left part of the screen:

Figure 69. Menu Bar - Sidebar Control Button




- **Collapse**: Click the **Sidebar Control Button** when the **Views Sidebar** is expanded to collapse it. This will automatically expand the **View Area**, providing more screen space.
- Expand: Click the Sidebar Control Button when the Views Sidebar is collapsed to expand it, revealing the available Views.

File Menu

The **File** drop-down menu contains general menu items for importing, exporting, and saving patient records, accessing settings, and exiting the application.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 42 of 393

Figure 70. Menu Bar - File Menu

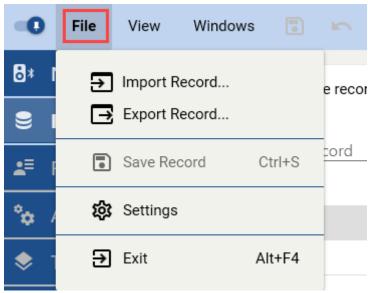


Import Record

This option allows you to import a patient's record from a folder on your PC, local network, or USB drive.

To import a patient's record:

- 1. Click the File menu.
- 2. Click the **Import Record** option to open a Windows Explorer Window:
 - 2.1. Navigate to the desired folder.
 - 2.2. Select one .nrr, .res, or .hl5 file (only one file at a time).
 - 2.3. Click the **Open** button in the bottom right corner.

Figure 71. Import Record - Open Record

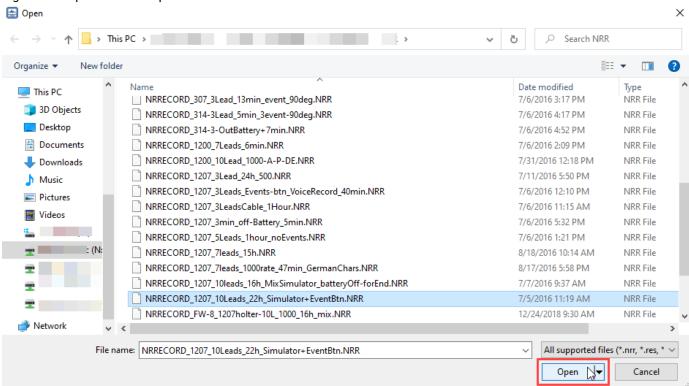


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 43 of 393

A new window for entering the initial information for the patient's record will appear.


3. Fill in the patient's personal information, record information, medications and indications data, and set basic scanning criteria. To accomplish this, complete the following steps:

Figure 72. Import Record - Record Infromation

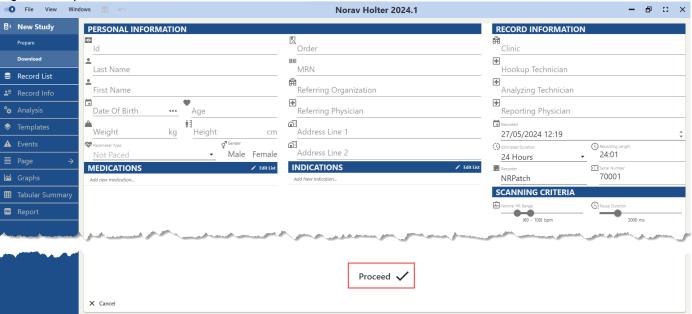

3.1. In the **Personal Information** section:


Figure 73. Import Record - Personal Information

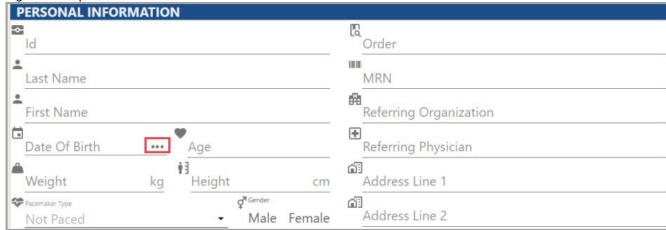

- 3.1.1. Click on each text field (**ID**,**First Name**,**Last Name**, **Age**, etc.) and type in the data according to your needs and workflows.
- 3.1.2. Click on the **Date Of Birth** field and type in the date or use the ellipsis button on the right to select from a calendar.
- 3.1.3. Select the pacemaker from the **Pacemaker Type**, if needed. When performing the import, you can specify the type of the patient's pacemaker if it wasn't specified earlier.
- 3.1.4. Select the patient's gender.
- 3.2. In the **Medications** section:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		


Figure 74. Import Record - Medications

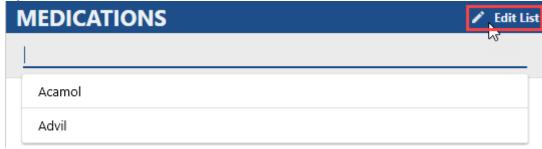

- 3.2.1. Click "Add New Medication" to open a drop-down list with predefined medications.
- 3.2.2. Click on an existing predefined medication, or type in a new medication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined medications.
- 3.2.3. To modify the list of predefined medications, click the **Edit List** icon located at the top right.

3.3. In the **Indications** section:


Figure 75. Import Record - Indications

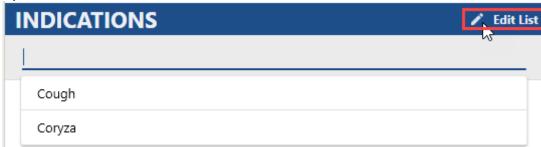

- 3.3.1. Click "Add New Indication" to open a drop-down list with predefined indications.
- 3.3.2. Click on an existing predefined indication, or type in a new indication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined indications.
- 3.3.3. To modify the list of predefined indications, click the **Edit List** icon located at the top right.

3.4. In the **Record Infromation** section:

Figure 76. Import Record - Record Infromation

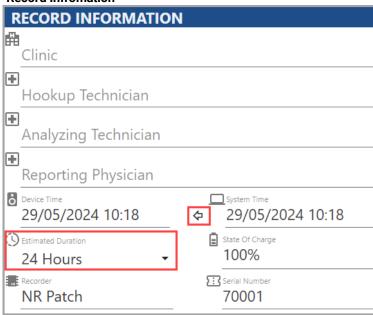


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 45 of 393

- 3.4.1. Click on each available text field (**Clinic**, **Hookup Technician**, etc.) and type in the data according to your needs and workflows.
- 3.4.2. Click the arrow button to synchronize **System Time** of your PC with the **Device Time**, if needed.
- 3.4.3. Select the **Estimated Duration** value for the record from the drop-down list (24 to 336 hours). The default value is set to 24 hours.

Please note that the data in other fields of this section cannot be modified.


3.5. In the **Scanning Criteria** section:

Figure 77. Import Records - Scanning Criteria

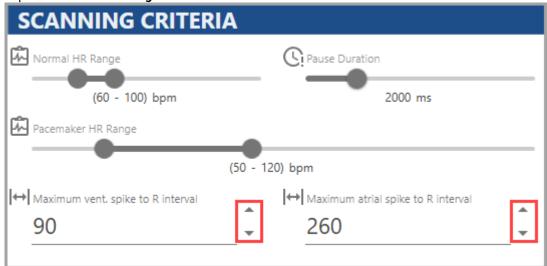


- 3.5.1. Drag the sliders to define the **Normal HR Range** and normal **Pause Duration** for the scanning process.
- 3.5.2. Drag the sliders to define the normal **Pacemaker HR Range**.
- 3.5.3. Set **Maximum vent. spike to R interval** and **Maximum atrial spike to R interval** by using the picker arrows on the right.
- 3.5.4. Set any additional parameters that may be available in this section.
- 4. Click **Proceed** in the bottom part of the **View Area**.

Figure 78. Import Records - Click Proceed to Complete

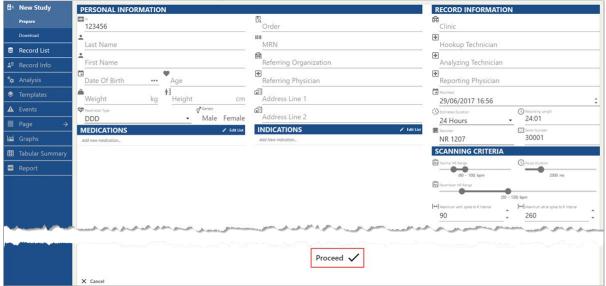

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 46 of 393	

Figure 79. Import Records - New Record Imported

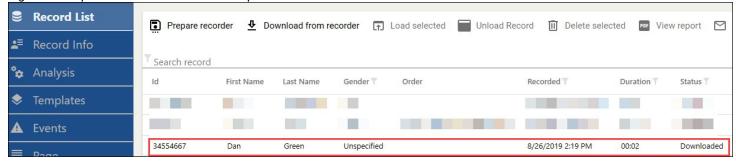


Export Record

This option allows you to export a patient's record to any folder on your PC, in your local network, or on a USB drive. Note that this option is available only when there is no patient's record loaded.

If this option is inactive, check the status of the record in the **Record List View** and unload a record if needed.

Figure 80. Export Record - Unload Record

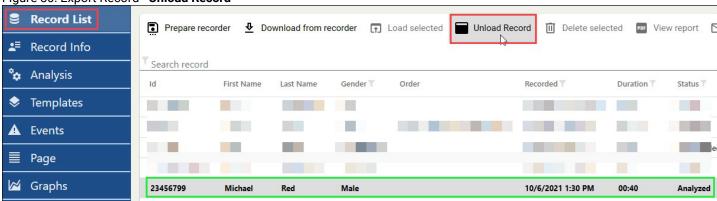

Follow these steps to export a patient's record. If there is no record loaded, disregard **Step 1** and **Step 2**:

- 1. Click the **Record List View** in the **Views Sidebar** on the left. You will see a patients' record list in the **View Area**.
- 2. Click **Unload Record** in the top action bar. If the unload is successful, the font formatting of the target patient's record changes from bold to regular.
- 3. Select the record you want to export by clicking on it in the **Record List**.
- 4. Click the **File** drop-down menu in the top left corner.


Figure 81. Export Record - Exporting Record 0 File View Norav Holter 2023.1 8 → Import Record.. ◆ Download from recorder Load selected Unload Record III Delete selected e recorder Export Record... •≡ Ctrl+S Save Record °¢ First Name Last Name Gender T **↓** Order Recorded T Duration T Status T **袋** Settings → Exit Alt+F4 12345987 Diary 10/6/2021 1:30 PM 23456799 Michael Red Male Analyzed 00:40

- 5. Click **Export Record** to open the Windows Explorer Window.
- 6. Navigate to the desired folder.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

7. Type in the desired name of the file archive in the **File name** text field.

Figure 82. Export Record - Saving Record

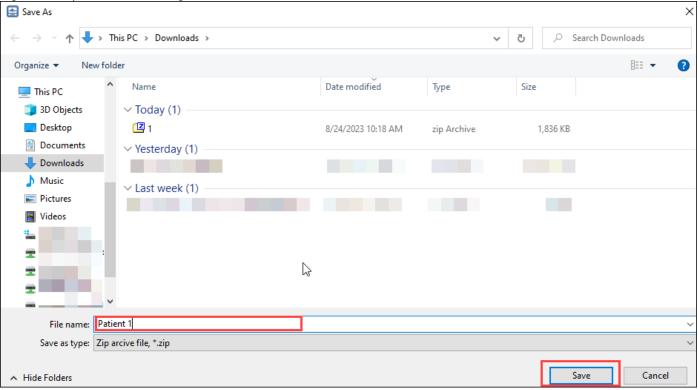


8. Click the **Save** button in the bottom right corner of the Windows Explorer Window.

You will promptly see a progress bar marking the successful completion of the record export.

Save Record

This option allows you to save any changes made to a loaded patient's record. To save a patient's record, follow these steps:

Figure 83. Save Record - Saving Record

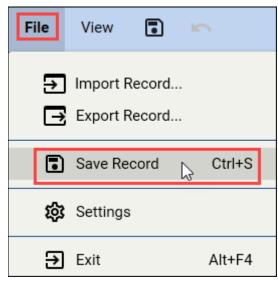

- 1. Click the **File** drop-down menu in the top left corner.
- 2. Click Save Record.

Alternatively, you can press the Ctrl + S keys on your keyboard to save the current patient's record at any time.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 48 of 393	

Settings


This option allows you to tailor the NH-301 software settings to meet your specific needs. For detailed instructions on how to modify these settings, please refer to the **Setup Process after Installation** (on page 11) section and follow the instruction, starting from the **Step 2**.

Exit

This option allows you to exit the NH-301 analysis system and close the application.

To exit the program:

Figure 84. Exit - Exiting Program

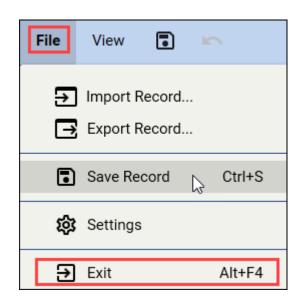

- 1. Click the **File** drop-down menu in the top left corner.
- 2. Click the **Exit** option.

Alternatively, you can press the **Alt + F4** keys on your keyboard to exit the NH-301 analysis system.


View Menu

The **View** menu allows you to switch between different **Views** (such as **Record List**, **Analysis View**, **Templates**, **Graph**, etc.), similar to the **Views Sidebar** functionality.

The **View Menu** can be in one of three different states:

• **Default State (No Records Loaded):** When no patient records are loaded in the system, only the **Record List** option is accessible.

Figure 85. View Menu - No Records Loaded

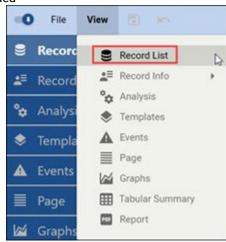


• Record Loaded (Not Analyzed): When a patient record is loaded into the system but it has never been analyzed

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 49 of 393	


within the application, only the **Record List** and **Record List** (with **General** and **Diary** subitems) options are accessible.

Figure 86. View Menu - Record Loaded

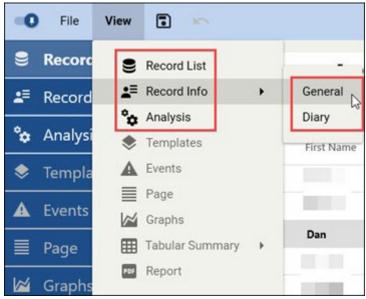

• **Record Loaded (Analyzed):** When a patient record is loaded into the system and has been analyzed, all **View** items and their subitems in the menu are active and accessible. You can switch between them freely.

Figure 87. View Menu - Record Analyzed

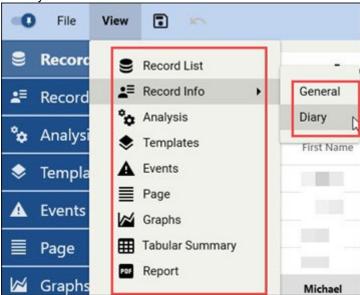


Windows Menu

Windows menu allows to activate **Secondary Screen Mode** facilitating more comfortable and efficient ECG analysis workflow. **Secondary Screen Mode** is a feature that allows for an additional display where you can place the **Page View**. This proves useful for quickly referring to the **Page View** while engaging with other **Views** on the main screen.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 50 of 393	

Figure 88. Page View - Secondary Screen Mode

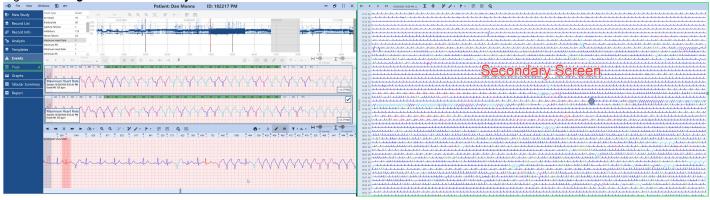


The Page View in Secondary Screen Mode replicates the functionality of the Page View strips in their respective Views.

To activate Secondary Screen Mode:

1. Select the **Windows** menu in the Menu Bar.

Figure 89. Page View - Activating Secondary Screen Mode via Menu Bar

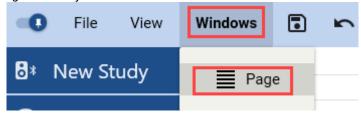

- 2. Choose the **Page** option from the menu. The **Page View** will open in a secondary window.
- 3. Adjust the secondary window settings as needed.

For further details, please, refer the **Secondary Screen Mode section** (on page 270).

Save Record

The **Save Record** button allows you to save a patient's record. Click this button to save the loaded patient's record. If there is no loaded record, this button will be inactive and displayed in grayscale.


Figure 90. Menu Bar - Save Record Button



Undo

The **Undo** button allows you to reverse the last action you took. Click the button to cancel the last action and return to the state before the action was taken. Alternatively, you can press the **Ctrl + Z** keys on your keyboard to undo the last action. If there is no loaded record or the record was loaded but no actions (like editing templates) have been taken, this button will be inactive and displayed in grayscale.

Figure 91. Menu Bar - Undo Button

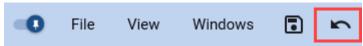

The **Undo** button can be used multiple times to undo a series of actions.


4.2. Title Bar Text

The **Title Bar Text** contains either the name of the application or the patient's identification data.

If no patient record is loaded, the Title Bar Text shows the name and version of the program.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 51 of 393	

Figure 92. Title Bar Text - No Record Loaded

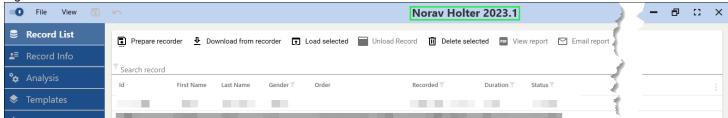

When a patient record is loaded, the Title Bar Text changes to display the patient's name and patient's ID.

Figure 93. Title Bar Text - Record Loaded



4.3. Window Control Buttons

The **Window Control Buttons** element includes the window's **Minimize**, **Restore/Maximize**, **Enter Full Screen**, and **Close** buttons, and is located in the top right corner of the window. These buttons enable you to customize the analysis system window size according to your preferences, change the window mode, or close the program.

Minimize

Click this button to reduce the window to an icon on the taskbar. The window is still running in the background, but it is not visible.

Figure 94. WIndow Control Buttons - Minimize Button



To restore the window, click on the program's icon on the Windows taskbar.

Restore/Maximize

Click this button to toggle between the maximized and restored states of a window. The maximized state is the state when a window fills the entire screen, and it cannot be resized or moved, though you can still see a title bar of the current window.

Figure 95. WIndow Control Buttons - Restore/Maximize Button



The restored size of the window is the unique size you set by dragging the edges of the window with the mouse cursor. The restored state of the window is an in-between state, when the window is neither minimized, nor maximized. You can freely resize and move the window in a restored state.

Enter Full Screen

Click this button to expand the window to the full screen. The **Enter Full Screen** button enables the window mode with the largest possible display space for the **View Area**.

Figure 96. WIndow Control Buttons - Enter Full Screen Button




As compared to the maximized state, you won't see a title bar in this mode, including the **Window Control Buttons** element. Moreover, the **Views Sidebar** element will be accessible only on hover.

To access the Views Sidebar items:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 52 of 393	


Figure 97. Enter Full Screen - Views Sidebar

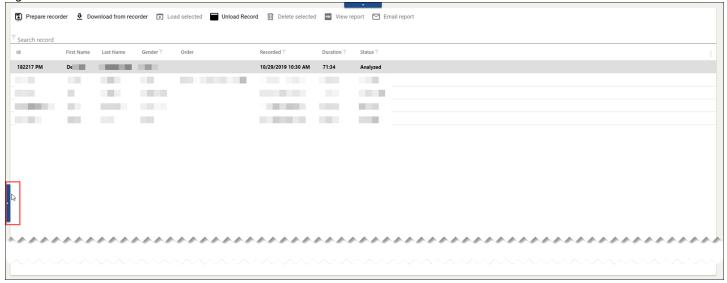

- 1. Hover over the handlebar on the left to expand the sidebar.
- 2. Click the **View** you want to switch to.

To exit the full screen mode:

Figure 98. Enter Full Screen - Exit Full Screen

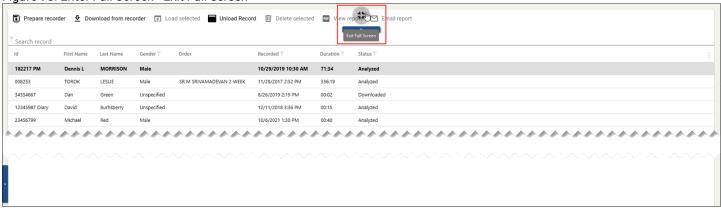

- 1. Hover over the handlebar on top of the screen.
- 2. The handlebar will slide down and reveal the **Exit Full Screen** button.
- 3. Click the Exit Full Screen button.

The window should switch to the restored state.

Close

Click this button to close the window and exit the NH-301 analysis system.

Figure 99. WIndow Control Buttons - Close Button



4.4. View Area

Unlike other interface elements, which remain largely consistent across screens (except being active or inactive under certain circumstances), **View Area** houses distinct content, data, and controls for each **View and View mode**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 53 of 393

Figure 100. View Area - Analysis View


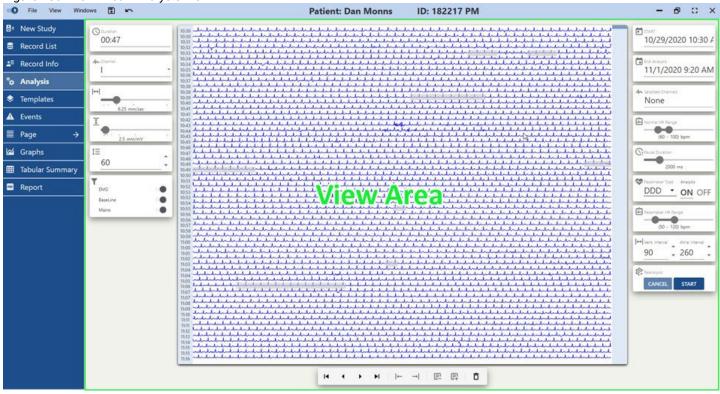


Figure 101. View Area - Analysis View

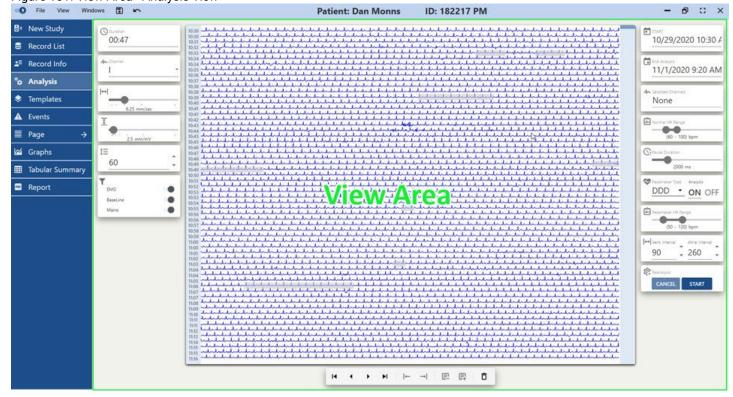

4.5. Views - Views Sidebar


The **Views Sidebar** allows you to switch between different **Views** by selecting them and their subitems directly in a sidebar. Fast navigation allows you to perform efficiently various tasks, from preparing the **Recorder** before a Holter test to generating and submitting the post-analysis report.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 54 of 393	

The **Views Sidebar** can be in one of three operational states:

• **Default State (No Records Loaded):** When no patient records are loaded in the system, only **Record List** and **New Study** options are accessible for managing records.

Figure 102. Views Sidebar - No Record Loaded

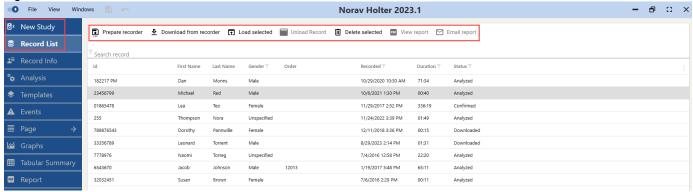

Record Loaded (Not Analyzed): When a patient record is loaded into the system but it has never been analyzed
within the application, only the New Study, Record List and Record Info (with General and Diary subitems)
options are accessible.

Figure 103. Views Sidebar - Record Loaded



• **Record Loaded (Analyzed):** When a patient record is loaded into the system and has been analyzed, all **View** items and their subitems in the menu are active and accessible. You can switch between them freely.

Figure 104. Views Sidebar - Record Loaded and Analysed




4.5.1. New Study

The **New Study** is the **View** that allows you to choose a **Recorder** you want to connect to the system and prepare for the Holter recording. When you click the **Prepare Recorder** button in the **Record List View**, you are redirected to the **New Study View**. In other words, you can initiate a new study either through the **New Study** or **Record List View**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 55 of 393	

Figure 105. New Study View - New Study View

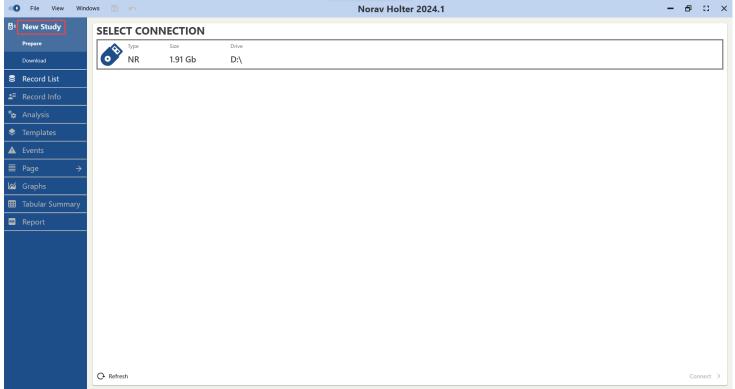


Initiating New Study: General Workflow

To initiate a new study:

- 1. Click the **Prepare Recorder** button on the top toolbar of the **Record List** or click the **New Study View** in the **Views Sidebar**. The **New Study View** is displayed, showcasing all available connections, including Memory Card, USB and Bluetooth connections.
- 2. **(Optionally)** Click the **Refresh** button in the bottom-left corner to refresh the **Select Connection** list and discover new connections, if applicable.
- 3. Select a connection.


Figure 106. Preparing Recorder - New Study

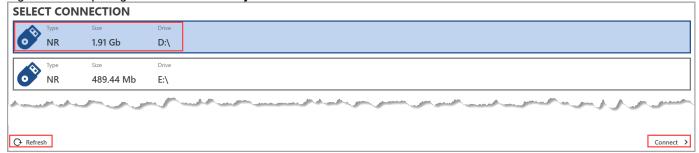

- 4. Click **Connect** in the bottom-right corner. The **Record Infromation** screen will appear.
- 5. Enter personal infromation. Click on each text field and type in the data as needed. To proceed, you must fill in at least one of the patient's identifiers: **Patient ID**, **First Name**, **Last Name**. Other fields are not mandatory to proceed.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 56 of 393	


Figure 107. Entering Record Information - Record Information

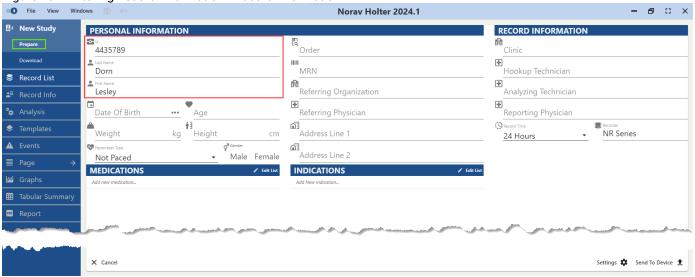



Note: Even if a patient has a pacemaker, selecting a **Pacemaker Type** from the drop-down menu during recorder setup is not mandatory. The device will capture the pacemaker signal regardless. This functionality ensures that the pacemaker signal is captured, even if the **Pacemaker Type** is not specified initially. Users have the flexibility to specify the pacemaker type at a later stage, such as during the download of the record or before analyzing it. While the ideal workflow involves specifying the **Pacemaker Type** at the setup stage, omitting this step will not compromise the quality of the ECG analysis.


Figure 108. Selecting Pacemaker Type - Pacemaker Type

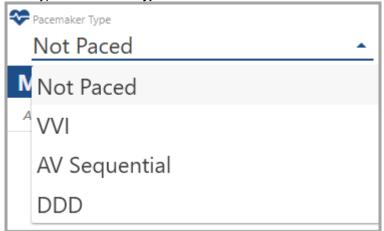

- 6. **(Optional)** Click on the **Date Of Birth** field and type in the date or use the ellipsis button on the right to select from a calendar.
- 7. (Optional) To add Medications:
 - 7.1. Click "Add New Medication" to open a drop-down list with predefined medications.
 - 7.2. Click on an existing predefined medication, or type in a new medication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined medications.
 - 7.3. To modify the list of predefined medications, click the **Edit List** icon located at the top right.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 57 of 393


Figure 109. Import Record - Medications

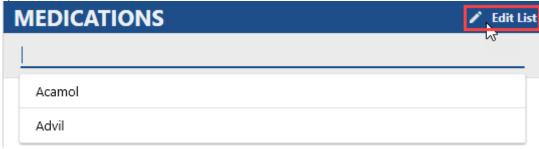

8. (Optional) To add Indications:

- 8.1. Click "Add New Indication" to open a drop-down list with predefined indications.
- 8.2. Click on an existing predefined indication, or type in a new indication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined indications.
- 8.3. To modify the list of predefined indications, click the **Edit List** icon located at the top right.


Figure 110. Import Record - Indications

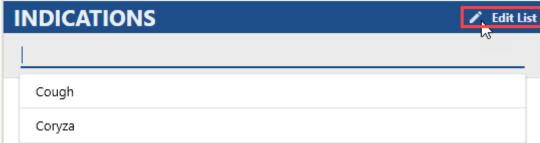

9. (Optional) In the Record Infromation section:

- 9.1. Click on each available text field (**Clinic**, **Hookup Technician**, etc.) and type in the data according to your needs and workflows.
- 9.2. Select the **Estimated Duration** value for the record from the drop-down list (24 to 336 hours). The default value is set to 24 hours.


Figure 111. New Study - Record Information

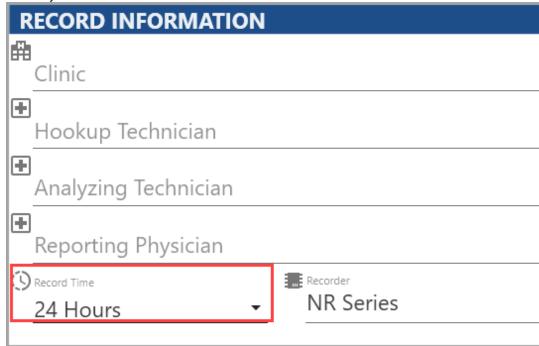

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 58 of 393

10. **(Optional) For Bluetooth connection:** If you're using a **Recorder** model equipped with Bluetooth, you may click the **Synchronize** arrow button to synchronize the device time with your system time. It is recommended to synchronize the device time settings periodically.

Figure 112. Record Settings - Synchronize Time

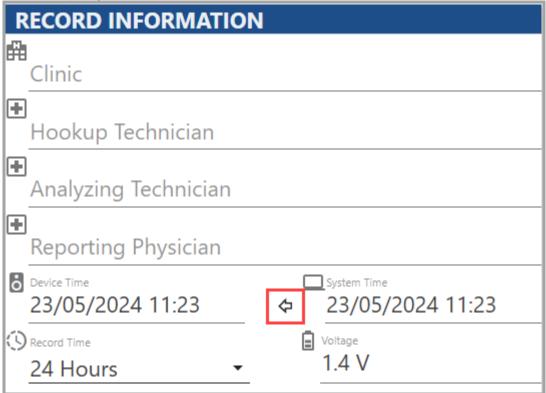

11. **(Optional)** For Bluetooth connection: If you're using a **Recorder** model equipped with Bluetooth, you may click the **Check ECG** button in the bottom-right corner of the screen to verify that the electrodes are properly connected and that the ECG signal will be recorded correctly. This is important because if the electrodes are not properly connected, the ECG signal may be distorted or unreadable.

Figure 113. Record Settings - Check ECG




Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 59 of 393	

Figure 114. Record Settings - ECG Streaming

Figure 115. Record Settings - Send To Device

File View Windows

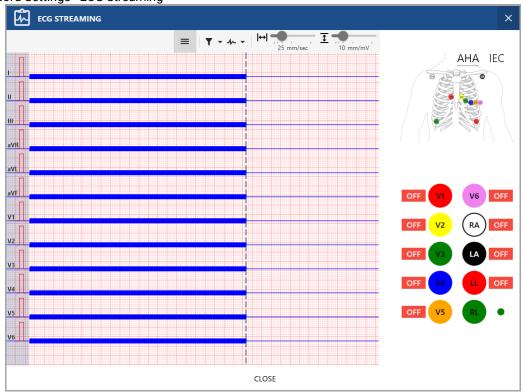


- 12. (Optional) If needed, click the Settings button located in the lower-right corner of the screen to modify the default settings of the ECG record before transmitting them to the **Recorder**. Additional information on adjusting the record settings is provided below.
- 13. Click **Send To Device** in the bottom-right corner to send the settings to the selected device. You will be briefly prompted by the **Uploading Configuration** progress bar. When it disappears, you will see a success message indicating that you have finished adjusting the record settings. If the **Send To Device** process fails, repeat the workflow from the start.

Norav Holter 2024.1


- 8 C ×

🗗 New Study PERSONAL INFORMATION RECORD INFORMATION Clinic 4435789 Dorn Hookup Technician MRN Record List Analyzing Technician Referring Organization Lesley

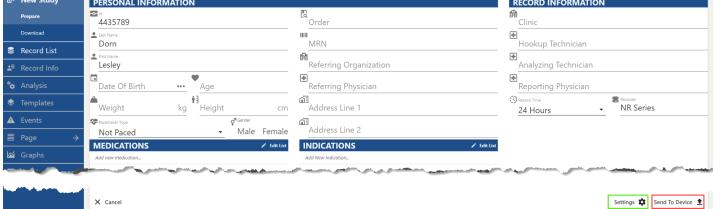

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 60 of 393

Figure 116. New Study - Upload Success

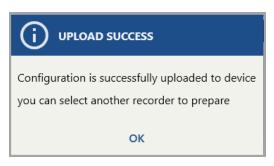


Adjusting Record Settings

To adjust the record settings, follow the steps listed below:

1. Click the **Settings** button located in the lower-right corner of the **Record Infromation** screen to modify the default settings of the ECG record. The **Record Settings** dialog box will appear.

Figure 117. Record Settings - Adjusting Record Settings

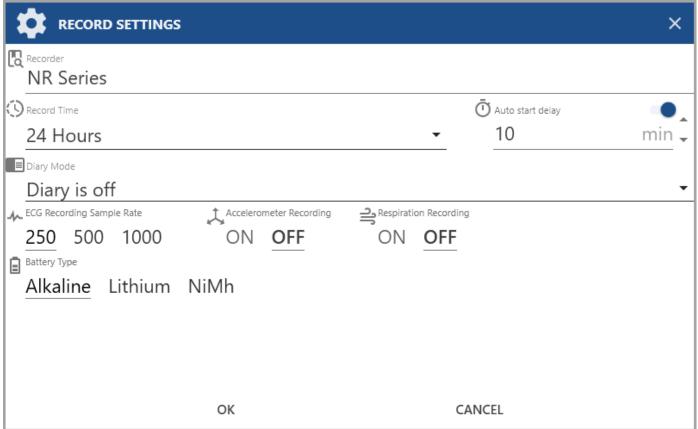




Note: You may notice that the **Record Infromation** screen and **Record Settings** dialog box layout for the Bluetooth and USB connection of the same recorder model differ. This difference mainly focuses on providing additional data but doesn't affect the core of the workflow.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 61 of 393	

Figure 118. Record Settings - Record Settings UI

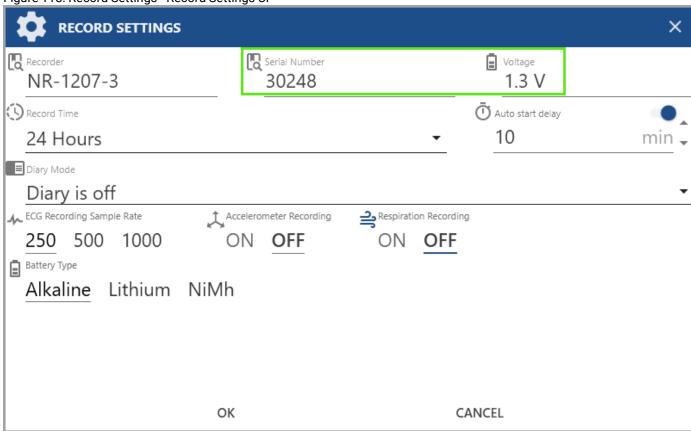

2. Select the **Record Time** value from the drop-down list, ranging from 24 to 336 hours, depending on the **Recorder** capabilities and requirements of the test.

Figure 119. Record Settings - Record Time

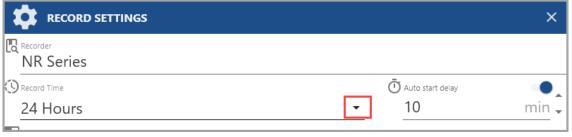



3. **(Optional)** If the selected **Record Time** or **ECG Recording Sample Rate** is not compatible with a specific battery type, a warning will be displayed. Select another time range or refer to the relevant **Recorder Manual** to check the **Record Time** parameters.

Figure 120. Record Settings - Warning

Selected record time with selected sample rate unsupported. Please, try to decrease sample rate or (and) record time.


- 4. Set **Auto Start Delay** to determine the timing for initiating the ECG recording:
 - Toggle **Auto Start Delay** OFF to disable the auto start delay feature.
 - Toggle **Auto Start Delay** ON and set the delay in minutes using the picker arrows on the right. You can increase or decrease the delay time by 5 minutes with each click.

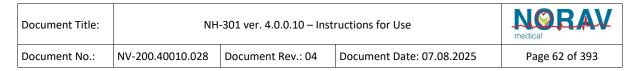

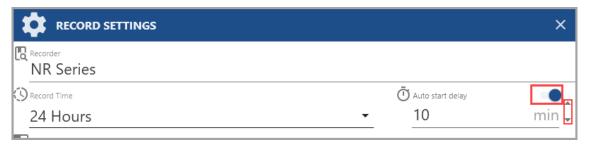


5. Select how the patient inputs diary events (different types of **Recorder** support different types of input; refer to the relevant manual, if needed):

Figure 122. Record Settings - Diary Mode

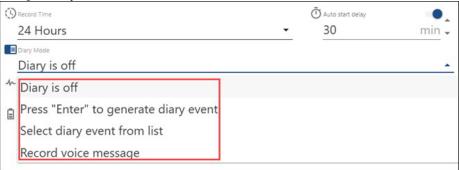

- Diary is off: A patient won't be able to make inputs.
- **Press "Enter" to generate diary event:** By pressing Enter on the **Recorder**, a patient will be able to register an **Event** without specifying its type.
- **Select a diary event from the list:** By pressing Enter on the **Recorder**, a patient will be able to choose an **Event** from a predefined list.
- **Record voice message:** By pressing Enter on the **Recorder**, a patient will be able to register an **Event** and make a brief voice description.
- 6. Click to select the **ECG Recording Sample Rate**: 250, 500, or 1000 Hz. The higher the sampling rate, the more detailed the ECG signal will be. However, a higher sampling rate also requires more memory and more battery capacity. If the selected ECG Recording sample rate is not compatible with a specific battery type, a warning will be displayed.


Figure 123. Record Settings - Sample Rate

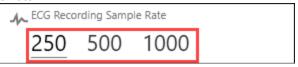

- 7. Click to set up **Accelerometer Recording**:
 - Select **ON** to enable the **Accelerometer Recording** during the test. Accelerometer recording can help to
 identify movement artifacts and remove them from the ECG signal, which can improve the accuracy of the
 interpretation.
 - Select OFF if there is no need to enable it.
- 8. Click to select **Respiration Recording**:
 - Select **ON** to enable the **Respiration Recording** during the test. Respiration recording can be used to assess the patient's respiratory rate and pattern. This information can be helpful in diagnosing certain heart conditions, such as sleep apnea.
 - Select **OFF** if there is no need to enable it.
- 9. Click to select the **Battery Type**: Alkaline, Lithium, NiMh. Choose the battery type considering the overall length of the Holter testing, quantity of leads, recorder model, etc. Please, refer to the relevant **Recorder** manual if needed.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 63 of 393	

Figure 124. Record Settings - Battery Type

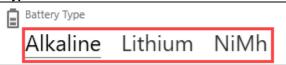


10. Click **OK** to save and apply changes to the record settings.

4.5.2. New Study: NR-314-P

The **New Study** is the **View** that allows you to choose a **Recorder** you want to connect to the system and prepare for the Holter recording. When you click the **Prepare Recorder** button in the **Record List View**, you are redirected to the **New Study View**. In other words, you can initiate a new study either through the **New Study** or **Record List View**.


Figure 125. New Study View - New Study View

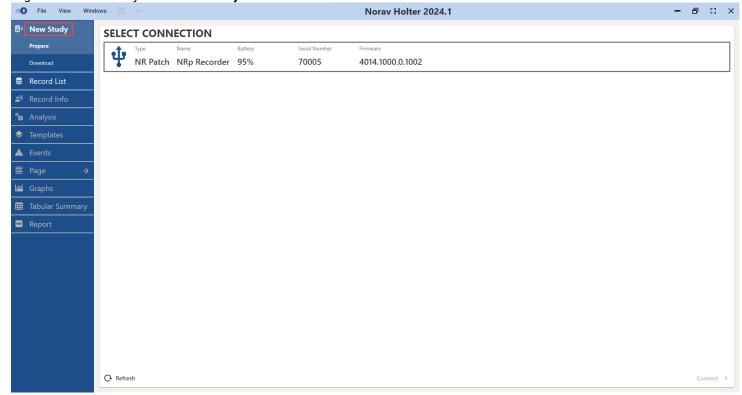

Initiating New Study: NR-314-P Workflow

To initiate a new study:

- 1. Click the **Prepare Recorder** button on the top toolbar of the **Record List** or click the **New Study View** in the **Views Sidebar**. The **New Study View** is displayed, showcasing all available connections, including Memory Card, USB and Bluetooth connections.
- 2. **(Optionally)** Click the **Refresh** button in the bottom-left corner to refresh the **Select Connection** list and discover new connections, if applicable.
- 3. Select a connection.

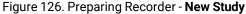

○ Refresh




Connect >

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical	
Document No.:	NV-200.40010.028	V-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025			

- 4. Click **Connect** in the bottom-right corner. The **Record Information** screen will appear.
- 5. Enter personal information. Click on each text field and type in the data as needed. To proceed, you must fill in at least one of the patient's identifiers: **Patient ID**, **First Name**, **Last Name**. Other fields are not mandatory to proceed.


Figure 127. Entering Record Information - Record Information

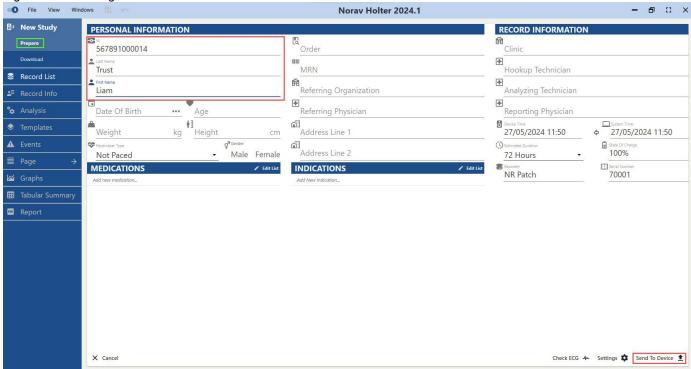



Note: Even if a patient has a pacemaker, selecting a **Pacemaker Type** from the drop-down menu during recorder setup is not mandatory. The device will capture the pacemaker signal regardless. This functionality ensures that the pacemaker signal is captured, even if the **Pacemaker Type** is not specified initially. Users have the flexibility to specify the pacemaker type at a later stage, such as during the download of the record or before analyzing it. While the ideal workflow involves specifying the **Pacemaker Type** at the setup stage, omitting this step will not compromise the quality of the ECG analysis.

Figure 128. Selecting Pacemaker Type - Pacemaker Type

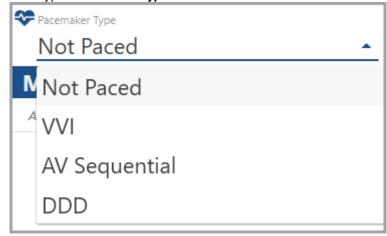

- 6. (**Optional**) Click on the **Date Of Birth** field and type in the date or use the ellipsis button on the right to select from a calendar.
- 7. (Optional) To modify Medications:
 - 7.1. Click "Add New Medication" to open a drop-down list with predefined medications.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 65 of 393	

- 7.2. Click on an existing predefined medication, or type in a new medication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined medications.
- 7.3. To modify the list of predefined medications, click the **Edit List** icon located at the top right.

Figure 129. Import Record - Medications



8. (Optional) To modify Indications:

- 8.1. Click "Add New Indication" to open a drop-down list with predefined indications.
- 8.2. Click on an existing predefined indication, or type in a new indication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined indications.
- 8.3. To modify the list of predefined indications, click the **Edit List** icon located at the top right.


Figure 130. Import Record - Indications



9. (Optional) In the Record Information section:

- 9.1. Click on each available text field (**Clinic**, **Hookup Technician**, etc.) and type in the data according to your needs and workflows.
- 9.2. Click the arrow button to synchronize **System Time** of your PC with the **Device Time**, if needed.
- 9.3. Select the **Estimated Duration** value for the record from the drop-down list (24 to 336 hours). The default value is set to 24 hours.

Figure 131. Record Settings - Record Information

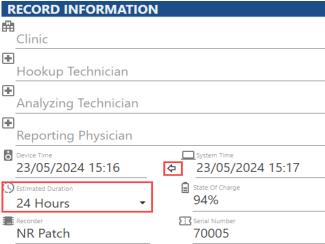


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

10. **(Optional) For Bluetooth connection:** If you have connected the **Recorder** to your PC via Bluetooth, you may click the **Check ECG** button in the bottom-right corner of the screen to verify that the electrodes are properly connected and that the ECG signal will be recorded correctly. This is important because if the electrodes are not properly connected, the ECG signal may be distorted or unreadable.


Figure 132. Record Settings - Check ECG



Figure 133. Record Settings - ECG Streaming

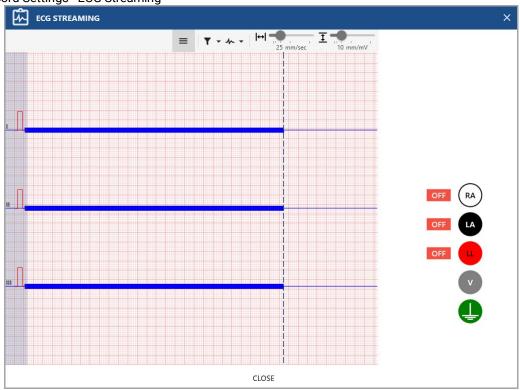


- 11. **(Optional)** If needed, click the **Settings** button located in the lower-right corner of the screen to modify the default settings of the ECG record before transmitting them to the **Recorder**. Additional information on adjusting the record settings is provided below.
- 12. Click **Send To Device** in the bottom-right corner to send the settings to the selected device. You will be briefly prompted by the **Uploading Configuration** progress bar. When it disappears, you will see a success message indicating that you have finished adjusting the record settings. If the **Send To Device** process fails, repeat the workflow from the start.

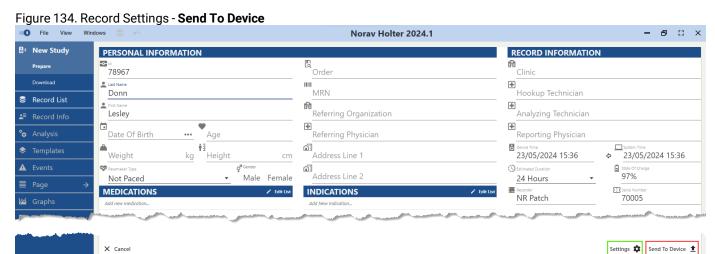

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

Figure 135. New Study - Upload Success

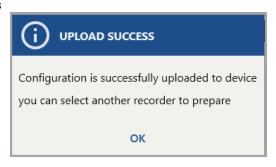


Adjusting Record Settings

To adjust the record settings, follow the steps listed below:

1. Click the **Settings** button located in the lower-right corner of the **Record Infromation** screen to modify the default settings of the ECG record. The **Record Settings** dialog box will appear.

Figure 136. Record Settings - Adjusting Record Settings

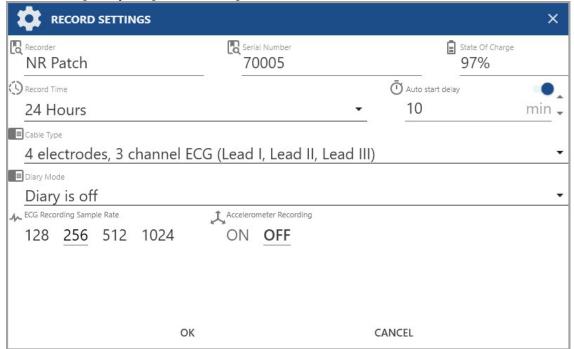



Note: You may notice that the **Record Infromation** screen and **Record Settings** dialog box layout for the Bluetooth and USB connection of the same recorder model differ. This difference mainly focuses on providing additional data but doesn't affect the core of the workflow.

2. Select the **Record Time** value from the drop-down list, ranging from 24 to 336 hours, depending on the **Recorder** capabilities and requirements of the test.

Figure 137. Record Settings - Record Time



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025		

3. **(Optional)** If the selected **Record Time** or **ECG Recording Sample Rate** is not compatible with a specific battery type, a warning will be displayed. Select another time range or refer to the relevant **Recorder Manual** to check the **Record Time** parameters.

Figure 138. Record Settings - Warning

Selected record time with selected sample rate unsupported. Please, try to decrease sample rate or (and) record time.


- 4. Set **Auto Start Delay** to determine the timing for initiating the ECG recording:
 - Toggle **Auto Start Delay** OFF to disable the auto start delay feature.
 - Toggle **Auto Start Delay** ON and set the delay in minutes using the picker arrows on the right. You can increase or decrease the delay time by 5 minutes with each click.

Figure 139. Record Settings - Auto Start Delay



5. Select the cable type from the **Cable Type** drop-down list:


Figure 140. Record Settings - Selecting Cable

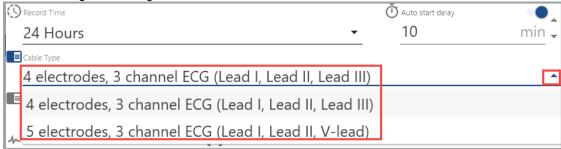

6. Select how the patient inputs diary events (different types of **Recorder** devices support different types of input; refer to the relevant manual, if needed):

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 69 of 393	

Figure 141. Record Settings - Diary Mode

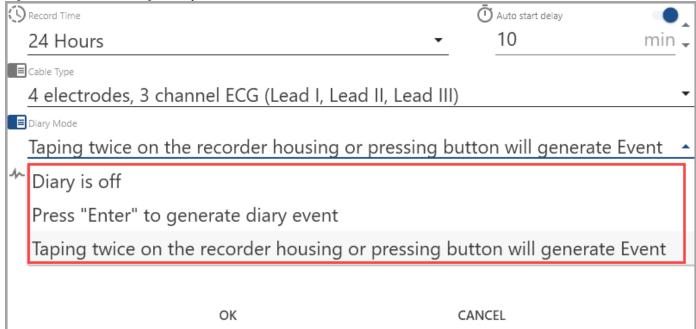

- Diary is off: A patient won't be able to make inputs.
- **Press "Enter" to generate diary event:** By pressing Enter on the **Recorder**, a patient will be able to register an **Event** without specifying its type.
- **Tapping twice on the recorder housing or pressig button will generate Event:** By pressing Enter on the **Recorder**, or tapping the **Recorder** case twice, a patient will be able to register an **Event**.
- 7. Click to select the **ECG Recording Sample Rate**: 128. 256, 512, or 1024 Hz. The higher the sampling rate, the more detailed the ECG signal will be. However, a higher sampling rate also requires more memory and more battery capacity. If the selected ECG Recording sample rate is not compatible with a specific battery type, a warning will be displayed.

Figure 142. Record Settings - Sample Rate




- 8. Click to set up **Accelerometer Recording**:
 - Select **ON** to enable the **Accelerometer Recording** during the test. Accelerometer recording can help to
 identify movement artifacts and remove them from the ECG signal, which can improve the accuracy of the
 interpretation.
 - Select **OFF** if there is no need to enable it.
- 9. Click **OK** to save and apply changes to the record settings.

4.5.3. Record List

The **Record List** is the initial **View** you see upon launching the NH-301 system. In this **View**, you can prepare the **Recorder** and manage patients' records imported and downloaded into the system.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 70 of 393	


Figure 143. Record List View - Record List

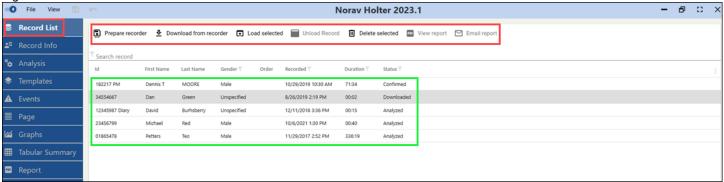

4.5.3.1. Using Top Toolbar

Using the top toolbar in this **View**, you can do the following:


Figure 144. Record List View - Top Toolbar

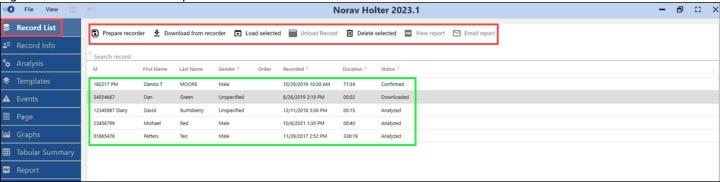

- **Prepare recorder:** Starts the recorder preparation procedure. More details in the Preparing Recoder section (on page 330).
- **Download from recorder:** Allows you to download a Holter recording into the system for the future analysis. You can download records from the **Memory Card** or directly from the **Recorder** via USB (if applicable). More details in the Downloading Holter Recording Data section (on page 361).
- Load selected: To load a Record you can double-click it in the list of records below the toolbar, or:
 - 1. Select a **Record** from the list. You can load **only one record** at a time.
 - 2. Click the **Load selected** button in the toolbar.
- Unload Record: Unloads a Record, so you could perform other actions or load the next Record.
- **Delete selected:** To permanently delete a **Record**:
 - 1. Select a **Record** from the list. You can also select multiple records using Ctrl+Click or Shift+Click keyboard shortcuts.
 - 2. Click the **Delete selected** button in the toolbar.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 71 of 393

Figure 145. Record List View - Deleting Records

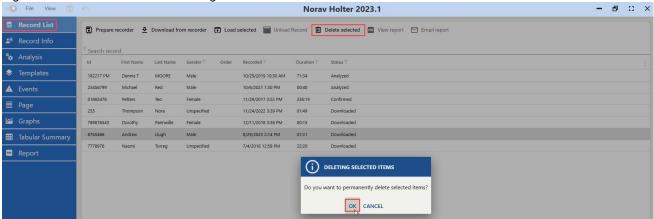

- 3. Click **OK** in the pop-up dialog box to confirm the action and permanently delete selected items. Click **Cancel** to discard the action.
- **View report**: Click the **View report** button to open an analysis report (in PDF, via the third-party viewer) associated with the selected **Record**, in a separate window.

Figure 146. Record List View - Viewing Report

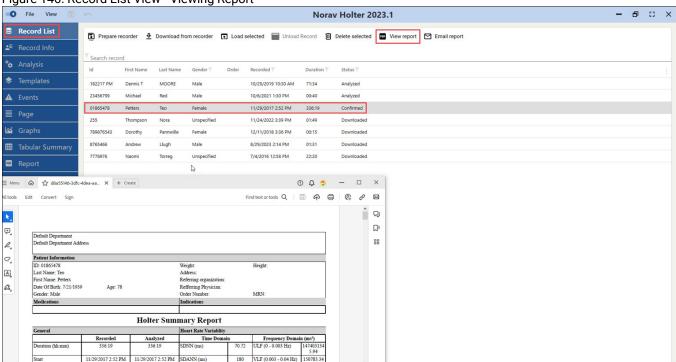


This button is active only for the records with the Confirmed status. A record obtains this status only when the analysis has been performed, and the report has been generated and confirmed.

• **Email report:** Click the **Email report** button to send an analysis report (in PDF) associated with the selected Record. You will see a window of your chosen email client with a new email containing this report.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	0010.028 Document Rev.: 04 Document Date: 07.08.2025		

Figure 147. Record List View - Emailing Report

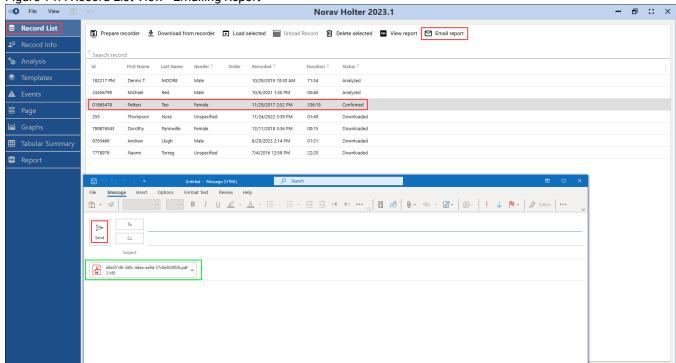


Fill in the required fields and click **Send** to send the report via email.

4.5.3.2. Using Search

This **View** also provides the capability to search, sort, and filter records. Please note, "Filter" icons remain inactive (gray-colored) when their settings remain default. After you customize any of these settings, search or filtering conditions, "Filter" icons become active (colored in black).

Figure 148. Record List View - Search and Filtering

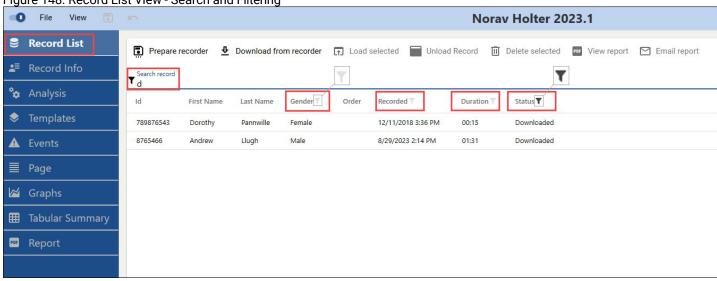


To find a **Record** in the **Record List**:

- 1. Click the **Search record** box.
- Start typing.
- 3. Continue typing until you see the result(s) you need. A real-time search engine will update the search results as you type, so you can see the results that match your query as you type it in.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 73 of 393

By default search box runs search through ID, First Name, Last Name, and Order columns.


Figure 149. Record List View - Search

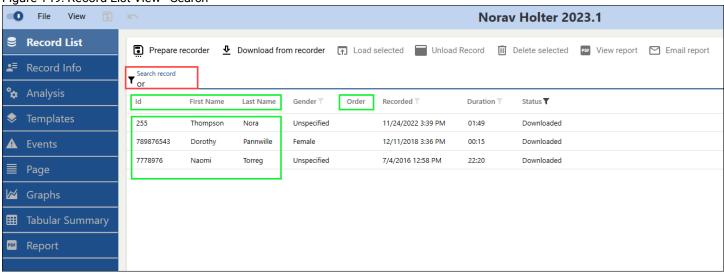

To change **Search record** settings:

1. Click a filter icon to the left of the **Search record** box.

Figure 150. Record List View - Search Settings

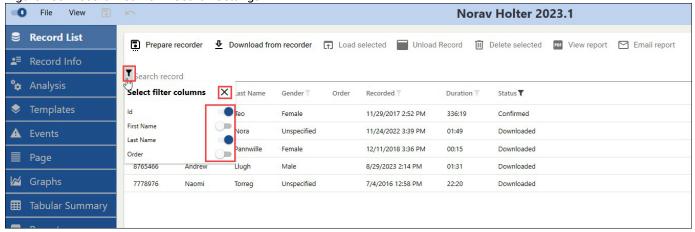

- 2. In the pop-up window, toggle **ON/OFF** default search parameters according to your needs.
- 3. Click the **Close** icon at the top right of the pop-up window or click anywhere else to close this window.

When you switch a toggle **OFF**, the **Search record** function excludes relevant column from the search.

4.5.3.3. Using Column Sorting

You can also sort records in the list using **Column Sorting**. **Column Sorting** toggles the sorting in a separate column between ascending and descending order. After switching the order in a separate column, all records in the list will be rearranged accordingly.


You can toggle column sorting order in each column you see under the **Search record** box.

To toggle column sorting order:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 74 of 393

1. Hover your mouse cursor over the header of a column you want to use for sorting (the ID column, for example).

Figure 151. Record List View - Column Sorting

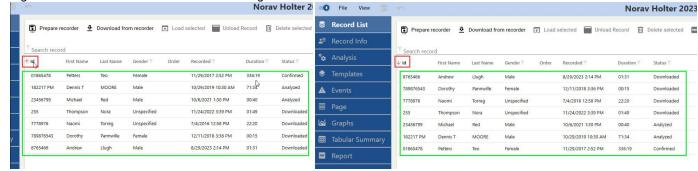


- 2. Click the column header to switch between the sorting orders the records will be rearranged in the corresponding order. The arrow icon to the left of the column header will change accordingly.
- 3. (Optional) Click the column header again to reverse the sorting.

To turn on additional columns or adjust the visibility of the displayed columns, if needed:

- 1. Click the vertical ellipsis menu on the right to access the **Parameters** drop-down menu.
- 2. Click on the relevant available toggles to turn them ON or OFF.
- 3. The **Record List** will be adjusted accordingly.

Figure 152. Record List View - Toggling Columns

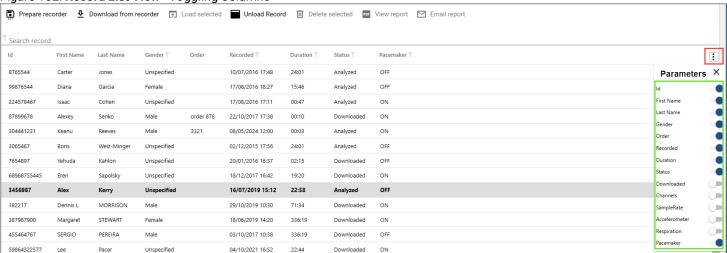

4.5.3.4. Using Filters

You can set various filters for **Gender**, **Recorded**, **Duration**, and **Status** columns in the **Record List** to view only records that match your criteria.

To customize **Gender** column filter settings:

- 1. Click a filter icon to the right of the column header.
- 2. In the pop-up window toggle **ON/OFF** filtering parameters according to your needs. Records with the parameters you toggled **OFF** will be excluded from the list.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 75 of 393

Figure 153. Record List View - Gender Filter ◆ Download from recorder Unload Record Till Delete selected Prepare recorder ↑ Load selected Search record Gender T ↑ Id Order Recorded T Duration First Name Last Name Status **Gender filters** 01865478 Petters Teo Female 017 2:52 PM 336:19 Confirmed MOORE Unspecified 019 10:30 AM 182217 PM Dennis T Male 71:34 Analyzed Male 21 1:30 PM 23456799 Michael Red Male 00:40 Analyzed Female 022 3:39 PM Downloaded 255 Thompson 01:49 Nora Unspe

- 3. Click the **Close** icon at the top right of the pop-up window or click anywhere else to close this window.
- 4. **(Optional)** To discard all changes and reset filter settings to default, hover over a filter icon and click the mouse wheel.

To customize **Recorded** column filter settings:

- 1. Click a filter icon to the right of the column header.
- 2. In the pop-up window toggle **ON/OFF** two parameters: **Date from** (Start Date) and **Date to** (End Date). These two parameters allow you to filter records by date range.

Figure 154. Record List View - Recorded Filter Prepare recorder ■ Download from recorder Load selected Unload Record III Delete selected PDF View report Search record ↑ Id First Name Recorded Y **Duration** T Status T Last Name Gender T Order 11/29/201 Recorded filters 01865478 Petters Female Confirmed 182217 PM MOORE Male Analyzed Dennis T 10/29/20 Date from 23456799 Michael Red Male 10/6/202 Analyzed 11/24/202 Date to 255 Thompson Nora Unspecified Downloaded 7778976 Naomi Torreg Unspecified 7/4/2016 12:58 PM 22:20 Downloaded 789876543 Dorothy Pannwille Female 12/11/2018 3:36 PM 00:15 Downloaded 01:31 8765466 Andrew Llugh Male 8/29/2023 2:14 PM Downloaded

- 3. Click active date fields and fill the date manually, or click the Calendar icon on the right and choose the date you need.
- 4. Click the **Close** icon at the top right of the pop-up window or click anywhere else to close this window.
- 5. **(Optional)** To discard all changes and reset filter settings to default, hover over a filter icon and click the mouse wheel.


As soon as you set **Date from** and **Date to** parameter values, the contents of the **Record List** will change accordingly. To customize **Duration** column filter settings:

1. Click a filter icon to the right of the column header.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 76 of 393

2. In the pop-up window toggle **ON/OFF** two parameters: **Duration from** (Minimum Duration) and **Duration to** (Maximum Duration). These two parameters allow you to filter records by their duration.

Figure 155. Record List View - Duration Filter

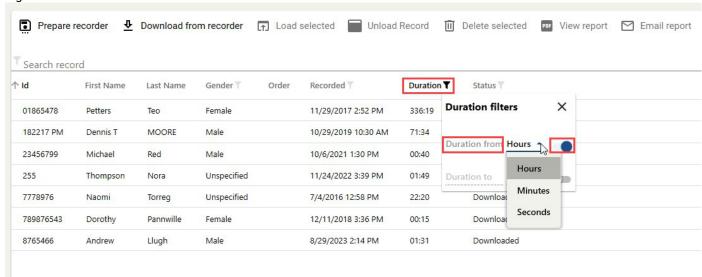

- 3. First, choose units for parameters toggled **ON** from the drop-down list on the right: **Hours, Minutes,** or **Seconds**.
- 4. Click active fields and fill the duration manually, using your keyboard.
- 5. Click the **Close** icon at the top right of the pop-up window or click anywhere else to close this window.
- 6. **(Optional)** To discard all changes and reset filter settings to default, hover over a filter icon and click the mouse wheel.

As soon as you set **Duration from** and **Duration to** parameter values, the contents of the **Record List** will change accordingly.

To customize **Status** column filter settings:

- 1. Click a filter icon to the right of the column header.
- 2. In the pop-up window toggle **ON/OFF** filtering parameters according to your needs. Records with the parameters you toggled **OFF** will be excluded from the list.

Figure 156. Record List View - Status Filter

Prepare	recorder <u></u>	Download fro	m recorder	↑ Load	selected Unload	Record III	Delete selected PDF View repo
Search reco	ord						
Id	First Name	Last Name	Gender T	Order	Recorded T	Duration \(\mathbb{T}\)	Status T
01865478	Petters	Teo	Female		11/29/2017 2:52 PM	336:19	Confir Status filters X
255	Thompson	Nora	Unspecified		11/24/2022 3:39 PM	01:49	Downl Downloaded
7778976	Naomi	Torreg	Unspecified		7/4/2016 12:58 PM	22:20	Downl Analyzed
789876543	Dorothy	Pannwille	Female		12/11/2018 3:36 PM	00:15	Downl Confirmed
8765466	Andrew	Llugh	Male		8/29/2023 2:14 PM	01:31	Downloaded

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 77 of 393

- 3. Click the **Close** icon at the top right of the pop-up window or click anywhere else to close this window.
- 4. **(Optional)** To discard all changes and reset filter settings to default, hover over a filter icon and click the mouse wheel.

4.5.4. Record Info


The **Record Info View** enables you to edit the patient's data and the diary of the downloaded or imported **Record**.

Figure 157. Record Info View - Record Info



You can navigate to the **Record Info View** using the **View Sidebar** on the left or the **View** menu in the **Menu bar**.

Figure 158. Record Info View - View Menu

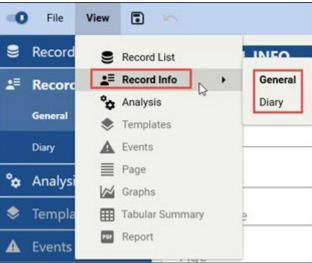

Most often, you will be automatically redirected to the **Record Info View** when you load a **Record from the Record List**.

You will also be prompted with the **Patient Information View**, which looks and works almost exactly as the **Record Info View**, in the following cases:

- When you import a Record. More details in the Import Record subsection (on page 42).
- When you download a **Record** from the **Recorder**. More details in the Downloading Holter Recording Data section (on page 361).

General View Mode

When you load a **Record** from the **Record List**, you are automatically redirected to the **General View Mode** of the **Record Info View**. Here you can review and edit the following sections of the patient information:

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 78 of 393


Figure 159. Record Info View - General View Mode



- Personal info, including ID, Last Name, First Name, Birth Date, etc.
- Record info, including Clinic, Hookup Technician, etc.
- Medications used by a patient.
- Indications for a patient.

To modify **Personal Info**:

Figure 160. Record Info View - Personal Info

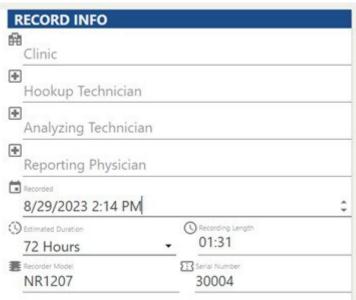

- 1. Click on each text field and type in the data according to your needs and workflows. For example, fill in **ID**, **Last Name**, **First Name**, **Birth Date**, **Age**, **Weight** and **Height** values.
- 2. **(Optional)** Click on the **Birth Date** field and type in the date or use the ellipsis button on the right to select from a calendar.
- 3. Select patient's **Gender**.

To modify **Record Info**:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 79 of 393

Figure 161. Record Info View - Record Info




- 1. Click on each text field and type in the data according to your needs and workflows. For example, fill in **Clinic, Hookup Technician** and other text fields.
- 2. (Optional) Click the **Recorded** field to enter the modified date and time when the record was launched. You can either type the date and time in the format DD/MM/YYYY HH:MM AM/PM or select a segment of the date and time string by clicking and holding the left mouse button. Then, use the picker arrows located to the right of the field to adjust the day, month, year, hour, minute, and AM/PM values as needed.
- 3. (Optional) If needed, select the estimated duration of the Record from the drop-down list (24 to 336 hours).

Please note that the data in the **Recording Length**, **Recorder Model**, and **Serial Number** fields cannot be modified.

To modify **Medications**:

Figure 162. Record Info View - Medications

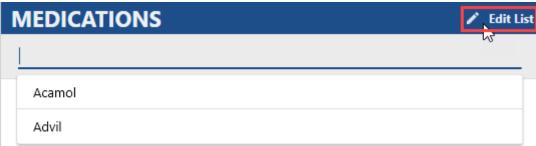

- 1. Click "Add New Medication" to open a drop-down list with predefined medications.
- 2. Click on an existing predefined medication, or type in a new medication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined medications.
- 3. To modify the list of predefined medications, click the **Edit List** icon located at the top right.

To modify **Indications**:


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 80 of 393

Figure 163. Record Info View - Indications



- 1. Click "Add New Indication" to open a drop-down list with predefined indications.
- 2. Click on an existing predefined indication, or type in a new indication and press **Enter** on your keyboard. Note that new entries made this way won't be stored in the list of predefined indications.
- 3. To modify the list of predefined indications, click the **Edit List** icon located at the top right.

Figure 164. Record Info View - Proceeding to Diary

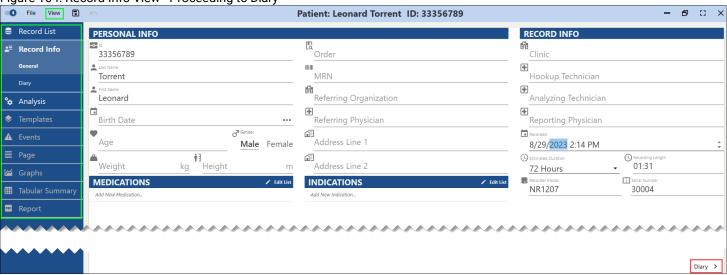


To proceed further within the **Record Info View**, click **Diary** at the bottom right. Otherwise, click any **View** in the **Views Sidebar** or in the **View** menu.

Diary View Mode

In the **Diary View Mode** you can add, review, modify, delete and sort **Diary Events** of a patient's **Record**.


Figure 165. Record Info View - Diary

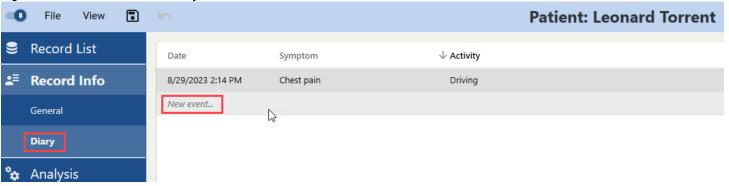

To add a **Diary Event**:


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 81 of 393

Figure 166. Record Info View - Adding Diary Event

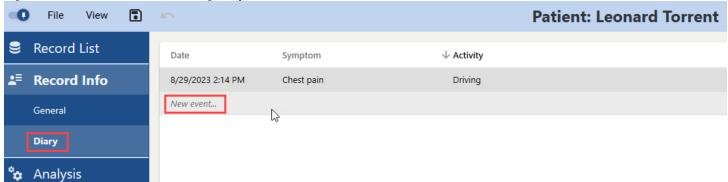


- 1. Click "New event". A new blank entry with a timestamp in the Date column will appear.
- 2. Click the blank text field in the **Symptom** column.

Figure 167. Record Info View - Adding Symptoms

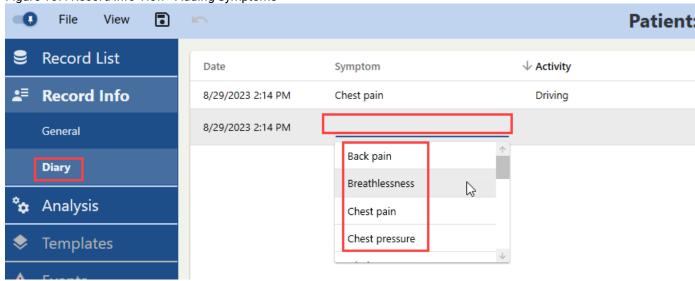

- 3. Choose a **Symptom** from a drop-down list of predefined symptoms or type in a symptom missing in the list.
- 4. Click the blank text field in the **Activity** column.

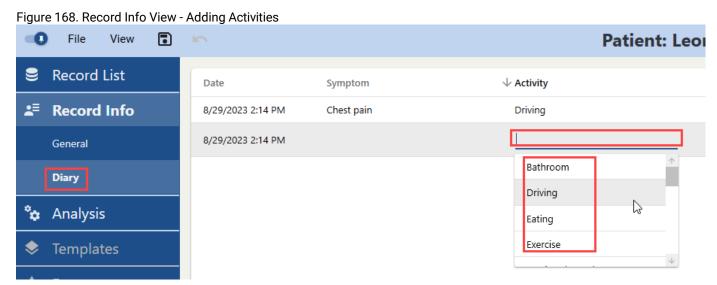


5. Choose an **Activity** from a drop-down list of predefined activities or type in an activity missing in the list.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 82 of 393

For example, after you click "New event", you may choose "Shortness of breath" in Step 3 and "Exercise" in Step 4. This will mean that at a given moment a patient experienced a shortness of breath while exercising.

To modify an existing **Diary Event**:

Figure 169. Record Info View - Editing Diary Event

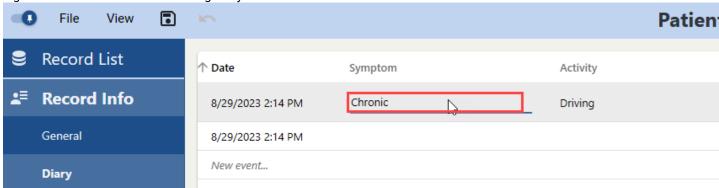

- 1. Click a text field in a respective column.
- 2. Use your keyboard to edit date and time in the **Date** column, or symptoms and activities in the **Symptom** or **Activity** columns respectively.


To delete a **Diary Event:**

- 1. Right-click the line with a Diary Event entry you want to delete and select the line. You can select multiple entries using Ctrl+Click or Shift+Click keyboard shortcuts.
- 2. Press **Delete** on your keyboard.

The entry will be deleted from the Diary.

To sort **Diary Events** by toggling between ascending and descending order:

Figure 170. Record Info View - Column Sorting

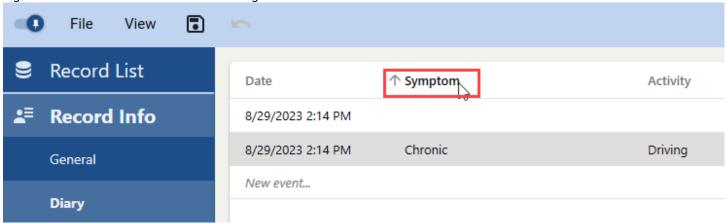


- 1. Hover your mouse cursor over the header of a column you want to use for sorting (the **Symptom** column, for example).
- 2. Click the column header to switch between the sorting orders **Diary Event** entries will be rearranged in the corresponding order. The arrow icon to the left of the column header will change accordingly.
- 3. (Optional) Click the column header again to reverse the sorting.

4.5.5. Analysis

Before analyzing a Holter recording, data must be uploaded either from the **Memory Card** or directly from the **Recorder**, if applicable. The NH-301 Holter analysis system scans and analyzes this data, offering a comprehensive toolkit for review, modification, and report generation. Within the **Analysis View**, users can define start and end points for analysis and opt to include or exclude specific (e.g., noisy) areas of the **Record**, as well as set analysis parameters.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 83 of 393

Figure 171. Analysis View - Analysis View

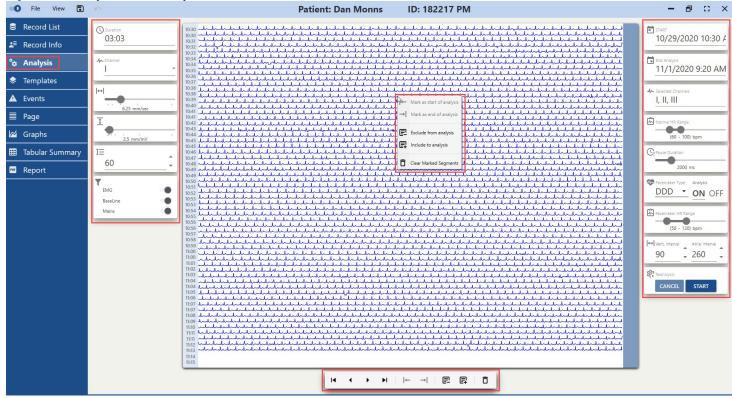


Signal Page Controls

The **Signal Page** displays a single page of data containing the electrical activity of the heart over a specific time frame. Signal timestamps are visible on the left, and a scrollbar is located on the right side of the **Signal Page**.

Analysis View enables adjustments to ECG signal visualization via control boxes situated on the left. These controls affect **only** the visualization seen on the **Signal Page**, which is centrally displayed on the screen.

Duration: Displays the overall duration (in minutes) of the ECG Record fragment shown on the current **Signal Page**. This parameter is not directly adjustable and is influenced by the **Signal Page Gain** and **Signal Page Lines** parameters, as well as by window size. Alter these settings to see automatic changes to the **Duration** value.


Figure 172. Analysis View - Duration Box

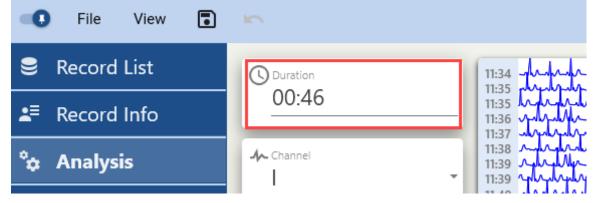

Channel: To select a channel you want to display on the **Signal Page**, choose it from a drop-down list on the right. The **Signal Page** updates accordingly.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 84 of 393

Figure 173. Analysis View - Channel Box

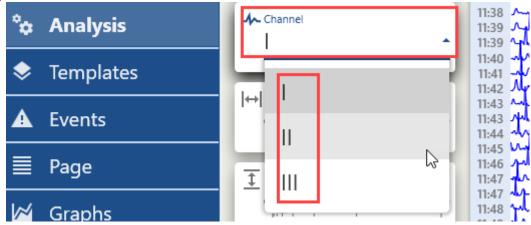


Signal Page Scale: Use the slider to adjust the ECG signal scale on the Signal Page. This action also affects the Duration value.

Figure 174. Analysis View - Signal Page Scale

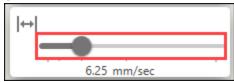

Signal Page Gain: Use the slider to adjust the ECG signal gain on the Signal Page.

Figure 175. Analysis View - Signal Page Gain

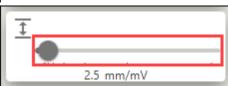


Signal Page Lines: Adjust the lines number using picker arrows on the right. This action changes the number of lines displayed on a **Signal Page** and consequently modifies the **Duration** value.

Figure 176. Analysis View - Signal Page Lines



Filters: Toggle **ON/OFF** three different filters designed to remove artifact components of various frequencies:

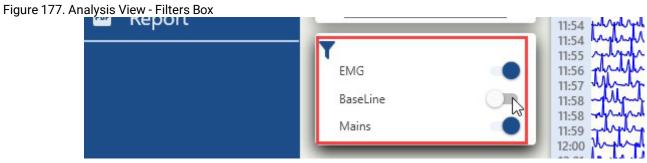

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 85 of 393	

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.


Action Bar and Context Menu Controls

The right-click context menu of the **Signal Page** and **Action Bar** offer identical functionalities, including defining start and end points for analysis, as well as including or excluding noisy fragments of the **Record**. Additionally, the **Action Bar** located at the bottom enables **Signal Page** pagination and navigation.

Context Menu


Mark as start of analysis: To set the analysis **Start**, hover over the desired point in the **Signal Page**, right-click, and select **Mark as start of analysis**.

Figure 178. Analysis View - Mark Start

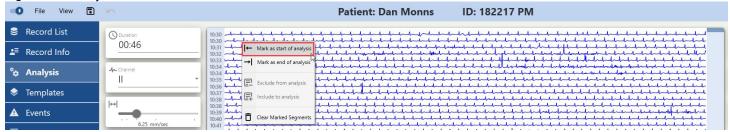

Mark as end of analysis: To set the analysis **End**, hover over the desired endpoint in the **Signal Page**, right-click, and select **Mark as end of analysis**.


Figure 179. Analysis View - Mark End

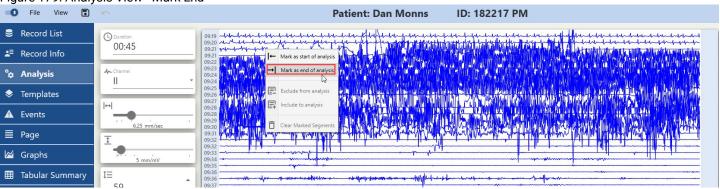

Exclude from analysis: Right-click at the point you want to designate as the starting point of the excluded noisy segment, and drag the cursor up to the desired end point. After selecting a continuous segment in this manner on the **Signal Page**, click **Exclude from analysis**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 86 of 393	

Figure 180. Analysis View - Exclude from Analysis

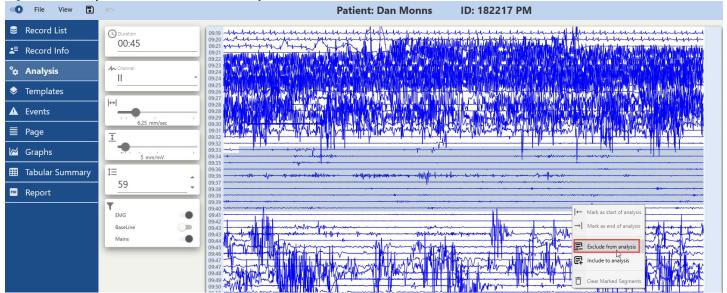


Include to analysis: Right-click at the desired starting point of the segment you wish to reinsert into the analysis, and drag the cursor up to the desired end point. After selecting a continuous segment within an excluded segment, click **Include to analysis**.




Note: The range you wish to re-include should fully overlap a previously excluded fragment. If it does not overlap an excluded area, the **Include to analysis** button and option remain inactive.



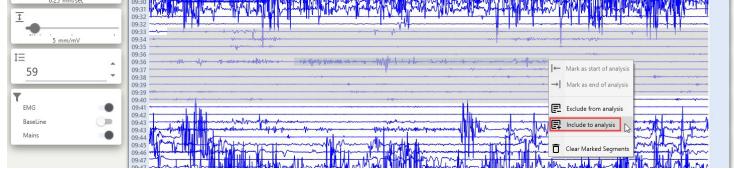

Clear marked segments: Select this option to clear all marked segments across the entire ECG Record in one click. It will clear marked **Start** and **End** points, exluded and included segments.

Figure 182. Analysis View - Clear Segments

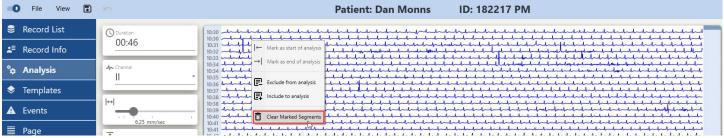

Action Bar

The **Action Bar** features the same functionalities as the Context Menu (on page 86), in addition to providing **Signal Page** navigation controls.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	V-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025			

Figure 183. Analysis View - Action Bar **Patient: Dan Monns** ID: 182217 PM Record List 00:46 10/29/2020 10:30 A ♣ Record Info Analysis 11/1/2020 10:05 AM Templates **A** Events None ■ Page ✓ Graphs III Tabular Summary 1= (Gi Pau 59 Report Pacemaker Type Analysis

DDD
ON OFF BaseLine |↔| Vent. interval ₊ 260 CANCEL START

First Page: Displays the first page of the **Record**.


Page Up: Navigates one page backward.

Page Down: Navigates one page forward.

Final Page: Displays the last page of the **Record**.

For quick navigation and review of the **Record**, use your mouse wheel or the scrollbar situated to the right of the **Signal Page**.

Figure 184. Analysis View - Navigation and Scroll Bar

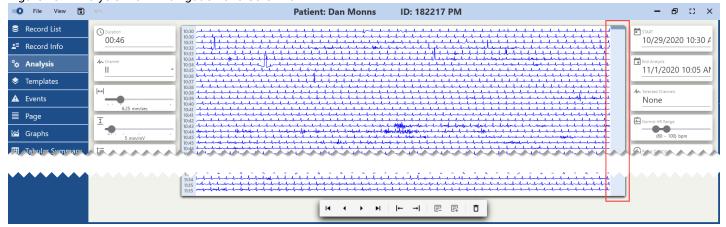




Note:

Split Events While Excluding Range Settings

The NH-301 analysis system settings contain a parameter that significantly influences the outcome of analyses involving excluded segments (on page 15). This parameter dictates how algorithms will process events containing **Exclude from analysis** areas:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 88 of 393	

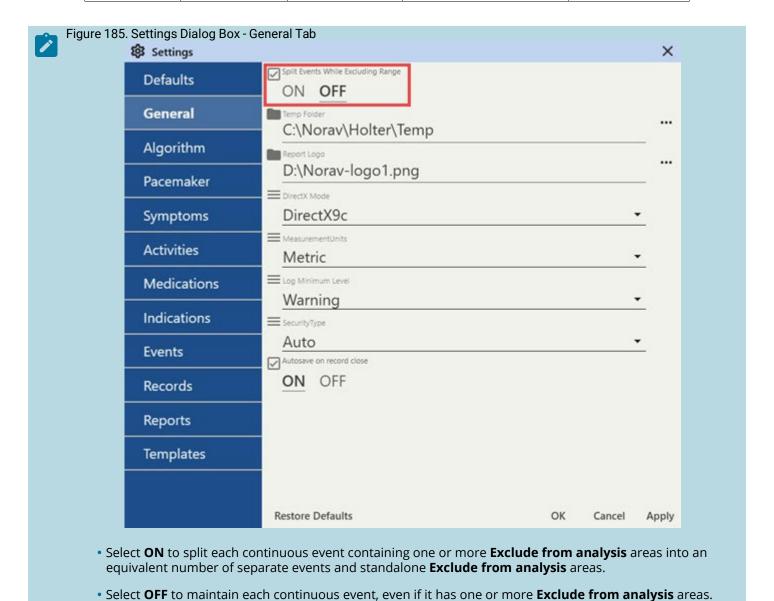

Analysis Controls

The core functionality of the **Analysis View** involves the setting of analysis parameters using control boxes located on the right.

Note that in this mode, event parameters such as duration, HRV, etc., will consider these **Exclude from analysis** areas. The event will not be fragmented into multiple events but will remain a single event.

If working with a **Record** that has already been analyzed (with **Analyzed** or **Confirmed** status), you will find that all **Analysis Controls** are inactive. A **Reanalysis** box will appear at the bottom right, containing two buttons: **PREPARE** and **START**. Click **PREPARE** to reactivate the **Analysis Controls**, particularly the **START** button, enabling re-analysis.

Figure 186. Analysis View - Analyse and Reanalyse




General Controls

START/End analysis: These two boxes display the start and end date and time of the Holter Record to be analyzed. These

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 89 of 393	


parameters are not adjustable.

Figure 187. Analysis View - Start and End Analysis

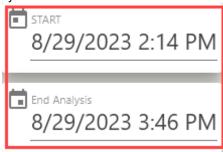


Selected Channels: Click this box to select from a dropdown list which ECG channels should be included in the analysis. Check the checkboxes next to all the channels you want to include in the analysis. You must choose at least **one** channel.

Figure 188. Analysis View - Selected Channels

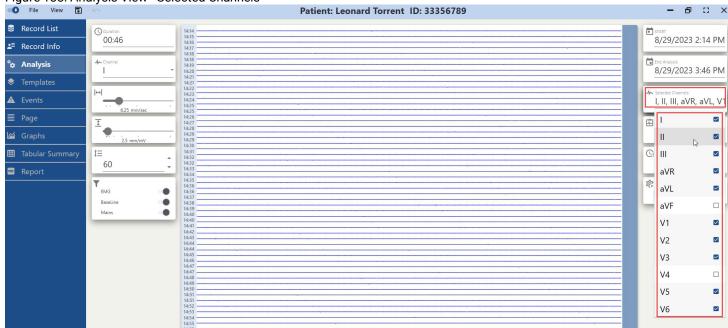

Normal HR Range: Use the double slider to define the minimum and maximum heart rate values. Any significant deviations from this range will be classified as Tachycardia or Bradycardia.

Figure 189. Analysis View - Normal HR Range

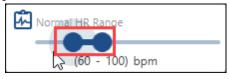

Pause Duration: Use this slider to set the maximum pause duration. Any significant deviations from this range will be classified as tachycardia.

Figure 190. Analysis View - Pause Duration



Pacemaker Controls

Pacemaker Type: Select a pacemaker type from the dropdown list (NONE, VVI, AV, DDD).

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 90 of 393

Figure 191. Analysis View - Pacemaker Type and Analysis



Analysis: Set pacemaker analysis to **ON** or **OFF** as needed.

Pacemaker HR Range: Use the double slider to set the minimum and maximum heart rate range initiated by a pacemaker.

Figure 192. Analysis View - Pacemaker HR Range



Vent. interval: Use the picker arrows on the right to set the time limit between the ventricular spike and the subsequent R-wave.



Atrial interval: Use the picker arrows on the right to set the time limit between the atrial spike and the subsequent R-wave.

Analysis/Reanalysis: Click START to initiate ECG Record analysis.

Figure 193. Analysis View - START and PREPARE Buttons





Note:


If the record you are working with hasn't been analyzed, you will see the **Analysis** box with only one button - **START**. If the **Record** has been previously analyzed, a **Reanalysis** box will appear with two buttons: **PREPARE** and **START**. Click **PREPARE** first to activate the control boxes, and then click **START** to initiate the **Analysis**.

After analysis completion, all **Views** in the **Views Sidebar** become accessible.

4.5.6. Templates

The **Templates View** enables efficient inspection of beat analysis results: review, edit, reclassify, overlay, merge, check, and delete individual beats or entire templates. This section will first cover the editing tools located in the upper pane, followed by an explanation of features in the lower pane.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 91 of 393	

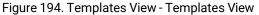



4.5.6.1. Reviewing Beat Morphologies

In the top pane of the **Templates View**, you will find different beat morphologies identified during the beat analysis. These morphologies are organized into separate boxes. Each box constitutes a template that contains beats with similar morphology, enabling you to work with groups of beats simultaneously.

4.5.6.1.1. Templates Pane Toolbar

Templates may be managed via the top toolbar, through drop-down menus, or by utilizing controls within each template box. Additionally, templates can be modified or removed through keyboard commands. In this section, we will concentrate on explaining the top toolbar controls. For ease of understanding, controls are combined into separate groups of interface elements.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 92 of 393	

Figure 195. Templates View - Templates Pane Toolbar



- 1. General Controls.
- 2. Templates Controls.
- 3. Layout Controls.
- 4. Filters.
- 5. Scale and Gain Controls.

General Controls


General Controls is a group of buttons representing different actions and modes for reviewing templates and beats.



Note:

If you set the **Checked Templates Panel** option (on page 37) in **File > Settings > Templates** tab to **ON**, checked templates will be displayed in a separate pane at the top of the screen. Otherwise, they will reside in the standard **Templates Pane**, adjacent to unchecked templates.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 93 of 393

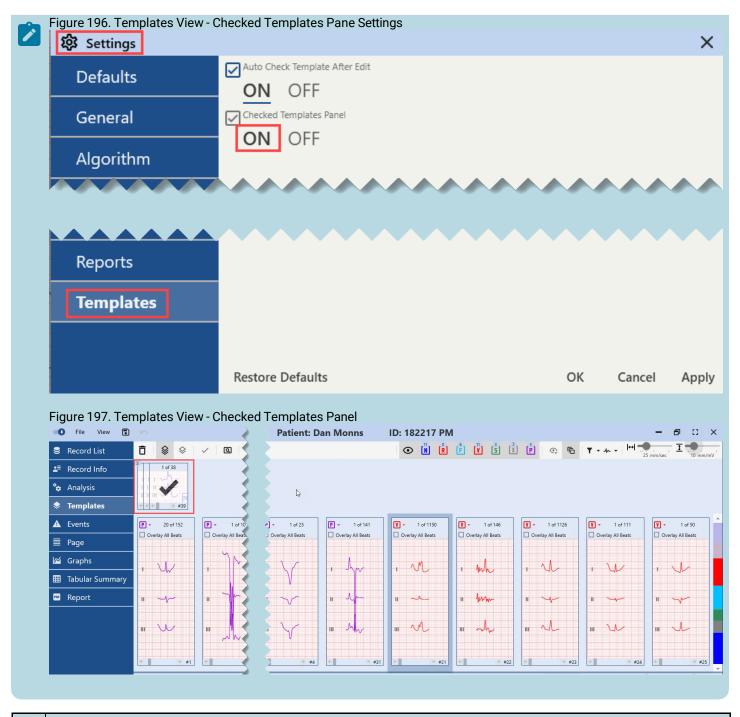

Icon	Description
Î	Delete : Allows you to remove a single beat or Template(s) . To delete multiple Templates , select them using Shift +Click or Ctrl+Click keyboard shortcuts.
\$	Template Mode : Allows you to work with the entire Template . Specifically, pressing Delete will remove the selected Template . This mode is also applicable when changing annotations through key bindings.
\$	Single Beat Mode : Enables you to work with a selected, individual beat within a Template .
~	Check Templates: Allows you to mark Templates as checked.
Q	Show in Page : Facilitates an immediate switch to Page View , revealing the precise location of the chosen beat within this particular Template in both the Signal Page and the ECG Strip .


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 94 of 393	

Figure 198. Templates View - Show in Page Button

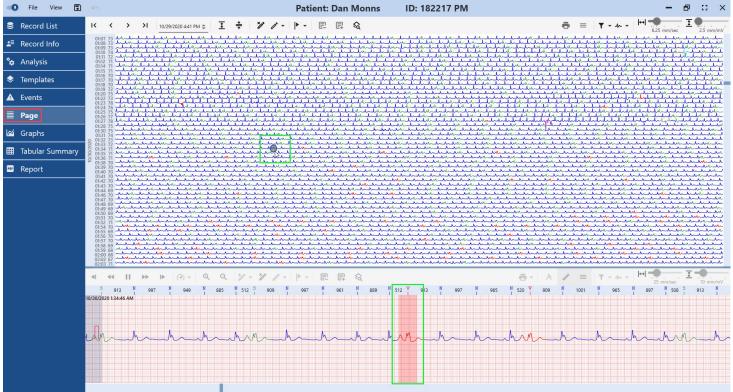

Templates Controls

Template Controls is a group of buttons that directly control the visibility of all template types, either as separate entities or collectively.

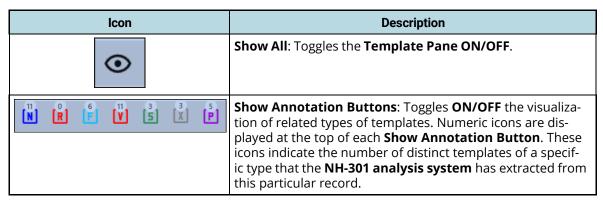

Layout Controls

Layout Controls are a set of buttons designed to **modify** the layout of the **Templates Pane** by adding or removing specific interface elements, as needed.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 95 of 393	

Figure 199. Templates View - Layout Controls OFF



I	con	Description
	Ē	Show Checked Templates : Toggles ON/OFF checked templates visualization alongside unchecked Templates in the general Templates Pane .
	(c)	Show Checked Templates Panel : Toggles ON/OFF a dedicated panel above the general Templates Pane , housing only checked templates.

Figure 200. Templates View - Layout Controls ON

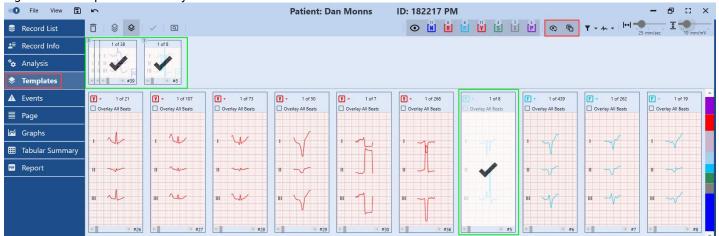


Filters

Filters facilitate the toggling of filtering and **ECG Channel** visualization rules for template boxes.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	01 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	ocument Rev.: 04 Document Date: 07.08.2025		Page 96 of 393	

Figure 201. Templates View - Filters

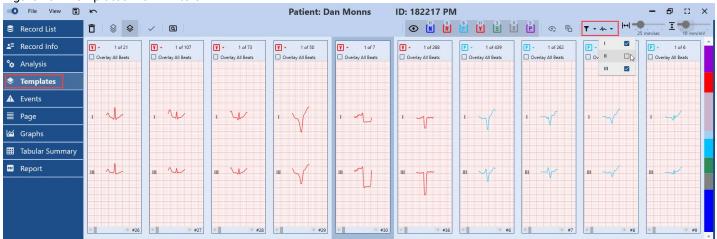


Icon	Description
T - An - (M) S See the D S Main S	Filter : Enables toggling of visualization ON/OFF with the applied EMG , Base Line , and Mains (on page 86) filters. To accomplish this, check or uncheck any number of checkboxes in the drop-down list.
	Channels : Allows toggling of ECG Record Channels display in Templates ON/OFF . To execute this, check or uncheck any number of checkboxes in the drop-down list, depending on the number of channels in a record and the quantity you wish to display.

Scale and Gain Controls

Drag **Scale and Gain Controls** sliders to set appropriate paper speed and amplitude for display in the template boxes.

Figure 202. Templates View - Scale and Gain Controls

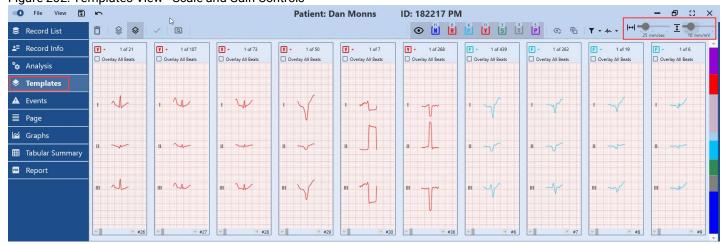


Scale and Gain Controls can help users identify and measure different ECG features more accurately, and analyze ECG data more efficiently by allowing them to quickly adjust the display of the template boxes to their needs.

4.5.6.1.2. Templates Pane

Reviewing numerous beats to ensure correct annotation and proper categorization into respective beat groups can be labor-intensive. The **NH-301 Holter** analysis system provides not only a highly accurate analysis algorithm but also extensive editing features, detailed below.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 97 of 393

Figure 203. Templates View - Templates Pane

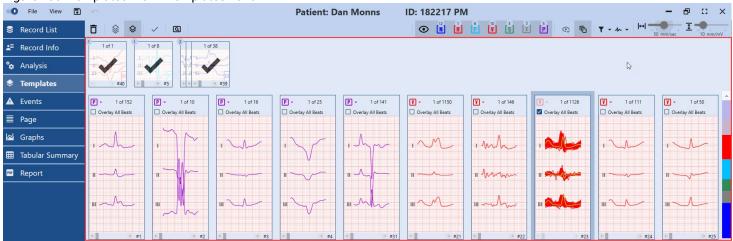

Each box functions as a template containing beats of similar morphology. Navigate through beats using either the arrow buttons inside the template box or your mouse scroll wheel.

Figure 204. Templates View - Templates Navigation

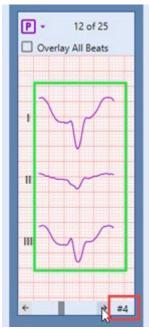


Pressing and holding the mouse button on an arrow enables quick cycling through all beats in that template. Alternatively, utilize a horizontal scrollbar at the lower section of the selected box by dragging the scroll thumb or clicking the scroll trough.




Note:


A number displayed in the bottom-right corner of each template indicates the template number, generated during both analysis and subsequent editing.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 98 of 393

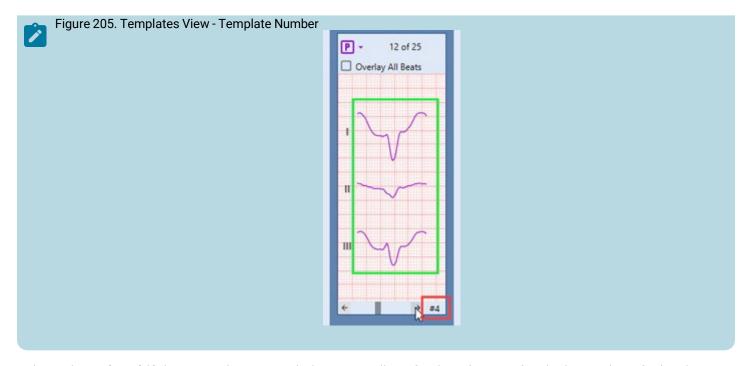

Utilizing the **Ctrl** or **Shift** key in combination with the mouse allows for the selection of multiple templates for batch processing:

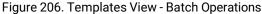

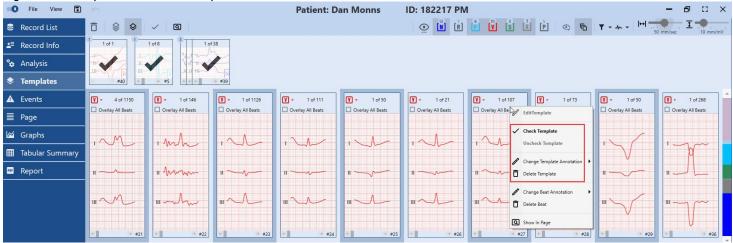

1. To select individual templates for modification, click on a template. Then press and hold the **Ctrl** key while clicking on each additional template you wish to select.

2.

To select a series of templates, click on the first template in the series. Press and hold the **Shift** key and click on the last template. All intervening templates will be automatically selected.

3.

You can now quickly perform batch operations, such as changing the annotation for all selected templates, deleting them, or checking them.

4.5.6.1.2.1. Overlay All Beats Feature


Swiftly assess all templates using the **Overlay All Beats** mode. This feature streamlines the review process and aids in the identification of outlier beats that deviate from the common pattern.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 99 of 393



To activate this feature:

• Check the **Overlay All Beats** checkbox located at the top of the template box.

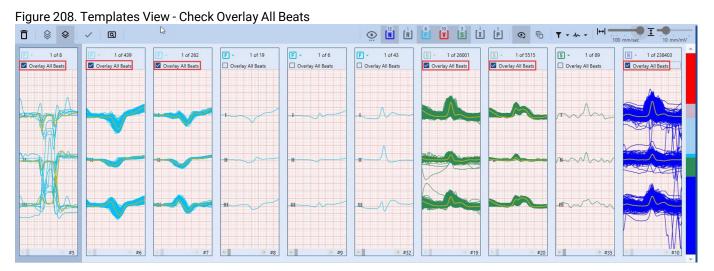


All beats within a given template are subsequently superimposed. The time required for this operation depends on the total number of beats in the template. Once superimposed, assess the alignment of the beats with the standard pattern displayed in the template box.

This example template box, depicted below, displays a range of beat shapes clustered together. Such inconsistencies may occur if beats have been manually renamed or templates merged.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 100 of 393

Figure 209. Templates View - Inconsistent Template Example

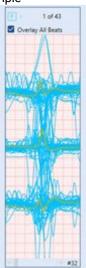

For optimal inspection of beat patterns, it is recommended to set the paper speed for the template view to 100 mm/s. This adjustment enlarges the template view for easier inspection.

Figure 210. Templates View - Scale Control



4.5.6.1.2.2. Merge Templates Feature

If your **Record** contains multiple templates, the **Merge Templates** feature can simplify your reports by combining templates with similar morphologies. Although the NH-301 software can handle a large number of templates, this feature offers an alternative for users who prefer streamlined reports.

To merge templates:

Figure 211. Templates View - Merge Templates



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 101 of 393

- 1. Choose two template boxes to merge.
- 2. Select the first template.
- 3. Start dragging the first template box onto the second one you have chosen in **Step 1**. The first box will become transparent during the drag, facilitating visual confirmation that the beat shapes align.
- 4. Release the mouse button to finalize the merge.



Note:

The **Merge Templates** feature does not affect the outcomes of beat or arrhythmia analysis. Its primary function is to reduce template count for easier viewing.


4.5.6.1.2.3. Reclassifying Templates Feature

While reviewing templates, you may find it necessary to reclassify them for a more detailed and accurate **ECG** analysis.

The **Templates Pane** provides several methods for reclassification:


1.Drop-down menu at the top left corner of each template box.

Figure 212. Templates View - Template Box Drop-Down

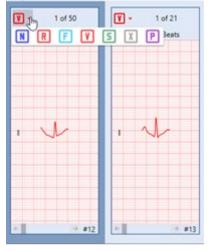

2.Right-click context menu.

Figure 213. Template View - Context Menu

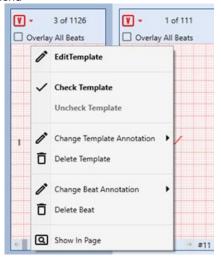

3. Keyboard shortcuts.


Further reclassification options are available in **Editing Templates** mode, which will be discussed in a subsequent section.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 102 of 393

To reclassify templates using the drop-down menu:

Hover over the drop-down list icon at the top-left corner of the template box you wish to reclassify. A reclassification menu will appear.

Figure 214. Templates View - Reclassification Using Drop-Down

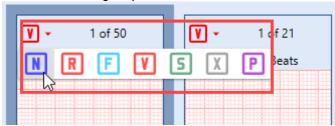

Select the arrhythmia type you wish to assign to the current template and click its button.

A "Processing" pop-up will briefly appear. Once it disappears, the template will be moved and grouped with templates of the same acquired type.



Note: This method does not support bulk actions.

To reclassify templates using the context menu:


- 3.1. Hover over the template box you wish to reclassify.
- 3.2. Right-click to expand the context menu.

Figure 215. Template View - Expand Context Menu



3.3. Hover over the **Change Template Annotation** option to expand the reclassification panel.

Figure 216. Templates View - Change Template Annotation

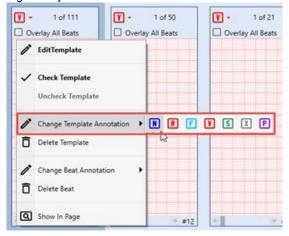

3.4. Select the arrhythmia type you wish to assign to the current template and click its button.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 103 of 393

3.5. A "Processing" pop-up will briefly appear. Once it disappears, the reclassification is complete, and the template will be moved and grouped accordingly.



Note: This method allows bulk actions. Multiple templates can be selected using **Ctrl+Click** or **Shift+Click** shortcuts to reclassify them in one action.

To reclassify templates using keyboard shortcuts:

- 3.1. Select the template box you wish to reclassify.
- 3.2. Use a standard keyboard shortcut. For a list of default shortcuts, refer below.
- 3.3. A "Processing" pop-up will briefly appear. Once it disappears, the reclassification is complete, and the template will be moved and grouped accordingly.



Note: This method also allows bulk actions using **Ctrl+Click** or **Shift+Click** shortcuts to reclassify them in one action.

Table 1. Templates Reclassification Keyboard Shortcuts

Action	But- ton	Keyboard Keys
Classify the template as Normal	N	Shift + N
Classify the template as R on T	R	Shift + R
Classify the template as Fusion	F	Shift + F
Classify the template as VPB	V	Shift + V
Classify the template as SVE	S	Shift + S
Classify template beats as Questionable	X	Shift + X
Classify template beats as Paced	P	Shift + P
Delete the template	Ō	Shift + Delete

Example: When you reclassify an **S**-annotated template to **N**-annotated, it will be moved and grouped with other **N**-templates in the **Templates Pane**.


4.5.6.1.2.4. Edit Template Feature

The **Edit Template** feature enables the viewing of every single beat within a template, facilitating the most thorough inspection possible.

To edit beats within a template, you have the following options:

Figure 217. Templates View - Edit Template Option

Document Title	e:	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical	
Document No.	:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 104 of 393

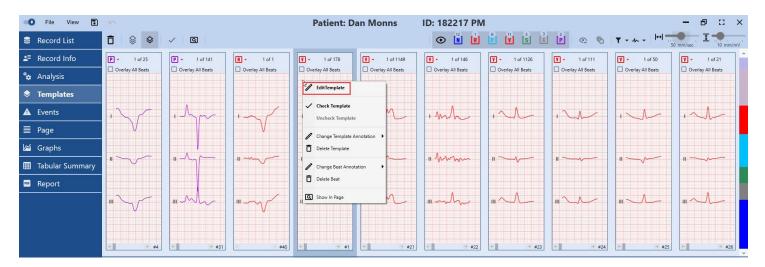


- · Double-click the template box.
- Alternatively, you can:
 - 1. Hover over the template box you want to edit.
 - 2. Right-click the box to expand the context menu.
 - 3. Select the **Edit Template** option from the context menu.

The screen layout changes automatically, as shown below.

Edit Template Mode

In the top pane of the **Edit Template** mode, you can view each beat of the template, facilitating the identification of beats that may be classified as noise or may be incorrectly labeled due to noise or artifacts. In the bottom pane, you can see the **Strip View**, identical to that in other views and modes, which will be explained later.



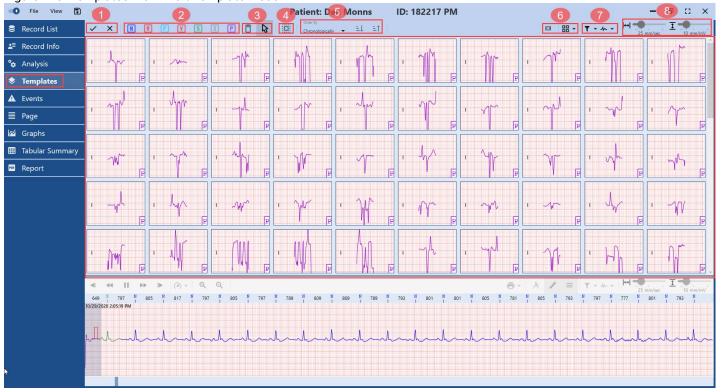

Beats within the template can be managed using the top toolbar or through keyboard controls. First, we will explain the top toolbar controls. To streamline the process, we will group them into categories of UI elements:

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 105 of 393

- 1. Save and Discard Changes Controls.
- 2. Reclassification Controls.
- 3. Mouse Function Controls.
- 4. Select All Control.
- 5. Sorting Controls.
- 6. Layout Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

Save and Discard Changes Controls

This group of controls enables exiting **Edit Template** mode, either saving or discarding all changes.

le	cons	Description
	<	Save: Click to save all changes you've made in the Edit Template mode and return to the Templates View .
	X	Cancel: Click to discard all changes you've made in the Edit Template mode and return to the Templates View.

Reclassification Controls

This group of controls allows you to reclassify beats individually or in bulk.

Figure 219. Template Edit - Reclassification Controls



Click one of these buttons to reclassify beat(s) using one of the available methods. Alternatively, refer to the **Templates Reclassification Keyboard Shortcuts** table below to change beat types via keyboard shortcuts. To select multiple beats, utilize **Ctrl + Click** or **Shift + Click** shortcuts.

You can either first click a reclassification button and then select beats to rename, or first select beats and then click the button:

- Option 1: To reclassify beats:
 - 8.1. Click one of the **Reclassification Controls**. This will activate a reclassification action: beats you mark in the next step will be reclassified to the beat type of your choice.


Figure 220. Template Edit - Reclassification Button Activated



8.2. Right-click beats in the **Beats Pane** one by one. This changes their annotation type and color.

Figure 221. Template Edit - Beats Pane

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 106 of 393

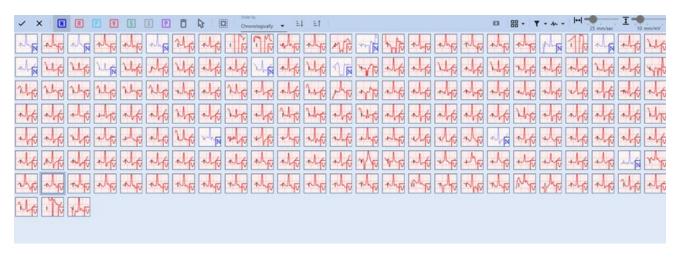

- **8.3. (Optional)** Scroll to review all the beats in the current template. The **Beats Pane** can accommodate up to 250 beats, while a template may contain thousands.
- 8.4. Click the **Save** button on the left of the top toolbar after completing reclassification.

Figure 222. Template Edit - Save Button



Option 2: To reclassify beats:

8.1. Select beats you want to reclassify using **Ctrl + Click** or **Shift + Click**. You can also click the **Select All** button in the top toolbar to select all beats.






- **8.2. (Optional)** Scroll to review all the beats in the current template. The **Beats Pane** can accommodate up to 250 beats, while a template may contain thousands.
- **8.3**. Click one of the **Reclassification Controls**. This immediately reclassifies all selected beats. Beat boxes will change their annotation type and color of the beat visualization.

Figure 224. Template Edit - Reclassification Button Activated



Figure 225. Template Edit - Beats Pane

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 107 of 393

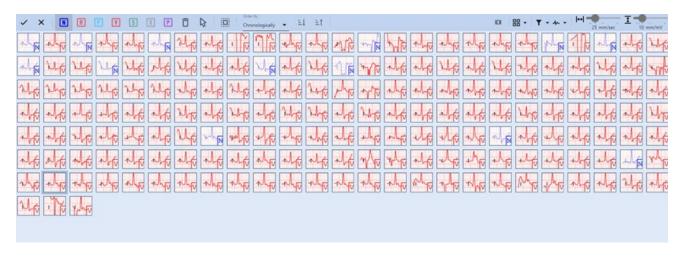

8.4. Click the **Save** button on the left of the top toolbar after completing reclassification.

Figure 226. Template Edit - Save Button



Example: When you reclassify a **V**-type beat into an **N**-type beat, the icon at the bottom left of the beat box changes from **V** to **N**, and the color switches from red to blue. Similarly, the color of the beat changes from red to blue.

Figure 227. Template Edit - Reclassification Example

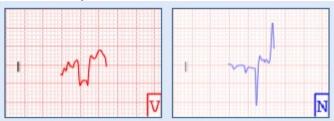


Table 2. Templates Reclassification Keyboard Shortcuts

Action	But- ton	Keyboard Keys
Classify the template as Normal	N	Shift + N
Classify the template as R on T	R	Shift + R
Classify the template as Fusion	F	Shift + F
Classify the template as VPB	V	Shift + V
Classify the template as SVE	5	Shift + S
Classify template beats as Questionable	X	Shift + X
Classify template beats as Paced	P	Shift + P
Delete the template	Ō	Shift + Delete

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 108 of 393

This group of controls allows you to toggle between the **Free Cursor** mode and **Delete** mode. These modes affect your interaction with beats in the **Edit Template** mode.

Icons	Description
1	Switch to Free Cursor button : Allows you to switch from beats annotation function or Delete mode to Free Cursor mode. After you have changed annotations or marked for deletion beats you wanted , you can deactivate the annotation feature by clicking this button. This protects against unintended changes due to a misclick.
	Switch to Delete button : Click to activate the Delete mode. Alternatively, use the Shift + Delete keyboard shortcut. The Delete mode operates as follows:
	 Upon activation, it marks all previously selected beats for deletion. If no beats are selected, none will be marked for deletion.
	After activation, right-click on additional beat boxes to mark them for deletion.

You can either first click the Switch to Delete button and then select beats, or first select beats and then click the button (additional beats can still be added):


- Option 1: To delete beats:
 - 1. Click the **Switch to Delete** button. This activates **Delete** mode: subsequent beats you select will be marked for deletion.

Figure 228. Template Edit - Delete Button



2. Right-click on beats in the **Beats Pane** individually. This marks them for deletion and changes their beat visualization.

Figure 229. Template Edit - Marking for Deletion

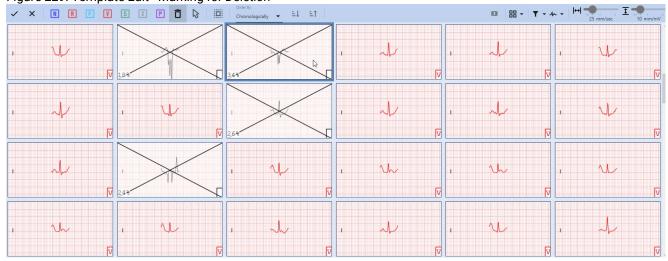

- 3. **(Optional)**Scroll to review and select all desired beats for deletion in the current template. The **Beats Pane** can accommodate up to 250 beats; templates may contain thousands.
- 4. Click the **Save** button on the left of the top toolbar after marking beats for deletion.


Figure 230. Template Edit - Save Button




Option 2: To delete beats:

1. Select beats you want to delete, using **Ctrl + Click** or **Shift + Click**. Alternatively, click the **Select All** button in the top toolbar to select all beats.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 109 of 393	

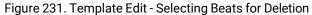

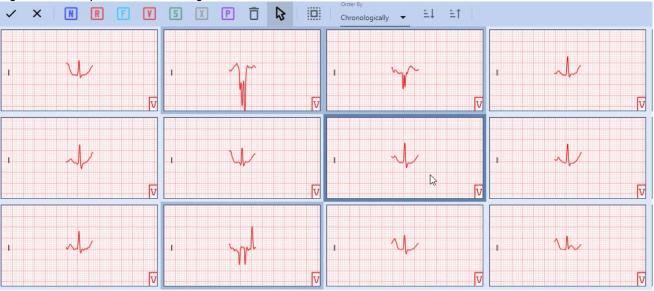

- 2. **(Optional)** Scroll down and up to review and select all the beats you want to delete in the current template. The **Beats Pane** can accommodate up to 250 beats; templates may contain thousands.
- 3. Click the **Switch to Delete** button. This activates **Delete** mode and marks all selected beats for deletion. Marked beat boxes will shift their visualization to grayscaled.


Figure 232. Template Edit - Marking Beats for Deletion

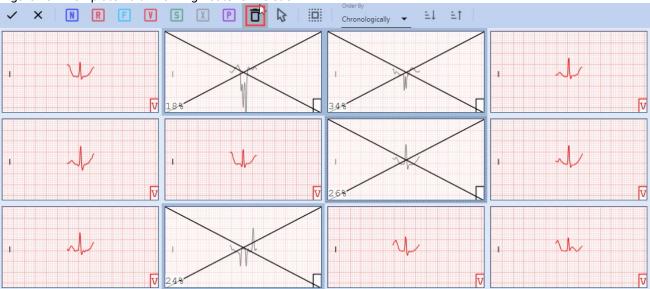

4. (Optional) Continue marking beats by right-clicking on them.

Figure 233. Template Edit - Right-click to Mark for Deletion

Document Title:	NH	-301 ver. 4.0.0.10 – Insti	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 110 of 393

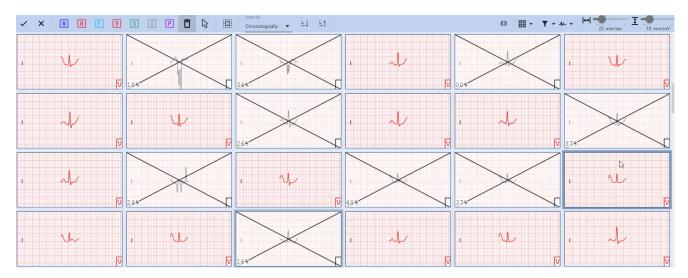

5. Click the **Save** button on the left of the top toolbar after marking beats for deletion.


Figure 234. Template Edit - Save Button



Example:

When a beat is marked for deletion, the beat box turns grayscaled and crossed.

Figure 235. Template Edit - Marking for Deletion

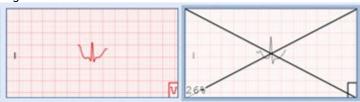

Select All Control

This control enables you to select all beats within the template for bulk action.

Figure 236. Template Edit - Select All Button



You may use the **Select All** button to reclassify or delete all beats at once within the template.

To reclassify beats:

- 1. Click the **Select All** button to select all beats within the current template.
- 2. Click one of the **Reclassification Controls**. This immediately reclassifies all selected beats to the beat type of your choice. Beat boxes will change their annotation type and the color of the beat visualization.

Figure 237. Template Edit - Reclassification Button Activated



3. Click the **Save** button on the left side of the top toolbar after completing the reclassification of beats.

Figure 238. Template Edit - Save Button

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical			
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025			



To delete beats:

- 1. Click the **Select All** button to select all beats within the current template.
- 2. Click the **Switch to Delete** button. This activates **Delete** mode and immediately marks all selected beats for deletion. Marked beat boxes will shift their visualization to grayscaled.

Figure 239. Template Edit - Delete Button



3. Click the **Save** button on the left side of the top toolbar after you have completed marking beats for deletion.

Figure 240. Template Edit - Save Button



Sorting Controls

The first control in this group, the **Order by** drop-down list, allows you to toggle between viewing beats sorted in chronological order, based on their appearance in the ECG Record, by their RR interval duration, or by beat prematurity.

Figure 241. Template Edit - Sorting Controls




Two icons on the right enable ascending or descending sorting, respectively.

To sort beats with the **Order by** drop-down list:

1. Click the drop-down list.

Figure 242. Template Edit - Order by Sorting

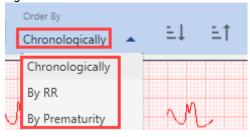


2. Select your desired sorting type. The order of the beats in the **Beats Pane** will change immediately.

To toggle a sorting order:

1. Click one of the buttons to switch sorting orders — all the beats will rearrange according to your choice.


Figure 243. Template Edit - Ascending and Descending Sorting

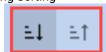

2. Click the second **Sorting Control** button to reverse the current sorting order.

Figure 244. Template Edit - Changing Sorting Order



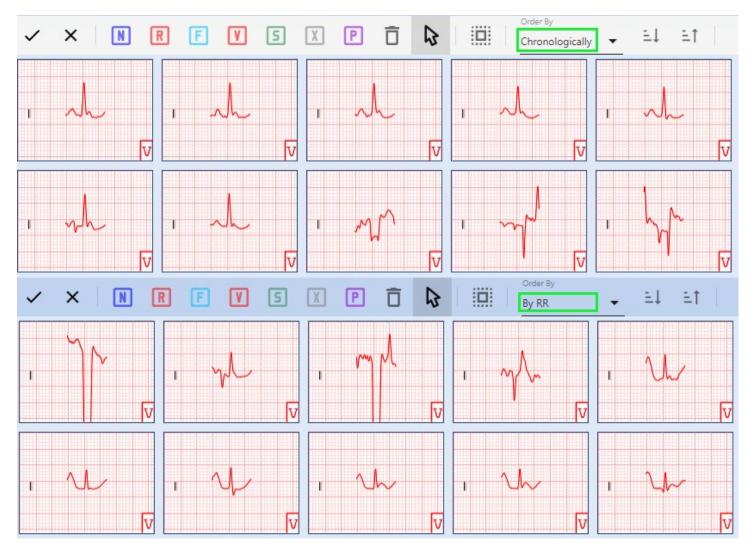

Layout Controls

This group of controls allows you to alter the layout of beat boxes and the **Beats Pane** to suit your reviewing needs and preferences.

Icons	Description					
	Show Single Beat button: Enables the visualization of adjacent QRS fragments within beat boxes, facilitating the assessment of larger fragments in each beat box without switching Views or modes.					
₩ *	Beats Pane Layout button: Specifies the number of beat boxes displayed per row and the number of rows within the Beats Pane (ranging from 1 to 250 beat boxes).					

To toggle adjacent QRS fragments ON/OFF:

1. Click the **Show Single Beat** button. The visualization in beat boxes will immediately change: an adjacent ECG fragment will appear in every single beat box.

To change the **Beats Pane** layout:

- 1. Hover over the **Beats Pane Layout** button to expand the Grid Control element. This element allows visual selection of the total number of beat boxes to display on one **Beats Pane** page.
- 2. Choose the exact number of rows and columns by hovering over the expanded Grid Control drop-down. Move your mouse cursor from left to right and from top to bottom to adjust the number of rows and columns.
- 3. Click the Grid Control element when you have chosen the desired row and column count. The **Beats Pane Layout** will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 113 of 393

This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

lcc	ons	Description				
•		Filter : This control enables you to toggle ON/OFF the visualization of beats with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the dropdown list. These filters improve the accuracy and reliability of ECG analysis.				
1	. •	Channel Selection: This control lets you select the channel(s) for display within the beat boxes, using the drop-down list (up to 12 channels, depending on the number of channels in the current Record).				

To toggle filters **ON/OFF**:

- 1. Hover over the **Filter** drop-down list to expand it.
- 2. Select the filter you wish to toggle. The visualization within some beat boxes will change immediately.

It is recommended to keep filters ON to provide accurate analysis:

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within beat boxes:


- 1. Hover over the **Channel Selection** drop-down list to expand it.
- 2. Select filters you wish to toggle. The visualization of beat boxes in the **Beats Pane** will change immediately.

Toggling **ON/OFF** ECG channels in beat templates enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise, and create custom beat template visualizations tailored to your particular needs.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 245. Template Edit - Scale and Gain Controls

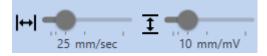


To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within beat boxes will change immediately.

Figure 246. Template View - Setting Paper Speed

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 114 of 393

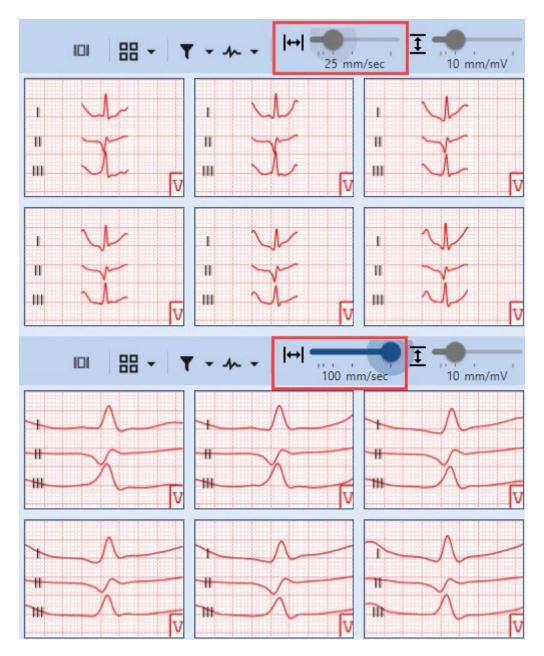


To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within beat boxes will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 115 of 393

Figure 247. Template View - Setting Amplitude

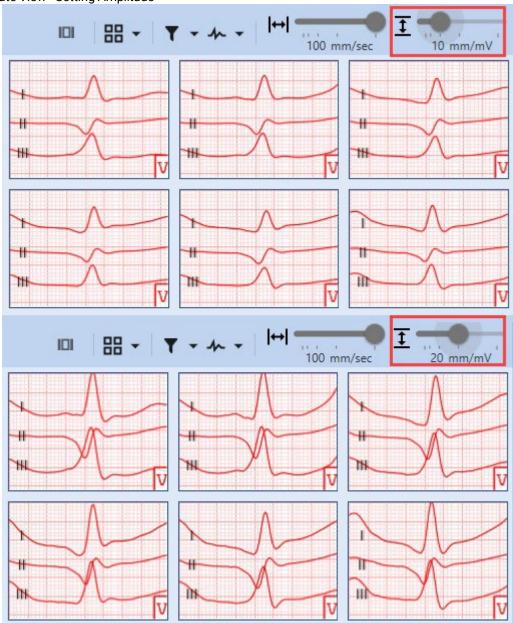

4.5.6.1.2.4.1. Strip View

A generic view of the **NH-301 Holter** application **Strip View** is displayed in the bottom pane of the **Templates View**, as well as in the **Events** and **Page Views**. The **Strip View** in the **Edit Template** mode has somewhat limited functionality compared to the identical **Strip View** in the "parent" **Templates View**. In the **Edit Template** mode, the main focus is on editing beats; therefore, the Strip View presents primarily visual data aimed at assisting with beat analysis, comparison, and classification.

Consequently, the **Strip View** in **Edit Template** mode has no support for the following capabilities:

- Inserting, deleting, and reclassifying beats.
- Creating user events.
- Excluding or including fragments in analysis.
- Show in Templates function.
- Show in Page function.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 116 of 393



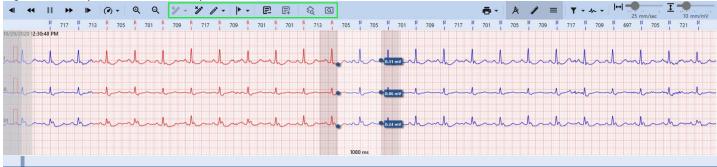


Figure 249. Strip View - Edit Template Mode Strip View





Note:

The ECG Strip grid is scaled accurately to millimeters, irrespective of the computer or monitor size in use. The **NH-301 Holter** analysis system automatically adjusts all application windows to align with the computer's graphical settings and monitor capabilities. This feature enables you to use any measuring tool, including specialized ECG rulers like the **Norav Medical ECG ruler**, to measure amplitude, cycles, frequency, and RR intervals.

4.5.6.1.2.4.1.1. Strip View Toolbar

The toolbar at the top of the **Strip View** in the **Edit Template** mode contains various controls that facilitate the scanning and reviewing of the ECG signal within the **Strip View** itself. For clarity, these controls are organized into distinct groups of interface elements:



Note: The **Strip View** in the **Edit Template** mode contains fewer controls in the toolbar than does a generic **Strip View** displayed in the **Page** and **Events Views**. Refer to the dedicated Strip View section (on page 137) for further details about generic ECG **Strip Views** in this application.

Figure 250. Strip View - Edit Template Mode Strip View



- 1. General Controls.
- 2. Printing Control.
- 3. Measuring Controls.
- 4. Filters.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 117 of 393

5. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate efficient ECG Strip scanning and review within the Strip View.

Icon	Description					
•	Step Backward button: Click to move the ECG Strip one step backward for scanning and reviewing.					
*	Scan Backward button: Click to initiate continuous backward scanning of the ECG Strip.					
П	Pause Scan button: Click to halt continuous scanning of the ECG Strip. For example, if you have activated a Scan Backward or a Scan Forward button, halt scanning by clicking the Pause Scan button.					
>>	Scan Forward button: Click to initiate continuous forward scanning of the ECG Strip.					
I	Step Forward button: Click to move the ECG Strip one step forward for scanning and reviewing.					
(d) +	Scan Speed button: Allows you to control the scanning speed. To set the scanning speed:					
	1. Hover over the Scan Speed icon to expand the drop-down list.					
	2. Click the desired scanning speed multiplier, ranging from ×1 to ×128.					
Q	Zoom In button: Click to adjust the scale of the waveform in the Strip View , focusing on specific areas or comparing different fragments of the ECG record.					
Q	Zoom Out button: Click to adjust the scale of the waveform in the Strip View . Use this button to zoom out if you had zoomed in earlier, or to view adjacent beats and fragments of the ECG Records.					




Note: You can also click and hold the strip to drag it sideways, allowing you to view adjacent beats and ECG fragments.

Printing Control

This button enables you to print a fragment of the ECG **Strip**. The printed copy will include a segment that fits the output format. The baseline of the printed version mirrors the center line of the Strip visible on your PC screen. The printed ECG Strip will accommodate as much ECG data as possible from the visible area around the center line.

Figure 251. Strip View - Printing Strip

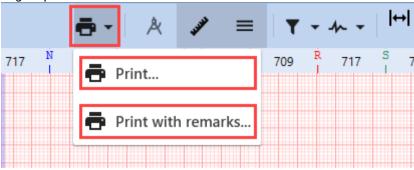


To print a fragment of the waveform in the **Strip View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Strip View (on page 118) to locate the desired area, if needed.
- 3. Hover over the **Printing Control** button to expand the drop-down list.
- 4. Click the **Print** option and navigate to the Print dialog box:


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 118 of 393

Figure 252. Strip View - Printing Dialog Box

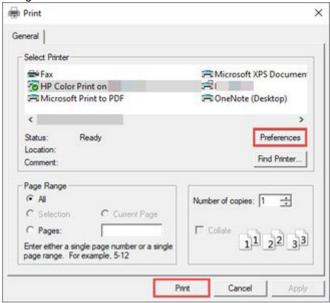

- 4.1. Choose a printer **available** in the **Select Printer** window.
- 4.2. Adjust other preferences according to your needs.
- 4.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.
- 5. (Optional) Click the **Print with remarks** option if you want to print a fragment of the waveform with your remarks:
 - 5.1. Complete the "Report remarks" field in the **Edit Report Remarks** pop-up.

Figure 253. Strip View - Printing with Remarks

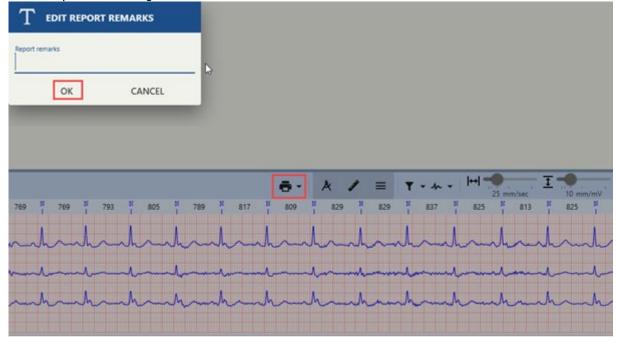


- 5.2. When you are complete with filling in you remarks, click OK.
- 5.3. (Optional) Click Cancel to abandon this action.
- 5.4. Choose a printer available in the **Select Printer** window.
- 5.5. Adjust other preferences according to your needs.
- 5.6. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	N©RAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 119 of 393	


Measuring Controls

Measuring Controls is a group of buttons designed to facilitate the indication and measurement of various waveform parameters within the **Strip View**. Namely, the most important tool in this group of controls is the **Caliper** tool. It allows you to measure different intervals and amplitudes on the ECG strip, which can be used to diagnose and assess a variety of heart conditions.

lcon	Description
*	Caliper toggle : Click to toggle ON/OFF the Caliper tool. The Caliper tool assists with measuring intervals and amplitudes, such as the RR interval, T-wave, and QRS complex amplitudes. Refer to the Measuring ECG with Caliper (on page 160) section for details.
Tricks	ECG Ruler toggle : Click to toggle ON/OFF the ECG Ruler , located right below the Strip View Toolbar . The ECG Ruler indicates the duration of RR intervals for adjacent beats and their morphology classification (i.e., N, R, F, V, etc.).
	Channel Numeration toggle: Click to toggle ON/OFF the channel numeration on the left side of the ECG Strip.

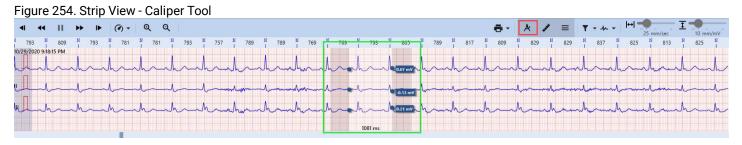

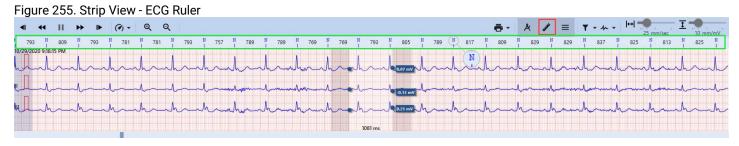

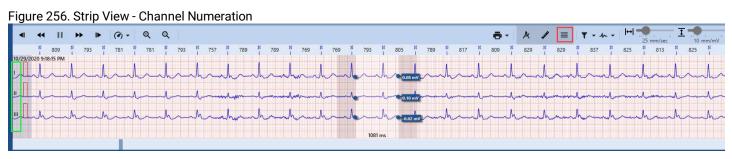

Filters

This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.


Icon	าร	Description
▼ .	•	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.
1	•	Channel Selection: This control lets you select the channel(s) for display within the Strip View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record).

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025			

To toggle filters **ON/OFF**:

1. Hover over the **Filter** drop-down list to expand it.

Figure 257. Strip View - Toggling Filters

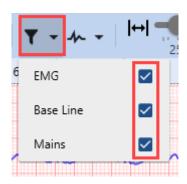

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.


It is recommended to keep filters **ON** to provide accurate analysis:

- **EMG filter:** Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Strip:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 258. Strip View - Toggling Channels

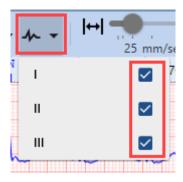

2. Select filters you wish to toggle. The visualization of the Strip will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 259. Template Edit - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within the Strip will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025			

Figure 260. Strip View - Paper Speed



To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within the Strip will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV medical	
Document No.:	NV-200.40010.028	IV-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025			


Figure 261. Strip View - Amplitude



4.5.6.1.2.4.1.2. Measuring ECG with Caliper

You may use the **Caliper** function to measure the RR interval, QRS complex duration and amplitude, QT interval, and other ECG waveform parameters.

Figure 262. Strip View - Caliper Tool

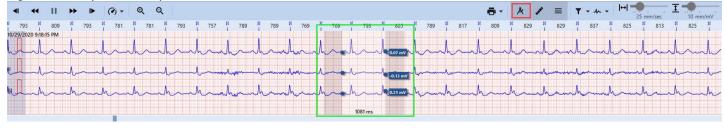

Caliper Design Overview

The **Caliper** tool in the **Strip View** displays two **measuring lines** (1) and a **measuring interval** (2) between them. To activate the **Caliper** tool, click the **Caliper** button in the **Strip View Toolbar**.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 123 of 393

Figure 263. Strip View - Caliper Measurement Lines and Interval



Each ECG channel in the Strip has a small box adjacent to the point where the right measurement line intersects the channel signal line. The value in this box indicates the amplitude difference between the two points demarcated by the measurement lines.

Figure 264. Strip View - Interval Length and Amplitude Difference

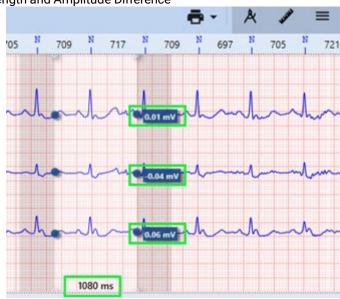

The interval length in milliseconds is displayed at the bottom of the measuring interval.

Moving Caliper Lines

You may move Caliper measurement lines either **simultaneously** or **independently**.

To move both lines simultaneously:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 124 of 393	

1. Position the cursor within the measuring interval. The cursor will change to a resize cursor.

Figure 265. Strip View - Moving Caliper Lines Simultaneously



- 2. Drag and drop the **Caliper** to the desired location.
- 3. (Optionally) Use the Left and Right arrow keys to fine-tune the position.

To move lines independently:

1. Position the cursor near one of the measurement lines until it changes to a resize cursor.

Figure 266. Strip View - Moving Caliper Lines Independently



- 2. Click to select the measurement line.
- 3. Drag and drop it to the new location.

Zoom Feature

To adjust the display scale of the waveform in the Strip, utilize the **Zoom In/Zoom Out** options:

Figure 267. Strip View - Zooming Options



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 125 of 393	

- Click the corresponding button in the **Strip View Toolbar**.
- Hold the Ctrl key while scrolling up (Zoom In) or down (Zoom Out) the mouse wheel.

Caliper Snapping Feature

The **Caliper Snapping** feature automatically aligns the **Caliper** measurement lines to the nearest R-peaks in the ECG signal. This functionality aids in the precise measurement of ECG wave duration and amplitude. Manual alignment can be challenging, hence **Caliper Snapping** enhances the accuracy, efficiency, and reproducibility of ECG interpretation.

To utilize the Caliper snapping feature:

- 1. Click the Caliper button in the Strip View Toolbar.
- 2. (Optional) Use the Left and Right arrow keys to make slight adjustments to the Caliper measurement lines.
- 3. To snap a **Caliper** measurement line to an R-spike:

Figure 268. Strip View - Caliper Snapping



- To snap any line:
 - 3.1. Hold down the **Alt** key.
 - 3.2. Position the cursor near one of the measurement lines until the cursor icon changes to a resize cursor.
 - 3.3. Click the measurement line to select it.
 - 3.4. Drag and drop it to the desired location.
- To snap the right measurement line:
 - 3.1. Hold down the **Right Alt** key.
 - 3.2. Use the Left and Right arrow keys to move the **Caliper**; the right measurement line will automatically snap to the R-peaks as you move.
- To snap the left measurement line:
 - 3.1. Hold down the Left Alt key.
 - 3.2. Use the Left and Right Arrow keys to move the Caliper; the left measurement line will automatically snap to the R-peaks as you move.

The **Caliper Snapping** function is instrumental in enabling a quick and accurate evaluation of various ECG parameters.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025			

4.5.6.1.2.5. Marking Reviewed Templates

The reviewed templates can be marked to help you distinguish between them and the templates not yet reviewed.

Figure 269. Templates View - Checked Templates Panel ON



To mark a template as reviewed:

1


Select a Template: Click a template to select it.

2.

(Optional) Select Multiple Templates:

• **Select Individual Templates**: Hold down the **Ctrl** key while clicking on additional templates. The system will highlight them with a darker frame.


Figure 270. Templates View - Selecting Individual Templates

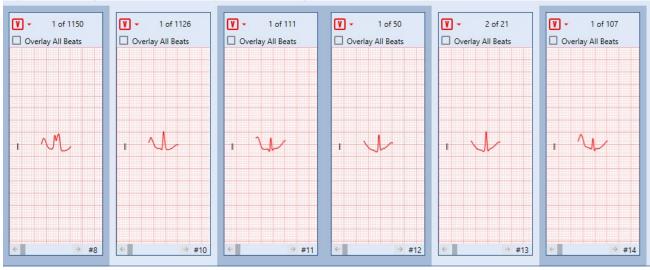

Select a Range of Templates:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 127 of 393	

Figure 271. Templates View - Selecting Range of Templates

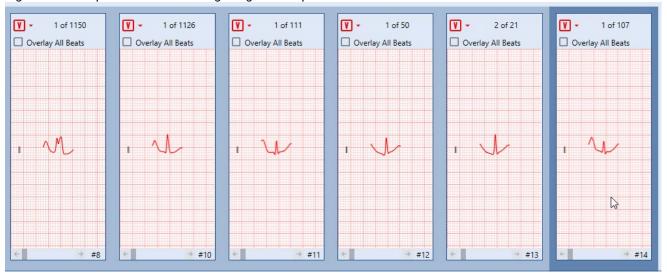


- 2.1. Identify the initial template in the range you wish to select.
- 2.2. Hold down the **Shift** key.
- 2.3. Click the initial template.
- 2.4. Identify the final template in your desired range.
- 2.5. Click the final template to complete the range selection.
- Select all templates:
 - 2.1. Use the **Ctrl+A** keyboard shortcut to select all templates displayed in the Templates Pane at once. Note: You can only select displayed templates. Adjust the types of templates displayed using the **Templates Controls** (on page 95) in the **Templates Pane Toolbar**.

3.

Mark as Reviewed:

• Right-click on a selected template and choose **Check Template** from the context menu.

Figure 272. Templates View - Check Template

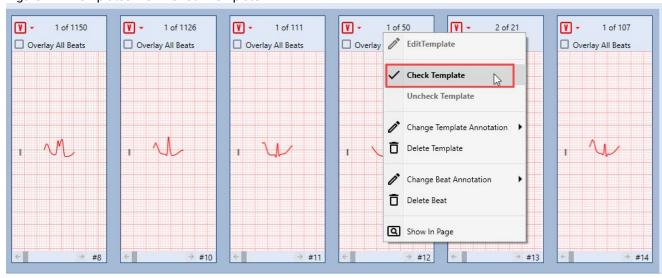

Click the Check Templates button located in the Templates Pane Toolbar at the top of the Templates View.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 128 of 393	

Figure 273. Templates View - Check Templates



• **(Optional)** You can also mark individual templates as checked by selecting them and pressing the Space key on your keyboard.

Figure 274. Templates View - Check Individual Template



Upon completion, the system will mark all selected templates as reviewed. If you have enabled the **Checked Templates**Panel (on page 38) under **File > Settings > Templates**, and the **Show Checked Templates Panel** button is toggled **ON**, these marked templates will be displayed in the panel at the top of the screen.

Figure 275. Templates View - Show Checked Templates Panel Button



To unmark a template marked as reviewed:

3.1. Hover over a template in the **Templates Pane** marked as checked. Verify that the **Show Checked Templates** button in the **Templates Pane Toolbar** is toggled ON. If it's not ON, checked templates will not be displayed in the **Templates Pane**.

Figure 276. Templates View - Show Checked Templates Button




3.2. Right-click on the checked template to expand the context menu.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV medical
Document No.:	NV-200.40010.028	010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 129 of 393


3.3. Click the **Uncheck Template** option. Alternatively, press the **Space** key on your keyboard. The checkmark will be removed, and the template will return to its default state.

Figure 277. Templates View - Uncheck Template Option

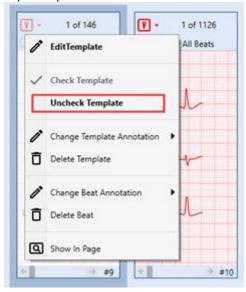

If the **Show Checked Templates Panel** button in the **Templates Pane Toolbar** is also toggled ON, the template will be removed from the **Checked Templates Panel** upon unchecking.

Figure 278. Templates View - Uncheck Template

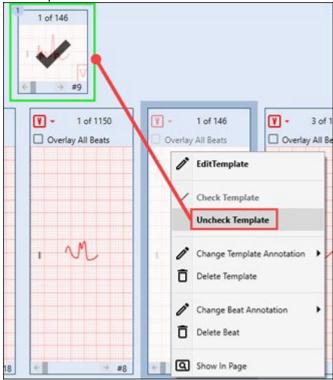


4.5.6.1.2.6. Context Menu Features

A context menu, accessible via right-click, enables you to quickly and easily access common actions, such as editing or deleting templates and beats, changing annotations, and marking templates as reviewed, etc.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 130 of 393	

Figure 279. Template View - Context Menu

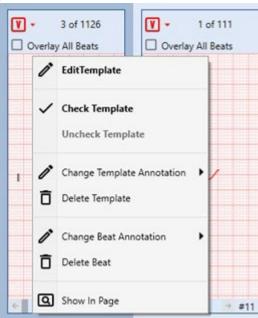


Edit Template

Allows you to switch to the **Edit Template** mode:

- 1. Hover over the template box you want to edit.
- 2. Right-click the box to expand the context menu.
- 3. Select the **Edit Template** option.

Figure 280. Templates View - Edit Template Option

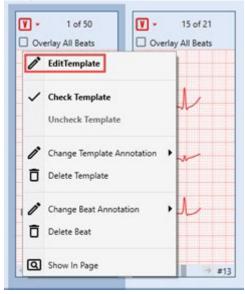

Check Template


Enables you to mark templates as reviewed:

- 3.1. Hover over the template box you want to mark as reviewed.
- 3.2. Right-click the box to expand the context menu.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 131 of 393	

3.3. Select the **Check Template** option.

Figure 281. Templates View - Check Template Option

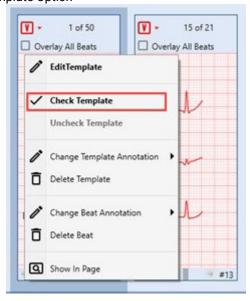


Uncheck Template

Allows you to unmark templates previously marked as reviewed:

- 3.1. Hover over the template box you want to unmark.
- 3.2. Right-click the box to expand the context menu.
- 3.3. Select the **Uncheck Template** option.

Figure 282. Templates View - Uncheck Template Option

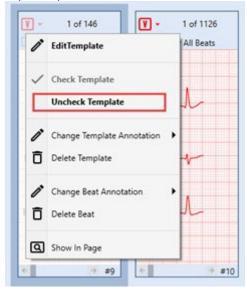


Change Template Annotation

Allows you to reclassify templates:

- 3.1. Hover over the template box you wish to reclassify.
- 3.2. Right-click the box to expand the context menu.
- 3.3. Hover over the **Change Template Annotation** option to expand the reclassification panel.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 132 of 393

Figure 283. Templates View - Change Template Annotation Option

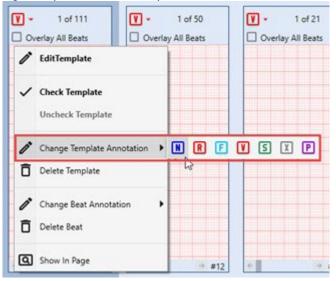

- 3.4. Select the arrhythmia type you wish to assign to the current template and click its button.
- 3.5. A "Processing" pop-up will briefly appear. Once it disappears, the reclassification is complete, and the template will be moved and grouped accordingly.




Note: This method allows bulk actions. Multiple templates can be selected using **Ctrl+Click** or **Shift+Click** shortcuts to reclassify them in one action.

Delete Template

Allows you to delete templates:

- 3.1. Select the template box you wish to delete.
- 3.2. (Optional) Select additional templates using Ctrl+Click or Shift+Click keyboard shortcuts, if needed.
- 3.3. Right-click selected box to expand the context menu.
- 3.4. Click the **Delete Template** option.

Figure 284. Templates View - Delete Template Option

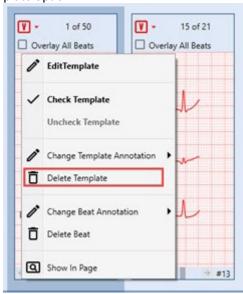

3.5. A "Processing" pop-up will briefly appear. Once it disappears, the deletion is complete, and the template(s) will be removed from the **Templates Pane**.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 133 of 393

Change Beat Annotation

Allows you to reclassify individual beats within a selected template box:

- 3.1. Click a template box.
- 3.2. Choose a beat you want to reclassify. Navigate through beats using one of the following methods:
 - Use the arrow buttons inside the template box. Pressing and holding the mouse button on an arrow facilitates rapid cycling through all beats within that template.
 - Utilize a scroll wheel.
 - Drag the scroll thumb on the horizontal scrollbar located at the lower section of the selected box.
 - Click the scroll trough in the horizontal scrollbar at the lower section of the template box.
- 3.3. Right-click the box to expand the context menu.
- 3.4. Hover over the **Change Beat Annotation** option to expand the reclassification panel.

Figure 285. Templates View - Change Beat Annotation Option

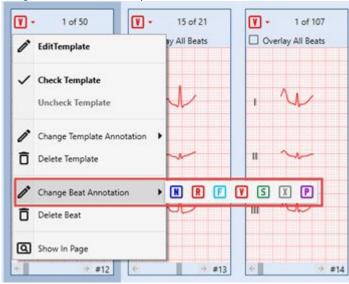

- 3.5. Select the arrhythmia type you wish to assign to the current beat and click its button.
- 3.6. A "Processing" pop-up will briefly appear. Once it disappears, the reclassification is complete, and the beat will be moved and grouped accordingly.



Note: This method doesn't allow bulk actions.

Delete Beat


Allows you to delete an individual beat:

- 3.1. Click a template box.
- 3.2. Choose a beat you want to delete. Navigate through beats using one of the following methods:
 - Use the arrow buttons inside the template box. Pressing and holding the mouse button on an arrow facilitates rapid cycling through all beats within that template.
 - Utilize a scroll wheel.
 - Drag the scroll thumb on the horizontal scrollbar located at the lower section of the selected box.
 - Click the scroll trough in the horizontal scrollbar at the lower section of the template box.
- 3.3. Right-click the box to expand the context menu.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 134 of 393

3.4. Click the **Delete Beat** option.

Figure 286. Templates View - Delete Beat Option

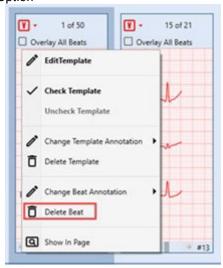

3.5. A "Processing" pop-up will briefly appear. Once it disappears, the deletion is complete, and the beat will be removed from the template.



Note: This method doesn't allow bulk actions.


Show In Page

Shifts focus to the selected ECG Data Segment in the **Page View**, enabling a transition from examining individual beats or templates to a broader, **Full Disclosure View** for comprehensive analysis:


- 3.1. Click a template box.
- 3.2. Choose a beat you want to focus on. Navigate through beats using one of the following methods:
 - Use the arrow buttons inside the template box. Pressing and holding the mouse button on an arrow facilitates rapid cycling through all beats within that template.
 - Utilize a scroll wheel.
 - Drag the scroll thumb on the horizontal scrollbar located at the lower section of the selected box.
 - Click the scroll trough in the horizontal scrollbar at the lower section of the template box.
- 3.3. Right-click the box to expand the context menu.
- 3.4. Click the **Show in Page** option.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025	

Figure 287. Templates View - Show In Page Option

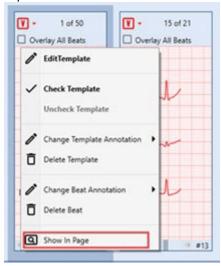


3.5. You will be redirected to the **Page View**. The beat you selected in **Step 2** will be marked with a circle in a **Full Disclosure Page** and selected in the **Strip View** below the **Full Disclosure Page**.

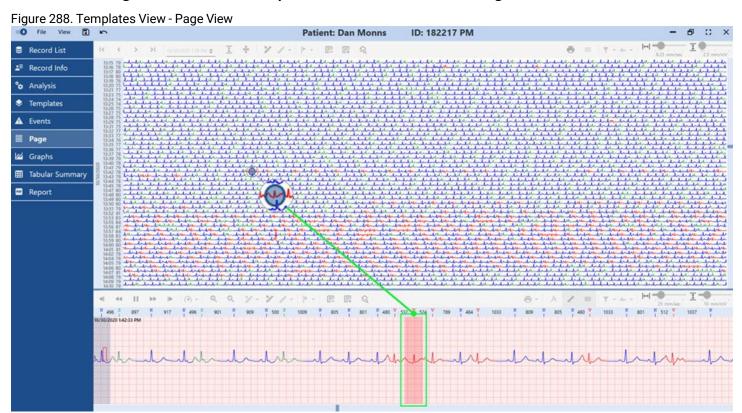


4.5.6.1.3. Strip View

The ECG **Strip View** is displayed in the bottom pane of the **Templates View** and in the **Edit Templates Mode**. It is also displayed in the **Events**, **Page**, and **Graphs Views**. The **Strip View** presents detailed data for the currently selected beat, as well as the adjacent ECG signal. Typically, the strip displays 12 to 16 seconds of ECG signal, depending on the monitor characteristics and ECG paper speed settings. You can utilize the **Strip View** for detailed analysis, measurements, beat reclassification, user event creation, and more.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 136 of 393

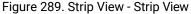

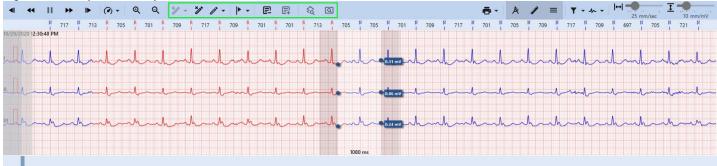




Note:

The ECG Strip grid is scaled accurately to millimeters, irrespective of the computer or monitor size in use. The **NH-301 Holter** analysis system automatically adjusts all application windows to align with the computer's graphical settings and monitor capabilities. This feature enables you to use any measuring tool, including specialized ECG rulers like the **Norav Medical ECG ruler**, to measure amplitude, cycles, frequency, and RR intervals.

4.5.6.1.3.1. Strip View Toolbar

The toolbar at the top of the **Strip View** contains various controls that facilitate the scanning and reviewing of the ECG signal within the **Strip View** itself. For ease of understanding, these controls are organized into distinct groups of interface elements:

Figure 290. Templates View - Strip View Toolbar

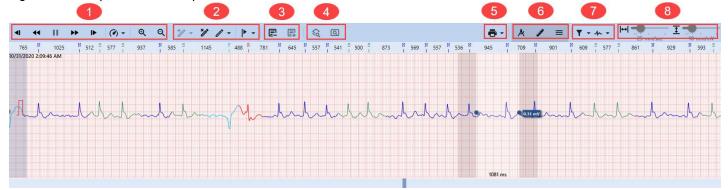

- 1. General Controls.
- 2. Beats and Events Controls.
- 3. Analysis Controls.
- 4. Views Switching Controls.
- 5. Printing Control.
- 6. Measuring Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate efficient ECG Strip scanning and review within the **Strip View**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 137 of 393	

lcon	Description
•	Step Backward button: Click to move the ECG Strip one step backward for scanning and reviewing.
*	Scan Backward button: Click to initiate continuous backward scanning of the ECG Strip.
I	Pause Scan button: Click to halt continuous scanning of the ECG Strip. For example, if you have activated a Scan Backward or a Scan Forward button, halt scanning by clicking the Pause Scan button.
*	Scan Forward button: Click to initiate continuous forward scanning of the ECG Strip.
•	Step Forward button: Click to move the ECG Strip one step forward for scanning and reviewing.
(d) +	Scan Speed button: Allows you to control the scanning speed. To set the scanning speed:
	1. Hover over the Scan Speed icon to expand the drop-down list.
	2. Click the desired scanning speed multiplier, ranging from ×1 to ×128.
Q	Zoom In button: Click to adjust the scale of the waveform in the Strip View , focusing on specific areas or comparing different fragments of the ECG record.
Q	Zoom Out button: Click to adjust the scale of the waveform in the Strip View . Use this button to zoom out if you had zoomed in earlier, or to view adjacent beats and fragments of the ECG Records.



Note: You can also click and hold the strip to drag it sideways, allowing you to view adjacent beats and ECG fragments.

Beats and Events Controls

Beats and Events Controls are a group of buttons that facilitate beats and events management in the Strip View.

lcon	Description					
* /* •	Insert Beat button: Allows you to insert new beats between existing ones. Further instructions are presented below.					
*/	Delete Beats button : Click to delete a selected beat.					
<i>o</i> * -	Change Beats Annotation button: Enables you to modify the annotation of selected beats.					
 	Create User Event button : Allows you to create a new user event, located at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates. Further instructions are presented below.					

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position in the **Strip View**.

To insert a beat:

- 1. Select a position in the **Strip View** using one of these methods:
 - Press Alt+Click in the **Strip View**: A vertical dotted line will appear, marking the desired position.
 - Press your mouse wheel in the **Strip View:** A vertical dotted line will appear, marking the desired position.
 - Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside
 the dotted line, a context menu will also expand, simplifying access to the **Insert Beat** option. **Note:** this key
 combination expands the context menu with only one option available **Insert Beat**.
- 2. Hover over the Insert Beat button in the Strip View Toolbar to expand the drop-down list.

Document Title:	NH	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

Figure 291. Strip View - Inserting Beats



3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

To delete beats:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 139 of 393

- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 292. Strip View - Deleting Beats



2. Click the **Delete Beats** button.

To change beat annotation:

- 1. Select a fragment of the ECG, which you wish to reclassify, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 140 of 393

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 293. Strip View - Changing Beats Annotation



- 2. Hover over the Change Beats Annotation button in the Strip View Toolbar to expand the drop-down list.
- 3. Click the appropriate type of morphology for the beat you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. The selected beat in the **Strip View** will immediately change color.

Table 3. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V
Classify as SVE	5	S
Classify as Questionable	X	x
Classify Paced	P	Р

To create a user event:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 141 of 393

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 142 of 393



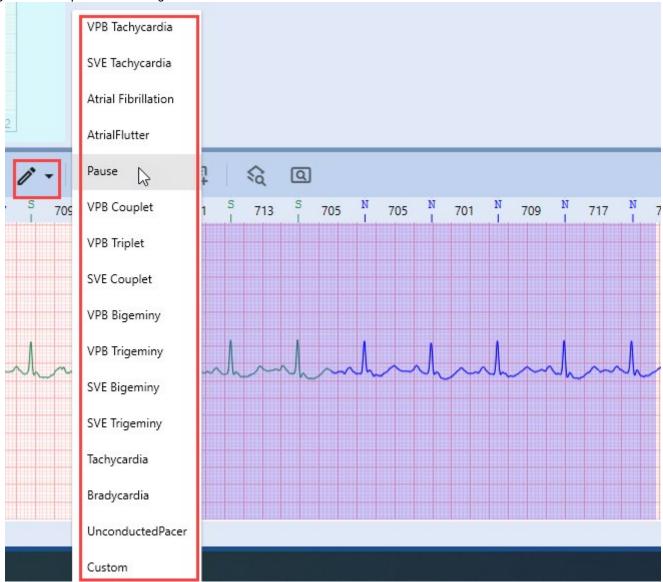

- 2. Hover over the **Create User Event** button or menu option to expand the drop-down list.
- 3. Click the relevant type of **Event** you wish to create in the selected area.

Analysis Controls

Analysis Controls is a group of buttons that enable you to exclude or re-include specific fragments of the ECG recording within the Strip View.

Icon	Description
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
戽	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

To exclude an ECG segment from analysis:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	N©RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 143 of 393

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.
 - 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 295. Strip View - Excluding from analysis



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 144 of 393

2. Click the **Exclude from analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

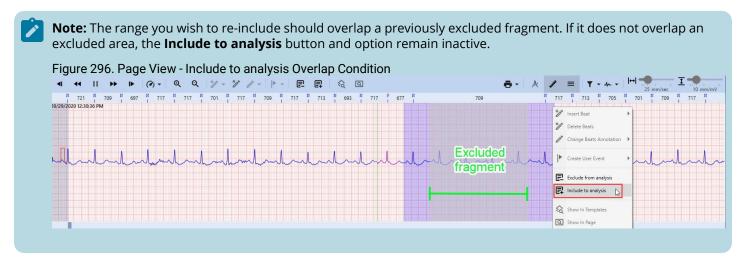

1. Select a fragment of the ECG, you want to re-include to analysis, using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 145 of 393

Figure 297. Strip View - Including to analysis



2. Click the **Include to analysis** button or select the option from the context menu.

Views Switching Controls

Views Switching Controls are a group of buttons that allow you to focus on a single beat you have selected in the **Strip View**, in the **Templates View** and **Page View**. This functionality enhances analysis quality and efficiency.

Icon	Description
€	Show In Templates button: Click this button to display the beat you have selected in the Strip View ; it will be highlighted in the relevant template within the Templates Pane .
Q	Show in Page : Facilitates an immediate switch to Page View , revealing the precise location of the chosen beat within this particular Template in both the Signal Page and the ECG Strip .

Figure 298. Strip View - Show In Templates




Printing Control

This button enables you to print a fragment of the ECG **Strip**. The printed copy will include a segment that fits the output format. The baseline of the printed version mirrors the center line of the Strip visible on your PC screen. The printed ECG Strip will accommodate as much ECG data as possible from the visible area around the center line.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 146 of 393


Figure 299. Strip View - Printing Strip

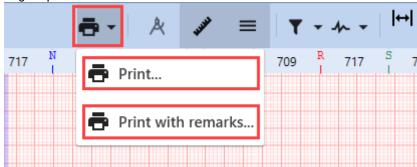

To print a fragment of the waveform in the **Strip View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Strip View (on page 138) to locate the desired area, if needed.
- 3. Hover over the **Printing Control** button to expand the drop-down list.
- 4. Click the **Print** option and navigate to the Print dialog box:

Figure 300. Strip View - Printing Dialog Box

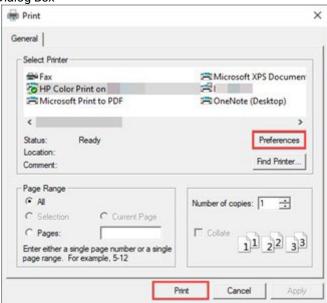


- 4.1. Choose a printer **available** in the **Select Printer** window.
- 4.2. Adjust other preferences according to your needs.
- 4.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.
- 5. (Optional) Click the Print with remarks option if you want to print a fragment of the waveform with your remarks:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

5.1. Complete the "Report remarks" field in the **Edit Report Remarks** pop-up.

Figure 301. Strip View - Printing with Remarks

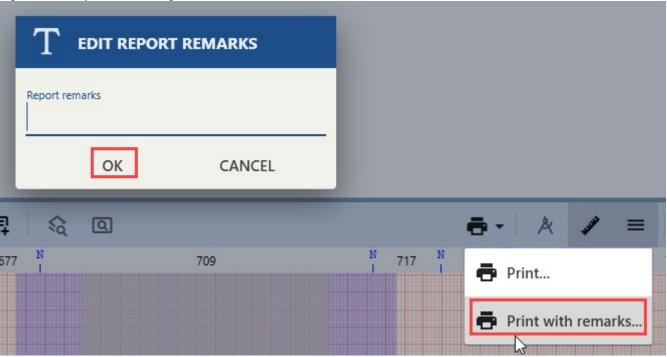

- 5.2. When you are complete with filling in you remarks, click OK.
- 5.3. (Optional) Click Cancel to abandon this action.
- 5.4. Choose a printer available in the **Select Printer** window.
- 5.5. Adjust other preferences according to your needs.
- 5.6. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Measuring Controls

Measuring Controlsare a group of buttons designed to facilitate the indication and measurement of various waveform parameters within the **Strip View**. Namely, the most important tool in this group of controls is the **Caliper** tool. It allows you to measure different intervals and amplitudes on the ECG strip, which can be used to diagnose and assess a variety of heart conditions. We will explain how to use the Caliper for ECG measurements (on page 160) in the following sections.

Icon	Description
Å	Caliper toggle : Click to toggle ON/OFF the Caliper tool. The Caliper tool assists with measuring intervals and amplitudes, such as the RR interval, T-wave, and QRS complex amplitudes. Refer to the Measuring ECG with Caliper (on page 160) section for details.
HARP	ECG Ruler toggle : Click to toggle ON/OFF the ECG Ruler , located right below the Strip View Toolbar . The ECG Ruler indicates the duration of RR intervals for adjacent beats and their morphology classification (i.e., N, R, F, V, etc.).
=	Channel Numeration toggle: Click to toggle ON/OFF the channel numeration on the left side of the ECG Strip.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 148 of 393





Figure 303. Strip View - ECG Ruler



Figure 304. Strip View - Channel Numeration



Filters

This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.


Icons	Description				
₹ -	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.				
h -	Channel Selection: This control lets you select the channel(s) for display within the Strip View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record).				

To toggle filters **ON/OFF**:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 149 of 393

1. Hover over the **Filter** drop-down list to expand it.

Figure 305. Strip View - Toggling Filters

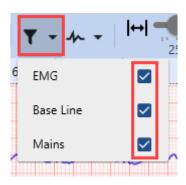

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.


It is recommended to keep filters **ON** to provide accurate analysis:

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Strip:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 306. Strip View - Toggling Channels

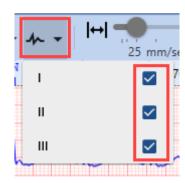

2. Select filters you wish to toggle. The visualization of the Strip will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 307. Strip View - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within the Strip will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 150 of 393


Figure 308. Strip View - Setting Paper Speed



To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within the Strip will change immediately.

Figure 309. Strip View - Setting Amplitude

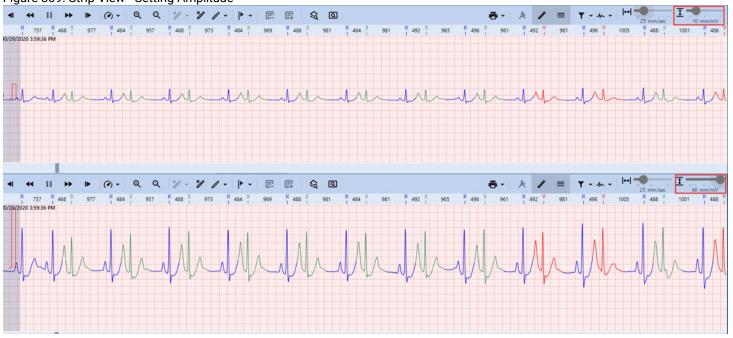


4.5.6.1.3.2. Context Menu Features

A **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions, such as inserting or deleting beats, changing annotations, creating user events, etc.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 151 of 393

Figure 310. Strip View - Context Menu

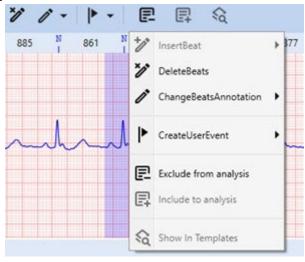

Insert Beat

Allows you to insert new beats between existing ones:



Note:

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position within the **Strip View**.

1. Select a position in the **Strip View** using one of these methods:


Figure 311. Strip View - Positioning Beat



- Hold Alt and click in the **Strip View**: A vertical dotted line will appear, marking the desired position. While holding Alt, click the right mouse button to expand the context menu.
- Press your mouse wheel in the **Strip View**: A vertical dotted line will appear, marking the desired position. Click again and hold the mouse wheel, and then press the right mouse button to expand the context menu.
- Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside the dotted line, a context menu will expand, simplifying access to the **Insert Beat** option.
- 2. Hover over the Insert Beat option in the Context Menu to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 152 of 393

Figure 312. Strip View - Insert Beat

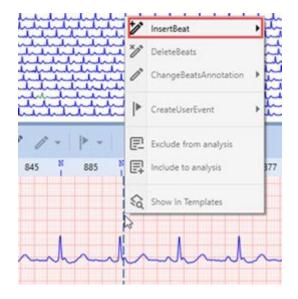

3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

Delete Beats

Allows you to delete a selected beat or a fragment of a waveform:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 153 of 393

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 313. Strip View - Delete Beats



2. Click the **Delete Beats** option in the context menu. The system will automatically recalculate events and templates following the deletion.

Change Beats Annotation

Enables you to modify the annotation of the selected beat:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Change Beats Annotation** option in the context menu to expand the drop-down list.

Figure 314. Strip View - Change Beats Annotation

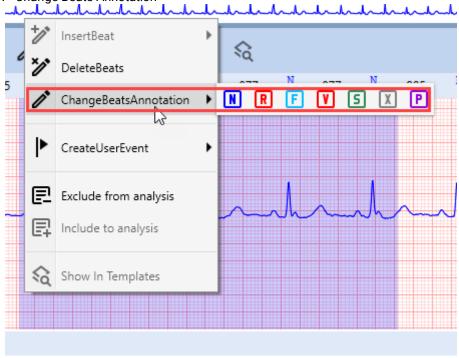

3. Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Reclassified beats in the **Strip View** will immediately change color.

Table 4. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 155 of 393

Table 4. Beats Reclassification Keyboard Shortcuts for Strip View (continued)

Action	But- ton	Keyboard Keys
Classify as SVE	5	S
Classify as Questionable	X	Х
Classify Paced	P	Р

Create User Event

Allows you to create a new user event at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates.

- 1. Select a fragment of the ECG you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Create User Event** option to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		



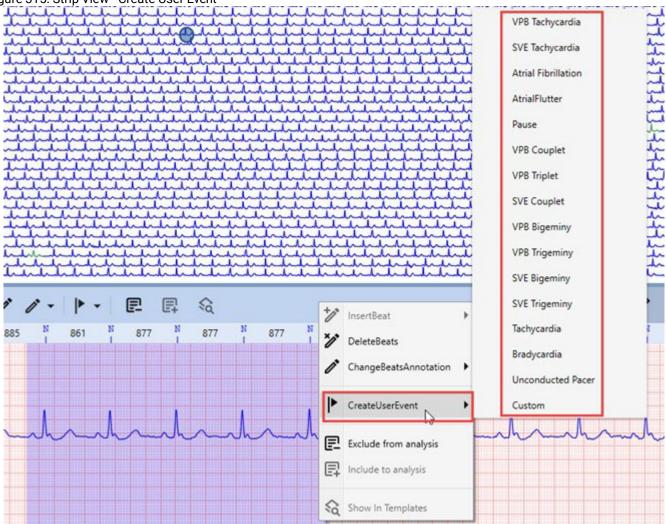

3. Click the relevant type of **Event** you wish to create in the selected area.

Exclude from analysis

Enables the exclusion of a selected fragment of an ECG record from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 157 of 393

- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 316. Strip View - Exclude from analysis

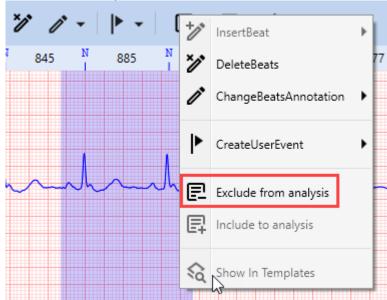

2. Click the **Exclude from Analysis** option in the context menu.

Include to analysis

Enables the re-inclusion of previously excluded fragments of an ECG record:

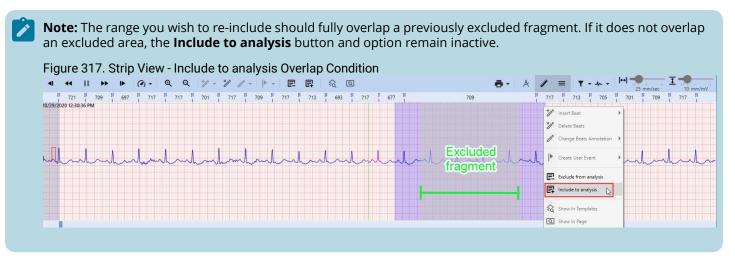

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

- 1. Select a fragment of the ECG you want to re-include in analysis, using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - 1.7. Right-click the selected fragment to expand the context menu.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the Ctrl key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 318. Strip View - Include to analysis



2. Click the **Include to analysis** option in the context menu.

Show In Templates

Select this option from the context menu to display the beat you have selected in the **Strip View**; it will be highlighted in the relevant template within the **Templates Pane**.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 159 of 393

Figure 319. Strip View - Show In Templates



4.5.6.1.3.3. Measuring ECG with Caliper

You may use the **Caliper** function to measure the RR interval, QRS complex duration and amplitude, QT interval, and other ECG waveform parameters. The **Caliper** also allows for editing the ECG record via the **Strip View**.

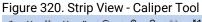



Caliper Design Overview

The **Caliper** tool in the **Strip View** displays two **measuring lines** and a **measuring interval** between them. To activate the **Caliper** tool, click the **Caliper** button in the **Strip View Toolbar**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 160 of 393

Figure 321. Strip View - Caliper Measurement Lines and Interval



Each ECG channel in the Strip has a small box adjacent to the point where the right measurement line intersects the channel signal line. The value in this box indicates the amplitude difference between the two points demarcated by the measurement lines.

Figure 322. Strip View - Interval Length and Amplitude Difference



The interval length in milliseconds is displayed at the bottom of the measuring interval.

Moving Caliper Lines


You may move Caliper measurement lines either **simultaneously** or **independently**.

To move both lines simultaneously:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 161 of 393	

1. Position the cursor within the measuring interval. The cursor will change to a resize cursor.

Figure 323. Strip View - Moving Caliper Lines Simultaneously

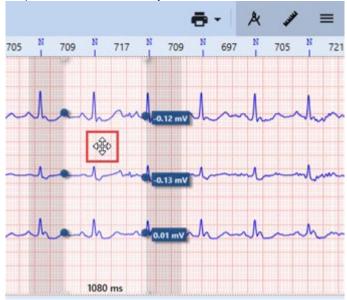


- 2. Drag and drop the **Caliper** to the desired location.
- 3. Optionally, use the Left and Right arrow keys to fine-tune the position.

To move lines independently:

1. Position the cursor near one of the measurement lines until it changes to a resize cursor.

Figure 324. Strip View - Moving Caliper Lines Independently

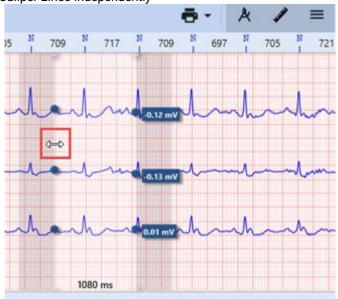

- 2. Click to select the measurement line.
- 3. Drag and drop it to the new location.

Zoom Feature

To adjust the display scale of the waveform in the Strip, utilize the **Zoom In/Zoom Out** options:

Figure 325. Strip View - Zooming Options



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 162 of 393

- Click the corresponding button in the Strip View Toolbar.
- Hold the Ctrl key while scrolling up (Zoom In) or down (Zoom Out) the mouse wheel.

Caliper Snapping Feature

The **Caliper Snapping** feature automatically aligns the **Caliper** measurement lines to the nearest R-peaks in the ECG signal. This functionality aids in the precise measurement of ECG wave duration and amplitude. Manual alignment can be challenging, hence **Caliper Snapping** enhances the accuracy, efficiency, and reproducibility of ECG interpretation.

To utilize the Caliper snapping feature:

- 1. Click the Caliper button in the Strip View Toolbar.
- 2. (Optional) Use the Left and Right arrow keys to make slight adjustments to the Caliper measurement lines.
- 3. To snap a **Caliper** measurement line to an R-spike:

Figure 326. Strip View - Caliper Snapping



- To snap any line:
 - 3.1. Hold down the **Alt** key.
 - 3.2. Position the cursor near one of the measurement lines until the cursor icon changes to a resize cursor.
 - 3.3. Click the measurement line to select it.
 - 3.4. Drag and drop it to the desired location.
- To snap the right measurement line:
 - 3.1. Hold down the **Right Alt** key.
 - 3.2. Use the Left and Right arrow keys to move the **Caliper**; the right measurement line will automatically snap to the R-peaks as you move.
- To snap the left measurement line:
 - 3.1. Hold down the Left Alt key.
 - 3.2. Use the Left and Right Arrow keys to move the Caliper; the left measurement line will automatically snap to the R-peaks as you move.


The **Caliper Snapping** function is instrumental in enabling a quick and accurate evaluation of various ECG parameters.

Editing Beats and Events Using Caliper

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 163 of 393

The **Caliper** tool enables you to edit specific fragments of the ECG waveform contained within the measurement lines of the **Caliper**. When you opt to delete or reclassify beats, or exclude a waveform fragment from analysis, the changes will be confined to the selected fragment, which may contain multiple beats. In essence, the **Caliper** allows you to edit a continuous section of an ECG record, demarcated by the measurement lines, in a single action.

Figure 327. Strip View - Editing Beats and Events Using Caliper

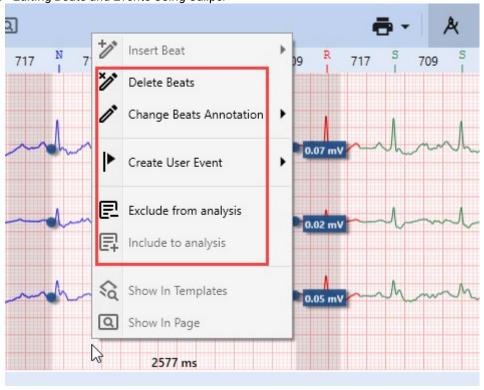

To edit a fragment of an ECG using the Caliper:

- 1. Position the **Caliper** and adjust its measurement lines as needed. For guidance, refer to the Caliper Design Overview (on page 162) section.
- 2. Right-click in the **Strip View** area to expand the context menu.
- 3. The **Caliper** tool's context menu offers several options within the **Strip View**:
 - **Delete Beats**: Click this option to remove all beats within the boundaries of the **Caliper's** measurement lines. This action will result in minor recalculations.



Note: Deleting beats may trigger the generation of **Pause** events. To annotate and exclude noisy segments, the **Exclude from analysis** option is preferable to **Delete Beats**.

- **Change Beats Annotation**: Hover over this option to display a drop-down list, then click the morphology type you wish to assign to the beats. The reclassified beats will instantly change color in the **Strip View**.
- **Create User Event**: Hover over this option to expand a drop-down list, and click the event type you wish to create in the selected area.
- **Exclude from analysis**: Click to exclude a noisy ECG segment within the **Caliper's** measurement lines from the analysis, without affecting other analytical outcomes.
- Include to analysis: Click to include previously excluded ECG fragments back into the analysis.



Note: The **Include in Analysis** option becomes active in the **Caliper** context menu only when the current **Caliper** measurement interval overlaps with a segment that was excluded earlier.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		


4.5.7. Events

The **Events View** facilitates the assessment of detected arrhythmias by offering quick navigation through example strips and trend overviews. To switch between different types of arrhythmias, click on various events in the **Events List**. Utilize the **Events Overview Panel** to assess, modify, and delete these events. For rapid identification of **HR/RR Trends**, refer to the designated section. Additionally, obtain more in-depth information via the **Strip View** and **Page View**, integrated within the Events View screen.

In this section, we will focus on explaining different areas of the **Events View** screen, including:

- 1. Events List.
- 2. HR/RR Trends View.
- 3. Events Overview Panel.
- 4. Strip View.
- 5. Page View.


Figure 328. Events View - Events View

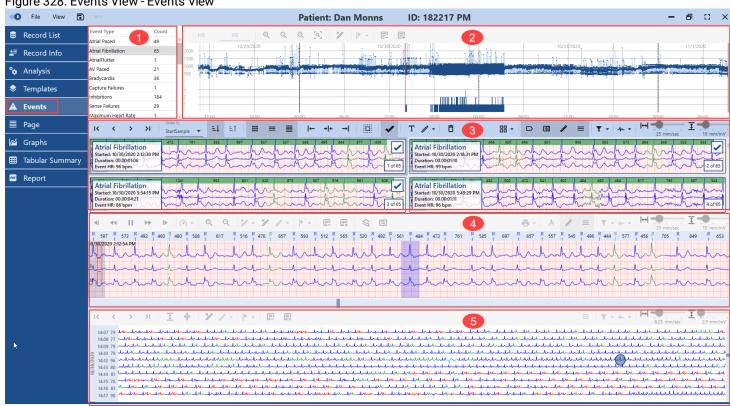


4.5.7.1. Events List

The **Events List** displays various arrhythmia events identified during the analysis of the **Recording**. By clicking on different arrhythmia types in the **Events List**, users can toggle between comprehensive overviews for each event type. These overviews populate the **HR/RR Trends View**, **Events Overview Panel**, **Strip View**, and **Page View** with data specific to the selected arrhythmia.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

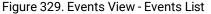

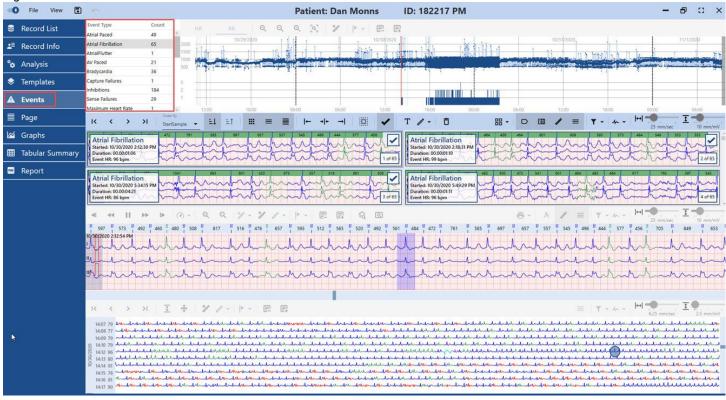

The **Events List** primarily serves to enable switching between arrhythmia types for further evaluation. This assessment utilizes the HR/RR Trends View (2), Events Overview Panel (3), Strip View (4), and Page View (5) tools and capabilities.

Figure 330. Events View - Assessment Tools



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 166 of 393

Figure 331. Events View - Events List Columns

Event Type	Count
Atrial Paced	49
Atrial Fibrillation	65
AtrialFlutter	1
AV Paced	21
Bradycardia	36
Capture Failures	1
Inhibitions	184
Sense Failures	29
Maximum Heart Rate	1

- 1. **Event Type:** The left column contains clickable names of different arrhythmia types.
- 2. **Count:** The right column shows the number of occurrences for each corresponding arrhythmia type in the **Recording**.

To switch between Event types:

1. Click on a relevant event type in the **Events List**. All related views and panels will adjust accordingly.

Upon clicking a specific arrhythmia type, the first occurrence of this event is automatically highlighted in the HR/RR Trends View, Events Overview Panel, Strip View, and Page View.


Figure 332. Events View - Event Highlighting



After selecting a particular arrhythmia type, to continue with the review of further occurrences:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		



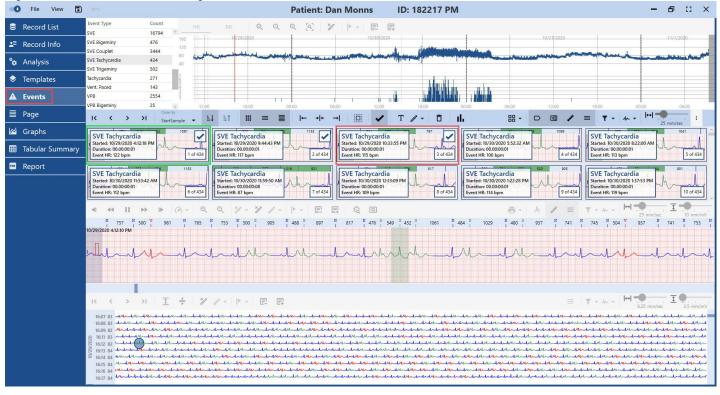


1. Click on any unselected **Example Strip** in the **Events Overview Panel** to switch to another event. All related views and panels will update accordingly.

For more details on the functionality of the Events Overview Panel (on page 168), please refer to its respective section.

4.5.7.2. Events Overview Panel

The **Events Overview Panel** serves as the primary workspace within the **Events View**, supplemented by **HR/RR Trends View**, **Strip View**, and **Page View** for a more detailed ECG analysis. It provides a wide array of capabilities such as reviewing, reclassifying, deleting, identifying, and marking specific events for the final report.

Figure 334. Events View - Events Overview Panel

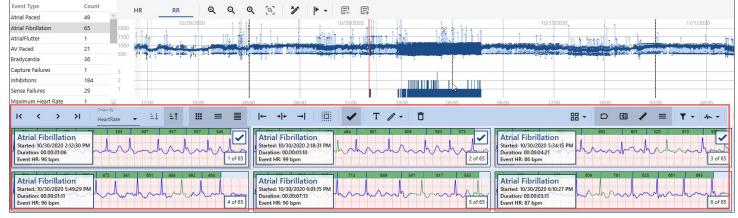

When clicking on any event type in the **Events List**, the **Events Overview Panel** populates with **Example Strips** corresponding to the selected arrhythmia type. Concurrently, **HR/RR Trends View**, **Strip View**, and **Page View** adjust their displays. By default, the system will automatically highlight the first sample of the chosen event type.


Users can interact with these **Example Strips** using the **Events Overview Panel**, context menus, and other available tools within the **Events View**.

The **Count** column in the **Events List** displays the total number of **Example Strips** for the selected event type, excluding Maximum and Minimum HR/RR events.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

Example: If the **Events Overview Panel** displays 36 **Example Strips**, that number will also appear in the **Events List** under the relevant arrhythmia type.

Figure 335. Events View - Example Strips in the Panel

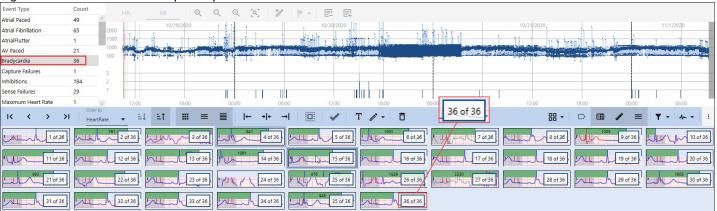



Note:

When selecting Maximum or Minimum HR/RR events from the **Events List**, you may observe that the **Count** value for these types is '1', yet the **Events Overview Panel** is filled with multiple **Example Strips**. As the Holter NH-301 analysis system captures numerous instances of these **Events**, often with identical or similar numerical values for Heart Rate and RR length, only one **Example Event** needs to be selected for the report as an indicative parameter.

To maintain analysis quality, the Holter NH-301 software allows users to evaluate these Maximum and Minimum HR/RR samples and select **one** most appropriate sample for the final **Report**.



Example: When dealing with **Maximum** and **Minimum HR/RR** strips, you should choose **only a single strip** to include in the **Report**.

In the accompanying image, the total number of **Example Strips** for Maximum HR is indicated as 316,254. Notably, there are at least two strips displaying a Heart Rate of 125 bpm, among others with Maximum HR values of 124 bpm.

Professionals are not required to scrutinize all the available **Example Strips**. The workflow specifies that only one **Event** example needs to be set as selected. Utilizing the quick check feature for **Maximum Heart Rate**, **Minimum Heart Rate**, **Maximum RR**, and **Minimum RR** events, the NH-301 software facilitates the identification of the most suitable **Event** sample for the report.

4.5.7.2.1. Events Overview Toolbar

The toolbar at the top of the **Events Overview Panel** in the **Strip View** contains various controls for facilitating the review and modification of different event types. For ease of understanding, these controls are organized into distinct groups of interface elements. The **Events Overview Toolbar** features three unique states, each designed to support specific analysis workflow requirements for different event types. This section will describe all three toolbar states, starting with

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 169 of 393

the "default" state.

Figure 337. Events View - Events Overview Toolbar



- 1. General Controls.
- 2. Sorting Controls.
- 3. Events View Layout Controls.
- 4. Example Strip Navigation Controls.
- 5. Events Editing Controls.
- 6. Example Strips Layout Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate navigation between **Example Strips** within the **Events Overview Panel**. When switching between **Example Strips**, the **HR/RR Trends View**, **Strip View**, and **Page View** adjust automatically.

Figure 338. Events View - Navigation within Events Overview Panel





Note: To navigate between **Example Strips** within the **Events Overview Panel**, one can also use the built-in scrollbar on the right side and select **Example Strips** by clicking on them.


lcon	Description
K	First Event button: Click to jump to the first Example Strip of the selected Event type within the Events Overview Panel.
<	Previous Event button: Click to navigate to the previous Example Strip of the selected Event type.
>	Next Event button: Click to navigate to the next Example Strip of the selected Event type.
۶I	Last Event button: Click to jump to the last Example Strip of the selected Event Type within the Events Panel Overview.



Note: By default, the software system will automatically select the first sample of the specific **Event** type according to either default or user-modified sorting rules.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

Figure 339. Events View - Default Selection

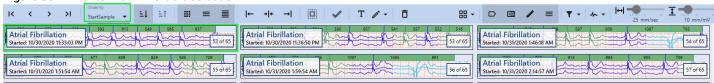

Sorting Controls

The first control in this group, the **Order by** drop-down list, enables toggling between **Example Strips** sorted by chronological order, **Event** duration, or **Heart Rate** value. Two icons on the right allow for ascending or descending sorting, respectively.


Figure 340. Events View - Sorting Controls



To adjust the sorting:

1. Click the **Order by** drop-down list.

Figure 341. Events View - Order by Sorting

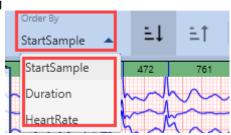


2. Select your preferred sorting type. The order of the **Example Strips** in the **Events Overview Panel** will update immediately.

To toggle a sorting order:

1. Click one of the buttons to switch sorting orders. The **Example Strips** will rearrange according to your choice.

Figure 342. Events View - Ascending and Descending Sorting

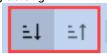

2. Click the second **Sorting Control** button to reverse the current sorting order.

Events View Layout Controls

This group of controls allows you to modify the **Events View** layout by toggling the **Events Overview Panel**, **Strip View**, and **Page View** located on the same **Events View** screen. The layout can be customized according to your ongoing needs: either expand or disable specific views as your review progresses.

Figure 343. Events View - Events View Layout Controls






Note: You can deactivate up to two of the three toggles at once. Simply choose any toggle you wish to remain activated, and disable the one or two others. All three toggles cannot be turned **OFF** simultaneously.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NØRAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

Icons	Description
:::	Events Overview Panel toggle: Click to turn ON/OFF the Events Overview Panel within the Events View . Both Strip View and Page View will resize automatically.

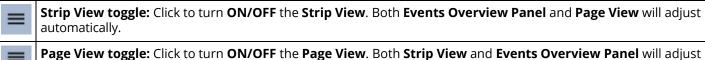

automatically.

Figure 344. Events View - Events Overview Panel ON IC C >> >I StartSamole → E↓ E↑ ||| || || || || || || ||- +|+ →| 0 / Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Started: 10/30/2020 2:12:30 PM Started: 10/30/2020 2:18:31 PN Duration: 00.00:01:10 Started: 10/30/2020 5:34:15 PM Started: 10/30/2020 5:49:29 PM Started: 10/30/2020 6:01:15 PM Duration: 00.00:01:11 Event HR: 96 bpm Duration: 00.00:01:06 Event HR: 96 bpm Duration: 00.00:04:21 Event HR: 86 bpm Duration: 00.00:07:13 Event HR: 90 bpm Event HR: 99 bpm Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Started: 10/30/2020 6:10:27 PM Duration: 00:00:03:11 Event HR: 87 bpm Started: 10/30/2020 6:15:38 PM Duration: 00.00:05:38 Event HR: 94 bpm Started: 10/30/2020 6:47:24 PM Duration: 00.00:02:12 Event HR: 98 bpm Started: 10/30/2020 6:24:55 PM Started: 10/30/2020 6:41:52 PN Duration: 00:00:01:23 Duration: 00.00:06:50 Event HR: 96 bpm Duration: 00.00:01:23 event HR: 96 bpm

Atrial Fibrillation

Event HR: 101 bpm

Started: 10/30/2020 7:11:19 PM Duration: 00.00:05:11

Atrial Fibrillation

Event HR: 97 born

Started: 10/30/2020 7:05:28 PM Duration: 00.00:05:22

Atrial Fibrillation

Event HR: 95 born

Started: 10/30/2020 6:50:33 PM Duration: 00.00:02:33 Atrial Fibrillation

Event HR: 97 bpm

Started: 10/30/2020 6:53:46 PM Duration: 00.00:11:03 Atrial Fibrillation

Event HR: 102 born

Started: 10/30/2020 7:18:09 PM Duration: 00.00:08:18

Figure 345. Events View - Page View OFF 1+1-I. K (> >I . EL ET III ■ ■ |- +|+ --| 0 T / ō 88 + D III / = Y-4-Atrial Fibrillation Atrial Fibrillation 1 Atrial Fibrillation 1/1 Atrial Fibrillation Atrial Fibrillation Started: 10/30/2020 2:12:30 PM Duration: 00.00:01:06 Event HR: 96 bpm Λ tarted: 10/30/2020 6:01:15 PM Started: 10/30/2020 5:49:29 Pt tarted: 10/30/2020 2:18:31 PM Duration: 00.00:01:10 Event HR: 99 bpm Duration: 00.00:04:21 Event HR: 86 bpm vent HR: 96 bpm Duration: 00.00:07:13 Event HR: 90 bpm Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Started: 10/30/2020 6:10:27 Pt itarted: 10/30/2020 6:15:38 PM Started: 10/30/2020 6:24:55 PM tarted: 10/30/2020 6:41:52 PM started: 10/30/2020 6:47:24 PM Duration: 00.00:03:11 Event HR: 87 bpm Duration: 00.00:05:38 Event HR: 94 bpm. Duration: 00.00:06:50 Event HR: 96 bpm ration: 00.00:01:23 ent HR: 96 bpm Duration: 00.00:02:12 Event HR: 98 bpm Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Atrial Fibrillation Started: 10/30/2020 6:50:33 PM arted: 10/30/2020 6:53:46 PM tarted: 10/30/2020 7:05:28 PM Started: 10/30/2020 7:18:09 PN Duration: 00.00:08:18 Event HR: 102 bpm Duration: 00.00:05:22 Event HR: 97 bpm Duration: 00.00:02:33 Event HR: 95 bpm 0 7-71-P 44 II >> 1000 10 V E 8 0 A. N 597 N 573 N 492 N 460 S 480 N 508 N 817 N 516 N 476 S 657 N 593 N 512 N 565 N 520 N 492 N 561 N 494 N 472 N 761 N 585 N 697 N 657 N 557 N 545 N 496 N 444 S 577 N 456 S 705 N 593 N 512 N 545 N 593 N 512 N 565 N 520 N 492 N 561 N 494 N 472 N 761 N 585 N 697 N 657 N 557 N 545 N 496 N 444 S 577 N 456 S 705 N 593 N 512 N 545 N 593 N 512 N 545 N 593 N 512 N 545 N 549 N 545 N 549 N 545 N 545


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

Figure 346. Events View - Events Overview Panel OFF



Example Strip Navigation Controls

Example Strip Navigation Controls comprise a set of buttons that enable centering, left-aligning, or right-aligning the **Example Strip(s)** within their designated window(s). These controls facilitate a comprehensive review of the **Example Strip**, which typically does not entirely fit into the **Example Strip** box.

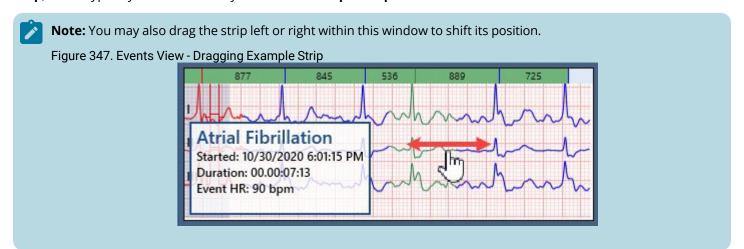

lcon	Description
 	Align Event Left button: Click to align the selected Example Strip(s) to the left.
→ ←	Center Event button: Click to center the selected Example Strip(s).
→I	Align Event Right button: Click to align the selected Example Strip(s) to the right.



Note: Bulk operations on **Example Strips** can be executed by either selecting them via the **Select All Button** or by selection through **Ctrl+Click** or **Shift+Click** actions.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 173 of 393



Events Editing Controls

Events Editing Controls consist of buttons designed to modify **Events**, mark them for inclusion in the **Report**, delete **Events**, or add notes. This group of controls houses main events modification features within the **Events Overview Toolbar**.

Figure 349. Events View - Events Editing Controls



Icon	Description
	Select All button: Click to select all Example Strips currently displayed within the Events Overview Panel. Click again to deselect all Example Strips. Alternatively, you can use the Ctrl + A keyboard shortcut.
	The Select All button enables bulk operations with Example Strips of the selected Event type. Features such as Set as Marked , Modify Events , Edit Label , Delete Selected Events , and Align Events (on page 173) can be applied to all strips of the selected Event type in one action.
~	Set as marked button: Click to mark Example Strip(s) you wish to include in the final Report (excluding Maximum/Minimum HR and Maximum/Minimum RR events).
	By default, if you don't mark any strips manually, for all Event types, excluding Maximum/Minimum HR and Maximum/Minimum RR , the first three Example Strips will be included in the Report .
¥	Set as selected button: Click to select one Maximum/Minimum HR and one Maximum/Minimum RR event Example Strip for inclusion in the final Report.
	For these types of events, the Set as Marked button is replaced by the Set as Selected button. This change indicates that you can only select a single event of these types for the Report .
	For Maximum/Minimum HR and Maximum/Minimum RR events, the software automatically selects only one Example Strip . Use this button to select a different Example Strip , if required.
Т	Edit Label button: Click to add or edit a note associated with the selected Example Strip(s) . Notes may be used for various purposes, such as documenting clinical context, identifying specific findings, and communicating these to other clinicians.
	Keep in mind that note length is constrained by the Example Strips visualization parameters in the final Report . Refer to the example below <i>(on page 175)</i> to find more details.
<i>o</i> -	Modify Event button: Click to reclassify (change the annotation) of the selected Example Strip(s) . Changes will be reflected in the Count column of the Events List for affected Event types .
	Keep in mind that the contents of the Modify Event drop-down list vary depending on the types of Events you select in the Events List .
Ō	Delete Selected Events: Click to delete selected Example Strips of the chosen Event type from the Events List . Changes will be reflected in the Count column of the Events List for the affected Event type.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	01 ver. 4.0.0.10 – Instructions for Use	
Document No.: NV-200.40010.028		Document Rev.: 04	Document Date: 07.08.2025	Page 174 of 393

- 1. Select **Example Strip(s)** of the chosen **Event** type you wish to include in the **Report**:
 - Click on an Example Strip to select it.
 - Utilize Ctrl + Click or Shift + Click to select multiple Example Strips. Alternatively, use the Select All button in the top toolbar or employ the Ctrl + A keyboard shortcut to select all strips.
- 2. **(Optional)** Scroll to review the strips you aim to mark. The **Events Overview Panel** can display multiple events of the same type, accommodating even thousands.
- 3. Click the **Set as Marked** button. A checked box icon will appear in the upper-right corner of each selected **Example Strip**.

To set an Example Strip as selected:

- 1. Navigate through the **Events Overview Panel** to locate the strip you intend to include in the report. The **Events Overview Panel** can display multiple events of the same type, accommodating even thousands.
- 2. Click on an **Example Strip** to select it.
- 3. Click the **Set as selected** button. A pin icon will appear in the upper-right corner of the chosen **Example Strip**.




Note: It is recommended to choose the maximum and minimum values as the final step of the review procedure. If subsequent changes are made, such as adding or deleting beats, these values might be automatically recalculated in the background. Consequently, you may need to reassess the maximum and minimum values.

To add or edit a note:

- 1. Select **Example Strip(s)** of the chosen **Event** type you wish to include in the **Report**:
 - Click on an **Example Strip** to select it.
 - Utilize Ctrl + Click or Shift + Click to select multiple Example Strips. Alternatively, use the Select All button in the top toolbar or employ the Ctrl + A keyboard shortcut to select all strips.
- 2. **(Optional)** Scroll to review the strips. The **Events Overview Panel** can display multiple events of the same type, accommodating even thousands.
- 3. Click the **Edit Label** button to open the **EDIT EVENT NOTE** dialog box. Alternatively, double-click on a single **Example Strip** to open a dialog box.


Figure 350. Events View - Edit Event Note Dialog Box

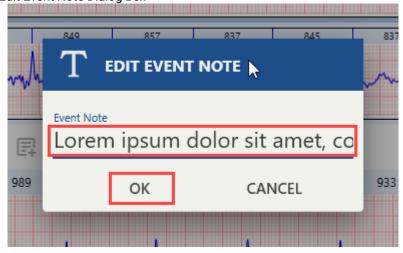

4. Enter or modify the note in the **Event Note** text field. Note that the length of the note is subject to a soft limit, governed by the **Example Strips'** visualization parameters in the final report. You may type in a substantial note, but be aware that it may not display correctly in the final report. Refer to the figure below for details on the note length soft cap.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 175 of 393

Figure 351. Events View - Example Strip Note Limitations

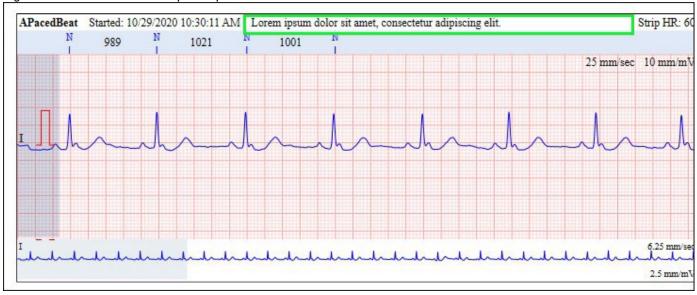

5. Click **OK** to save the note or **Cancel** to discard it.

To modify Event(s):

- 1. Select **Example Strip(s)** of the chosen **Event type** you want to modify:
 - Click on an **Example Strip** to select it.
 - Utilize Ctrl + Click or Shift + Click to select multiple **Example Strips**. Alternatively, use the **Select All** button in the top toolbar or employ the Ctrl + A keyboard shortcut to select all strips.
- 2. **(Optional)** Scroll to review and select all the strips you want to modify. The **Events Overview Panel** can display multiple events of the same type, accommodating even thousands.
- 3. Hover over the **Modify Event** button to expand a drop-down menu.
- 4. Choose a new **Event** type for the selected **Example Strips**. The values in the Count column of the **Events List** will change accordingly for the **Event** types affected by the action.



Note: Be aware that the contents of the **Modify Event** drop-down list will differ based on the types of **Events** chosen in the **Events List**. Each specific **Event** type has a distinct set of available modifications.

Example Strips Layout Controls

The **Example Strips Layout Controls** comprise a set of buttons that manage the layout of **Example Strips** within the **Events Overview Panel**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 176 of 393

Figure 352. Events View - Example Strips Layout Controls Scheme



lcon	Description
₩ *	Example Strips Layout button: Determines the number of Example Strips displayed per row and the number of rows within the Events Overview Panel , with options ranging from 1 to 40 strips.
	Example Strips Text Box toggle : Click to toggle ON/OFF a text box in the bottom left corner of the Example Strip . This box contains Event type, Event time stamp, and other basic data about each Event occurrence. When adding a note (on page 175) to an Example Strip via the Edit Label button, the note appears at the bottom of this text box.
13	Example Strips Numerating Box toggle: Click to toggle ON/OFF a text box located in the bottom right corner of the Example Strip . This box displays a sequential count of the Example Strip as it was chronologically identified by the software and the total number of Events of the same type.
ATTEN	ECG Ruler toggle : Click to toggle ON/OFF the ECG Ruler at the top of each Example Strip , which shows the duration of RR intervals for adjacent beats within the Event .
\equiv	Channel Numeration toggle: Click to toggle ON/OFF channel numeration on the left side of the Example Strip.

To change the Example Strips layout:

- 4.1. Hover over the **Example Strips Layout** button to expand the drop-down menu.
- **4.2.** Navigate through the dropdown to select the desired total number of **Example Strips** to display within the **Events Overview Panel**. Adjust the number of rows and columns by moving the cursor diagonally from the top left to the bottom right.

Figure 353. Events View - Changing Events Overview Panel Layout



4.3. Click on the preferred cell icon in the dropdown menu to apply the layout changes.



Note: If you wish to display numerous **Example Strips** within the **Events Overview Panel**, consider toggling OFF Strip View and Page View (on page 171) to preserve the readability of UI elements within the **Example Strips**.

Document Title:	nt Title: NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	V-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 177 of 393

Filters facilitate the toggling of filtering and **ECG Channel** visualization rules for **Example Strips**.

Figure 354. Events View - Filters



lcon	Description
T = 46 = INO SI Bee the D SI Main SI	Filter : Enables toggling of visualization ON/OFF with the applied EMG , Base Line , and Mains filters. To accomplish this, check or uncheck any number of checkboxes in the drop-down list.
	Channels : Allows toggling of ECG Record Channels display in Example Strips ON/OFF . To execute this, check or uncheck any number of checkboxes in the drop-down list, depending on the number of channels in a record and the quantity you wish to display.

Scale and Gain Controls

Drag **Scale and Gain Controls** sliders to set appropriate paper speed and amplitude for display in the Example Strips.

Figure 355. Events View - Scale and Gain Controls




Doc	cument Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Doc	cument No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 178 of 393

data more efficiently by allowing them to quickly adjust the display of the template boxes to their needs.

4.5.7.2.1.1. Events Overview Toolbar for SVE Type Events


For the purpose of prematurity refinement, when you select any **SVE** type events like **SVE**, **SVE Bigeminy**, **SVE Couplet**, **SVE Tachycardia**, **SVE Trigeminy**, an additional **Refine Prematurity** button in the **Events Overview Toolbar** appears. Prematurity refinement is a feature of the **NH-301 Holter** software that can be used to improve the accuracy of **SVE** events classification. Prematurity refinement works by reclassifying events that are not very premature (do not occur very early in the cardiac cycle) as normal. This can reduce the number of false-positive classifications.



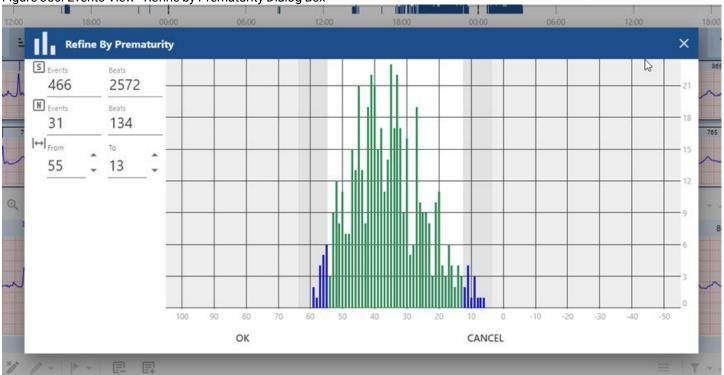

When you click the **Refine Prematurity** button, the **Refine By Prematurity** dialog box pops up. The dialog box contains an event type distribution graph with the adjustable refinement sliders on the left and right margins of the grid.


Figure 357. Events View - Refine Prematurity Button

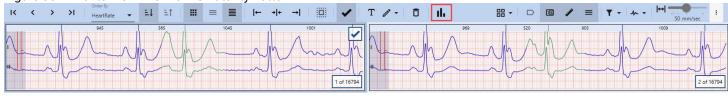

The prematurity is defined by the time difference between the premature beat and previous normal beats. A premature beat occurs sooner than the expected time of the next normal beat. The X axis represents the "percentage of prematurity" of the premature beats comprising the event, in comparison to normal beats.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	Oocument No.: NV-200.40010.028		Document Date: 07.08.2025	Page 179 of 393

Figure 358. Events View - Refine by Prematurity Dialog Box Elements

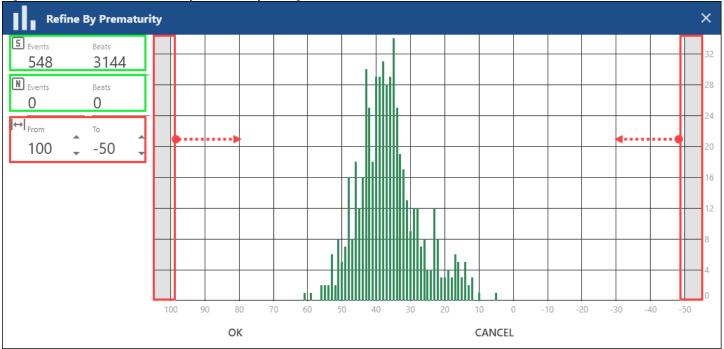

In the top left corner, you can see how many **Events** of the selected type were identified by the **NH-301 Holter** software and how many **Beats** comprise these events.

Below that, you can see how many **Events** will be reclassified as Normal and how many **Beats** will be affected after you set up and confirm bulk reclassification via this feature.

Finally, **From** and **To** parameters represent the position of the refinement sliders on the X-axis. Default values: From = 100 which equals to the left margin of the graph; To = -50 which equals the right margin of the graph. You can move the sliders to change the default values.

To refine selected SVE type events:

1. Click the **Refine Prematurity** button in the **Events Overview Toolbar**. The **Refine By Prematurity** dialog box will pop up.


Figure 359. Events View - Click Refine Prematurity Button



- 2. Move refinement sliders on the left and right in relevant directions to refine detected **Events** by prematurity. This will decrease the number of S-events and S-beats in the top left corner and increase the number of N-events and N-beats. Graph bars affected by refinement sliders will also change their color from green to blue. To move the sliders to the desired position:
 - Click spinner arrows near to the **From** and **To** elements.
 - Click and drag refinement sliders with your mouse cursor.

Document Title:	nent Title: NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 180 of 393

Figure 360. Events View - Refine By Prematurity

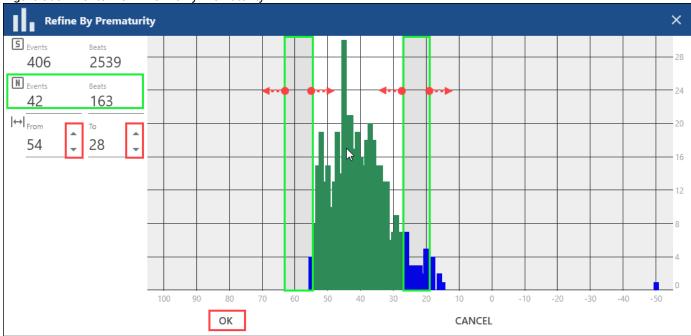

3. Click the **OK** button to confirm the refinement after you have positioned the sliders as required. The value in the **Count** column of the **Events List** for the selected **Events** type will change accordingly. To discard refinement, click **Cancel**.

Figure 361. Events View - Events List Count Change

Event Type	Count
Pause	27
ST	11
SVE	16786
SVE Bigeminy	475
SVE Couplet	3437
SVE Tachycardia	411
SVE Trigeminy	502
Tachycardia	271
Vent. Paced	143

4.5.7.2.1.2. Events Overview Toolbar for Min/Max HR/RR Events

For Maximum/Minimum Heart Rate and Maximum/Minimum RR events, you need to choose only one Example Strip for inclusion in the final Report. Users are not supposed to modify, delete, or add notes to these specific types of Events due to their special role in ECG Recording analysis. In line with this workflow logic, the Events Overview Toolbar for Maximum/Minimum Heart Rate and Maximum/Minimum RR events has inactive Sorting Controls, while the Events Editing Controls are simplified to one Set as Selected button, as opposed to the five buttons on the default Toolbar.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 181 of 393

Figure 362. Events View - Default Events Overview Toolbar



For **Maximum/Minimum HR** and **Maximum/Minimum RR** events, the software automatically selects only one **Example Strip**. Use the **Set as Selected** button to choose a different **Example Strip**, if required.


Figure 363. Events View - Min-Max HR-RR Events Overview Toolbar




To set an Example Strip as selected:

- 1. Navigate through the **Events Overview Panel** to locate the strip you intend to include in the report. The **Events Overview Panel** can display multiple events of the same type, accommodating even thousands.
- 2. Click on an **Example Strip** to select it.
- 3. Click the **Set as selected** button. A pin icon will appear in the upper-right corner of the chosen **Example Strip**.

Figure 364. Events View - Set as selected

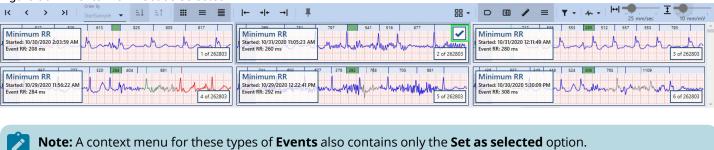

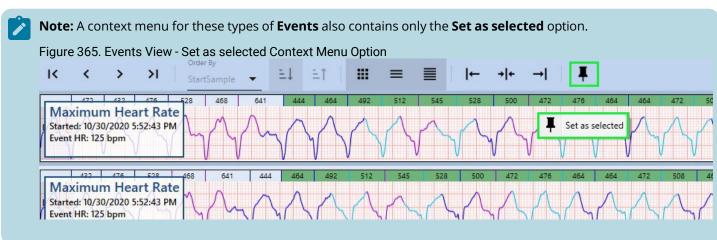


4.5.7.2.2. Context Menu Features

A **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions available on the **Events Overview Panel**.

For all event types, excluding **Maximum Heart Rate**, **Minimum Heart Rate**, **Maximum RR**, and **Minimum RR** events, the context menu allows for adding notes, setting **Example Strips** as marked, and modifying or deleting events.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 182 of 393

Figure 366. Events View - Default Context Menu

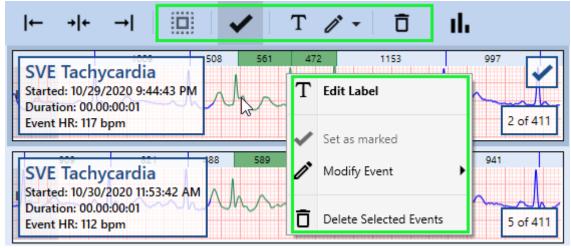

The context menu for **Maximum Heart Rate**, **Minimum Heart Rate**, **Maximum RR**, and **Minimum RR** events contains only the **Set as selected** option.

Figure 367. Events View - Min-Max HR-RR Context Menu

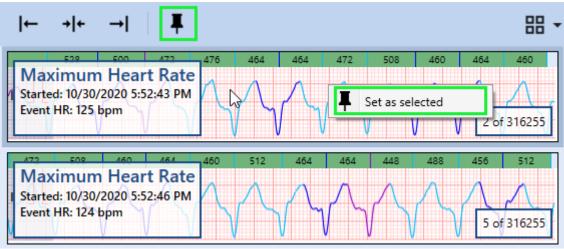


To unveil the context menu, right-click within the **Example Strip** box of your choice.

4.5.7.3. HR/RR Trends View

The **HR/RR Trends View** displays two graphs that show the heart rate and RR interval changes over time, respectively. This feature is instrumental for visualizing heart rate trends. The **RR Trend View** is more specific, as it distinguishes between different types of heart rhythms. The **HR Trend View** is more general, but it can be used to track changes in heart rate over a longer period of time. Underneath the **Trends View**, a histogram illustrates the frequency of selected arrhythmia type occurrences.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 183 of 393

Figure 368. Events View - HR/RR Trends

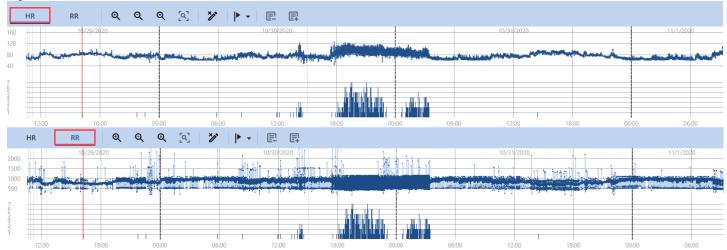


The **HR/RR Trends View** is a tool that aids in assessing and obtaining additional data about arrhythmia events by presenting heart rate and RR interval changes over time.

Figure 369. Events View - Trends View Toolbar



When hovering over any point on the graph within the **HR/RR Trends View**, a tooltip displays the heart rate or RR interval values.

Figure 370. Events View - HR-RR Text Box

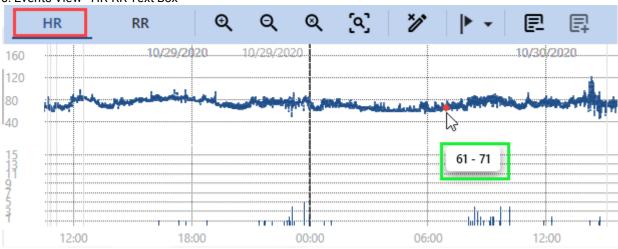

To toggle between HR and RR views in the Trends View:

1. Click either the **HR** or **RR** button in the upper-left corner of the **Trends View**. The graph visualization will change accordingly.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 184 of 393

Icon	Description
Q	Zoom In button: Click to adjust the scale of the HR or RR graph in the HR/RR Trends View , focusing on specific areas of the graph. You can click this button multiple times.


Q	Zoom Out button: Click to adjust the scale of the HR or RR graph in the HR/RR Trends View . Use this button to zoom out if you have zoomed in previously, or to view adjacent areas of the graph. You can click this button multiple times.
Q	Full Extent button: Click to zoom out instantly and see the entire HR/RR Trends graph.
[0]	Zoom to Extent button: Click to zoom in instantly to the selected area of the graph. The area will be displayed at the maximum available zoom.
/	Delete Beats button : Click to delete beats within the selected area.
 	Create User Event button : Allows you to create a new user event, located within the selected area. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may change their annotations.
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

To zoom into the selected area of the graph to its full extent:

- 1. Select the fragment you wish to exclude from analysis using one of the following methods:
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Trend View** graph and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.
 - 1.3. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Click the **Zoom To Extent** button. The selected fragment of graph will be instantly displayed with the maximum zoom.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 185 of 393

Figure 371. Events View - Zoom To Extent Button

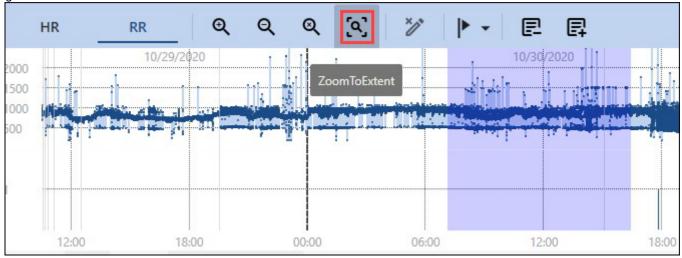

3. (Optionally) To revert the zoom, you may click the **Zoom Out** or the **Full Extent** buttons.

To delete beats:

- 1. Select a fragment you wish to delete, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the graph using one of these methods:

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Trend View** graph and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 186 of 393

2. Click the **Delete Beats** button or select the option from the context menu.

Figure 372. Events View - Trends View Delete Beats

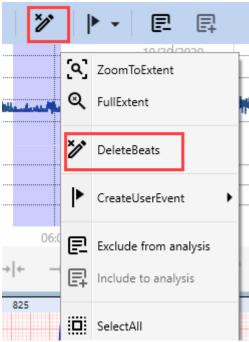


To create a user event:

- 1. Select a fragment of the graph, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the graph using one of these methods:
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the Ctrl key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Trend View** graph and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.
 - 1.3. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Create User Event** button or menu option to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 187 of 393

Figure 373. Events View - Trends View Create User Event

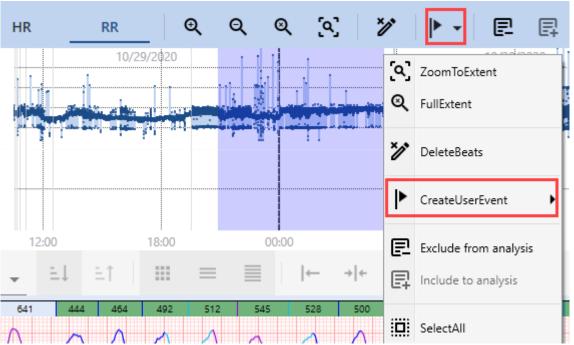

3. Click the relevant type of **Event** you wish to create in the selected area.

To exclude an ECG segment from analysis:


- 1. Select the fragment you wish to exclude from analysis using one of the following methods:
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Trend View** graph and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 188 of 393

2. Click the **Exclude from Analysis** button or select the same option from the context menu.

Figure 374. Events View - Trends View Exclude from analysis

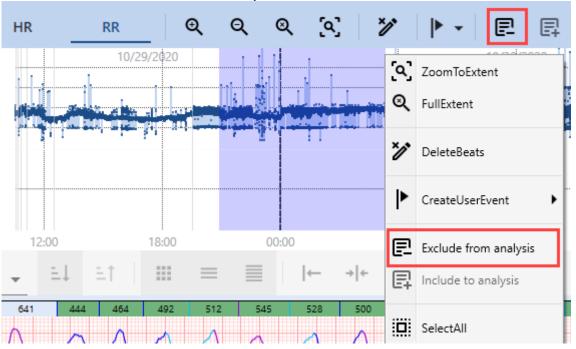

To re-include an ECG segment into analysis:

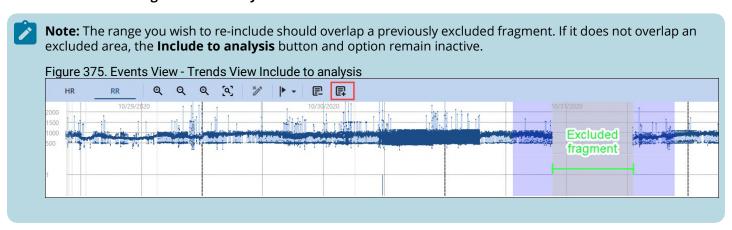

- 1. Select the fragment you wish to re-include using one of the following methods:
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Trend View** graph and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	0.40010.028 Document Rev.: 04 Document Date: 07.08.2025			

- 1.3. (Optional) If you are zoomed in, you can move the graph strip by clicking and dragging it.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Click the **Include to analysis** button or select the option from the context menu.


Context Menu Features

The **Context Menu**, accessible via right-click, enables quick and easy access to common actions found on the **HR/RR Trends View Toolbar**.

The Context Menu contains identical features to the HR/RR Trends View Toolbar.

The sole distinction between the **Context Menu** and the **HR/RR Trends View Toolbar** is the presence of the **Select All** feature. Utilize this option in the menu to select the entire graph.


Figure 376. Events View - Trends View Context Menu

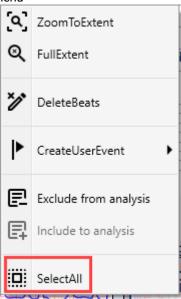

Histogram

Below the **Trend View**, you can see the **Histogram Panel**, showcasing the number of occurrences of the selected arrhythmia type. The **Histogram Panel** adds an additional visual dimension (on page 379), enhancing the ECG analysis process.


Figure 377. Events View - Histogram

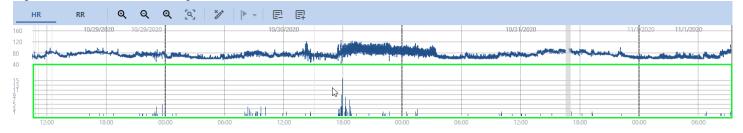


4.5.7.4. Strip View

The ECG **Strip View** is displayed in the bottom pane of the **Templates View** and in the **Edit Templates Mode**. It is also displayed in the **Events**, **Page**, and **Graphs Views**. The **Strip View** presents detailed data for the currently selected beat, as well as the adjacent ECG signal. Typically, the strip displays 12 to 16 seconds of ECG signal, depending on the monitor characteristics and ECG paper speed settings. You can utilize the **Strip View** for detailed analysis, measurements, beat reclassification, user event creation, and more.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 190 of 393

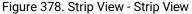

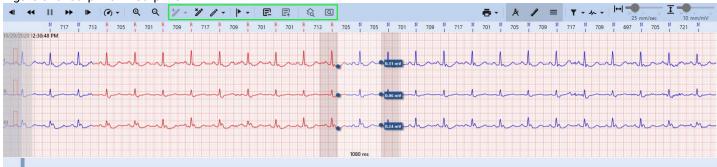




Note:

The ECG Strip grid is scaled accurately to millimeters, irrespective of the computer or monitor size in use. The **NH-301 Holter** analysis system automatically adjusts all application windows to align with the computer's graphical settings and monitor capabilities. This feature enables you to use any measuring tool, including specialized ECG rulers like the **Norav Medical ECG ruler**, to measure amplitude, cycles, frequency, and RR intervals.

4.5.7.4.1. Strip View Toolbar

The toolbar at the top of the **Strip View** contains various controls that facilitate the scanning and reviewing of the ECG signal within the **Strip View** itself. For ease of understanding, these controls are organized into distinct groups of interface elements:

Figure 379. Templates View - Strip View Toolbar

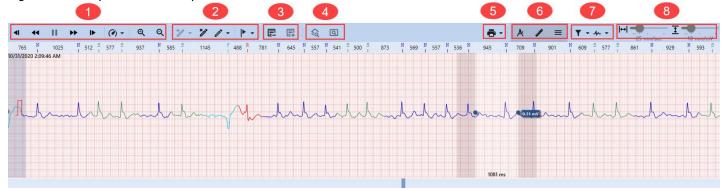

- 1. General Controls.
- 2. Beats and Events Controls.
- 3. Analysis Controls.
- 4. Views Switching Controls.
- 5. Printing Control.
- 6. Measuring Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate efficient ECG Strip scanning and review within the **Strip View**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 191 of 393	

lcon	Description
•	Step Backward button: Click to move the ECG Strip one step backward for scanning and reviewing.
*	Scan Backward button: Click to initiate continuous backward scanning of the ECG Strip.
I	Pause Scan button: Click to halt continuous scanning of the ECG Strip. For example, if you have activated a Scan Backward or a Scan Forward button, halt scanning by clicking the Pause Scan button.
*	Scan Forward button: Click to initiate continuous forward scanning of the ECG Strip.
•	Step Forward button: Click to move the ECG Strip one step forward for scanning and reviewing.
(d) +	Scan Speed button: Allows you to control the scanning speed. To set the scanning speed:
	1. Hover over the Scan Speed icon to expand the drop-down list.
	2. Click the desired scanning speed multiplier, ranging from ×1 to ×128.
Q	Zoom In button: Click to adjust the scale of the waveform in the Strip View , focusing on specific areas or comparing different fragments of the ECG record.
Q	Zoom Out button: Click to adjust the scale of the waveform in the Strip View . Use this button to zoom out if you had zoomed in earlier, or to view adjacent beats and fragments of the ECG Records.



Note: You can also click and hold the strip to drag it sideways, allowing you to view adjacent beats and ECG fragments.

Beats and Events Controls

Beats and Events Controls are a group of buttons that facilitate beats and events management in the Strip View.

Icon	Description
/ // -	Insert Beat button: Allows you to insert new beats between existing ones. Further instructions are presented below.
*/	Delete Beats button : Click to delete a selected beat.
<i>o</i> * -	Change Beats Annotation button : Enables you to modify the annotation of selected beats.
 	Create User Event button : Allows you to create a new user event, located at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates. Further instructions are presented below.

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position in the **Strip View**.

To insert a beat:

- 1. Select a position in the **Strip View** using one of these methods:
 - Press Alt+Click in the **Strip View**: A vertical dotted line will appear, marking the desired position.
 - Press your mouse wheel in the **Strip View:** A vertical dotted line will appear, marking the desired position.
 - Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside
 the dotted line, a context menu will also expand, simplifying access to the **Insert Beat** option. **Note:** this key
 combination expands the context menu with only one option available **Insert Beat**.
- 2. Hover over the Insert Beat button in the Strip View Toolbar to expand the drop-down list.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 192 of 393

Figure 380. Strip View - Inserting Beats



3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

To delete beats:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	0.40010.028 Document Rev.: 04 Document Date: 07.08.2025			

- 1.3. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 381. Strip View - Deleting Beats



2. Click the **Delete Beats** button.

To change beat annotation:

- 1. Select a fragment of the ECG, which you wish to reclassify, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 382. Strip View - Changing Beats Annotation



- 2. Hover over the Change Beats Annotation button in the Strip View Toolbar to expand the drop-down list.
- 3. Click the appropriate type of morphology for the beat you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. The selected beat in the **Strip View** will immediately change color.

Table 5. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	E	F
Classify as VPB	V	V
Classify as SVE	5	S
Classify as Questionable	X	X
Classify Paced	P	Р

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 195 of 393

To create a user event:

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	V-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025			



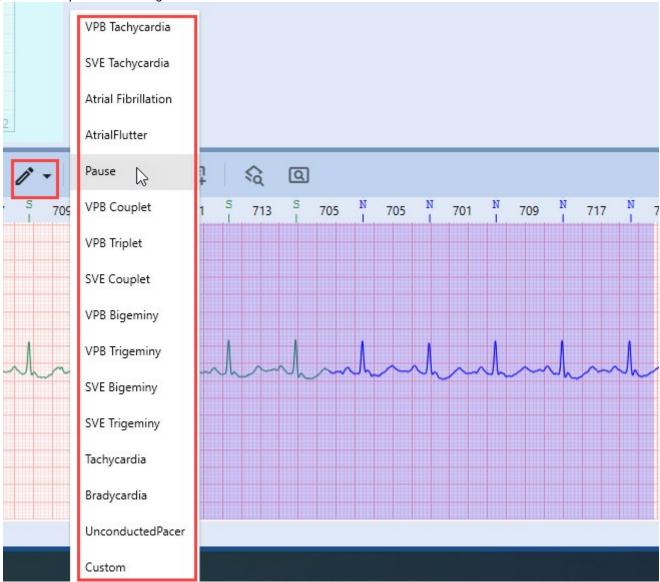

- 2. Hover over the **Create User Event** button or menu option to expand the drop-down list.
- 3. Click the relevant type of **Event** you wish to create in the selected area.

Analysis Controls

Analysis Controls is a group of buttons that enable you to exclude or re-include specific fragments of the ECG recording within the Strip View.

lcon	Description
圍	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
目	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

To exclude an ECG segment from analysis:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 197 of 393

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 384. Strip View - Excluding from analysis



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

2. Click the **Exclude from analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

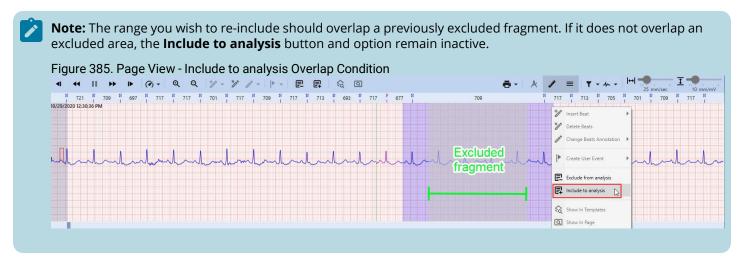

1. Select a fragment of the ECG, you want to re-include to analysis, using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Strip View (on page 192) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 199 of 393

Figure 386. Strip View - Including to analysis



2. Click the **Include to analysis** button or select the option from the context menu.

Views Switching Controls

Views Switching Controls are a group of buttons that allow you to focus on a single beat you have selected in the **Strip View**, in the **Templates View** and **Page View**. This functionality enhances analysis quality and efficiency.

Icon	Description
\$ a	Show In Templates button: Click this button to display the beat you have selected in the Strip View ; it will be highlighted in the relevant template within the Templates Pane .
Q	Show in Page : Facilitates an immediate switch to Page View , revealing the precise location of the chosen beat within this particular Template in both the Signal Page and the ECG Strip .

Figure 387. Strip View - Show In Templates




Printing Control

This button enables you to print a fragment of the ECG **Strip**. The printed copy will include a segment that fits the output format. The baseline of the printed version mirrors the center line of the Strip visible on your PC screen. The printed ECG Strip will accommodate as much ECG data as possible from the visible area around the center line.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 200 of 393


Figure 388. Strip View - Printing Strip

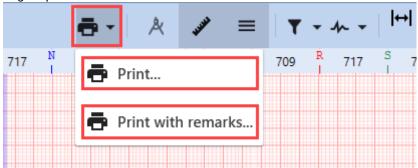

To print a fragment of the waveform in the **Strip View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Strip View (on page 192) to locate the desired area, if needed.
- 3. Hover over the **Printing Control** button to expand the drop-down list.
- 4. Click the **Print** option and navigate to the Print dialog box:

Figure 389. Strip View - Printing Dialog Box

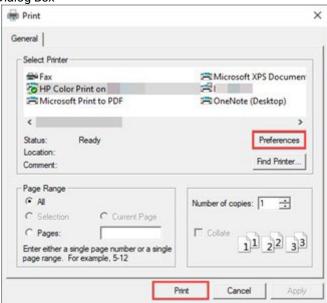


- 4.1. Choose a printer **available** in the **Select Printer** window.
- 4.2. Adjust other preferences according to your needs.
- 4.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.
- 5. (Optional) Click the Print with remarks option if you want to print a fragment of the waveform with your remarks:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 201 of 393

5.1. Complete the "Report remarks" field in the **Edit Report Remarks** pop-up.

Figure 390. Strip View - Printing with Remarks

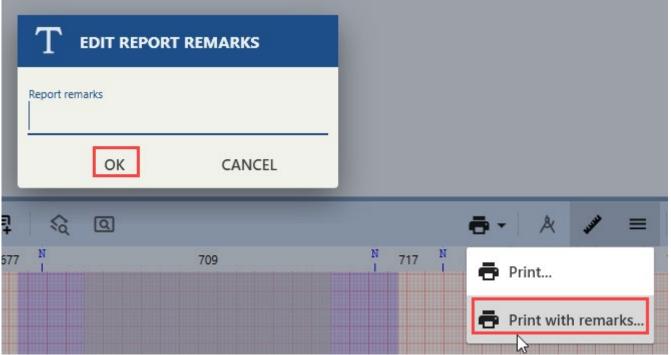

- 5.2. When you are complete with filling in you remarks, click OK.
- 5.3. (Optional) Click Cancel to abandon this action.
- 5.4. Choose a printer available in the **Select Printer** window.
- 5.5. Adjust other preferences according to your needs.
- 5.6. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Measuring Controls

Measuring Controlsare a group of buttons designed to facilitate the indication and measurement of various waveform parameters within the **Strip View**. Namely, the most important tool in this group of controls is the **Caliper** tool. It allows you to measure different intervals and amplitudes on the ECG strip, which can be used to diagnose and assess a variety of heart conditions. We will explain how to use the Caliper for ECG measurements (on page 160) in the following sections.

lcon	Description
A	Caliper toggle : Click to toggle ON/OFF the Caliper tool. The Caliper tool assists with measuring intervals and amplitudes, such as the RR interval, T-wave, and QRS complex amplitudes. Refer to the Measuring ECG with Caliper (on page 160) section for details.
*****	ECG Ruler toggle : Click to toggle ON/OFF the ECG Ruler , located right below the Strip View Toolbar . The ECG Ruler indicates the duration of RR intervals for adjacent beats and their morphology classification (i.e., N, R, F, V, etc.).
	Channel Numeration toggle: Click to toggle ON/OFF the channel numeration on the left side of the ECG Strip.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 202 of 393





Figure 392. Strip View - ECG Ruler



Figure 393. Strip View - Channel Numeration



Filters

This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.


Icon	Description				
₹ .	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.				
1	Channel Selection: This control lets you select the channel(s) for display within the Strip View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record).				

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 203 of 393

To toggle filters **ON/OFF**:

1. Hover over the **Filter** drop-down list to expand it.

Figure 394. Strip View - Toggling Filters

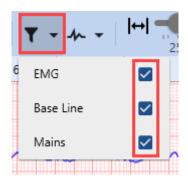

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.


It is recommended to keep filters **ON** to provide accurate analysis:

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Strip:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 395. Strip View - Toggling Channels

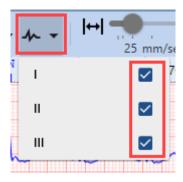

2. Select filters you wish to toggle. The visualization of the Strip will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 396. Strip View - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within the Strip will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 204 of 393


Figure 397. Strip View - Setting Paper Speed



To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within the Strip will change immediately.

Figure 398. Strip View - Setting Amplitude

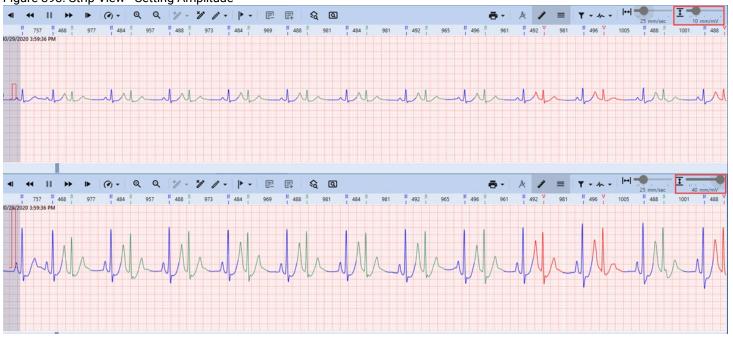


4.5.7.4.2. Context Menu Features

A **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions, such as inserting or deleting beats, changing annotations, creating user events, etc.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 205 of 393

Figure 399. Strip View - Context Menu

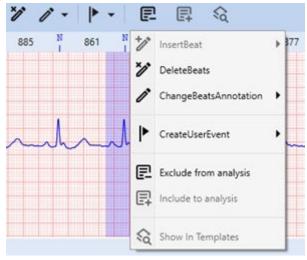

Insert Beat

Allows you to insert new beats between existing ones:



Note:

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position within the **Strip View**.

1. Select a position in the **Strip View** using one of these methods:


Figure 400. Strip View - Positioning Beat



- Hold Alt and click in the **Strip View**: A vertical dotted line will appear, marking the desired position. While holding Alt, click the right mouse button to expand the context menu.
- Press your mouse wheel in the **Strip View**: A vertical dotted line will appear, marking the desired position. Click again and hold the mouse wheel, and then press the right mouse button to expand the context menu.
- Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside the dotted line, a context menu will expand, simplifying access to the **Insert Beat** option.
- 2. Hover over the Insert Beat option in the Context Menu to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 206 of 393

Figure 401. Strip View - Insert Beat

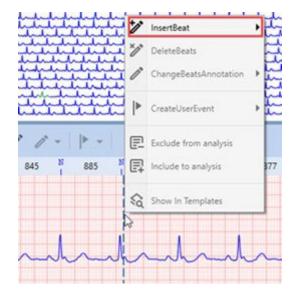

3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

Delete Beats

Allows you to delete a selected beat or a fragment of a waveform:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 207 of 393

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 402. Strip View - Delete Beats

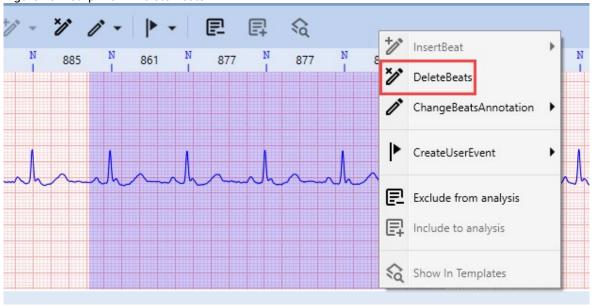

2. Click the **Delete Beats** option in the context menu. The system will automatically recalculate events and templates following the deletion.

Change Beats Annotation

Enables you to modify the annotation of the selected beat:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 208 of 393

- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Change Beats Annotation** option in the context menu to expand the drop-down list.

Figure 403. Strip View - Change Beats Annotation

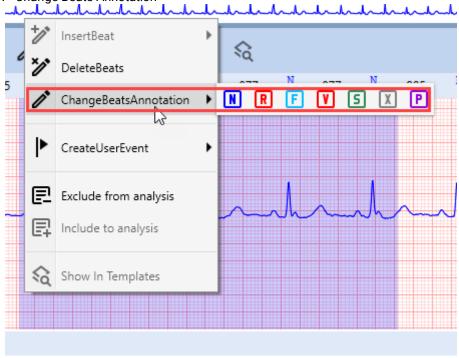

3. Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Reclassified beats in the **Strip View** will immediately change color.

Table 6. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 209 of 393

Table 6. Beats Reclassification Keyboard Shortcuts for Strip View (continued)

Action	But- ton	Keyboard Keys
Classify as SVE	5	S
Classify as Questionable	X	х
Classify Paced	P	Р

Create User Event

Allows you to create a new user event at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates.

- 1. Select a fragment of the ECG you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Create User Event** option to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 210 of 393



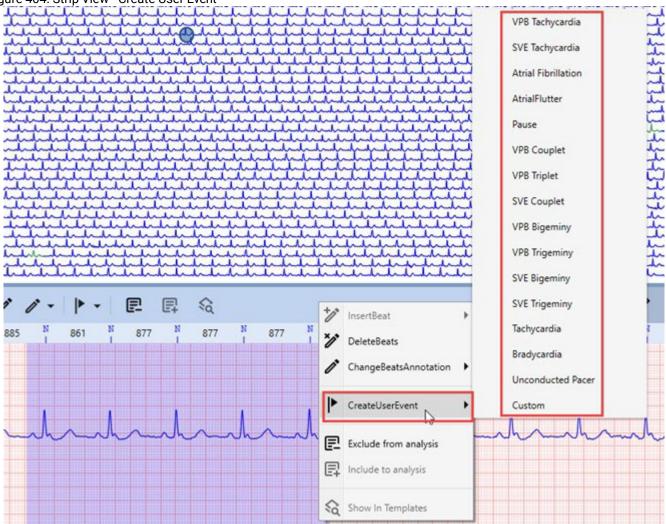

3. Click the relevant type of **Event** you wish to create in the selected area.

Exclude from analysis

Enables the exclusion of a selected fragment of an ECG record from analysis:

1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 211 of 393

- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 405. Strip View - Exclude from analysis

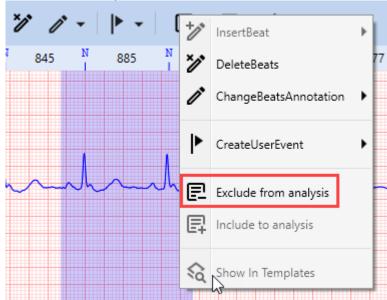

2. Click the **Exclude from Analysis** option in the context menu.

Include to analysis

Enables the re-inclusion of previously excluded fragments of an ECG record:

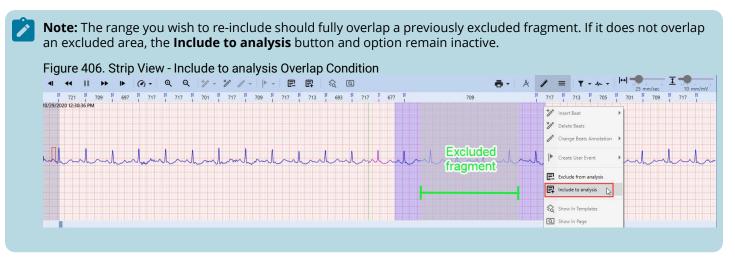

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 212 of 393

- 1. Select a fragment of the ECG you want to re-include in analysis, using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - 1.7. Right-click the selected fragment to expand the context menu.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the Ctrl key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 407. Strip View - Include to analysis



2. Click the **Include to analysis** option in the context menu.

Show In Templates

Select this option from the context menu to display the beat you have selected in the **Strip View**; it will be highlighted in the relevant template within the **Templates Pane**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 213 of 393

Figure 408. Strip View - Show In Templates



4.5.7.4.3. Measuring ECG with Caliper

You may use the **Caliper** function to measure the RR interval, QRS complex duration and amplitude, QT interval, and other ECG waveform parameters. The **Caliper** also allows for editing the ECG record via the **Strip View**.



Caliper Design Overview

The **Caliper** tool in the **Strip View** displays two **measuring lines** and a **measuring interval** between them. To activate the **Caliper** tool, click the **Caliper** button in the **Strip View Toolbar**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 214 of 393

Figure 410. Strip View - Caliper Measurement Lines and Interval



Each ECG channel in the Strip has a small box adjacent to the point where the right measurement line intersects the channel signal line. The value in this box indicates the amplitude difference between the two points demarcated by the measurement lines.

Figure 411. Strip View - Interval Length and Amplitude Difference



The interval length in milliseconds is displayed at the bottom of the measuring interval.

Moving Caliper Lines

You may move Caliper measurement lines either **simultaneously** or **independently**.

To move both lines simultaneously:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 215 of 393

1. Position the cursor within the measuring interval. The cursor will change to a resize cursor.

Figure 412. Strip View - Moving Caliper Lines Simultaneously



- 2. Drag and drop the **Caliper** to the desired location.
- 3. Optionally, use the Left and Right arrow keys to fine-tune the position.

To move lines independently:

1. Position the cursor near one of the measurement lines until it changes to a resize cursor.

Figure 413. Strip View - Moving Caliper Lines Independently



- 2. Click to select the measurement line.
- 3. Drag and drop it to the new location.

Zoom Feature

To adjust the display scale of the waveform in the Strip, utilize the **Zoom In/Zoom Out** options:

Figure 414. Strip View - Zooming Options



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 216 of 393

- Click the corresponding button in the **Strip View Toolbar**.
- Hold the Ctrl key while scrolling up (Zoom In) or down (Zoom Out) the mouse wheel.

Caliper Snapping Feature

The **Caliper Snapping** feature automatically aligns the **Caliper** measurement lines to the nearest R-peaks in the ECG signal. This functionality aids in the precise measurement of ECG wave duration and amplitude. Manual alignment can be challenging, hence **Caliper Snapping** enhances the accuracy, efficiency, and reproducibility of ECG interpretation.

To utilize the Caliper snapping feature:

- 1. Click the Caliper button in the Strip View Toolbar.
- 2. (Optional) Use the Left and Right arrow keys to make slight adjustments to the Caliper measurement lines.
- 3. To snap a **Caliper** measurement line to an R-spike:

Figure 415. Strip View - Caliper Snapping



- To snap any line:
 - 3.1. Hold down the **Alt** key.
 - 3.2. Position the cursor near one of the measurement lines until the cursor icon changes to a resize cursor.
 - 3.3. Click the measurement line to select it.
 - 3.4. Drag and drop it to the desired location.
- To snap the right measurement line:
 - 3.1. Hold down the **Right Alt** key.
 - 3.2. Use the Left and Right arrow keys to move the **Caliper**; the right measurement line will automatically snap to the R-peaks as you move.
- To snap the left measurement line:
 - 3.1. Hold down the Left Alt key.
 - 3.2. Use the Left and Right Arrow keys to move the Caliper; the left measurement line will automatically snap to the R-peaks as you move.


The **Caliper Snapping** function is instrumental in enabling a quick and accurate evaluation of various ECG parameters.

Editing Beats and Events Using Caliper

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 217 of 393

The **Caliper** tool enables you to edit specific fragments of the ECG waveform contained within the measurement lines of the **Caliper**. When you opt to delete or reclassify beats, or exclude a waveform fragment from analysis, the changes will be confined to the selected fragment, which may contain multiple beats. In essence, the **Caliper** allows you to edit a continuous section of an ECG record, demarcated by the measurement lines, in a single action.

Figure 416. Strip View - Editing Beats and Events Using Caliper

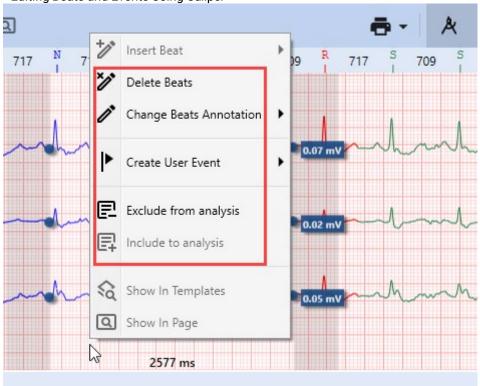

To edit a fragment of an ECG using the Caliper:

- 1. Position the **Caliper** and adjust its measurement lines as needed. For guidance, refer to the Caliper Design Overview (on page 216) section.
- 2. Right-click in the **Strip View** area to expand the context menu.
- 3. The **Caliper** tool's context menu offers several options within the **Strip View**:
 - **Delete Beats**: Click this option to remove all beats within the boundaries of the **Caliper's** measurement lines. This action will result in minor recalculations.



Note: Deleting beats may trigger the generation of **Pause** events. To annotate and exclude noisy segments, the **Exclude from analysis** option is preferable to **Delete Beats**.

- **Change Beats Annotation**: Hover over this option to display a drop-down list, then click the morphology type you wish to assign to the beats. The reclassified beats will instantly change color in the **Strip View**.
- **Create User Event**: Hover over this option to expand a drop-down list, and click the event type you wish to create in the selected area.
- **Exclude from analysis**: Click to exclude a noisy ECG segment within the **Caliper's** measurement lines from the analysis, without affecting other analytical outcomes.
- Include to analysis: Click to include previously excluded ECG fragments back into the analysis.




Note: The **Include in Analysis** option becomes active in the **Caliper** context menu only when the current **Caliper** measurement interval overlaps with a segment that was excluded earlier.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 218 of 393

4.5.7.5. Page View

The **Page View Strip** allows for previewing and scanning the ECG Recording as **Full Disclosure** within the **Events View**. It enables the highlighting of elements in **Full Disclosure**, creation of user events, reclassification or deletion of beats, and exclusion of noisy segments. In conjunction with other components of the **Events View** — including the **Events List** (on page 165), **Events Overview Panel** (on page 168), **HR/RR Trends View** (on page 183), and **Strip View** (on page 136), it supports a thorough review and analysis of events detected in the ECG Record.

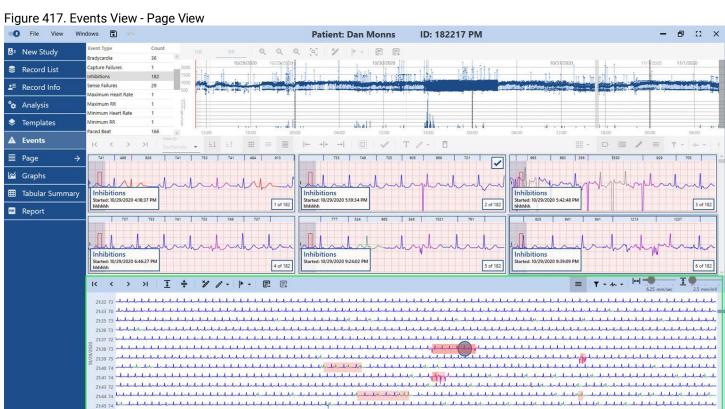

4.5.7.5.1. Page View Toolbar

The **Page View Toolbar** at the top of the **Page View** in the **Events View** contains various controls for facilitating the scanning and review of the ECG Recording, including beat deletion and reclassification, adding user events, excluding noise segments, and more. For ease of understanding, these controls are organized into distinct groups of interface elements.



- 1. General Controls.
- 2. Page View Layout Controls.
- 3. Editing Controls.
- 4. Events Highlighting Control.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 219 of 393

- 5. Filters.
- 6. Scale and Gain Controls.

General Controls

General Controls comprise a group of buttons designed to facilitate navigation within the **Page View** on the **Events View** screen

Figure 419. Events View - Page View General Controls

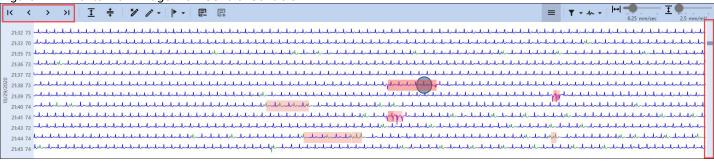




Note: For navigation within the **Page View**, the scroll function of your mouse or the built-in scrollbar to the right

Icon	Description
K	First Page: Click to display the first page of the Record .
<	Page Up: Click to navigate to the previous page of the Record.
>	Page Down: Click to navigate to the next page of the Record.
>1	Final Page: Click to display the last page of the Record , which may contain noisy segments.

Page View Layout Controls

Page View Layout Controls comprise a group of buttons designed to alter the number of waveform lines displayed in the **Page View** on the **Events View** screen.

Figure 420. Events View - Page View Layout Controls

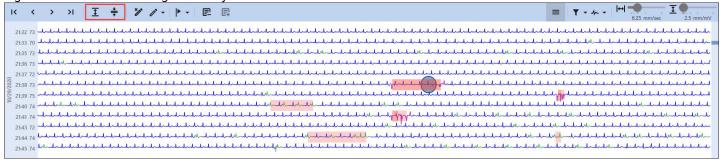


lcon	Description
<u>+</u>	Decrease Lines Per Page button: Click to reduce the number of lines per Page, enhancing focus on specific ECG lines. This feature allows for adjusting the density of lines within the Page View . Repeated clicks reduce lines to a minimum of one per Page.
→ ←	Increase Lines Per Page button: Click to add more lines per Page, enabling a broader segment of the ECG Record to be reviewed. Adjust the density of lines within the Page View as needed. However, an overpopulated Page View may hinder waveform analysis due to a visual overload.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 220 of 393

Editing Controls

Editing Controls is a group of buttons allowing you to reclassify and delete beats, create user events, and exclude noisy segments from the Record.

Figure 421. Page View - Page View Editing Controls

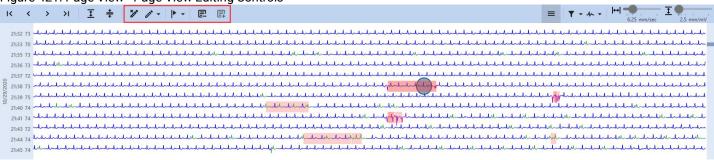

lcon	Description
*/	Delete Beats button : Click to delete beats within the selected area.
<i>o</i> * -	Change Beats Annotation button: Enables you to modify the annotation of selected beats.
•	Create User Event button : Allows you to create a new user event, located within the selected area. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may change their annotations.
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
目	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

By default, when you click on any **Example Strip** within the **Events Overview Panel**, **HR/RR Trends** graph, or select a beat in the **Strip View**, the corresponding area of the ECG Record is highlighted with a circle in the **Page View**. Similarly, clicking on the **Page View** causes the **Strip View** and **HR/RR Trends** graphs to adjust correspondingly. The **Events Overview Panel** remains unaffected by this action.

To delete a beat:

- 1. Select a fragment of the ECG you wish to delete, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 221 of 393

- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 422. Events View - Page View Delete Beats

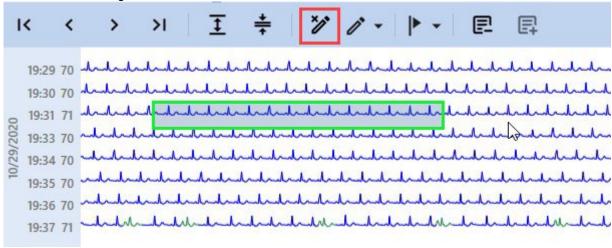

2. Click the **Delete Beats** button or the same option in the context menu.

To change beats annotation:

- 1. Select a fragment of the ECG using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Page View *(on page 220)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 222 of 393

- Using the right mouse button:
 - 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.
 - 1.3. **(Optional)** Scan the Page View *(on page 220)* to locate the points of interest, if necessary.
 - 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Change Beats Annotation** button in the **Page View Toolbar** or the same option in the context menu to expand the drop-down list.

Figure 423. Events View - Page View Change Beats Annotation

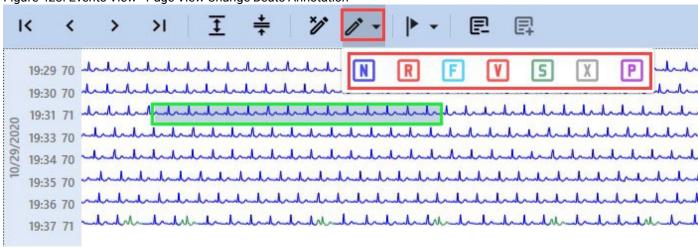

3. Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Selected beats will immediately change color.

Table 7. Beats Reclassification Keyboard Shortcuts for Page View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V
Classify as SVE	S	S
Classify as Questionable	X	x
Classify Paced	P	Р

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 223 of 393

To create a user event:

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Page View *(on page 220)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the Create User Event button or menu option to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 224 of 393

Figure 424. Events View - Page View Create User Event

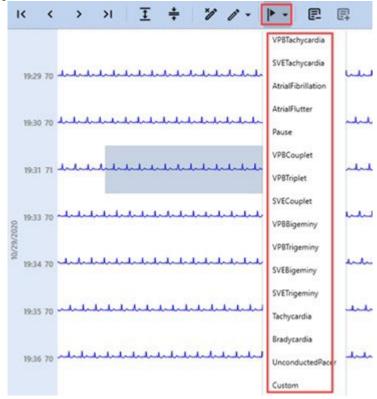

3. Click the relevant type of **Event** you wish to create in the selected area.

To exclude an ECG segment from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 225 of 393


- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 425. Events View - Page View Exclude from analysis

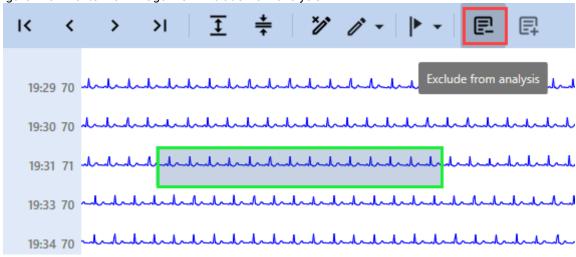

2. Click the **Exclude from Analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

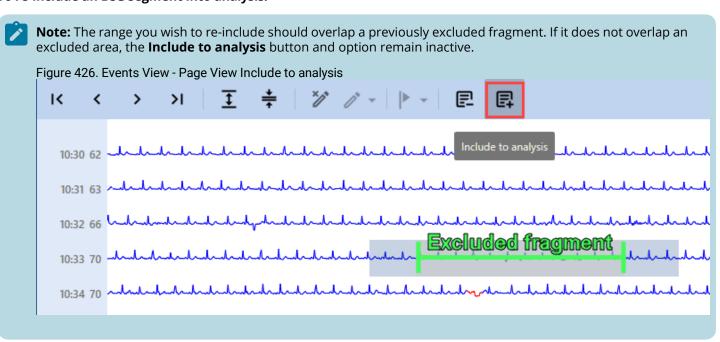

- 1. Select a fragment of the ECG you want to re-include in analysis, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.

Document Title:	NH	-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 226 of 393

- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.


Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Click the **Include to analysis** button or select the same option from the context menu.

Printing Control

This button enables you to print a selected fragment of the ECG Record.

Figure 427. Page View - Printing Control

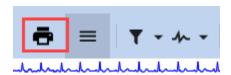


To print a fragment of the waveform in the **Page View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Page View *(on page 232)* to locate the desired area, if needed.
- 3. Click the **Printing Control** and navigate to the Print dialog box:

Document Title:	NH	301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 227 of 393

Figure 428. Strip View - Printing Dialog Box

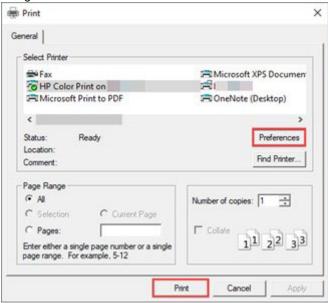

- 3.1. Choose a printer available in the Select Printer window.
- 3.2. Adjust other preferences according to your needs.
- 3.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Events Highlighting Control

Toggle this button **ON** to highlight **Events** occurrences within the **Page View**. Toggle **OFF** to stop highlighting **Events**. Please keep in mind that in some cases different **Events** may overlap in the **ECG Recording**.

Figure 429. Events View - Page View Highlight Events Button OFF



Figure 430. Events View - Page View Highlight Events Button ON



Filters

This group of controls allows you to apply various filters affecting waveform visualization and enhancing the accuracy and reliability of ECG analysis.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 228 of 393

Icons	Description
₹ -	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.
4 •	Channel Selection: This control lets you select the channel(s) for display within the Page View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record .

To toggle filters ON/OFF:

1. Hover over the **Filter** drop-down list to expand it.

Figure 431. Events View - Toggling Filters

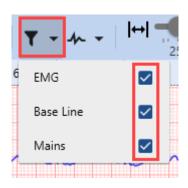

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.


It is recommended to keep filters **ON** to provide accurate analysis:

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Page View:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 432. Events View - Toggling Channels

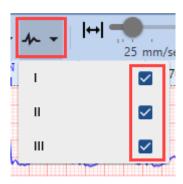

2. Select channels you wish to toggle. The visualization within the **Page View** will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting waveform visualization and enhancing the accuracy and reliability of ECG analysis.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	01 ver. 4.0.0.10 – Instructions for Use	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 229 of 393

Figure 433. Events View - Scale and Gain Controls




To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 1.56 up to 25 mm/sec. The visualization within the **Page View** will change immediately.

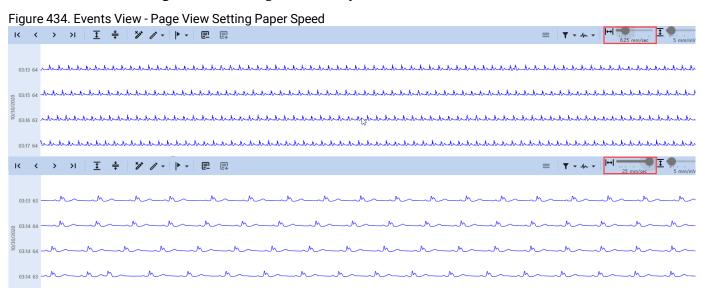

To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 0.5 up to 80 mm/mV. The visualization within the **Page View** will change immediately.

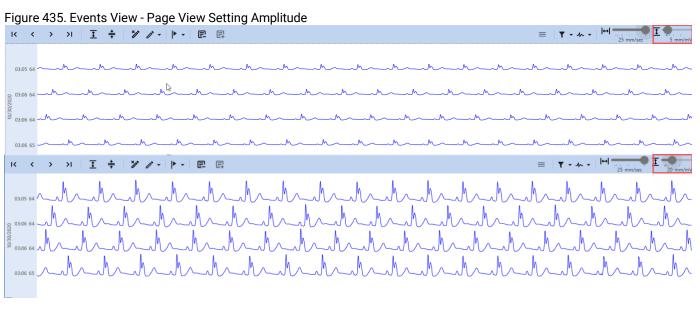


4.5.7.5.2. Context Menu Features

The **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions available on the **Page View Toolbar**.

The **Context Menu** mirrors the **Page View Toolbar**, with menu options carrying the same annotations and capabilities as buttons on the Toolbar *(on page 219)*.

Document Title:	NH	-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 230 of 393



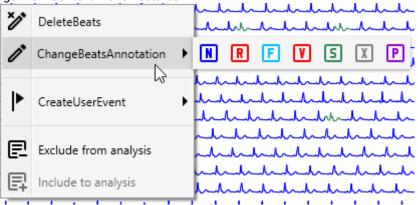

4.5.8. Page

The **Page View** displays the entire ECG data as **Full Disclosure**, enabling back-and-forth navigation for the complete Recording review. ECG sections in the **Page View** are viewable one channel at a time. The **Page View** also supports Secondary Screen Mode (on page 270) (or Dual Screen Mode), adding additional level of comfort for reviewing and editing the Recording in the **Page View**.

The **Page View** screen houses two main elements: the **Signal Page** in the top part, and the **Strip View** in the lower part. In the top area of the **Signal Page** you can see the **Signal Page Toolbar** - a collection of controls allowing you to scan and review the entire **ECG Recording**.

review the entire **ECG Recording**.

Figure 437. Page View - Page View


Patient: Dan Monns ID: 182217 PM

Patient: Dan Monns ID: 182217 PM

Figure 437. Page View - Page View

Patient: Dan Monns ID: 182217 PM

Figure 437. Page View - Pa

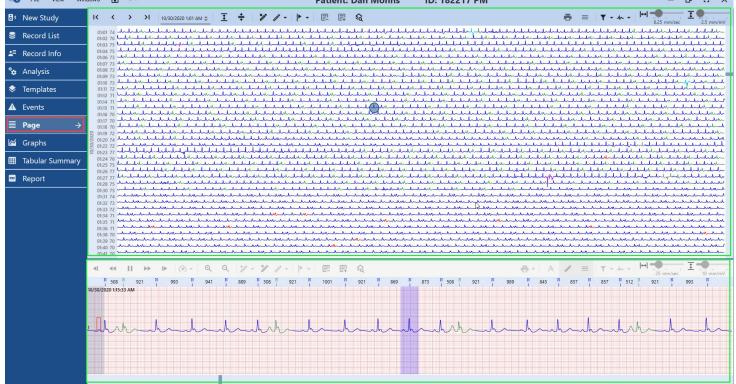


A green circle on the **Page** indicates current position in the **ECG Record**, shown in the **Strip View** below the Page.

4.5.8.1. Page View Toolbar

The **Page View Toolbar** contains controls for facilitating the scanning and review of large ECG Recording fragments, beats deletion and reclassification, adding user events, excluding noise segments and more. For ease of understanding, these controls are organized into distinct groups of interface elements.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	01 ver. 4.0.0.10 – Instructions for Use	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 231 of 393

Figure 438. Page View - Page View Toolbar

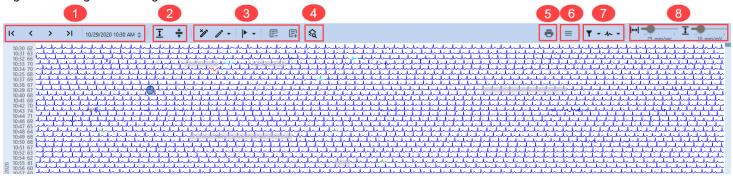


- 1. General Controls.
- 2. Page View Layout Controls.
- 3. Editing Controls.
- 4. Show in Templates Control.
- 5. Printing Control.
- 6. Events Highlighting Control.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate navigation within the **Page View**.

Figure 439. Page View - General Controls

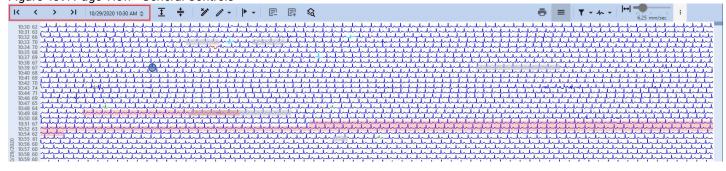




Note: To navigate within the **Page View**, you can also use the built-in scrollbar on the right.

Icon	Description
K	First Page: Click to display the first page of the Record.
<	Page Up: Click to navigate to the previous page of the Record.
>	Page Down: Click to navigate to the next page of the Record.
>1	Final Page: Click to display the last page of the Record . Last pages of a Record tend to have noisy segments.
10/28/0020 10/30 AM (\$	Time Navigation Control: Click arrows on the right to navigate the ECG Record by switching between its time stamps.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 232 of 393

Page View Layout Controls comprise a group of buttons designed to alter the number of waveform lines displayed in the **Page View**.

Figure 440. Page View - Page View Layout Controls

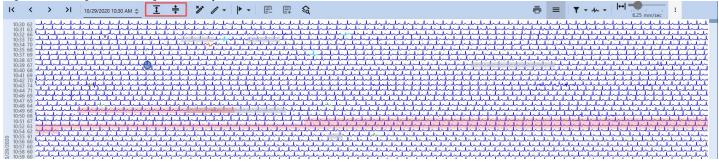


Icon	Description
<u>‡</u>	Decrease Lines Per Page button: Click to reduce the number of lines per Page, enhancing focus on specific ECG lines. This feature allows for adjusting the density of lines within the Page View . Repeated clicks reduce lines to a minimum of one per Page.
*	Increase Lines Per Page button: Click to add more lines per Page, enabling a broader segment of the ECG Record to be reviewed. Adjust the density of lines within the Page View as needed. However, an overpopulated Page View may hinder waveform analysis due to a visual overload.

Editing Controls

Editing Controls is a group of buttons allowing you to reclassify and delete beats, create user events, and exclude noisy segments from the Record.

Figure 441. Page View - Editing Controls

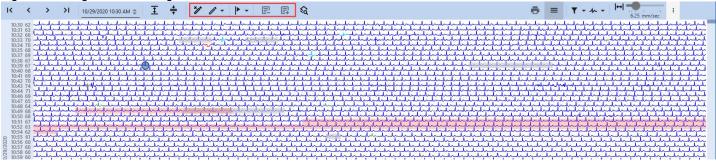

Icon	Description
/	Delete Beats button : Click to delete beats within the selected area.
<i>-</i>	Change Beats Annotation button : Enables you to modify the annotation of selected beats.
 	Create User Event button : Allows you to create a new user event, located within the selected area. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may change their annotations.
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
目	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

By default, when you click on a Record waveform within the **Page View** the corresponding area of the **Strip View** is highlighted.

To delete a beat:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 233 of 393	

- 1. Select a fragment of the ECG you wish to delete, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 442. Page View - Delete Beats

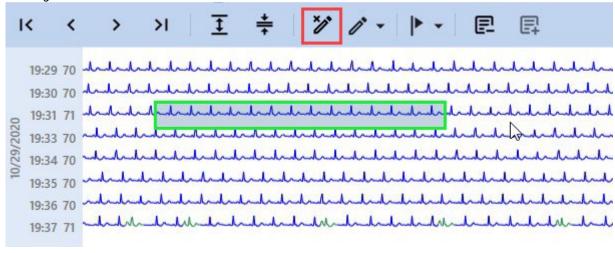

2. Click the **Delete Beats** button.

To change beats annotation:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 234 of 393		

- 1. Select a fragment of the ECG, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Page View *(on page 220)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the Change Beats Annotation button in the Page View Toolbar to expand the drop-down list.

Figure 443. Page View - Change Beats Annotation

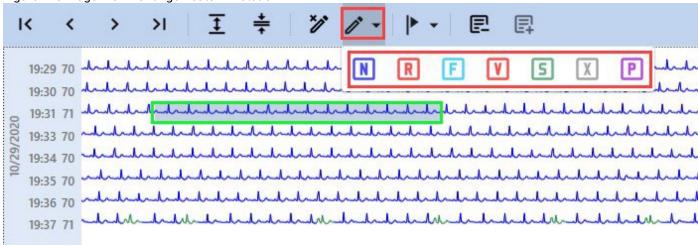

3. Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Selected beats will immediately change color.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

Table 8. Beats Reclassification Keyboard Shortcuts

for Page View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V
Classify as SVE	S	S
Classify as Questionable	X	х
Classify Paced	P	Р

To create a user event:

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.


Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the Create User Event button or menu option to expand the drop-down list.

Figure 444. Page View - Create User Event

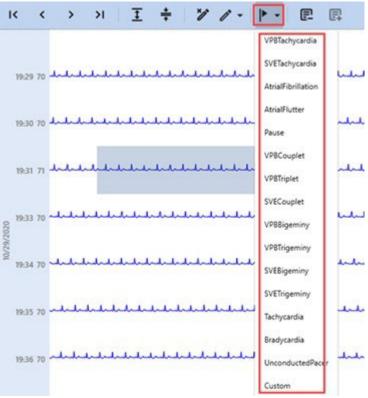

3. Click the relevant type of **Event** you wish to create in the selected area.

To exclude an ECG segment from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 237 of 393	

- 1.5. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:


- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 445. Page View - Exclude from analysis

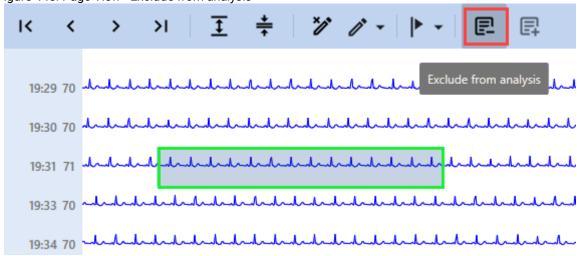

2. Click the **Exclude from Analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

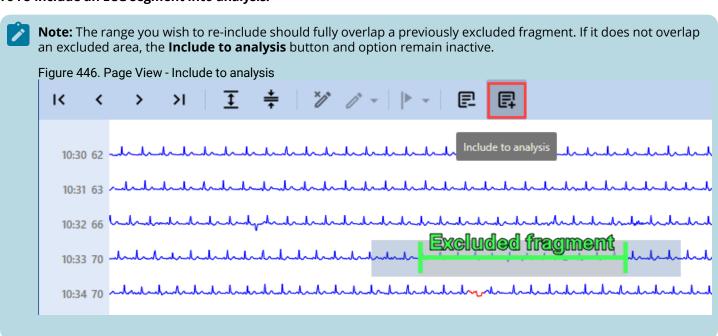

- 1. Select a fragment of the ECG, you want to re-include in analysis, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 238 of 393	

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Page View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Page View (on page 220) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Click the **Include to analysis** button or select the option from the context menu.

Show in Templates Control

This control allows you to focus on a single beat you have selected in the **Page View**, in the **Templates View**.

Icon	Description
⟨ Ø	Show In Templates button: Click this button to display the beat you have selected in the Page View ; it will be highlighted in the relevant template within the Templates Pane (on page 97).

Events Highlighting Control

Toggle this button **ON** to highlight **Events** occurrences within the **Page View**. Toggle **OFF** to stop highlighting **Events**. Please keep in mind that in some cases different **Events** may overlap in the **ECG Recording**.

Figure 447. Page View - Highlight Events Button OFF



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025	

Figure 448. Page View - Highlight Events Button ON



Filters


This group of controls allows you to apply various filters affecting waveform visualization and enhancing the accuracy and reliability of ECG analysis.

Icons	Description					
	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.					
+	Channel Selection: This control lets you select the channel(s) for display within the Page View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record .					

To toggle filters ON/OFF:

1. Hover over the **Filter** drop-down list to expand it.

Figure 449. Page View - Toggling Filters

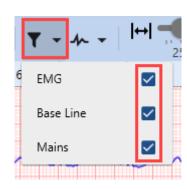

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.

It is recommended to keep filters **ON** to provide accurate analysis:


- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Page View:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 240 of 393

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 450. Page View - Toggling Channels

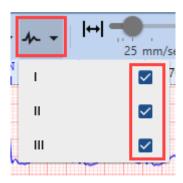

2. Select channels you wish to toggle. The visualization within the **Page View** will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting waveform visualization and enhancing the accuracy and reliability of ECG analysis.


Figure 451. Page View - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 1.56 up to 25 mm/sec. The visualization within the **Page View** will change immediately.

Figure 452. Page View - Page View Setting Paper Speed

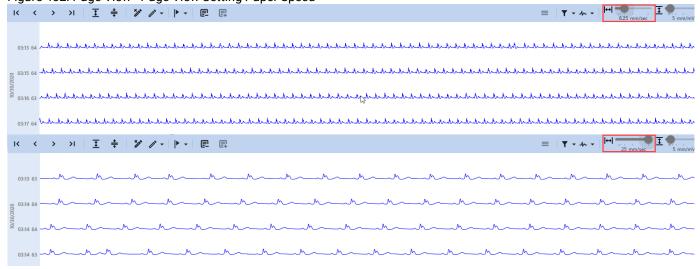


Document Title:	NH	NORAV medical			
Document No.:	NV-200.40010.028	0.40010.028 Document Rev.: 04 Document Date: 07.08.2025			

To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 0.5 up to 80 mm/mV. The visualization within the **Page View** will change immediately.


Figure 453. Page View - Setting Amplitude

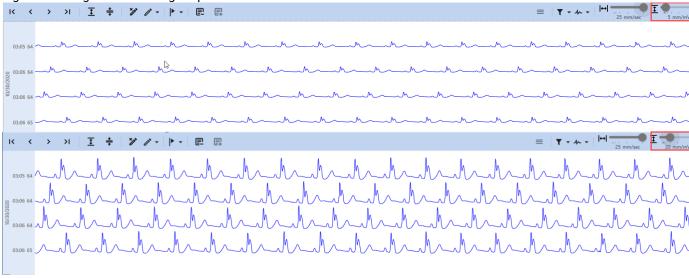

4.5.8.2. Strip View

The ECG **Strip View** is displayed in the bottom pane of the **Templates View** and in the **Edit Templates Mode.** It is also displayed in the **Events** and **Page Views**. The **Strip View** presents detailed data for the currently selected beat, as well as the adjacent ECG signal. Typically, the strip displays 12 to 16 seconds of ECG signal, depending on the monitor characteristics and ECG paper speed settings. You can utilize the **Strip View** for detailed analysis, measurements, beat reclassification, user event creation, and more.

Figure 454. Strip View - Strip View

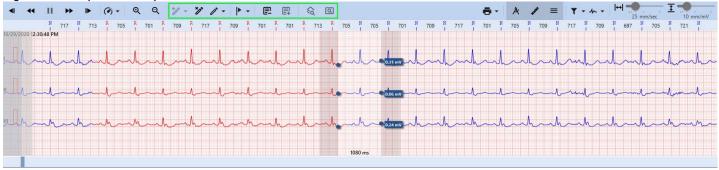



Note:

The ECG Strip grid is scaled accurately to millimeters, irrespective of the computer or monitor size in use. The **NH-301 Holter** analysis system automatically adjusts all application windows to align with the computer's graphical settings and monitor capabilities. This feature enables you to use any measuring tool, including specialized ECG rulers like the **Norav Medical ECG ruler**, to measure amplitude, cycles, frequency, and RR intervals.

4.5.8.2.1. Strip View Toolbar

The toolbar at the top of the **Strip View** contains various controls that facilitate the scanning and reviewing of the ECG signal within the **Strip View** itself. For ease of understanding, these controls are organized into distinct groups of interface elements:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 242 of 393





- 1. General Controls.
- 2. Beats and Events Controls.
- 3. Analysis Controls.
- 4. Views Switching Controls.
- 5. Printing Control.
- 6. Measuring Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate efficient ECG Strip scanning and review within the **Strip View**.

Icon	Description						
■	Step Backward button: Click to move the ECG Strip one step backward for scanning and reviewing.						
44	Scan Backward button: Click to initiate continuous backward scanning of the ECG Strip.						
П	Pause Scan button: Click to halt continuous scanning of the ECG Strip. For example, if you have activated a Scan Backward or a Scan Forward button, halt scanning by clicking the Pause Scan button.						
**	Scan Forward button: Click to initiate continuous forward scanning of the ECG Strip.						
I	Step Forward button: Click to move the ECG Strip one step forward for scanning and reviewing.						
(d) +	Scan Speed button: Allows you to control the scanning speed. To set the scanning speed:						
	1. Hover over the Scan Speed icon to expand the drop-down list.						
	2. Click the desired scanning speed multiplier, ranging from ×1 to ×128.						
Q	Zoom In button: Click to adjust the scale of the waveform in the Strip View , focusing on specific areas or comparing different fragments of the ECG record.						
Q	Zoom Out button: Click to adjust the scale of the waveform in the Strip View . Use this button to zoom out if you had zoomed in earlier, or to view adjacent beats and fragments of the ECG Records.						



Note: You can also click and hold the strip to drag it sideways, allowing you to view adjacent beats and ECG fragments.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025			

Beats and Events Controls

Beats and Events Controls are a group of buttons that facilitate beats and events management in the Strip View.

Icon	Description
/ /* -	Insert Beat button: Allows you to insert new beats between existing ones. Further instructions are presented below.
*/	Delete Beats button : Click to delete a selected beat.
<i>I</i> * -	Change Beats Annotation button : Enables you to modify the annotation of selected beats.
 	Create User Event button : Allows you to create a new user event, located at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates. Further instructions are presented below.

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position in the **Strip View**.

To insert a beat:

- 1. Select a position in the **Strip View** using one of these methods:
 - Press Alt+Click in the **Strip View**: A vertical dotted line will appear, marking the desired position.
 - Press your mouse wheel in the **Strip View:** A vertical dotted line will appear, marking the desired position.
 - Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside
 the dotted line, a context menu will also expand, simplifying access to the **Insert Beat** option. **Note:** this key
 combination expands the context menu with only one option available **Insert Beat**.
- 2. Hover over the Insert Beat button in the Strip View Toolbar to expand the drop-down list.

Figure 456. Strip View - Inserting Beats



3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

To delete beats:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025		

• Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 457. Strip View - Deleting Beats



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Rev.: 04 Document Date: 07.08.2025			

To change beat annotation:

- 1. Select a fragment of the ECG, which you wish to reclassify, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 458. Strip View - Changing Beats Annotation



2. Hover over the Change Beats Annotation button in the Strip View Toolbar to expand the drop-down list.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 246 of 393

3. Click the appropriate type of morphology for the beat you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. The selected beat in the **Strip View** will immediately change color.

Table 9. Beats Reclassification Keyboard Shortcuts

for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V
Classify as SVE	S	S
Classify as Questionable	X	x
Classify Paced	P	Р


To create a user event:

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. (Optional) Scan the Strip View (on page 243) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 247 of 393

- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 459. Strip View - Creating User Event

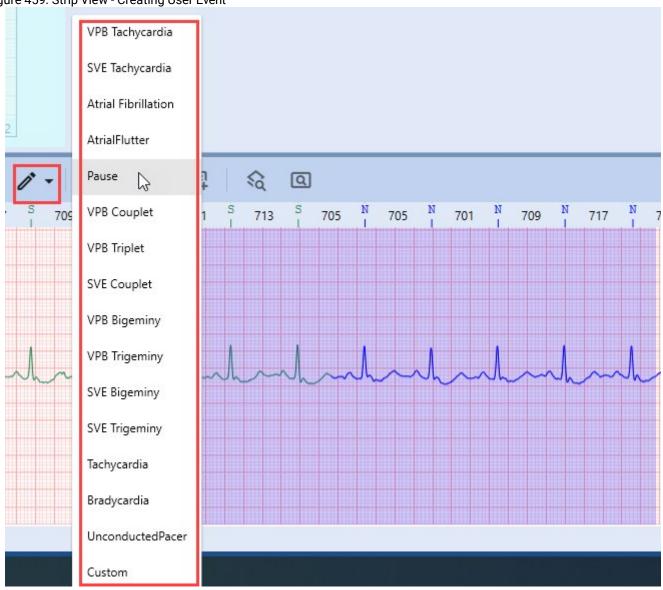

- 2. Hover over the Create User Event button or menu option to expand the drop-down list.
- 3. Click the relevant type of **Event** you wish to create in the selected area.

Analysis Controls

Analysis Controls is a group of buttons that enable you to exclude or re-include specific fragments of the ECG recording within the Strip View.

lcon	Description
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
目	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 248 of 393	

To exclude an ECG segment from analysis:

1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the Ctrl key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 460. Strip View - Excluding from analysis



Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 249 of 393

2. Click the **Exclude from analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

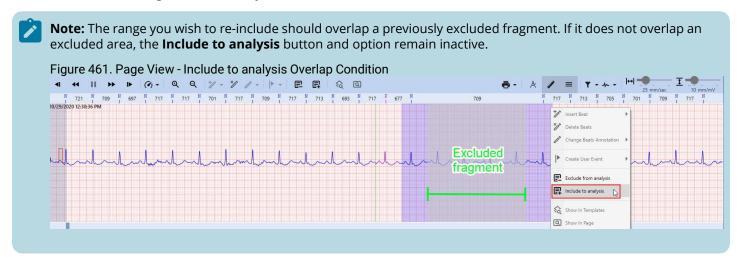

- 1. Select a fragment of the ECG, you want to re-include to analysis, using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) Scan the Strip View (on page 243) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Hold down the **Ctrl** key.
 - 1.3. Click the initial point.
 - 1.4. Identify the final point in your desired range.
 - 1.5. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
 - 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
 - Using the right mouse button:
 - 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
 - 1.2. Drag the cursor to set the final point.
 - 1.3. **(Optional)** Scan the Strip View (on page 243) to locate the points of interest, if necessary.
 - 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 462. Strip View - Including to analysis

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 250 of 393



2. Click the **Include to analysis** button or select the option from the context menu.

Views Switching Controls

Views Switching Controls are a group of buttons that allow you to focus on a single beat you have selected in the **Strip View**, in the **Templates View** and **Page View**. This functionality enhances analysis quality and efficiency.

Icon	Description
\$	Show In Templates button: Click this button to display the beat you have selected in the Strip View ; it will be highlighted in the relevant template within the Templates Pane .
Q	Show in Page : Facilitates an immediate switch to Page View , revealing the precise location of the chosen beat within this particular Template in both the Signal Page and the ECG Strip .




Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 251 of 393

Printing Control

This button enables you to print a fragment of the ECG **Strip**. The printed copy will include a segment that fits the output format. The baseline of the printed version mirrors the center line of the Strip visible on your PC screen. The printed ECG Strip will accommodate as much ECG data as possible from the visible area around the center line.


Figure 464. Strip View - Printing Strip

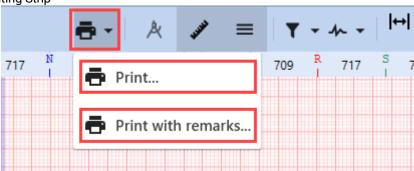

To print a fragment of the waveform in the **Strip View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Strip View *(on page 243)* to locate the desired area, if needed.
- 3. Hover over the **Printing Control** button to expand the drop-down list.
- 4. Click the **Print** option and navigate to the Print dialog box:


Figure 465. Strip View - Printing Dialog Box

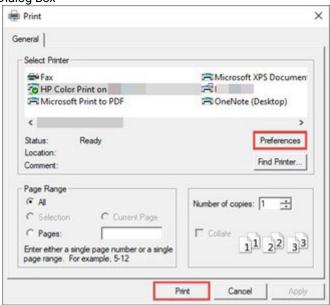

- 4.1. Choose a printer **available** in the **Select Printer** window.
- 4.2. Adjust other preferences according to your needs.
- 4.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.
- 5. **(Optional)** Click the **Print with remarks** option if you want to print a fragment of the waveform with your remarks:
 - 5.1. Complete the "Report remarks" field in the **Edit Report Remarks** pop-up.

Figure 466. Strip View - Printing with Remarks

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 252 of 393

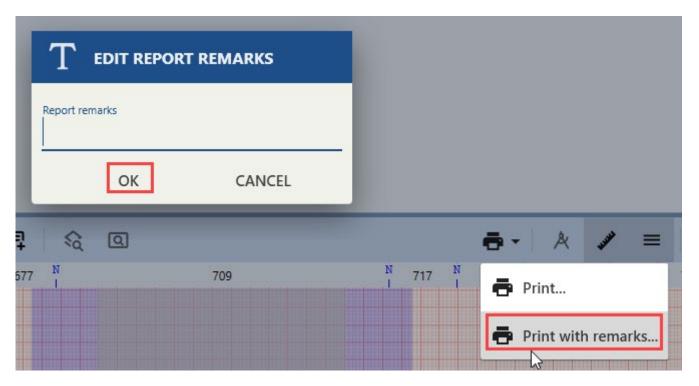

- 5.2. When you are complete with filling in you remarks, click OK.
- 5.3. (Optional) Click Cancel to abandon this action.
- 5.4. Choose a printer available in the **Select Printer** window.
- 5.5. Adjust other preferences according to your needs.
- 5.6. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Measuring Controls


Measuring Controlsare a group of buttons designed to facilitate the indication and measurement of various waveform parameters within the **Strip View**. Namely, the most important tool in this group of controls is the **Caliper** tool. It allows you to measure different intervals and amplitudes on the ECG strip, which can be used to diagnose and assess a variety of heart conditions. We will explain how to use the Caliper for ECG measurements (on page 160) in the following sections.

Icon	Description
Å	Caliper toggle : Click to toggle ON/OFF the Caliper tool. The Caliper tool assists with measuring intervals and amplitudes, such as the RR interval, T-wave, and QRS complex amplitudes. Refer to the Measuring ECG with Caliper (on page 160) section for details.
1999	ECG Ruler toggle : Click to toggle ON/OFF the ECG Ruler , located right below the Strip View Toolbar . The ECG Ruler indicates the duration of RR intervals for adjacent beats and their morphology classification (i.e., N, R, F, V, etc.).
≡	Channel Numeration toggle: Click to toggle ON/OFF the channel numeration on the left side of the ECG Strip.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 253 of 393





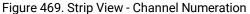



Filters


This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Icons	Description
₹ -	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG , Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.
* -	Channel Selection: This control lets you select the channel(s) for display within the Strip View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record).

To toggle filters **ON/OFF**:

1. Hover over the **Filter** drop-down list to expand it.

Figure 470. Strip View - Toggling Filters

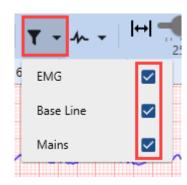

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.

It is recommended to keep filters **ON** to provide accurate analysis:

- **EMG filter:** Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.


Document Title:	NH	NORAV medical
Document No.:	NV-200.40010.028	Page 254 of 393

• **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Strip:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 471. Strip View - Toggling Channels

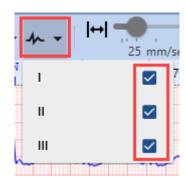

2. Select filters you wish to toggle. The visualization of the Strip will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 472. Strip View - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within the Strip will change immediately.

Figure 473. Strip View - Setting Paper Speed



Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV medical
Document No.:	NV-200.40010.028	7-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		

To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within the Strip will change immediately.


Figure 474. Strip View - Setting Amplitude



4.5.8.2.2. Context Menu Features

A **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions, such as inserting or deleting beats, changing annotations, creating user events, etc.

Figure 475. Strip View - Context Menu

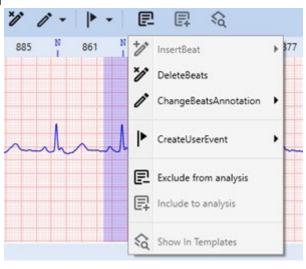

Insert Beat

Allows you to insert new beats between existing ones:




Note:

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position within the **Strip View**.


1. Select a position in the **Strip View** using one of these methods:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 256 of 393

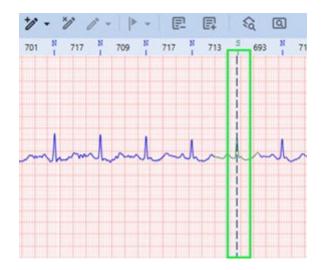

- Hold Alt and click in the **Strip View**: A vertical dotted line will appear, marking the desired position. While holding Alt, click the right mouse button to expand the context menu.
- Press your mouse wheel in the **Strip View**: A vertical dotted line will appear, marking the desired position. Click again and hold the mouse wheel, and then press the right mouse button to expand the context menu.
- Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside
 the dotted line, a context menu will expand, simplifying access to the **Insert Beat** option.
- 2. Hover over the Insert Beat option in the Context Menu to expand the drop-down list.

Figure 477. Strip View - Insert Beat

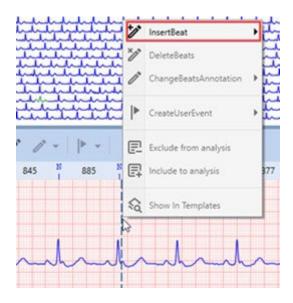

3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

Delete Beats

Allows you to delete a selected beat or a fragment of a waveform:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - · Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 257 of 393

- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.


• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 478. Strip View - Delete Beats

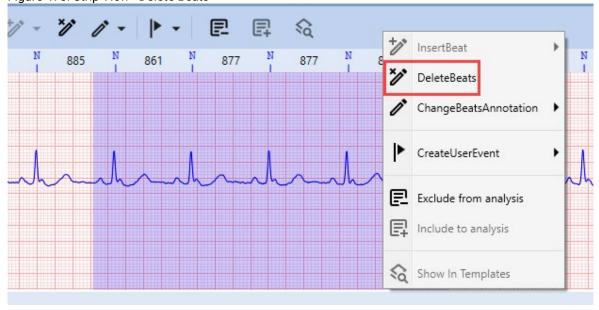

2. Click the **Delete Beats** option in the context menu. The system will automatically recalculate events and templates following the deletion.

Change Beats Annotation

Enables you to modify the annotation of the selected beat:

1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 258 of 393	

- · Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
- Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Change Beats Annotation** option in the context menu to expand the drop-down list.

Figure 479. Strip View - Change Beats Annotation

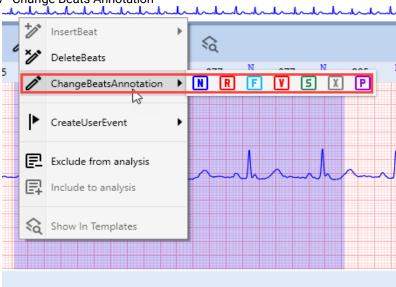

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025		

^{3.} Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Reclassified beats in the **Strip View** will immediately change color.

Table 10. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V
Classify as SVE	5	S
Classify as Questionable	X	Х
Classify Paced	P	Р

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 260 of 393	

Create User Event

Allows you to create a new user event at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates.

- 1. Select a fragment of the ECG you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View *(on page 138)* to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Create User Event** option to expand the drop-down list.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 261 of 393	



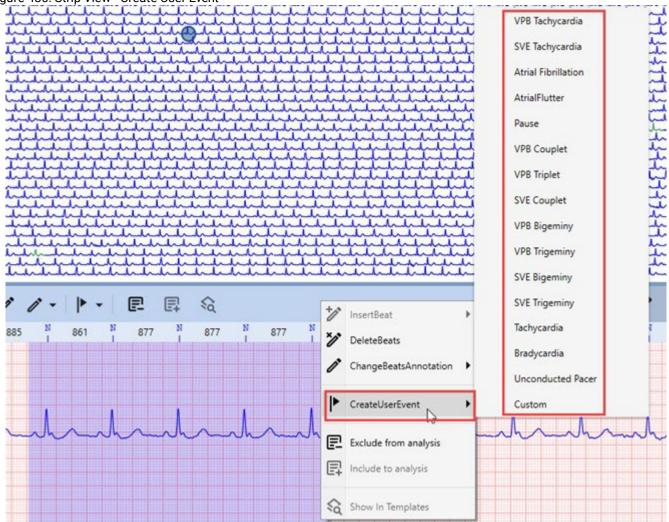

3. Click the relevant type of **Event** you wish to create in the selected area.

Exclude from analysis

Enables the exclusion of a selected fragment of an ECG record from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV medical
Document No.:	NV-200.40010.028	.200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		

- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 481. Strip View - Exclude from analysis

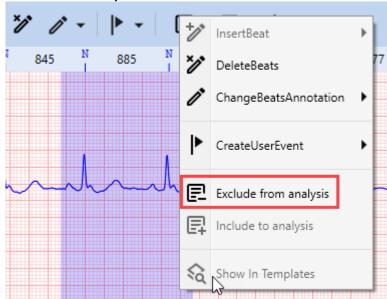

2. Click the **Exclude from Analysis** option in the context menu.

Include to analysis

Enables the re-inclusion of previously excluded fragments of an ECG record:

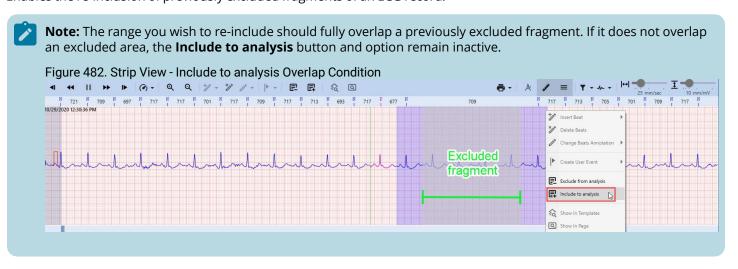

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 263 of 393

1. Select a fragment of the ECG you want to re-include in analysis, using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the Ctrl key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 483. Strip View - Include to analysis



2. Click the **Include to analysis** option in the context menu.

Show In Templates

Select this option from the context menu to display the beat you have selected in the **Strip View**; it will be highlighted in the relevant template within the **Templates Pane**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 264 of 393	

Figure 484. Strip View - Show In Templates



4.5.8.2.3. Measuring ECG with Caliper

You may use the **Caliper** function to measure the RR interval, QRS complex duration and amplitude, QT interval, and other ECG waveform parameters. The **Caliper** also allows for editing the ECG record via the **Strip View**.



Caliper Design Overview

The **Caliper** tool in the **Strip View** displays two **measuring lines** and a **measuring interval** between them. To activate the **Caliper** tool, click the **Caliper** button in the **Strip View Toolbar**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 265 of 393

Figure 486. Strip View - Caliper Measurement Lines and Interval



Each ECG channel in the Strip has a small box adjacent to the point where the right measurement line intersects the channel signal line. The value in this box indicates the amplitude difference between the two points demarcated by the measurement lines.

Figure 487. Strip View - Interval Length and Amplitude Difference



The interval length in milliseconds is displayed at the bottom of the measuring interval.

Moving Caliper Lines

You may move Caliper measurement lines either **simultaneously** or **independently**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 266 of 393	

To move both lines simultaneously:

1. Position the cursor within the measuring interval. The cursor will change to a resize cursor.

Figure 488. Strip View - Moving Caliper Lines Simultaneously




- 2. Drag and drop the **Caliper** to the desired location.
- 3. Optionally, use the Left and Right arrow keys to fine-tune the position.

To move lines independently:

1. Position the cursor near one of the measurement lines until it changes to a resize cursor.

Figure 489. Strip View - Moving Caliper Lines Independently

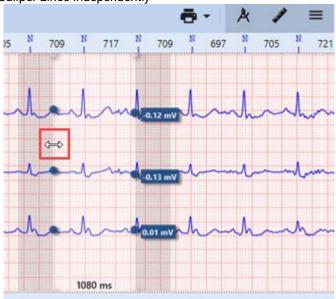

- 2. Click to select the measurement line.
- 3. Drag and drop it to the new location.

Zoom Feature

To adjust the display scale of the waveform in the Strip, utilize the **Zoom In/Zoom Out** options:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 267 of 393



- Click the corresponding button in the Strip View Toolbar.
- Hold the Ctrl key while scrolling up (Zoom In) or down (Zoom Out) the mouse wheel.

Caliper Snapping Feature

The **Caliper Snapping** feature automatically aligns the **Caliper** measurement lines to the nearest R-peaks in the ECG signal. This functionality aids in the precise measurement of ECG wave duration and amplitude. Manual alignment can be challenging, hence **Caliper Snapping** enhances the accuracy, efficiency, and reproducibility of ECG interpretation.

To utilize the Caliper snapping feature:

- 1. Click the Caliper button in the Strip View Toolbar.
- 2. (Optional) Use the Left and Right arrow keys to make slight adjustments to the Caliper measurement lines.
- 3. To snap a **Caliper** measurement line to an R-spike:

Figure 491. Strip View - Caliper Snapping



- To snap any line:
 - 3.1. Hold down the **Alt** key.
 - 3.2. Position the cursor near one of the measurement lines until the cursor icon changes to a resize cursor.
 - 3.3. Click the measurement line to select it.
 - 3.4. Drag and drop it to the desired location.
- To snap the right measurement line:
 - 3.1. Hold down the **Right Alt** key.
 - 3.2. Use the Left and Right arrow keys to move the **Caliper**; the right measurement line will automatically snap to the R-peaks as you move.
- To snap the left measurement line:
 - 3.1. Hold down the **Left Alt** key.
 - 3.2. Use the Left and Right Arrow keys to move the Caliper; the left measurement line will automatically snap to the R-peaks as you move.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 268 of 393	

The **Caliper Snapping** function is instrumental in enabling a quick and accurate evaluation of various ECG parameters.

Editing Beats and Events Using Caliper

The **Caliper** tool enables you to edit specific fragments of the ECG waveform contained within the measurement lines of the **Caliper**. When you opt to delete or reclassify beats, or exclude a waveform fragment from analysis, the changes will be confined to the selected fragment, which may contain multiple beats. In essence, the **Caliper** allows you to edit a continuous section of an ECG record, demarcated by the measurement lines, in a single action.

Figure 492. Strip View - Editing Beats and Events Using Caliper

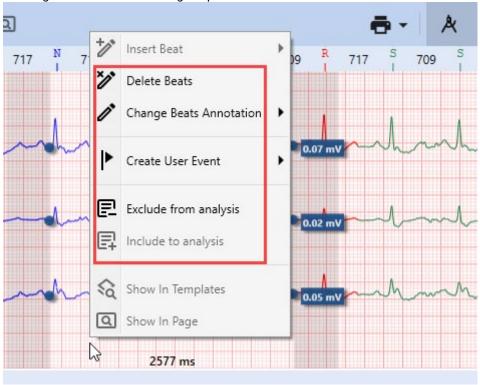

To edit a fragment of an ECG using the Caliper:

- 1. Position the **Caliper** and adjust its measurement lines as needed. For guidance, refer to the Caliper Design Overview (on page 267) section.
- 2. Right-click in the **Strip View** area to expand the context menu.
- 3. The **Caliper** tool's context menu offers several options within the **Strip View**:
 - **Delete Beats**: Click this option to remove all beats within the boundaries of the **Caliper's** measurement lines. This action will result in minor recalculations.



Note: Deleting beats may trigger the generation of **Pause** events. To annotate and exclude noisy segments, the **Exclude from analysis** option is preferable to **Delete Beats**.

- **Change Beats Annotation**: Hover over this option to display a drop-down list, then click the morphology type you wish to assign to the beats. The reclassified beats will instantly change color in the **Strip View**.
- **Create User Event**: Hover over this option to expand a drop-down list, and click the event type you wish to create in the selected area.
- **Exclude from analysis**: Click to exclude a noisy ECG segment within the **Caliper's** measurement lines from the analysis, without affecting other analytical outcomes.
- Include to analysis: Click to include previously excluded ECG fragments back into the analysis.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	301 ver. 4.0.0.10 – Instructions for Use	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 269 of 393



Note: The **Include in Analysis** option becomes active in the **Caliper** context menu only when the current **Caliper** measurement interval overlaps with a segment that was excluded earlier.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 270 of 393

4.5.8.3. Context Menu Features

The **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions available on the **Page View Toolbar**.

The **Context Menu** mirrors the **Page View Toolbar**, with menu options carrying the same annotations and capabilities as buttons on the Toolbar (on page 232).


Figure 493. Page View - Context Menu

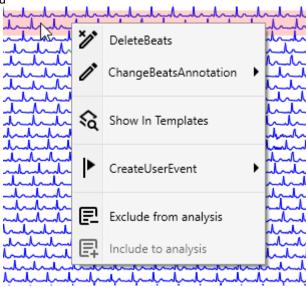

4.5.8.4. Secondary Screen Mode

The **NH-301 Holter** software supports **Secondary Screen Mode** to facilitate a more comfortable and efficient ECG analysis workflow. **Secondary Screen Mode** is a feature that allows for an additional display where you can place the **Page View**. This proves useful for quickly referring to the **Page View** while engaging with other **Views** on the main screen. The **Page View** in **Secondary Screen Mode** replicates the functionality of the **Page View** strips in their respective **Views**.

Figure 494. Page View - Secondary Screen Mode

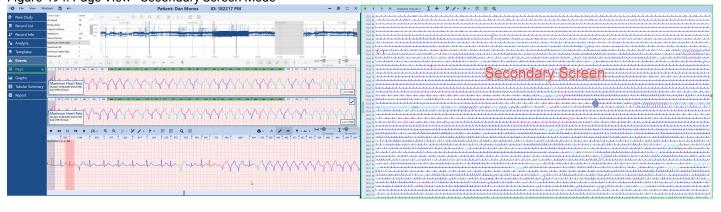

Views Interaction

The **Page View** in **Secondary Screen Mode** mirrors the functionalities of **Page View** elements in their respective **Views**. It offers similar interactions with other components of the specific **View**. Activating the **Secondary Screen Mode** renders the **Page View** in the **Views Sidebar** inactive.

View	Description
Analysis	Combines Page View and Strip View functionalities.

Document Title:	NH	-301 ver. 4.0.0.10 – Insti	01 ver. 4.0.0.10 – Instructions for Use	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 271 of 393

Tem- plates	Retains common Page View functionalities.
Events	Retains common Page View functionalities.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 272 of 393


View	Description
Graphs (ST tab)	Retains common Page View functionalities. Click HR , RR graphs, or the Strip View to navigate to the corresponding beat in the Page View .
Tabular Summary	Combines Page View and Strip View functionalities. Clicking any line in the Tabular report navigates to the relevant beat in the Page View and Strip View in the secondary window.

Activating Secondary Screen Mode

To activate Secondary Screen mode:


- Using the Menu Bar:
 - 1. Select the **Windows** menu in the Menu Bar.

Figure 495. Page View - Activating Secondary Screen Mode via Menu Bar

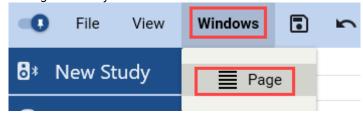

- 2. Choose the **Page** option from the menu. The **Page View** will open in a secondary window.
- 3. Adjust the secondary window settings as needed.
- Using Views Sidebar:
 - 1. Click the arrow icon next to the Page View option in the Views Sidebar. The Page View will appear in a secondary window.

Figure 496. Page View - Activating Secondary Screen Mode via Views Sidebar

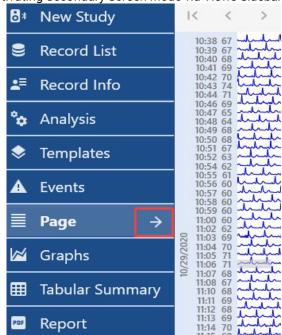


2. Adjust the secondary window settings as needed.

To deactivate Secondary Screen Mode:

1. Click the **X** (Close) icon in the top-right corner of the secondary window.

Figure 497. Page View - Deactivating Secondary Screen Mode

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 273 of 393

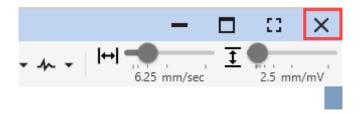


.

4.5.9. Graphs

The **Graphs View** facilitates the examination of **HRV** analysis results and the preview of **ST** trends, as well as **QT** and **QTc** analysis. **HRV** graphs illustrate both the time domain analysis outcomes and the frequency domain spectrum.



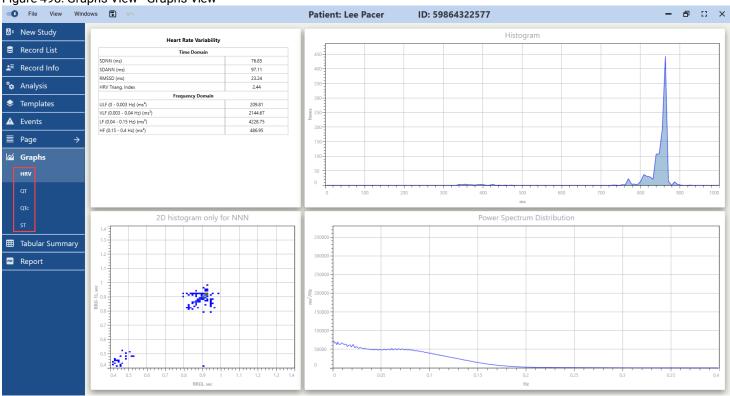


The **HRV**, **ST**, **QT**, and **QTc** tabs under the **Graphs View** enable the selection of the page displaying **HRV**, **QT**, and **QTc** analysis or the visualization of **ST** trends.

HRV Analysis

The **Heart Rate Variability** table lists standard HRV Time Domain and Frequency Domain analysis metrics, such as SDNN, SDANN, RMSSD, and more.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 274 of 393

Figure 499. Graphs View - HRV Tab

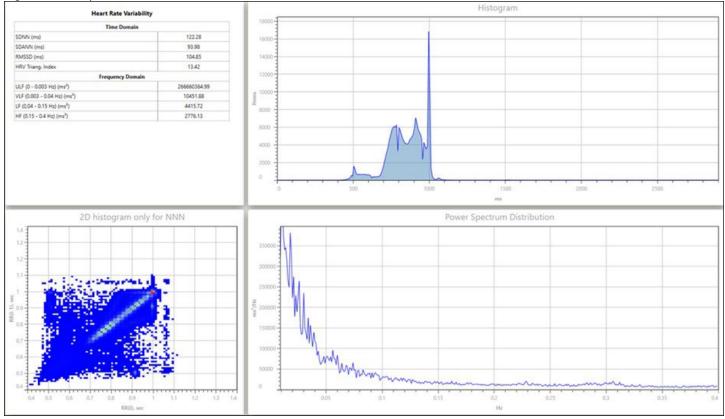

The **Histogram** illustrates the distribution of RR Intervals with identical durations. The graph's shape offers immediate insight into the variability.


The **Lorenz Plot** or **Scatter Diagram** (bottom left), based on the RR Intervals of consecutive **N**-annotated beats (with neither S nor V preceding or following the actual beat), presents a graphical representation of the variability. The X-axis denotes Interval (i), while the Y-axis corresponds to Interval (i-1). This plot provides a swift assessment of variability; a smaller diagonal line suggests lower variability. Outlier dots indicate extreme variations, such as SVTs or Pauses.

The **Power Spectrum Distribution** graph depicts the frequency domain analysis.

QT


The QT analysis offers insights into ventricular repolarization and can assist in identifying potential abnormalities linked to conditions such as Long QT Syndrome.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028		Page 275 of 393	

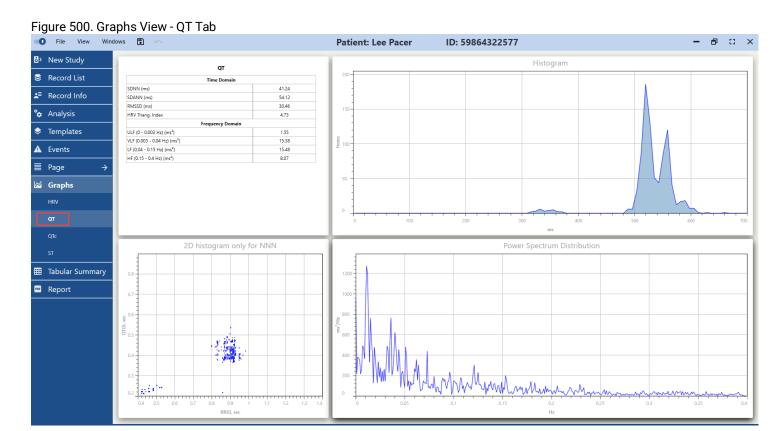

QTc


The **QTc** analysis provides insights into ventricular repolarization and can help identify potential abnormalities that might be associated with conditions like Long QT Syndrome. The QTc interval is a corrected version of the QT interval. By comparing the calculated QTc interval to the reference ranges, healthcare professionals can assess if it falls within the expected range, potentially indicating a normal or abnormal repolarization process.

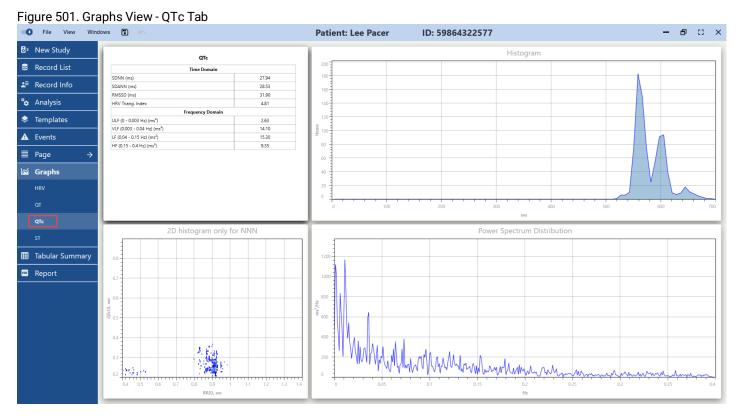

Document Title:	e: NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 276 of 393

ST

This page represents the **ST** calculations for each ECG channel. The top trend on the page displays the **HR** or **RR** intervals. To switch between trends, click on RR/HR buttons in the upper-right corner.

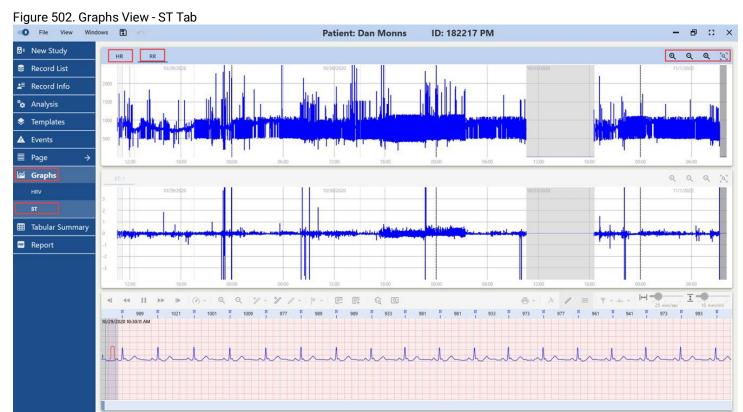


You can conduct a more thorough review of the graphs using the scaling buttons located in the top-right corner.


lcon	Description
Q	Zoom In button: Click to adjust the scale of the HR or RR graph in the HR/RR Trends View , focusing on specific areas of the graph. You can click this button multiple times.
Q	Zoom Out button: Click to adjust the scale of the HR or RR graph in the HR/RR Trends View . Use this button to zoom out if you have zoomed in previously, or to view adjacent areas of the graph. You can click this button multiple times.
ত্	Full Extent button: Click to zoom out instantly and see the entire HR/RR Trends graph.
[d]	Zoom to Extent button: Click to zoom in instantly to the selected area of the graph. The area will be displayed at the maximum available zoom.

4.5.9.1. Strip View

The ECG **Strip View** is displayed in the bottom pane of the **Templates View** and in the **Edit Templates Mode**. It is also displayed in the **Events**, **Page**, and **Graphs Views**. The **Strip View** presents detailed data for the currently selected beat, as well as the adjacent ECG signal. Typically, the strip displays 12 to 16 seconds of ECG signal, depending on the monitor characteristics and ECG paper speed settings. You can utilize the **Strip View** for detailed analysis, measurements, beat reclassification, user event creation, and more.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 277 of 393

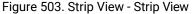

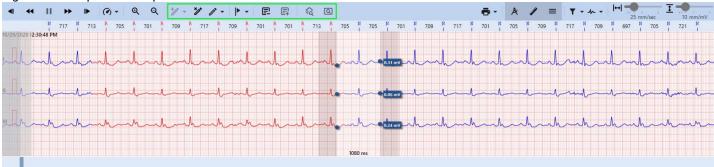




Note:

The ECG Strip grid is scaled accurately to millimeters, irrespective of the computer or monitor size in use. The **NH-301 Holter** analysis system automatically adjusts all application windows to align with the computer's graphical settings and monitor capabilities. This feature enables you to use any measuring tool, including specialized ECG rulers like the **Norav Medical ECG ruler**, to measure amplitude, cycles, frequency, and RR intervals.

4.5.9.1.1. Strip View Toolbar

The toolbar at the top of the **Strip View** contains various controls that facilitate the scanning and reviewing of the ECG signal within the **Strip View** itself. For ease of understanding, these controls are organized into distinct groups of interface elements:

Figure 504. Templates View - Strip View Toolbar

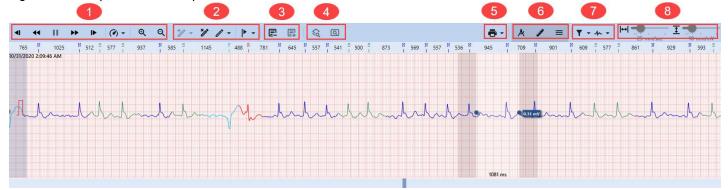

- 1. General Controls.
- 2. Beats and Events Controls.
- 3. Analysis Controls.
- 4. Views Switching Controls.
- 5. Printing Control.
- 6. Measuring Controls.
- 7. Filters.
- 8. Scale and Gain Controls.

General Controls

General Controls is a group of buttons designed to facilitate efficient ECG Strip scanning and review within the **Strip View**.

Document Title:	ent Title: NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	0.40010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 278 of 393

lcon	Description
•	Step Backward button: Click to move the ECG Strip one step backward for scanning and reviewing.
*	Scan Backward button: Click to initiate continuous backward scanning of the ECG Strip.
I	Pause Scan button: Click to halt continuous scanning of the ECG Strip. For example, if you have activated a Scan Backward or a Scan Forward button, halt scanning by clicking the Pause Scan button.
*	Scan Forward button: Click to initiate continuous forward scanning of the ECG Strip.
•	Step Forward button: Click to move the ECG Strip one step forward for scanning and reviewing.
(d) +	Scan Speed button: Allows you to control the scanning speed. To set the scanning speed:
	1. Hover over the Scan Speed icon to expand the drop-down list.
	2. Click the desired scanning speed multiplier, ranging from ×1 to ×128.
Q	Zoom In button: Click to adjust the scale of the waveform in the Strip View , focusing on specific areas or comparing different fragments of the ECG record.
Q	Zoom Out button: Click to adjust the scale of the waveform in the Strip View . Use this button to zoom out if you had zoomed in earlier, or to view adjacent beats and fragments of the ECG Records.



Note: You can also click and hold the strip to drag it sideways, allowing you to view adjacent beats and ECG fragments.

Beats and Events Controls

Beats and Events Controls are a group of buttons that facilitate beats and events management in the Strip View.

Icon	Description
/ // -	Insert Beat button: Allows you to insert new beats between existing ones. Further instructions are presented below.
*/	Delete Beats button : Click to delete a selected beat.
<i>(</i> * -	Change Beats Annotation button : Enables you to modify the annotation of selected beats.
 	Create User Event button : Allows you to create a new user event, located at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates. Further instructions are presented below.

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position in the **Strip View**.

To insert a beat:

- 1. Select a position in the **Strip View** using one of these methods:
 - Press Alt+Click in the **Strip View**: A vertical dotted line will appear, marking the desired position.
 - Press your mouse wheel in the **Strip View:** A vertical dotted line will appear, marking the desired position.
 - Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside
 the dotted line, a context menu will also expand, simplifying access to the **Insert Beat** option. **Note:** this key
 combination expands the context menu with only one option available **Insert Beat**.
- 2. Hover over the Insert Beat button in the Strip View Toolbar to expand the drop-down list.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 279 of 393

Figure 505. Strip View - Inserting Beats



3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

To delete beats:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. (Optional) Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 280 of 393

- 1.3. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 506. Strip View - Deleting Beats



2. Click the **Delete Beats** button.

To change beat annotation:

- 1. Select a fragment of the ECG, which you wish to reclassify, using one of the following methods:
 - · Click an individual beat to select it.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 281 of 393

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 507. Strip View - Changing Beats Annotation



- 2. Hover over the Change Beats Annotation button in the Strip View Toolbar to expand the drop-down list.
- 3. Click the appropriate type of morphology for the beat you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. The selected beat in the **Strip View** will immediately change color.

Table 11. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	E	F
Classify as VPB	V	V
Classify as SVE	5	S
Classify as Questionable	X	X
Classify Paced	P	Р

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 282 of 393

To create a user event:

- 1. Select a fragment of the ECG, which you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it.
 - Select a fragment of the waveform:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.


Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 508. Strip View - Creating User Event

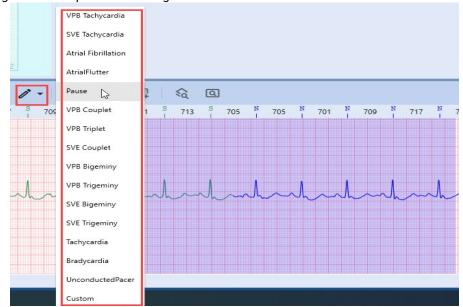

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 283 of 393

- 2. Hover over the **Create User Event** button or menu option to expand the drop-down list.
- 3. Click the relevant type of **Event** you wish to create in the selected area.

Analysis Controls

Analysis Controls is a group of buttons that enable you to exclude or re-include specific fragments of the ECG recording within the Strip View.

Icon	Description
	Exclude from analysis button: Enables the exclusion of a selected fragment of an ECG record from analysis.
目	Include to analysis button: Enables the re-inclusion of previously excluded fragments of an ECG record.

To exclude an ECG segment from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. (Optional) Scan the Strip View (on page 278) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 509. Strip View - Excluding from analysis

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 284 of 393



2. Click the **Exclude from analysis** button or select the same option from the context menu.

To re-include an ECG segment into analysis:

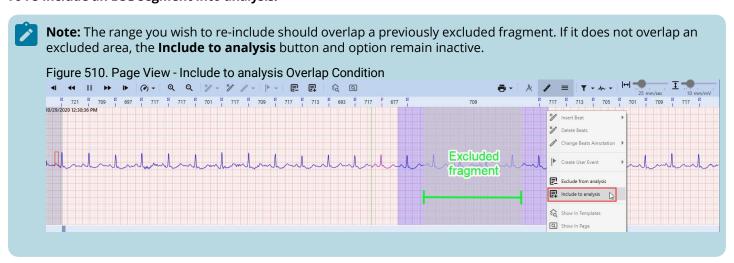

- 1. Select a fragment of the ECG, you want to re-include to analysis, using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in your desired range.
 - 1.5. (Optional) Scan the Strip View (on page 278) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - Using Ctrl + Click:
 - 1.1. Identify the initial point in the range you wish to select.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 285 of 393

- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 278) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 511. Strip View - Including to analysis



2. Click the **Include to analysis** button or select the option from the context menu.

Views Switching Controls

Views Switching Controls are a group of buttons that allow you to focus on a single beat you have selected in the **Strip View**, in the **Templates View** and **Page View**. This functionality enhances analysis quality and efficiency.

Icon	Description
€	Show In Templates button: Click this button to display the beat you have selected in the Strip View ; it will be highlighted in the relevant template within the Templates Pane .
Q	Show in Page : Facilitates an immediate switch to Page View , revealing the precise location of the chosen beat within this particular Template in both the Signal Page and the ECG Strip .

Figure 512. Strip View - Show In Templates


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	N©RAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 286 of 393



Printing Control

This button enables you to print a fragment of the ECG **Strip**. The printed copy will include a segment that fits the output format. The baseline of the printed version mirrors the center line of the Strip visible on your PC screen. The printed ECG Strip will accommodate as much ECG data as possible from the visible area around the center line.

Figure 513. Strip View - Printing Strip

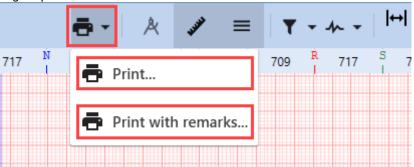


To print a fragment of the waveform in the **Strip View**:

- 1. Identify the area you want to print.
- 2. **(Optional)** Scan the Strip View (on page 278) to locate the desired area, if needed.
- 3. Hover over the **Printing Control** button to expand the drop-down list.
- 4. Click the **Print** option and navigate to the Print dialog box:


Figure 514. Strip View - Printing Dialog Box

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 287 of 393

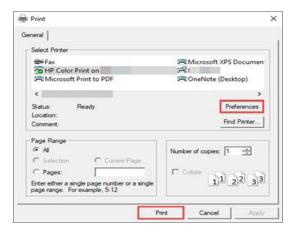

- 4.1. Choose a printer available in the Select Printer window.
- 4.2. Adjust other preferences according to your needs.
- 4.3. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.
- 5. (Optional) Click the Print with remarks option if you want to print a fragment of the waveform with your remarks:
 - 5.1. Complete the "Report remarks" field in the **Edit Report Remarks** pop-up.

Figure 515. Strip View - Printing with Remarks

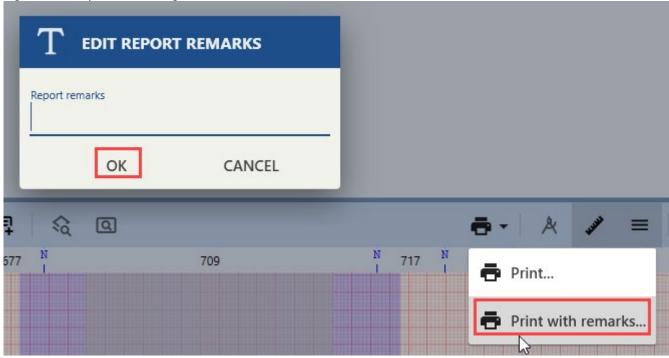

- 5.2. When you are complete with filling in you remarks, click OK.
- 5.3. (Optional) Click Cancel to abandon this action.
- 5.4. Choose a printer available in the **Select Printer** window.
- 5.5. Adjust other preferences according to your needs.
- 5.6. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Measuring Controls

Measuring Controls are a group of buttons designed to facilitate the indication and measurement of various waveform parameters within the **Strip View**. Namely, the most important tool in this group of controls is the **Caliper** tool. It allows you to measure different intervals and amplitudes on the ECG strip, which can be used to diagnose and assess a variety of heart conditions. We will explain how to use the Caliper for ECG measurements (on page 160) in the following sections.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 288 of 393

lcon			Des	cription	
Å	Caliper toggle : Click plitudes, such as the <i>(on page 160)</i> section	RR interval, T-wa	FF the Caliper tool. ave, and QRS comp	The Caliper tool assists with lex amplitudes. Refer to the	measuring intervals and an Measuring ECG with Caliper
ALIEN .	ECG Ruler toggle : Cl Ruler indicates the d etc.).	ck to toggle ON uration of RR int	OFF the ECG Rule ervals for adjacent	r , located right below the Str beats and their morphology	ip View Toolbar . The ECG classification (i.e., N, R, F, V,
	Channel Numeratio	n toggle: Click t	o toggle ON/OFF th	ne channel numeration on the	e left side of the ECG Strip.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	ocument Rev.: 04 Document Date: 07.08.2025	





Figure 517. Strip View - ECG Ruler



Figure 518. Strip View - Channel Numeration



Filters

This group of controls allows you to apply various filters affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.


Icons	Description
T -	Filter : This control enables you to toggle ON/OFF the visualization of the waveform with applied EMG, Base Line , and Mains filters. To activate or deactivate these filters, select or deselect any number of checkboxes from the drop-down list. These filters improve the accuracy and reliability of ECG analysis.
n -	Channel Selection: This control lets you select the channel(s) for display within the Strip View , using the dropdown list (up to 12 channels, depending on the number of channels in the current Record).

To toggle filters **ON/OFF**:

1. Hover over the **Filter** drop-down list to expand it.

Figure 519. Strip View - Toggling Filters

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 290 of 393

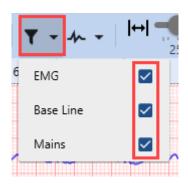

2. Select the filter you wish to toggle. The visualization of the waveform will change immediately.


It is recommended to keep filters **ON** to provide accurate analysis:

- EMG filter: Eliminates high-frequency ECG signal components.
- Baseline filter: Removes low-frequency ECG signal components.
- **Mains filter**: Eradicates 50 or 60 Hz power line interference. This interference can be caused by the electrical equipment in the environment.

To toggle the visualization of channels within the Strip:

1. Hover over the **Channel Selection** drop-down list to expand it.

Figure 520. Strip View - Toggling Channels

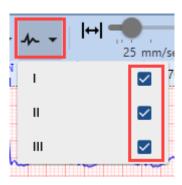

2. Select filters you wish to toggle. The visualization of the Strip will change immediately.

Toggling **ON/OFF** ECG channels enhances the accuracy and efficiency of ECG analysis. This feature allows you to focus on specific channels, compare them, filter out noise and more.

Scale and Gain Controls

This group of controls enables you to adjust the paper speed and amplitude, affecting beat visualization and enhancing the accuracy and reliability of ECG analysis.

Figure 521. Strip View - Scale and Gain Controls



To change the paper speed:

1. Drag the slider to your desired position, setting the paper speed within the range from 6.25 up to 100 mm/sec. The visualization within the Strip will change immediately.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 291 of 393


Figure 522. Strip View - Setting Paper Speed



To change the amplitude:

1. Drag the slider to your desired position, setting the amplitude within the range from 2.5 up to 40 mm/mV. The visualization within the Strip will change immediately.

Figure 523. Strip View - Setting Amplitude

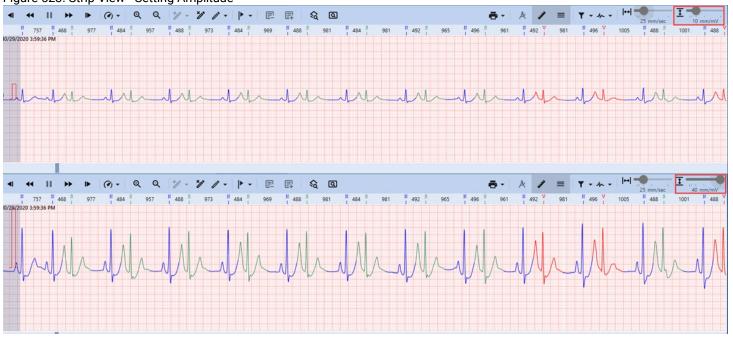


4.5.9.1.2. Context Menu Features

A **Context Menu**, accessible via right-click, enables you to quickly and easily access common actions, such as inserting or deleting beats, changing annotations, creating user events, etc.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 292 of 393

Figure 524. Strip View - Context Menu

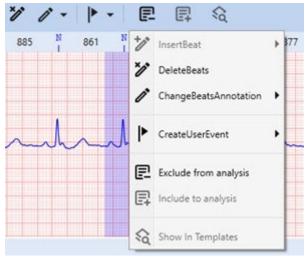

Insert Beat

Allows you to insert new beats between existing ones:



Note:

By default, when you click the **Strip View**, you select an entire beat. Inserting a beat is not equivalent to beat reclassification, so it requires an alternative method for selecting a position within the **Strip View**.

1. Select a position in the **Strip View** using one of these methods:


Figure 525. Strip View - Positioning Beat



- Hold Alt and click in the **Strip View**: A vertical dotted line will appear, marking the desired position. While holding Alt, click the right mouse button to expand the context menu.
- Press your mouse wheel in the **Strip View**: A vertical dotted line will appear, marking the desired position. Click again and hold the mouse wheel, and then press the right mouse button to expand the context menu.
- Press Alt+Right Click in the **Strip View**: A vertical dotted line will appear to mark the desired position. Alongside the dotted line, a context menu will expand, simplifying access to the **Insert Beat** option.
- 2. Hover over the Insert Beat option in the Context Menu to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 293 of 393

Figure 526. Strip View - Insert Beat

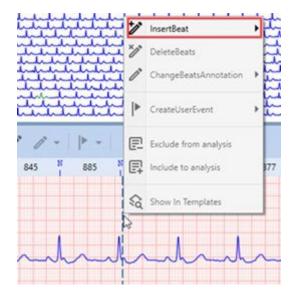

3. Click the appropriate type of morphology for the insertion. The NH-301 software will automatically determine which fragment of the waveform to use for this purpose. A specific segment of the ECG waveform in the **Strip View** will immediately change color.

Delete Beats

Allows you to delete a selected beat or a fragment of a waveform:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 294 of 393

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 527. Strip View - Delete Beats

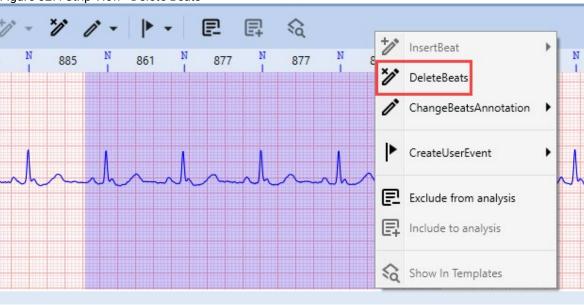

2. Click the **Delete Beats** option in the context menu. The system will automatically recalculate events and templates following the deletion.

Change Beats Annotation

Enables you to modify the annotation of the selected beat:

- 1. Select a fragment of the ECG, which you wish to delete, using one of the following methods:
 - · Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 295 of 393

- 1.4. Identify the final point in your desired range.
- 1.5. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. (Optional) Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Change Beats Annotation** option in the context menu to expand the drop-down list.

Figure 528. Strip View - Change Beats Annotation

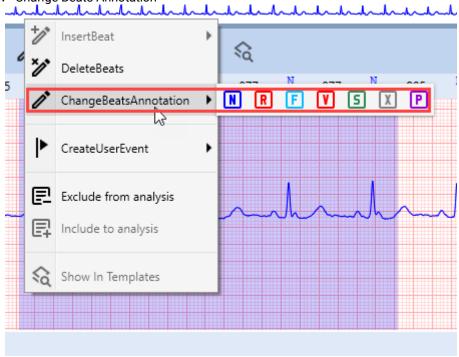

3. Click the appropriate type of morphology for the beats you wish to reclassify. Alternatively, you may utilize keyboard shortcuts as outlined in the table below. Reclassified beats in the **Strip View** will immediately change color.

Table 12. Beats Reclassification Keyboard Shortcuts for Strip View

Action	But- ton	Keyboard Keys
Classify as Normal	N	N
Classify as R on T	R	R
Classify as Fusion	F	F
Classify as VPB	V	V

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 296 of 393

Table 12. Beats Reclassification Keyboard Shortcuts for Strip View (continued)

Action	But- ton	Keyboard Keys
Classify as SVE	5	S
Classify as Questionable	X	Х
Classify Paced	P	Р

Create User Event

Allows you to create a new user event at the selected beat or ECG fragment. After adding a user event, the corresponding fragment of the ECG waveform will be recalculated, and some beats may shift between templates.

- 1. Select a fragment of the ECG you wish to reclassify as a continuous event, using one of the following methods:
 - Click an individual beat to select it. Right-click the selected fragment to expand the context menu.
 - Select a fragment of the waveform using one of these methods:

Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:


- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.
- 2. Hover over the **Create User Event** option to expand the drop-down list.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 297 of 393



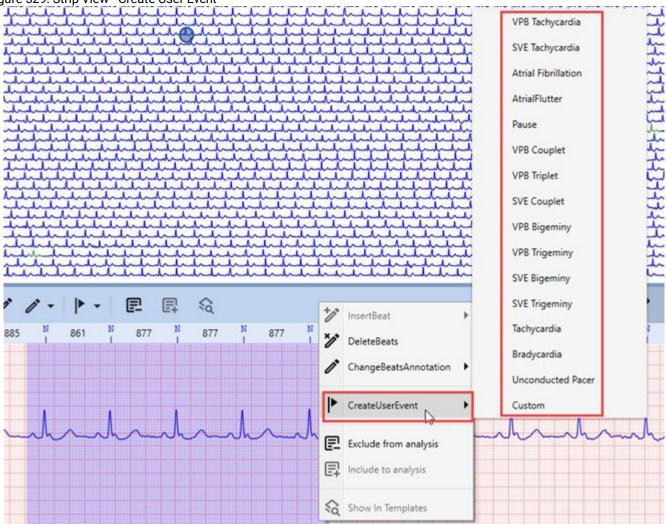

3. Click the relevant type of **Event** you wish to create in the selected area.

Exclude from analysis

Enables the exclusion of a selected fragment of an ECG record from analysis:

- 1. Select the ECG fragment you wish to exclude from analysis using one of the following methods:
 - Using Shift + Click:
 - 1.1. Identify the initial point in the range you wish to select.
 - 1.2. Click the initial point.
 - 1.3. Hold down the **Shift** key.
 - 1.4. Identify the final point in the range.
 - 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
 - 1.6. Click the final point to complete the range selection.
 - 1.7. Right-click the selected fragment to expand the context menu.

Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use			
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 298 of 393	

- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.


Figure 530. Strip View - Exclude from analysis

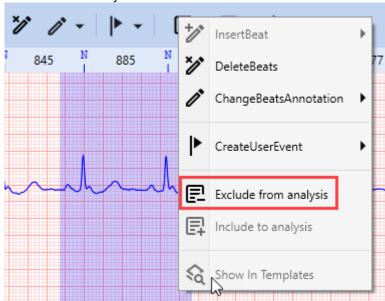

2. Click the **Exclude from Analysis** option in the context menu.

Include to analysis

Enables the re-inclusion of previously excluded fragments of an ECG record:

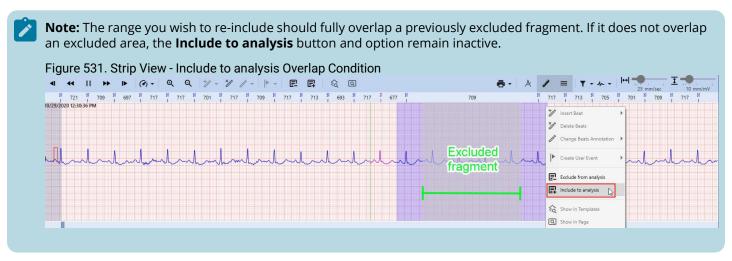

Document Title:	NH	NO RAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 299 of 393

1. Select a fragment of the ECG you want to re-include in analysis, using one of the following methods:

• Using Shift + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Click the initial point.
- 1.3. Hold down the **Shift** key.
- 1.4. Identify the final point in the range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Click the final point to complete the range selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using Ctrl + Click:

- 1.1. Identify the initial point in the range you wish to select.
- 1.2. Hold down the **Ctrl** key.
- 1.3. Click the initial point.
- 1.4. Identify the final point in your desired range.
- 1.5. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.6. Drag the cursor to the final point and release the mouse button to complete the selection.
- 1.7. Right-click the selected fragment to expand the context menu.

• Using the right mouse button:

- 1.1. Click the **Strip View** and hold the right mouse button to designate the initial point in the range you wish to select.
- 1.2. Drag the cursor to set the final point.
- 1.3. **(Optional)** Scan the Strip View (on page 138) to locate the points of interest, if necessary.
- 1.4. Release the right mouse button to complete the range selection. A context menu will appear.

Figure 532. Strip View - Include to analysis



2. Click the **Include to analysis** option in the context menu.

Show In Templates

Select this option from the context menu to display the beat you have selected in the **Strip View**; it will be highlighted in the relevant template within the **Templates Pane**.

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 300 of 393


Figure 533. Strip View - Show In Templates



4.5.9.1.3. Measuring ECG with Caliper

You may use the **Caliper** function to measure the RR interval, QRS complex duration and amplitude, QT interval, and other ECG waveform parameters. The **Caliper** also allows for editing the ECG record via the **Strip View**.



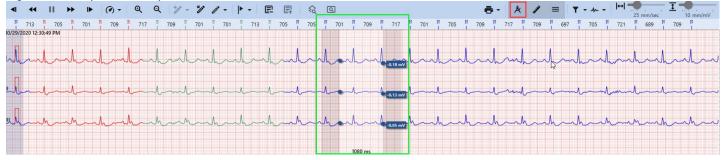

Caliper Design Overview

The **Caliper** tool in the **Strip View** displays two **measuring lines** and a **measuring interval** between them. To activate the **Caliper** tool, click the **Caliper** button in the **Strip View Toolbar**.

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 301 of 393

Figure 535. Strip View - Caliper Measurement Lines and Interval



Each ECG channel in the Strip has a small box adjacent to the point where the right measurement line intersects the channel signal line. The value in this box indicates the amplitude difference between the two points demarcated by the measurement lines.

Figure 536. Strip View - Interval Length and Amplitude Difference



The interval length in milliseconds is displayed at the bottom of the measuring interval.

Moving Caliper Lines

You may move Caliper measurement lines either **simultaneously** or **independently**.

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 302 of 393

To move both lines simultaneously:

1. Position the cursor within the measuring interval. The cursor will change to a resize cursor.

Figure 537. Strip View - Moving Caliper Lines Simultaneously



- 2. Drag and drop the **Caliper** to the desired location.
- 3. Optionally, use the Left and Right arrow keys to fine-tune the position.

To move lines independently:

1. Position the cursor near one of the measurement lines until it changes to a resize cursor.

Figure 538. Strip View - Moving Caliper Lines Independently



- 2. Click to select the measurement line.
- 3. Drag and drop it to the new location.

Zoom Feature

To adjust the display scale of the waveform in the Strip, utilize the **Zoom In/Zoom Out** options:

Figure 539. Strip View - Zooming Options

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 303 of 393



- Click the corresponding button in the **Strip View Toolbar**.
- Hold the Ctrl key while scrolling up (Zoom In) or down (Zoom Out) the mouse wheel.

Caliper Snapping Feature

The **Caliper Snapping** feature automatically aligns the **Caliper** measurement lines to the nearest R-peaks in the ECG signal. This functionality aids in the precise measurement of ECG wave duration and amplitude. Manual alignment can be challenging, hence **Caliper Snapping** enhances the accuracy, efficiency, and reproducibility of ECG interpretation.

To utilize the Caliper snapping feature:

- 1. Click the Caliper button in the Strip View Toolbar.
- 2. (Optional) Use the Left and Right arrow keys to make slight adjustments to the Caliper measurement lines.
- 3. To snap a **Caliper** measurement line to an R-spike:

Figure 540. Strip View - Caliper Snapping



- To snap any line:
 - 3.1. Hold down the **Alt** key.
 - 3.2. Position the cursor near one of the measurement lines until the cursor icon changes to a resize cursor.
 - 3.3. Click the measurement line to select it.
 - 3.4. Drag and drop it to the desired location.
- To snap the right measurement line:
 - 3.1. Hold down the **Right Alt** key.
 - 3.2. Use the Left and Right arrow keys to move the **Caliper**; the right measurement line will automatically snap to the R-peaks as you move.
- To snap the left measurement line:
 - 3.1. Hold down the **Left Alt** key.
 - 3.2. Use the Left and Right Arrow keys to move the Caliper; the left measurement line will automatically snap to the R-peaks as you move.

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 304 of 393

The **Caliper Snapping** function is instrumental in enabling a quick and accurate evaluation of various ECG parameters.

Editing Beats and Events Using Caliper

The **Caliper** tool enables you to edit specific fragments of the ECG waveform contained within the measurement lines of the **Caliper**. When you opt to delete or reclassify beats, or exclude a waveform fragment from analysis, the changes will be confined to the selected fragment, which may contain multiple beats. In essence, the **Caliper** allows you to edit a continuous section of an ECG record, demarcated by the measurement lines, in a single action.

Figure 541. Strip View - Editing Beats and Events Using Caliper



To edit a fragment of an ECG using the Caliper:

- 1. Position the **Caliper** and adjust its measurement lines as needed. For guidance, refer to the Caliper Design Overview *(on page 302)* section.
- 2. Right-click in the **Strip View** area to expand the context menu.
- 3. The **Caliper** tool's context menu offers several options within the **Strip View**:
 - **Delete Beats**: Click this option to remove all beats within the boundaries of the **Caliper's** measurement lines. This action will result in minor recalculations.



Note: Deleting beats may trigger the generation of **Pause** events. To annotate and exclude noisy segments, the **Exclude from analysis** option is preferable to **Delete Beats**.

- **Change Beats Annotation**: Hover over this option to display a drop-down list, then click the morphology type you wish to assign to the beats. The reclassified beats will instantly change color in the **Strip View**.
- **Create User Event**: Hover over this option to expand a drop-down list, and click the event type you wish to create in the selected area.
- **Exclude from analysis**: Click to exclude a noisy ECG segment within the **Caliper's** measurement lines from the analysis, without affecting other analytical outcomes.
- Include to analysis: Click to include previously excluded ECG fragments back into the analysis.

Document Title:	NH	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 305 of 393




Note: The **Include in Analysis** option becomes active in the **Caliper** context menu only when the current **Caliper** measurement interval overlaps with a segment that was excluded earlier.

Document Title:	NH	NORAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 306 of 393

4.5.10. Tabular Summary

The **Tabular Summary View** displays the **Hourly Tabular Report** table with all arrhythmias detected by the software during the analysis. This table is useful for predicting whether certain arrhythmias appeared only under certain circumstances. It also features the **Pacemaker Tabular Report**, representing the pacemaker-analysis hourly statistics (available only if pacemaker detection has been enabled in the recorder).

Figure 542. Tabular Summary View - Tabular Summary View

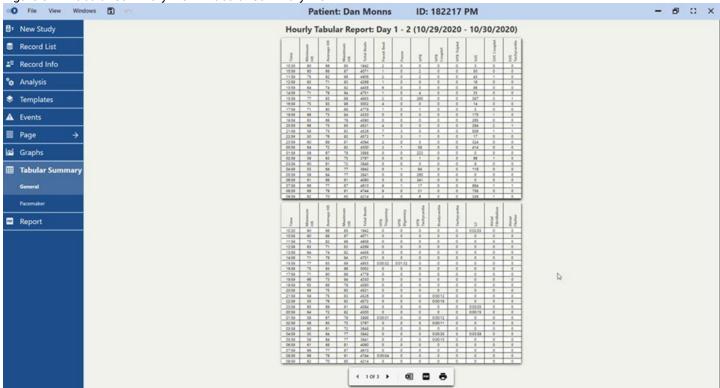

General Tab

The **General Tab** of the **Tabular Summary View** houses the **Hourly Tabular Report** table with all arrhythmias detected by the software during the analysis.

Figure 543. Tabular Summary View - General Tab


110	143. Tabl	ılar Sumr	ilary viev	v Gener	arrab								
	Hou	rely T	abul	ar Ro	nort	Day	1 _ 2	(10/	29/2	n 20 .	10/	30/20	1201
	1100	arry i	abui	ai ite	por t.	Day	1 - 2	(10)	23/2	020	10/.	30/20	120)
	Time	Minimum HR	Average HR	Maximum HR	Total Beats	Paced Beat	Pause	VPB	VPB Couplet	VPB Triplet	SVE	SVE Couplet	SVE Tachycardia
- 1	10:30	60	66	85	1942	2	0	0	0	0	3	0	0
- 1	10:59	60	68	87	4071	1	0	2	0	0	50	0	0
- 1	11:59	75	82	98	4908	2	0	2	0	0	43	1	0
	12:50	63	71	93	4250	- 1	0		0	9	10	0	0
-1	13:59	64	N 74	82	4458	6	0	3	0	0	98	0	0
ц	14.59	71	- 5 73	94	4751	-		4	•	•	- 23		
	15:59	77	83	89	4955	2	0	265	0	0	387	3	1
	16:59	75	83	98	5002	4	0	0	0	0	14	0	0
	17:59	71	80	88	4779	1	0	1	0	0	5	0	0
	18:59	66	73	94	4330	0	0	0	0	0	170	1	0
									_	-			



Note:

Click any line in the **Hourly Tabular Report**. You will be redirected to the **Page View** to the position of the event you have clicked on in the **Hourly Tabular Report**.

Document Title:	NH	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 307 of 393

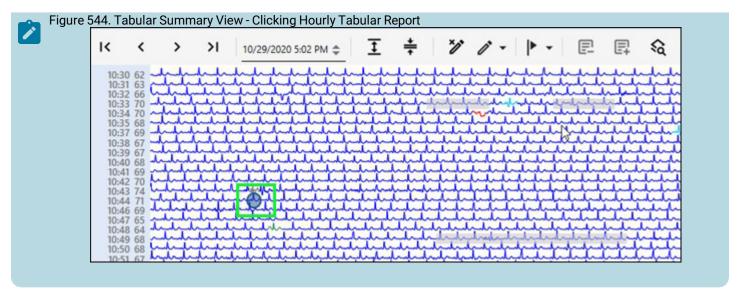


You can also perform the following actions in this **View**:

lcon	Description
POF	Navigation Controls : Click arrows to switch between pages.
ط	Export To Excel button : Click to export the table to Excel.
•	Export To PDF button : Click to export the table to PDF.
•	Print Table button: Click to print the table.

To print the **Hourly Tabular Report**:

- 1. Click the **Print Table** button and navigate to the Print dialog box.
- 2. Choose a printer **available** in the **Select Printer** window.
- 3. Adjust other preferences according to your needs.
- 4. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.

Figure 545. Tabular Summary View - Printing Dialog Box

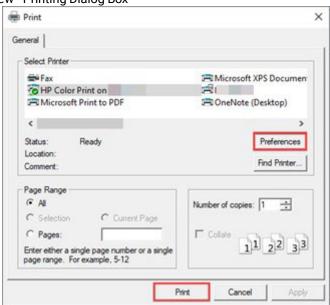


Document Title:	cument Title: NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 308 of 393		

Pacemaker Tab

The **Pacemaker Tabular Report** displays the pacemaker-analysis hourly statistics. This report becomes available only if the pacemaker detection has been enabled in the recorder.

Figure 546. Tabular Summary View - Pacemaker Tab

rigu	gure 546. Tabular Summary View - Pacemaker Tab												
Patient: Dan Monns								ID:	1822	17 PN	/ I		
	Pacemaker Tabular Report: Day							y 1 - 3 (10/29/2020 - 10/31/2020)				20)	
	Time	Total Beats	Paced Beat	Atrial Paced	Vent. Paced	AV Paced	% Paced	% Atrial Paced	% Vent. Paced	% AV Paced	Sense Failure	Capture Failure	Inhibitions
	10:30	1942	2	1	0	1	0.1	< 0.1	0	< 0.1	0	0	0
	10:50	4071	1	1	٥	٥	2.01	2.01	٥	2.01	0	0	0
	11:59	4908	2 ,	1	2	0	< 0.1	< 0.1	< 0.1	< 0.1	0	0	0
	12:59	4269	1	15 1	U	Ü	< 0.1	< 0.1	0	< 0.1	U	0	U
	13:59	4458	6	3	2	0	0.1	< 0.1	< 0.1	< 0.1	0	0	0
	14:59	4751	1	2	1	1	< 0.1	< 0.1	< 0.1	< 0.1	3	0	0
	15:59	4955	2	0	2	0	< 0.1	0	< 0.1	0	0	0	1
	16:59	5002	4	1	4	2	< 0.1	< 0.1	< 0.1	< 0.1	2	0	2

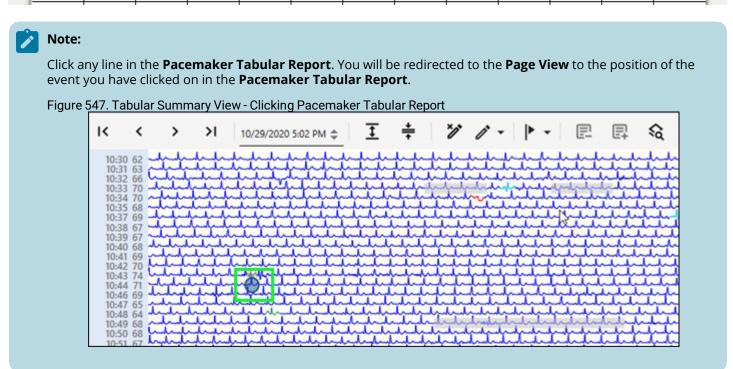

You can also perform the following actions in this **View**:


Icon	Description
POF	Navigation Controls : Click arrows to switch between pages.
Ø I	Export To Excel button : Click to export the table to Excel.
ē	Export To PDF button : Click to export the table to PDF.
ō	Print Table button : Click to print the table.

To print the **Pacemaker Tabular Report**:

Document Title:	ocument Title: NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 309 of 393		

- 1. Click the **Print Table** button and navigate to the Print dialog box.
- 2. Choose a printer available in the Select Printer window.
- 3. Adjust other preferences according to your needs.
- 4. Click **Print** at the bottom of the dialog box to execute the printing of the waveform fragment.


Figure 548. Tabular Summary View - Printing Dialog Box

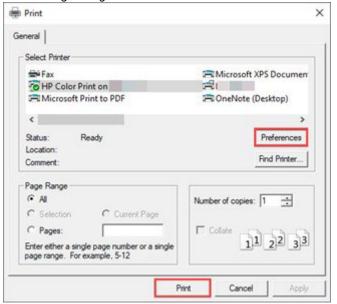

4.5.11. Report

The **Report** represents the final stage of the ECG analysis process. The NH-301 Holter software offers a full range of customization features that are both comprehensive and user-friendly, enabling users to tailor the **Report** to their individual requirements and preferences. Within the **Report View**, the final document can be personalized by toggling sections such as: **Summary** and **Conclusion**; **Events** while setting their order; **HRV**, **QT**, **QTc**, **ST**, **HR Trend**, or **RR Trend** Charts; it's also possible to include **Full Disclosure**, specifying start and end periods, selecting channels, and more. For clarity, the interface elements of this view are categorized into distinct groups.

Figure 549. Report View - Report View

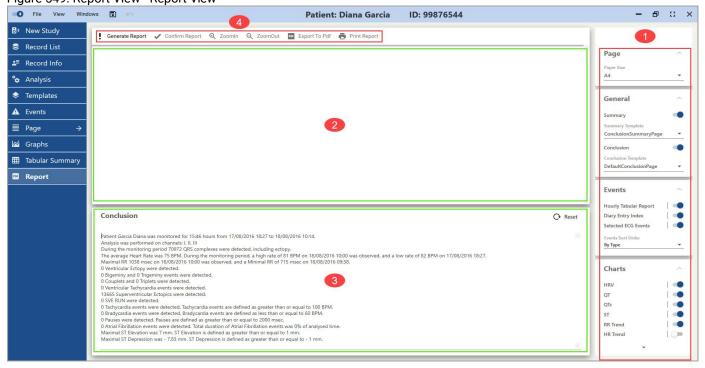

Document Title:	cument Title: NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 310 of 393		

- 1. Report Customization Boxes.
- 2. Summary Area
- 3. Conclusion Area
- 4. Report View Toolbar



Note: You can customize the **Report** with your institution's department name and address. To do that, please refer to the Setup Process after Installation (on page 13) section.

Report Customization Boxes

The **Report Customization Boxes** enable toggling ON or OFF entire sections of the **Report** in its final form.

To include a section in the **Report**, activate the toggle adjacent to the respective section. Sections can be included or excluded at any time. After modifying customization settings, a new **Report** generation is required to reflect the changes.


You can customize your **Report** through settings in the following boxes:

- 4.1. Page
- 4.2. General
- 4.3. Events
- 4.4. Charts
- 4.5. Full Disclosure

Page

Click the **Paper Size** drop-down list to select the **Report** format from the available options.


Figure 550. Report View - Page

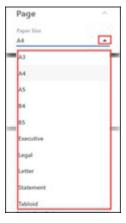


General

Click the **Summary** toggle to turn the section ON/OFF. This is the first section of the **Report** (if the section is ON), which provides a formatted table with key ECG analysis data.


Figure 551. Report View - Summary

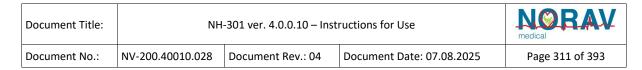

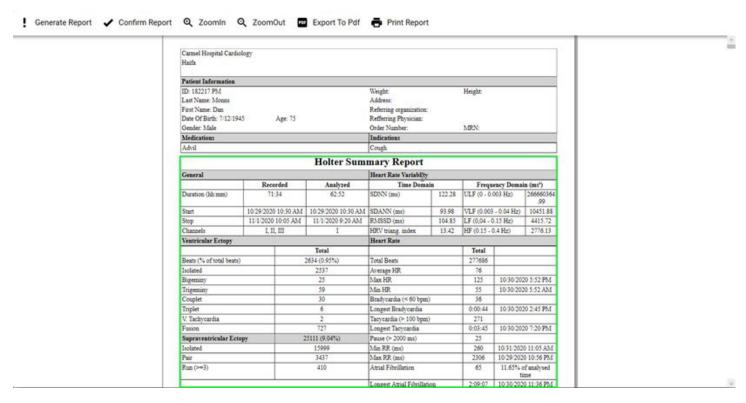

Click the **Summary Template** drop-down list to choose a summary template option from available options.


Figure 552. Report View - Summary Template

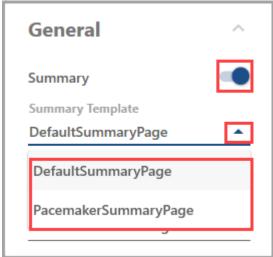

Click the **Conclusion** toggle to turn the section ON/OFF. This is the section of the **Report** (if the section is ON), which provides a default conclusion based on key ECG analysis data.


Figure 553. Report View - Conclusion

Document Title:	NH	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 312 of 393

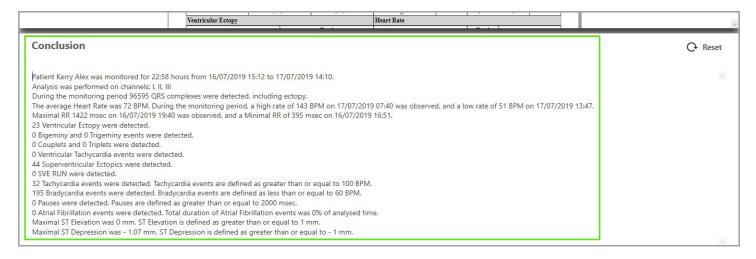

Click the **Conclusion Template** drop-down list to choose a conclusion template option from the available options.

Figure 554. Report View - Conclusion Template

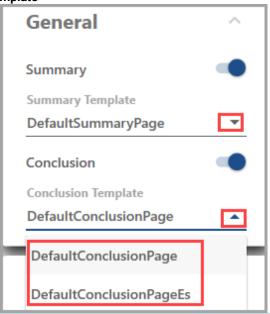




Note: If you turn OFF the **Summary**, when you generate a report, the **Conclusion** section will be displayed in the top part of the screen, in the **Summary** area.

Additional templates can be added to the relevant summary templates folder or conclusion templates folder. The path to this folder can be adjusted via the **File** drop-down menu, at **File>Settings>Reports**. You can contact our Support Team to obtain your own custom templates.

Figure 555. Report View - Templates Settings

Document	Title:	NH	NORAV medical		
Document	No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 313 of 393

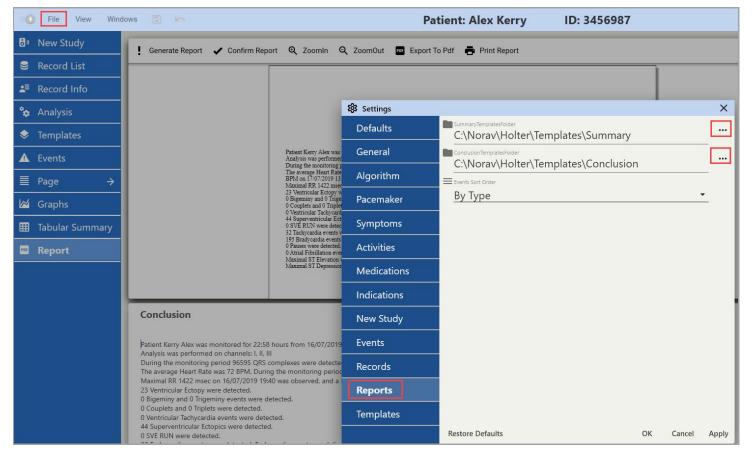

Events

Click relevant toggles ON/OFF to include or exclude the following subsections:

4.1. **Hourly Tabular Report**: This formatted table includes arrhythmia events identified during analysis, detailing a range of parameters related to event occurrences. This section is similar to the **Tabular Summary View** but with non-interactive table entries.

Figure 556. Report View - Hourly Tabular Report

	Hourly Tabular Report: Day 1 - 2 (10/29/2020 - 10/30/2020)											
Time	Minimum HR	Average HR	Maximum HR	Total Beats	Paced Beat	Pause	VPB	VPB Couplet	VPB Triplet	SVE	SVE Couplet	SVE Tachycardia
10:30	60	66	85	1942	2	0	0	0	0	3	0	0
10:59	60	68	87	4071	1	0	2	0	0	50	0	0
11:59	75	82	98	4908	2	0	2	0	0	43	1	0
12:59	63	71	83	4269	1	0	0	0	0	18	0	0
13:59	64	74	82	4458	6	0	3	0	0	98	0	0
14:59	71	79	94	4751	1	0	4	0	0	33	0	0
15:59	77	83	89	4955	2	0	265	0	0	387	3	1
16:59	75	83	98	5002	4	0	0	0	0	14	0	0
17.50	71	0.0	0.0	4770	1	0	1	0	0	5	0	0

Document Title:	Oocument Title: NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 314 of 393		

4.2. **Pacemaker Tabular Report**: This section presents a formatted table of pacemaker events detected during analysis, similarly detailing event occurrences. Like in the **Hourly Tabular Report**, entries here are not interactive.

Figure 557. Report View - Pacemaker Tabular Report

Pacemaker Tabular Report: Day 1 - 3 (10/29/2020 - 10/31/2020)												
Time	Total Beats	Paced Beat	APacedBeat	VPacedBeat	AVPacedBeat	PacedBeatPerc ent	APacedBeatPe	VPacedBeatPercent	AVPacedBeatP ercent	Failure To Sens e	FailureToCapt ure	FailureToPace
10:30	1942	2	1	0	1	0.1	< 0.1	0	< 0.1	0	0	0
10:59	4071	1	1	0	0	< 0.1	< 0.1	0	< 0.1	0	0	0
11:59	4908	2	1	2	0	< 0.1	< 0.1	< 0.1	< 0.1	0	0	0
12:59	4269	1	1	0	0	< 0.1	< 0.1	0	< 0.1	0	0	0
13-59	4458	6	3	2	0	0.1	< 0.1	< 0.1	< 0.1	0	0	0



4.3. **Diary Entry Index**: Lists all diary entries made during the ECG recording.

Figure 559. Report View - Diary Entry Index


Diary Entry Index									
	Date	Time	Symptom	Activity					
1	7/4/2016	12:58 PM	Heart racing						
2	7/4/2016	12:58 PM	Back pain						
3	7/4/2016	12:58 PM	Back pain						

4.4. **Selected ECG Events** Displays events marked (selected) for inclusion in the final **Report** while reviewing Events in the Events View *(on page 174)*.

Figure 560. Report View - Selected ECG Events


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 315 of 393			

Selected Events

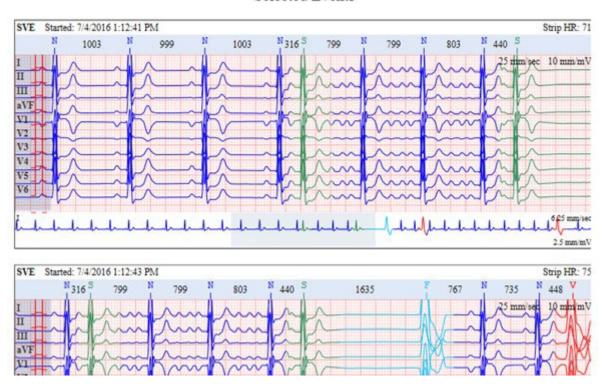

Click the **Events Sort Order** drop-down list to choose a sorting option from available options.

Figure 561. Report View - Events Sort Order

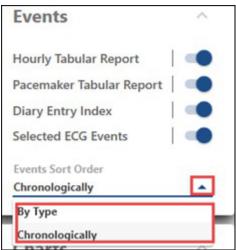


Charts

The **Charts** box allows the inclusion of **HRV** and **ST**, **QT** and **QTc**, and either the **RR Trend** or **HR Trend** graphs in the **Report**.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 316 of 393


Figure 562. Report View Charts Box

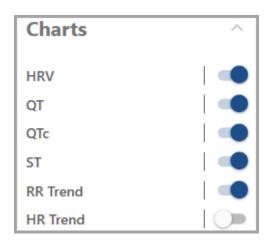



Note: Only one Trend (HR or RR) can be included at a time; selecting one will automatically deselect the other.

Figure 563. Report View - Charts

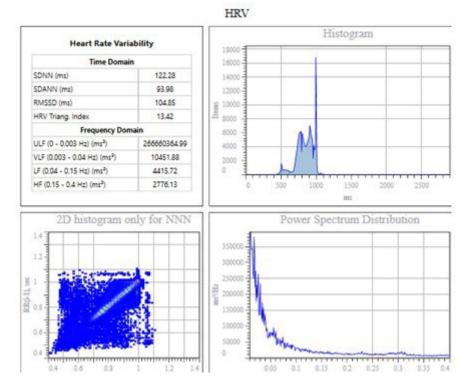


Full Disclosure

Click the **Full Disclosure** toggle to turn the **Full Disclosure** section ON/OFF. The **Full Disclosure** box also allows you to fine-tune its parameters for the Report.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 317 of 393


Figure 564. Report View - Full Disclosure

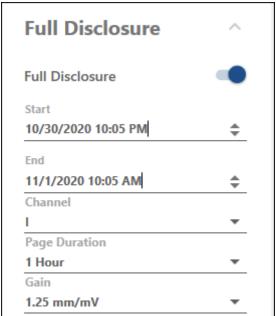

To adjust the **Full Disclosure** section configuration for the final Report:

4.1. **Start and End options**: Click the picker arrows on the right to adjust the timing and length of the **Full Disclosure** segment to include in the **Report**.

Figure 565. Report View - Start and End options

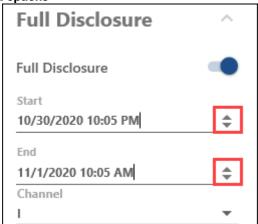

4.2. **Channel** Click the **Channel** drop-down list to choose which channel to display, with only one channel shown at a time in the **Full Disclosure** section.

Figure 566. Report View - Channel



4.3. **Page Duration** Click the **Page Duration** drop-down list to select one of the available options. This parameter will define the length of the **Full Disclosure** fragment to include in the **Report**.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 318 of 393

Figure 567. Report View - Page Duration



4.4. **Gain** Click the **Gain** drop-down list to select the gain scale for the **Full Disclosure** from the available options.


Figure 568. Report View - Gain

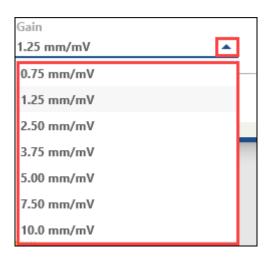

Summary Area

Review the generated **Report** using the scroll function or scroll bar, and zoom in/out as needed. The **Report View Toolbar** provides tools for report review and confirmation.


Figure 569. Report View - Report Area

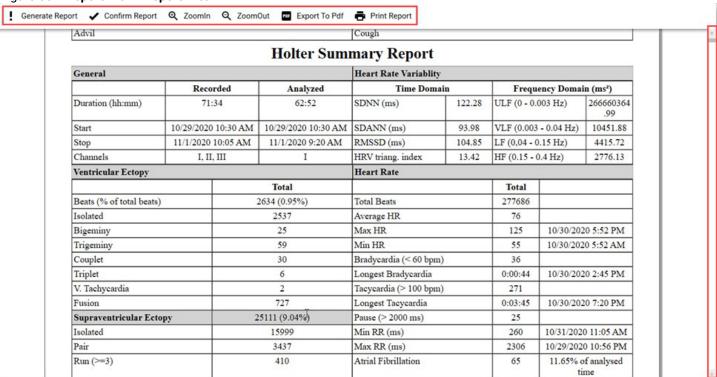

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 319 of 393

Summary Area displays a detailed **Holter Summary Report** once it's generated. The report in the **Summary Area** includes all the essential data obtained from the ECG record during the **Analysis**, including sections turned ON using **Report Customization Boxes** (on page 309).


Figure 570. Report View - Report Customization Boxes

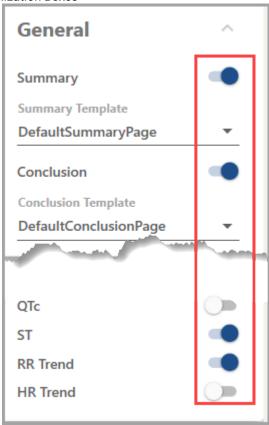

To review and confirm the **Report** using **Summary Area**:

- 4.1. Customize the report layout using the **Report Customization Boxes** (on page 309). Switch relevant toggles and adjust other parameters according to your needs.
- 4.2. Click on the **Generate Report** button. Wait until the report is generated and appears on the screen.

Figure 571. Report View - Report Generation and Confirmation

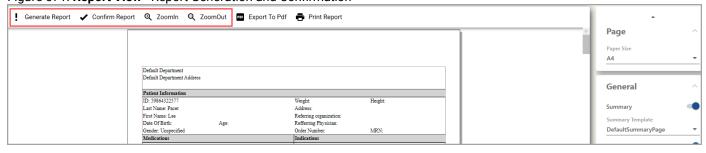

- 4.3. Review the generated **Report** using the scroll function or scroll bar, and **Zoom In/Zoom Out** buttons in the **Report View Toolbar** as needed.
- 4.4. If you have no more changes to make to the report after the review, click **Confirm Report** in the **Report View Toolbar**.




Note: You can adjust the **Summary** and **Conclusion** sections size proportionally by dragging and dropping the splitter bar between them.:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 320 of 393



- 4.4.1. To resize the section, move the cursor over the splitter bar until it changes to a vertical resize cursor.
- 4.4.2. Click the splitter bar and drag-and-drop it to the desired position. You can readjust the position at any moment.

Figure 572. Report View - Report

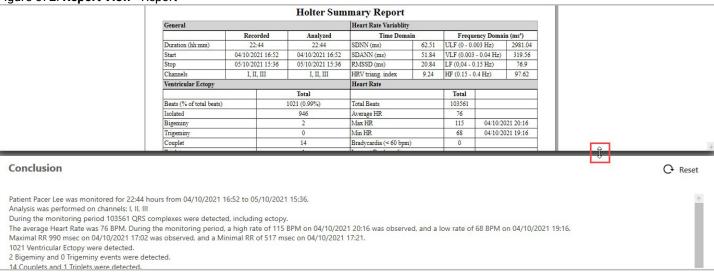

Conclusion Area

The **Conclusion Area** is designed to reflect the key findings of an ECG test. The **Conclusion** is an automatically generated list of key observations extracted from the ECG analysis and displayed in this area. It is typically reviewed and augmented by a qualified healthcare professional. It encapsulates the ECG test data, including patient history, test indications, results, and any diagnoses or recommendations. You may confirm the **Report** once you finish reviewing the **Conclusion**.

Figure 573. Report View - Conclusion




The **Conclusion** is autogenerated along the standard ECG analysis workflow supported by the NH-301 software:

- 4.1. Load a Record: Import or load the ECG record into the system via Record List View (on page 71).
- 4.2. **Analyze**: Perform the analysis on the loaded record using the **Analysis View** (on page 83).
- 4.3. **Go to the Report View**: Navigate to the **Report View**. When you click on the **Report** on the left, the **Conclusion** using the ECG parameters acquired during the analysis is automatically generated and displayed in the **Conclusion Area**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 321 of 393

Figure 574. Report View - Report Workflow

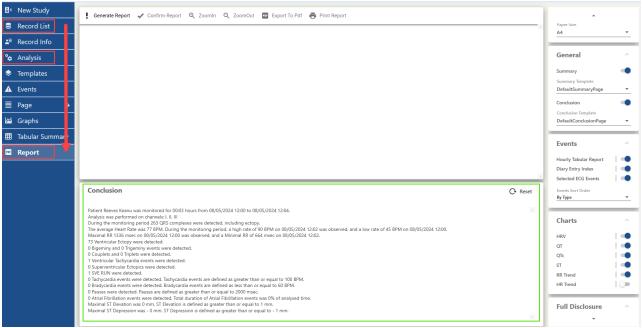


If you have saved the generated **Conclusion** and then navigate to the **Templates View** or **Events View** to make changes (like editing templates or adding events), you will see a pop-up when you return to the Report View. This pop-up will prompt you to decide whether to save or discard the changes made to the **Conclusion**, which has been recalculated because of the modifications in**Templates** and **Events**.

Here you can do the following:

- Save Changes: Click Yes to save the changes after the Conclusion was recalculated.
- Discard Changes: Click No to discard the changes.

Figure 575. Report View - Update Conclusion

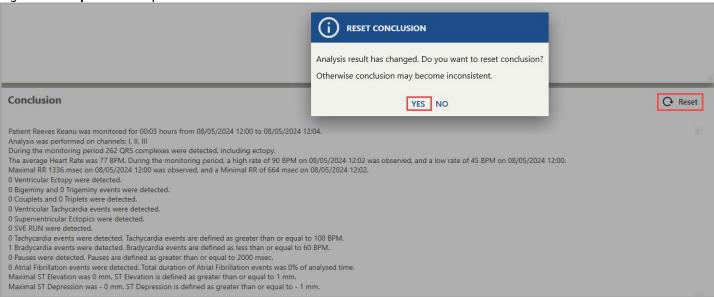


You can also modify the **Conclusion** text manually by following these steps:

- 4.1. Click at any desired point within the **Conclusion Area** to place a cursor.
- 4.2. Type in the text you need.
- 4.3. (Optional) Save the **Record** by clicking the **Save** icon in the **Top Toolbar**.

Figure 576. Report View - Edit and Revert Conclusion

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 322 of 393

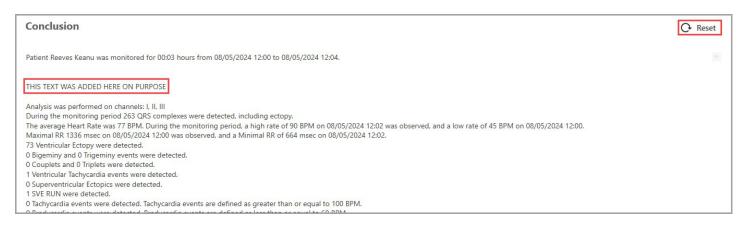

To undo (discard) the manual alterations made in the **Conclusion Area**:

4.1. Click the **Reset** button in the upper-right corner of the **Conclusion Area**. The contents of the **Conclusion** will revert to the last saved state where the **Conclusion** was recalculated.



Note: Changes in content resulting from recalculating the **Conclusion** due to modifications in **Templates** and **Events** cannot be undone using this feature. This feature only affects the information you have personally entered into the **Conclusion**, such as additional notes or observations that were typed in.



Note: Both functions—conclusion recalculation due to changes in analysis outcomes (such as editing templates or events) and manual changes reset—are "saving insensitive." You can edit templates or change the text of the **Conclusion**, then save the record, unload it, load it again, return to the **Report View**, and discard the changes you made before the last record saving.

Report View Toolbar

The **Report View Toolbar** facilitates report generation, confirmation, zooming, exporting to PDF, and printing.

Figure 577. Report View - Report View Toolbar

Generate Report ✔ Confirm Report ④ ZoomIn Q ZoomOut Export To Pdf 🖶 Print Report

Icon	Description
■ Generate Report	Generate Report button: Click to start the report generation process. The report will include all sections toggled ON in the Report Customization Boxes . Once the generation is complete, you'll receive a preview of the Report to review before finalizing your conclusion.
✓ Confirm Report	Confirm Report button: Click this to confirm and save the Report . It's advisable to do this only after reviewing the generated Report and submitting your conclusion.
⊙ Zoomln	Zoom In button: Click to zoom in on specific parts of the Report in the Report Area .
Q ZoomOut	Zoom Out button: Click to zoom out in the Report Area . Use this button to zoom out if you had zoomed in earlier.
Export To Pdf	Export To PDF button : This allows you to export and save the Report as a PDF. In the dialog box, click Save in the bottom-right corner to start the export. An 'Exporting' progress bar will indicate the process and disappear once the export is complete.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 323 of 393

Print Report	Print Report button : Click to open the Print dialog box for printing the Report .
	Within the Print dialog box:
	4.1. Choose a printer available in the Select Printer window.
	4.2. Adjust other preferences according to your needs.
	4.3. Click Print at the bottom of the dialog box to print the Report .

4.5.11.1. Report Confirmation Workflow

Confirmation is the final step in the ECG analysis process. All prior steps must be completed before confirming the **Report**.

To confirm the report:

- 1. Customize the report using the **Report Customization Boxes** in the **Report View**.
- 2. Click the **Generate Report** button. Once the report generation is complete, a report preview is provided.
- 3. Review the generated report using the **Report View Toolbar** and **Report Area** capabilities.
- 4. If necessary, fill in the **Conclusion** section.
- 5. Click the **Confirm Report** button to finalize the report.

Once the report is confirmed, its status in the **Record List View** changes to 'Confirmed', and you can view or email the PDF directly from this view.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 324 of 393

5. Preparing for Holter Recording

In this chapter, you'll find an overview of how to prepare and perform a **Holter Recording** after you have completed initial software installation and setup. For more detailed information about the setup and operation of the particular **Recorder** model, refer to the relevant operating manual.

To set up a **Holter Recording** process, follow the appropriate steps listed below. The exact order of the steps may vary depending on the **Holter Recorder** model and the type of connection to the computer where the NH-301 analysis system is installed:

- 1. Prepare the patient.
- 2. Connect the **Recorder** or the recorder memory card to the computer where the NH-301 analysis system is installed.
- 3. Enter the patient data.
- 4. Adjust the recording duration limit and other parameters.
- 5. Upload the patient data and the recording parameters to the **Recorder**.
- 6. Initiate the Recorder.
- 7. Verify or modify the data in the **Recorder**.
- 8. Start recording.

5.1. Preparing Patient

The signal quality of a Holter recording depends on the quality of electrodes in use and on the quality of skin preparation procedures. Ensure that the electrodes you use did not exceed their shelf life. Quality issues with patient's skin preparation and electrode condition, as well as incorrect electrode positioning may cause extra ECG artifacts and noise.

5.1.1. Preparing Skin

Follow these basic instructions to ensure correct electrode placement and high-quality recording outcomes:



Warning:

Do not clean the skin with an electrode spray, which is commonly used for exercise tests. Sprays of this type are not suitable for disposable electrodes.

- 1. Locate the correct electrode locations according to the illustrations in the Placing Electrodes (on page 323) section or as described in the relevant Recorder operating manual.
- 2. If necessary, clip hair at the electrode sites (or shave sites, if needed).
- 3. Thoroughly scrub each electrode area to remove dead skin, oil, and dirt. For proper cleaning, use an abrasive skin cleaner paste consisting of soap and pumice.
- 4. Cleanse the scrubbed area with physician-approved alcohol.
- 5. Dry the electrode placement sites.

5.1.2. Placing Electrodes

Incorrect electrode positioning may cause extra ECG artifacts and noise, potentially affecting analysis workflows and outcomes. All electrodes should be of the same brand and type, to minimize noise. Suggested electrode placement schemes are displayed in the following subsections. However, making the final placement decisions is up to the physician in charge.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 325 of 393




Note: You can use NR devices ECG display screen (on page 358) or NH-301 Holter software system capabilities (on page 59) related to device models equipped with Bluetooth to verify a proper patient hookup. At the same time, do not use these verification capabilities as diagnostic tools.

General instructions:

- 1. Use only high-quality disposable electrodes, designed for use with **Holter Recorders**. Wet gel electrodes are recommended. Refer to the **ANSI/AAMI EC12:2000 Standard** for safety, performance, and labeling requirements for the disposable electrodes, and guidelines for reliable patient connections.
- 2. Connect the **Recorder** input lead snap-on contacts with the electrode press-on studs. When you first place the electrode on the patient's chest, the electrode gel may be squeezed out, which can lead to poor electrode impedance.
- 3. Place the electrodes on the correct and properly cleansed positions.
- 4. If you use lead lock or clip lock electrodes, be sure to use the lock or clip to relieve stress on each lead wire. Otherwise, tape each lead wire into a stress loop to help prevent movement of the electrode. This ensures that body movements do not cause additional artifacts.

Figure 578. Placing Electrodes - Stress Loop Forming

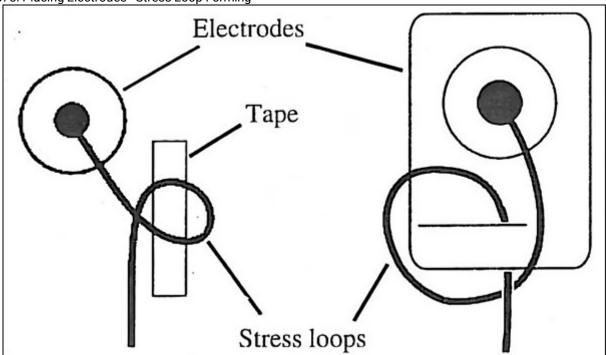



Note:

The **Norav Holter Recorders** are compatible with ECG cables of varying lead counts. The NH-301 Holter analysis system is designed to automatically recognize the number of channels being recorded.


5.1.2.1. Recommended Electrode Placement: 3 Channels, 7-Lead ECG Cable

Channel	Color	Placement
Ch 1 –	White	Right border of manubrium of the sternum
Ch 1 +	Red	Left anterior axillary line on the 6th costal arch
Ch 2 –	Black	Left border of manubrium of the sternum
Ch 2 +	Brown	Approximately 1 inch left of the xiphoid process

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 326 of 393

Ch 3 – Blue Right midclavicular lin	e on the 7th costal arch
-------------------------------------	--------------------------


Ch 3 +	Orange	Left midclavicular line on the 7th costal arch
GND	Green	Lower right costal arch

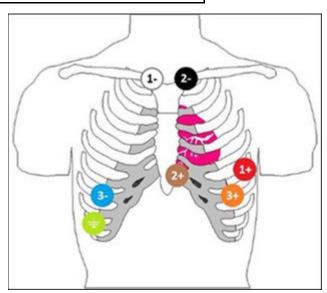

5.1.2.2. Alternative Electrode Placement: 3 Channels, 7-Lead ECG Cable

Channel	Color	Placement
Ch 1 –	White	Right midclavicular
Ch 1 +	Red	Left anterior axillary line on the 5th costal arch
Ch 2 –	Black	Left midclavicular
Ch 2 +	Brown	Approximately 1 inch right of the xiphoid process
Ch 3 –	Blue	Center of manubrium of the sternum
Ch 3+	Orange	Left midclavicular line on the 5th costal arch
GND	Green	Lower right costal arch

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028 Document Rev.: 04		Document Date: 07.08.2025	Page 327 of 393

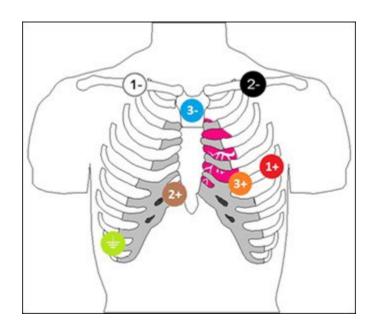


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 328 of 393

Channel	Electrode Colors	Standard 12- Lead Equivalent
Ch 1	White-Red	Modified V5
Ch 2	Black–Brown	Modified V1
Ch 3	Blue-Orange	Lead III

5.1.2.3. Recommended Electrode Placement: 3 Channels, 5-Lead ECG Cable

Channel	Color	Placement
Ch 1 –	Red	Center of manubrium of the sternum
Ch 2 -		
Ch 1 +	Brown	Approximately 2 inches left of the xiphoid process
Ch 2 +	Black	Left anterior axillary line on the 8th costal arch
Ch 3+		
Ch 3 –	White	Right midclavicular line on the 7th costal arch
GND	Green	Lower right costal arch

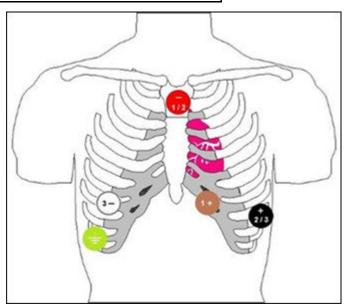


5.1.2.4. Alternative Electrode Placement: 3 Channels, 5-Lead ECG Cable

Channel	Color	Placement
Ch 1 – Ch 2 –	Red	Center of manubrium of the sternum
Ch 1 +	Brown	Left anterior axillary line on the 5th costal arch

Document Title:	NH	-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 329 of 393

Ch 2 + Ch 3 +	Black	Left midclavicular line on the 5th costal arch
Ch 3 –	White	Right midclavicular

Channel	Color	Placement
GND	Green	Lower right costal arch

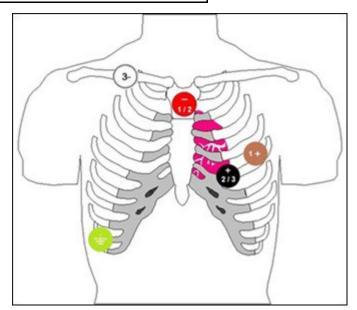


Channel	Electrode Colors	Standard 12-Lead Equiva- lent
Ch 1	Red-Brown	CM5
Ch 2	Red-Black	aFV
Ch 3	White-Black	Lead III

5.1.2.5. Recommended Electrode Placement: 3 Channels, 3-Lead ECG Cable


Channel	Color	Placement
Ch 1 –	White	Right midclavicular
Ch 2 –		
Ch 1 +	Red	Left anterior axillary line on the 6th costal arch
Ch 2 + Ch 3 -	Black	Left midclavicular

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use					
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 330 of 393			

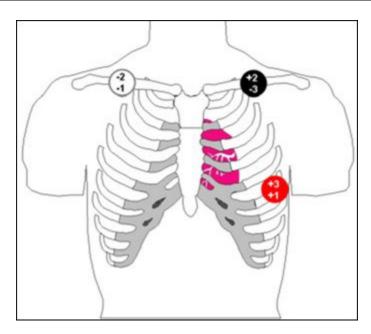

5.1.2.6. Alternative Electrode Placement: 3 Channels, 3-Lead ECG Cable

Channel	Electrode Colors	Standard 12-Lead Equiva- lent
Ch 1	White-Red	Lead II
Ch 2	Red-Black	Lead I
Ch 3	White-Black	Lead III

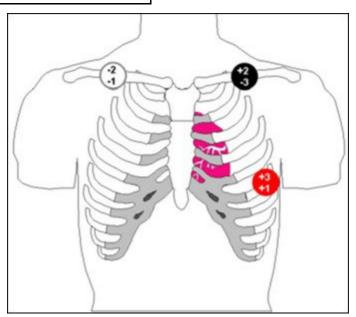


5.1.2.7. Recommended Electrode Placement: 12 Channels, 10-Lead ECG Cable

#	AHA Color	AHA Lead	IEC Color	IEC Lead	Placement
1	Red	V1	Red	C1	Fourth intercostal space at the right border of the sternum
2	Yellow	V2	Yellow	C2	Fourth intercostal space at the left border of the sternum
3	Green	V3	Green	C3	Midway between location V2 and V4

Document Title:	NH	NO RAV
Document No.:	NV-200.40010.028	Page 331 of 393

4	Blue	V4	Brown	C4	At the midclavicular line in the fifth intercostal space
---	------	----	-------	----	----------------------------------------------------------


#	AHA Color	AHA Lead	IEC Color	IEC Lead	Placement
5	Orange	V5	Black	C5	At the anterior axillary line on the same horizontal level as V4
6	Violet	V6	Violet	C6	At the midaxillary line on the same horizontal level as V4 and V5
7	Black	LA	Yellow	L	Left shoulder
8	Red	LL	Green	F	Lower edge of the rib cage, or at the lev- el of the umbilicus at the midclavicular line
9	Green	RL	Black	N	Lower edge of the rib cage, or at the lev- el of the umbilicus at the midclavicular line
10	White	RA	Red	R	Right shoulder

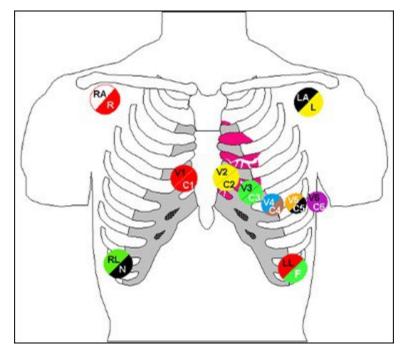

5.1.2.8. Recommended Electrode Placement: 2 Channels, 5-Lead ECG Cable

Channel	Color	Placement
Ch 1 –	White	Right border of manubrium of the sternum
Ch 1 +	Red	Left anterior axillary line on the 6th costal arch
Ch 2 –	Black	Left border of manubrium of the sternum
Ch 2 +	Brown	Approximately 1 inch left of the xiphoid process
GND	Green	Lower right costal arch

Document Title:	NH	NO RAV		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 332 of 393

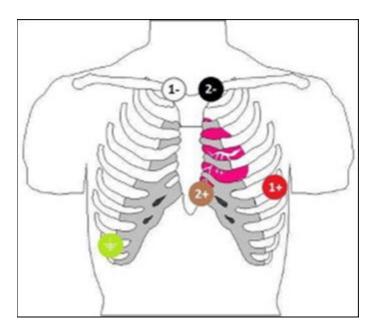

5.2. Preparing Recorder

After you have finished preparing the patient, it's time to set up your **Recorder** device for a new Holter recording.

To prepare a **Recorder** for a new Holter recording, follow these steps:

- 1. **Connecting the Recorder**: Connect the **Recorder** to the computer where the NH-301 analysis system is installed. You have three options:
 - 1. Connection via USB:
 - 1.1. Confirm that a **Memory Card** is installed in the recorder.
 - 1.2. Disconnect the ECG cable from the **Recorder** and connect a USB cable instead.
 - 1.3. Plug the USB cable into a USB port on the computer.
 - 2. Connection via Memory Card Reader:
 - 2.1. Eject the **Memory Card** from the **Recorder**.
 - 2.2. Place the **Memory Card** into the **Card Reader**.
 - 2.3. Connect the **Card Reader** to a USB port on the computer.
 - 3. Connection via Bluetooth:
 - 3.1. To connect a **Recorder** using Bluetooth pairing, please refer to the Pairing Norav Devices via Bluetooth (on page 333) section for detailed instructions, depending on the device model.
- 2. **Verifying Recorder Memory**: Before initiating a new recording, ensure the **Recorder** memory does not contain a previous recording. If it does, you'll need to download all records and clear the memory:
 - 1. Open Windows Explorer by clicking its icon on the Windows Taskbar.

Figure 579. Verifying Memory - Opening Windows Explorer




- 2. Click on the relevant drive that represents your **Recorder** memory.
- 3. Check for .nrr or .res files. If these files exist, proceed to **Step 3** below to download and clear them; otherwise proceed to the next section (on page 346).
- 3. (Optional) Downloading and Clearing Previous Records:

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 333 of 393


1. Download the record:

Figure 580. Preparing Recorder - Downloading Records

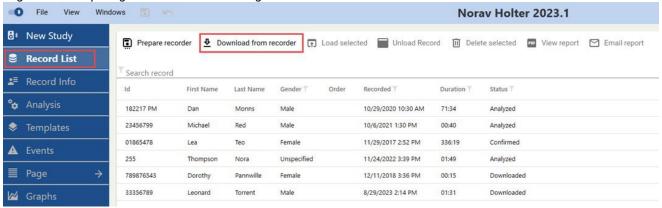

- 1.1. Click the **Record List View** in the **Views Sidebar** on the left.
- 1.2. Click the **Download from recorder** button on the top toolbar. A pop-up window will appear.
- 1.3. Select and double-click **one** record to download. Alternatively, click the **Download** button in the bottom right corner. If multiple records are recognized (as multiple recorders can be connected to a PC), a list of records will be displayed for selection. Note that you can select **only one record** at a time.

Figure 581. Preparing Recorder - Download Record

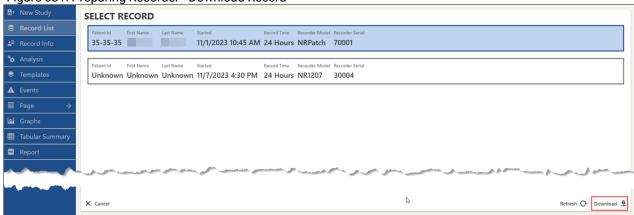


If you have connected additional **Recorders** or disconnected some of those previously plugged in, click **Refresh** in the bottom right corner to update the accessible records list.

- 1.4. The **Record Infromation**window will appear. Validate and modify the patient's information if needed; refer to Step 3 in the Menu Bar section (on page 43) on how to modify the patient's data.
- 1.5. Click **Proceed** to finish downloading the record. A "Downloading..." progress bar will appear. If only one record was recognized, the download is finished and the **Record List** will be displayed.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 334 of 393

Figure 582. Preparing Recorder - Click Proceed

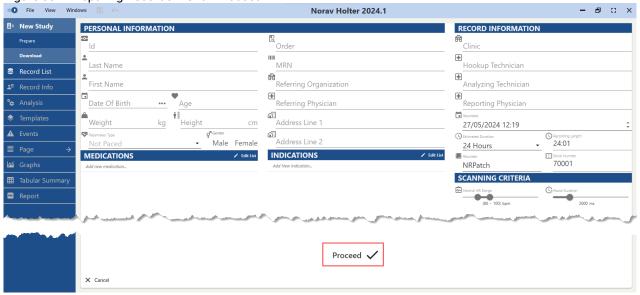

You can also:

- Click **Cancel** in the bottom left corner to skip all changes and get back to the **Record List View** immediately.
- Click **Record List View** to navigate back to the **Select Record**screen.
- 1.6. **(Optional)** If multiple records were recognized in **Substep III**, a list of records will be displayed again for selection. You can choose another record from the list if needed, and repeat the procedure starting from the **Substep III** here.
- 2. Clear the **Recorder** memory:
 - 2.1. Open Windows Explorer by clicking its icon on the Windows Taskbar.


Figure 583. Clearing Memory - Opening Windows Explorer



- 2.2. Click on the relevant drive that represents your **Recorder** memory.
- 2.3. Select any .nrr, or .res files to delete them. If there are more than one file, use Ctrl + Click or Shift + Click to select all relevant files.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 335 of 393

Figure 584. Clearing Memory - Deleting Files

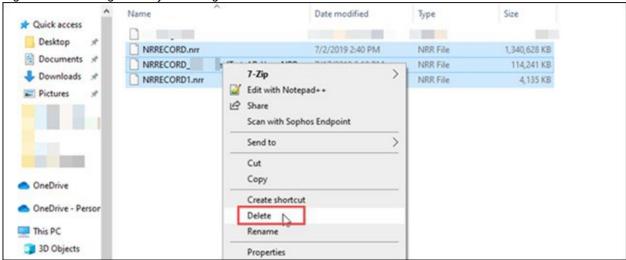

2.4. Right-click and select Delete from the context menu, or press Delete on your keyboard to clear the **Recorder**memory. All the selected files will be moved to the **Recycle Bin**.



Note: Regularly inspect the **Holter Recorder**, particularly the ECG cable leads, for signs of wear or damage to prevent issues during patient hook-up.

After completing these steps, proceed to the NH-301 analysis system to enter record information, adjust record settings, and more (on page 346).

5.2.1. Pairing Noray Devices via Bluetooth

Before starting the Preparing Recorder (on page 330) procedure, you may opt to connect a **Recorder** to your computer via Bluetooth pairing. This section will guide you through the pairing process of various NR device models with your computer.

Depending on the default pairing method of the specific NR **Recorder** you may need to follow one of the following sequences.

5.2.1.1. Pairing NR-314-P Device Model

The NR-314-P model, due to its compact size and design, has no Memory Card slot, unlike most other **NR Recorder** models. The NR-314-P model features USB and Bluetooth connection methods.




Note: The NR-314-P device model is compatible with **Bluetooth Low Energy (BLE) dongles only**. Other device models may employ both standard and low energy dongles.

To connect an NR-314-P device via Bluetooth:

- 1. Turn the device ON.
- 2. On your computer navigate to **Settings>Devices>Bluetooth & other devices**.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 336 of 393

Figure 585. Pairing Devices - Bluetooth and other devices Screen

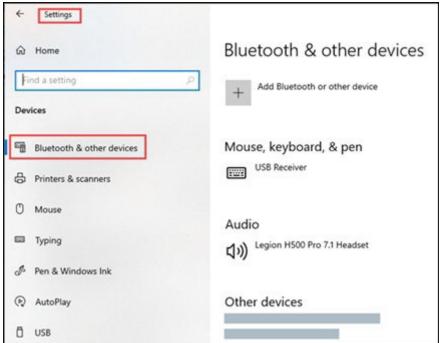


3. Click Add Bluetooth or other device.

Figure 586. Pairing Devices - Add Bluetooth Device

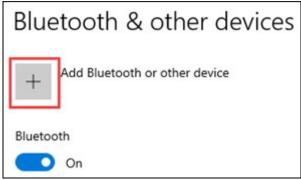

4. Select the **Bluetooth** option.


Figure 587. Pairing Devices - Add a device

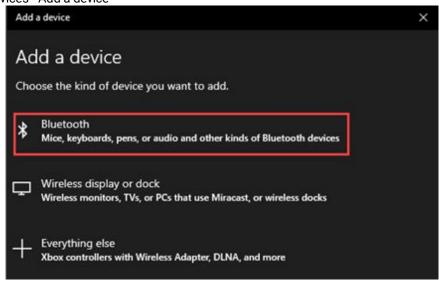

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 337 of 393


- 5. Click an NRpXXXXX option (or a similar entry) in the Add a device list. You will be prompted to enter the PIN code.
- 6. Enter the following PIN: **120474**. If this PIN doesn't work, please contact the Support Team.

Figure 588. Pairing Devices - Entering PIN

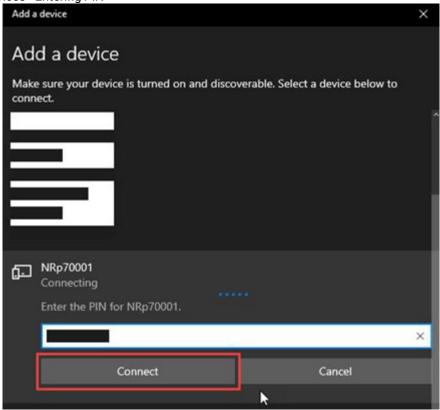

- 7. Click **Connect** to initiate pairing.
- 8. If you see the 'Paired' message on your computer, the process is complete. Otherwise, repeat from **Step 1** or consult the device manual.

Figure 589. Pairing Devices - Success

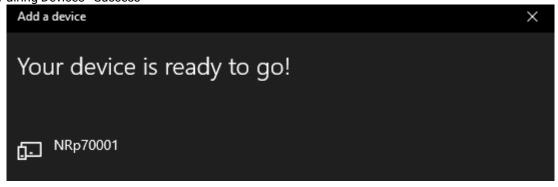



Note:

You may encounter a device selection and connection issue when attempting to select the NR-314-P in the **Select Connection** list of the **New Study View**. Generally, the procedure follows these steps:

- 1. Turn the device ON. The green indicator light should start blinking.
- 2. Wait until the green light stops blinking.
- 3. Select the NR-314-P device from the **Select Connection** list.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 338 of 393



If you proceed to **Step 3** before meeting the condition of **Step 2**, you may encounter a Bluetooth connection issue: the application might lose connection with the device, and the device name in the Windows **Bluetooth and other devices** list may change.

To resolve the connection issue, disable and then re-enable Bluetooth in Windows:

- 1. Select the **Start** button, then select **Settings > Devices > Bluetooth & other devices**.
- 2. Select the Bluetooth toggle and turn it OFF.
- 3. Turn the Bluetooth toggle ON.

Alternatively, to resolve the connection issue, disconnect and then reconnect the device dongle:

- 1. Remove the dongle.
- 2. Wait 5 seconds before reconnecting the dongle to your computer.

After resolving the Bluetooth pairing issue, adhere to the general procedure described above, ensuring compliance with the **Step 2** condition.

The NR-314-P recorder's LED indicator functions as detailed in the table below. Refer to this table for accurate interpretation of the NR-314-P recorder's LED indicator behaviors.

Action/Indication	Description	LED Indicator Behavior
Switch ON (NR-314-P disconnected from the docking station and has no recordings)	Press the button firmly , then release it.	Slowly blinking green light during initialization. Solid green after initialization is complete.
Switch OFF (before starting a new recording or uploading a record configuration file)	Press and hold the button for approximately 15 seconds until the indicator turns OFF completely, then release.	The LED indicator starts blinking green , then turns OFF when the device is turned OFF.
Start a new recording	After you have turned the NR-314-P recorder ON and the initialization is complete, press and hold the button for 3 seconds, then release.	The LED indicator begins blinking green slowly . The recording will be initiated in 30 seconds. After 60 more seconds, the green light will start glimmering faster with longer intervals , meaning the recording is in progress.
		Note: Note that when you turn the NR-314-P recorder ON, it will start the recording automatically according to the set Auto Start Delay parameter (10 minutes by default) even if you don't take any action.
Record User Event	Press the button until the device beeps, then release, or double-click the button.	The LED indicator turns solid blue , and after approximately 30 more seconds, goes OFF.
Stop recording	Press and hold the button for 15 seconds until the LED indicator turns OFF, then release.	The LED indicator turns solid blue , then goes OFF when the recording is stopped.
Record session is finished	-	The LED indicator displays slowly blinking red-to-green light and goes OFF after 5 minutes. Connect the device to a PC via USB to download a record.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 339 of 393

Data download	the docking station (USB connection).	When you connect the NR-314-P recorder to the docking station, the LED indicator starts flashing green . In a few seconds, the LED indicator turns solid blue . Now you may
		download the data from the recorder.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 340 of 393


Action/Indication	Description	LED Indicator Behavior
Switch ON (NR-314-P disconnected from	Press the button firmly , then release it.	Blinking green light during initialization. Solid green after initialization is complete.
the docking station and has a recording)		Immediately after that, the LED indicator of the NR-314-P recorder starts blinking from green to red , and turns OFF after a while.
		Note: Download or delete the recording from the NR-314-P recorder to turn it ON.
Battery is fully charged when con- nected via USB	The NR-314-P recorder is docked to the docking station (USB connection).	The LED indicator displays solid blue light.
Battery is charging when connected via USB	The NR-314-P recorder is docked to the docking station (USB connection).	The LED indicator displays blinking blue light.
Battery low	The NR-314-P recorder is connected via USB or Bluetooth.	The LED indicator displays solid red light. Connect the device to the docking station.
Hardware issue - non-critical	The NR-314-P recorder is connected via USB or Bluetooth.	The LED indicator displays blinking red light and goes off after 2 minutes. Please contact the Support Team.
Hardware issue - crit- ical	The NR-314-P recorder is connected via USB or Bluetooth.	The LED indicator displays blinking red light and goes OFF after 5 seconds. Disconnect the device from the charging station and send it to Norav lab for repair
Device is undocked and connected via Bluetooth	The NR-314-P recorder is connected via Bluetooth.	The LED indicator displays solid blue light.
Data transmission via Bluetooth	The NR-314-P recorder is connected via Bluetooth.	The LED indicator displays blinking blue light.

5.2.1.2. Pairing Norav Recorders Using Various Pairing Options

Some NR device models provide three device-computer pairing options to suit your preferences:

- Auto
- Manual
- Passcode

Figure 590. Pairing Devices - Bluetooth Pairing Options

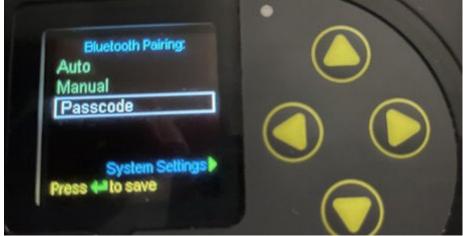

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 341 of 393

Bluetooth option in the **System Settings** menu, allowing you to switch among the aforementioned pairing methods.

Pairing with an Auto option:


- 1. Turn the device ON.
- 2. On your computer navigate to **Settings>Devices>Bluetooth & other devices**.

Figure 591. Pairing Devices - Bluetooth and other devices Screen



- 3. Click Add Bluetooth or other device.
- 4. Select the **Bluetooth** option.


Figure 592. Pairing Devices - Add a device

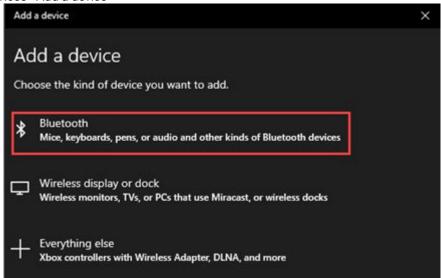

5. Click an **NR Recorder** option (or a similar entry) in the **Add a device** list.


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 342 of 393

Figure 593. Pairing Devices - Pairing a Device

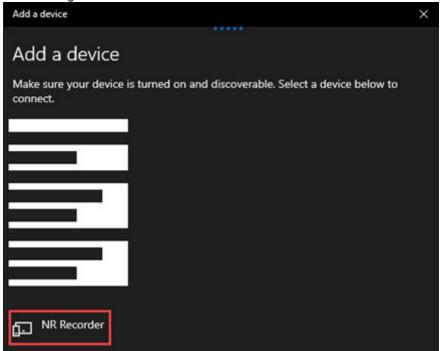

6. If you see the 'Paired' message on your computer, the process is complete. Otherwise, repeat from **Step 1** or consult the device manual.

Figure 594. Pairing Devices - Success

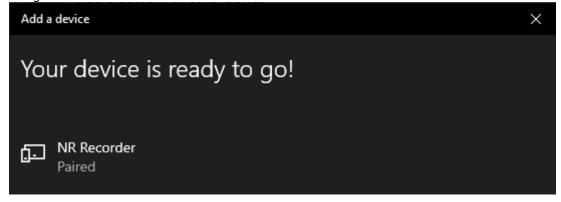

Pairing with a Manual option:

- 1. Turn the device ON.
- 2. Access the device's main menu.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 343 of 393

Figure 595. Pairing Devices - Device Main Menu Example



3. Go to **Settings**.


Figure 596. Pairing Devices - Device Settings Menu Example



- 4. Select **Pairing** to see the Bluetooth Pairing screen on your device.
- 5. On your computer navigate to **Settings>Devices>Bluetooth & other devices**.


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 344 of 393

Figure 597. Pairing Devices - Bluetooth and other devices Screen

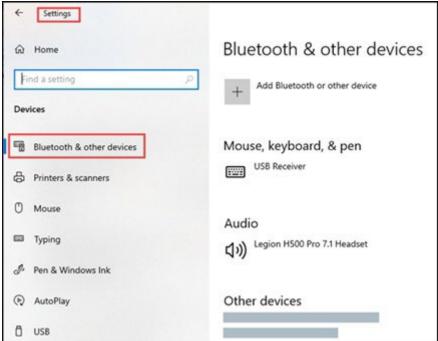

- 6. Click Add Bluetooth or other device.
- 7. Select the **Bluetooth** option.

Figure 598. Pairing Devices - Add a device

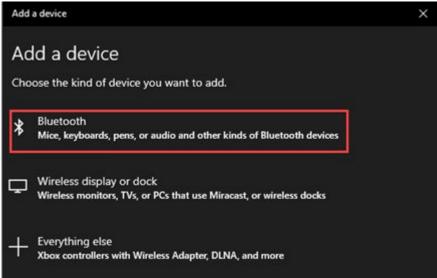

- 8. Click an NR Recorder option (or a similar entry) in the Add a device list.
- 9. If you see the 'Paired' message on your computer, the process is complete. Otherwise, repeat from **Step 1** or consult the device manual.

Pairing with a Passcode option:

- 1. Turn the device ON.
- 2. Access the device's main menu.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 345 of 393

Figure 599. Pairing Device - Device Main Menu Example



3. Go to **Settings Menu**.

Figure 600. Pairing Device - Device Settings Menu Example



4. Select **Pairing** to see the Bluetooth Pairing screen on your device.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 346 of 393	

Figure 601. Pairing Devices - Pairing via PIN Code



5. On your computer navigate to **Settings>Devices>Bluetooth & other devices**.


Figure 602. Pairing Devices - Bluetooth and other devices Screen



- 6. Click Add Bluetooth or other device.
- 7. Select the **Bluetooth** option.


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 347 of 393	

Figure 603. Pairing Devices - Add a device

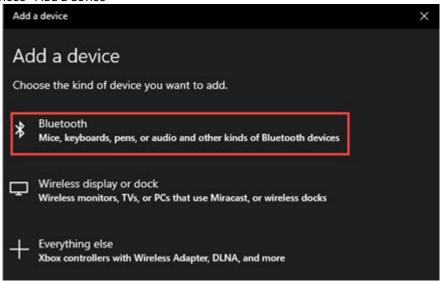

- 8. Click an NR Recorder option (or a similar entry) in the Add a device list.
- 9. Follow the on-screen instructions on your computer to input a PIN code.
- 10. Press the **Enter** button on your device to confirm the connection. Click **Connect** in the **Add a device** window of your computer. You may perform these actions in any order.

Figure 604. Pairing Devices - Pairing with PIN

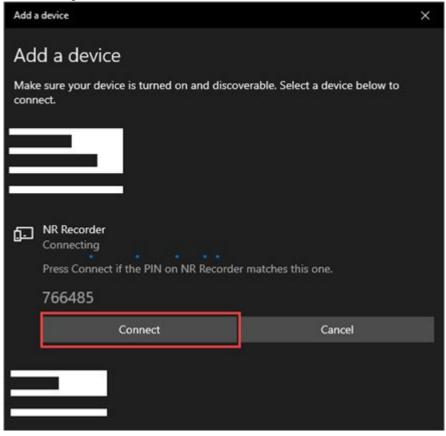

11. If you see the 'Paired' message on your computer, the process is complete. Otherwise, repeat from **Step 1** or consult the device manual.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 348 of 393


5.2.1.3. Pairing Norav Recorder via PIN-Based Connection

Some NR device models offer a device-computer pairing by entering a PIN code on your computer. These models, unlike those offering three different pairing options (refer to the section above for more details (on page 337)), have no **Pairing** option in their **Settings Menu**, and no **Bluetooth** option in the **System Settings** menu.

To pair your device with your computer:


- 1. Turn the device ON.
- 2. On your computer navigate to **Settings>Devices>Bluetooth & other devices**.

Figure 605. Pairing Devices - Bluetooth and other devices Screen

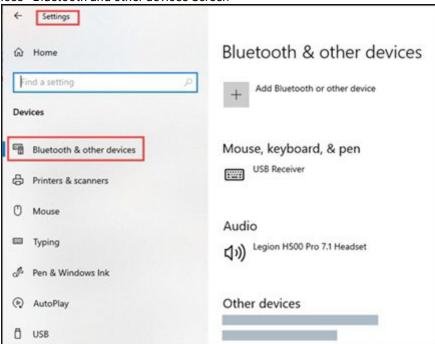

- 3. Click Add Bluetooth or other device.
- 4. Select the **Bluetooth** option.

Figure 606. Pairing Devices - Add a device

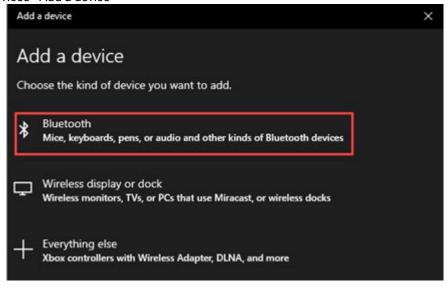


5. Click an **NR-XXXX-X-XXXXX** option (or a similar entry) in the **Add a device** list. You will be prompted to enter the PIN code.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 349 of 393

6. Enter the following PIN: 12345. If this PIN doesn't work, please contact the Support Team.

Figure 607. Pairing Devices - Pairing via PIN

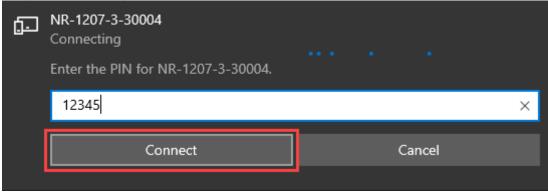

- 7. Click **Connect** to initiate pairing.
- 8. If you see the 'Paired' message on your computer, the process is complete. Otherwise, repeat from **Step 1** or consult the device manual.

5.3. Entering Patient Information

After you have finished preparing the patient and your **Recorder**, you need to enter patient information.

To enter **Patient Information**, follow these steps:


1. Click the **Prepare Recorder** button on the top toolbar of the **Record List View**. The **Patient Information** window is displayed.

Figure 608. Record List View - Prepare Recorder



Enter patient information. Click on each text field and type in the data according to your needs and workflows. To
proceed, you must fill in at least one of the patient's identifiers: Patient ID, First Name, Last Name. Other text
fields are not mandatory to proceed.

Figure 609. Entering Patient Information - Patient Information

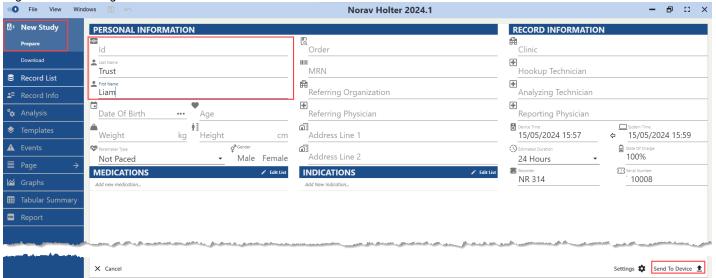

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 350 of 393	

- 3. **(Optional)** Click on the **Birth Date** field and type in the date, or use the ellipsis button on the right to select from a calendar.
- 4. Click **Next** at the bottom right of the screen.


After completing the steps above, you can begin Adjusting Record Settings (on page 347).

5.4. Adjusting Record Settings

After you have entered the patient information, you need to adjust the **Record Settings**. Please note that all underlined values (like ON/OFF) in this **View** are default values.

To adjust the Record Settings, follow the steps listed below:


Figure 610. Record Settings - Adjusting Record Settings

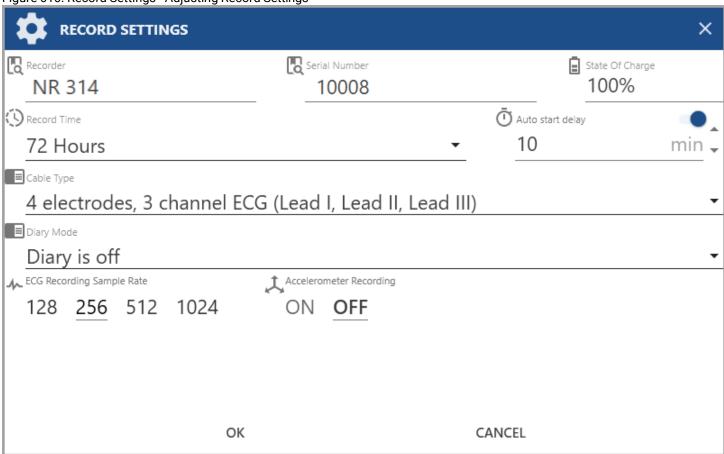

- 1. Select the **Record Time** from the drop-down list on the right, ranging from 24 to 336 hours, depending on the **Recorder** capabilities and requirements of the test. Refer to the relevant **Recorder** manual to check the longest possible **Record Time**.
- 2. **(Optional)** If the selected **Record Time** or **ECG Recording Sample Rate** is not compatible with a specific battery type, a warning will be displayed. Refer to the relevant **Recorder Manual** to check the **Record Time** parameters.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 351 of 393	

Figure 611. Record Settings - Warning Example

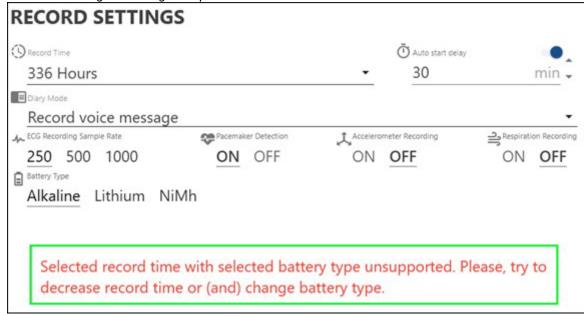


3. Set Auto start delay:

- Toggle Auto start delay OFF to disable the auto start delay feature.
- Toggle **Auto start delay** ON and set the delay in minutes using the picker arrows on the right. You can increase or decrease the delay time by 5 minutes with each click.

Figure 612. Record Settings - Auto start delay


RECORD SETTINGS

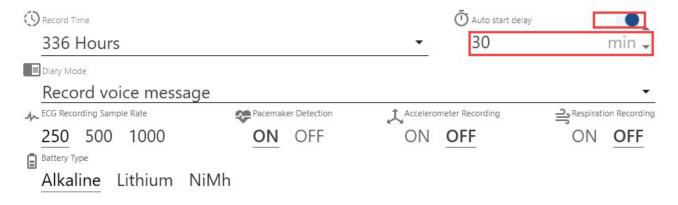

4. Select how the patient inputs diary events:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 352 of 393

Figure 613. Recorder Settings - Diary Mode

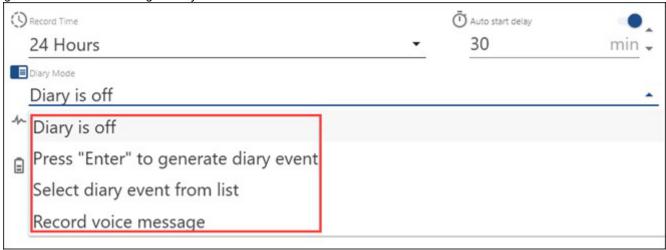

- **Diary is off:** A patient won't be able to make inputs.
- **Press "Enter" to generate diary event:** By pressing Enter on the **Recorder**, a patient will be able to register an **Event** without specifying its type.
- **Select a diary event from the list:** By pressing Enter on the **Recorder**, a patient will be able to choose an **Event** from a predefined list.
- **Record voice message:** By pressing Enter on the **Recorder**, a patient will be able to register an **Event** and make a brief voice description.

Note that different types of **Recorders** support different types of input for diary events. Please, refer to the relevant **Recorder** manual if needed.

5. Click to select the **ECG Recording Sample Rate**: 250, 500 or 1000 Hz. The higher the sampling rate, the more detailed the ECG signal will be. However, a higher sampling rate also requires more memory and more battery capacity. If the selected ECG Recording sample rate is not compatible with a specific battery type, a warning will be displayed.

Figure 614. Recorder Settings - Sample Rate



6. Click to select the **Battery Type**: Alkaline, Lithium, NiMh. Choose the battery type considering the overall length of the Holter testing, quantity of leads, **Recorder** model, etc. Please, refer to the relevant **Recorder** manual if needed.

Figure 615. Recorder Settings - Battery Type



7. Click to select **Pacemaker Detection**:

Figure 616. Recorder Settings - Pacemaker, Accelerometer, Respiration Options



Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	40010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 353 of 393


- Select **ON** to detect a pacemaker during the test and analysis. This is important for the interpretation of the ECG recordings, as the presence of a pacemaker can affect the appearance of the ECG waves.
- Select **OFF** if there is no need to detect a pacemaker for this patient.
- 8. Click to select Accelerometer Recording:
 - Select **ON** to enable the **Accelerometer Recording** during the test. Accelerometer recording can help to identify movement artifacts and remove them from the ECG signal, which can improve the accuracy of the interpretation.
 - Select **OFF** if there is no need to enable it.
- 9. Click to select **Respiration Recording**:
 - Select **ON** to enable the **Respiration Recording** during the test. Respiration recording can be used to
 assess the patient's respiratory rate and pattern. This information can be helpful in diagnosing certain heart
 conditions, such as sleep apnea.
 - Select **OFF** if there is no need to enable it.
- 10. **(Optional)** If you're using a **Recorder** model equipped with Bluetooth, you may click the **Synchronize** button to synchronize device clock with your computer clock. It is recommended to synchronize the device clock settings periodically. For the **NR-314-P** model, it is the easiest way of syncing as the device has no display and navigation buttons.

Figure 617. Record Settings - Synchronize



11. **(Optional)** If you're using a **Recorder** model equipped with Bluetooth, you may click the **Check ECG** button to verify that the electrodes are properly connected and that the ECG signal is being recorded correctly. This is important because if the electrodes are not properly connected, the ECG signal may be distorted or unreadable.

Figure 618. Record Settings - Check ECG

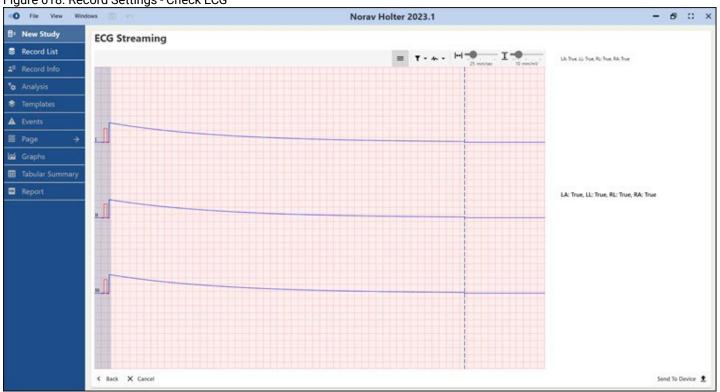

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 354 of 393

12. Click **Send To Device** in the bottom right corner to send the settings to the selected device. You will be briefly prompted by the **Uploading Configuration** progress bar. When it disappears, you will see a success message indicating that you have finished adjusting the **Record Settings**. If the **Send To Device** process fails, repeat the workflow from the start.

Figure 619. Record Settings - Send To Device

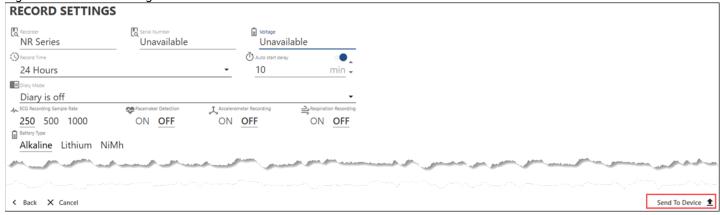


Figure 620. Record Settings - Uploading Configuration



You can also use the following buttons at every step:

- Click **Cancel** in the bottom left corner to skip all changes.
- Click **Back** to navigate back to the previous screen.

After completing the steps above, you may begin Initiating Recording.

5.5. Adjusting Record Settings: NR-314-P Model Workflow


After you have entered the patient information, you need to adjust the **Record Settings**. Please note that all underlined values (like ON/OFF) in this **View** are default values.

As the Norav Holter NR-314-P is designed to be compact and lightweight for discreet and comfortable wear, it has some distinctions in adjusting the **Record Settings** workflow.

To adjust the Record Settings, follow the steps listed below:


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 355 of 393

Figure 621. Record Settings - Adjusting Record Settings

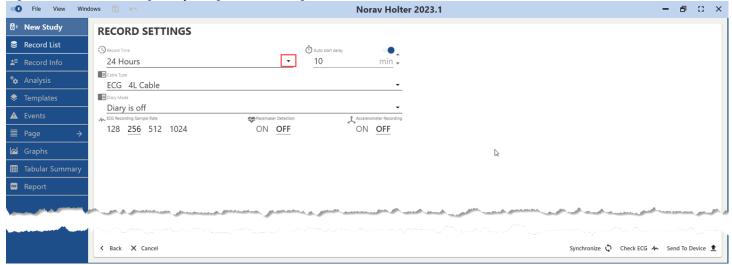

- 1. Select the **Record Time** from the drop-down list on the right, ranging from 24 to 336 hours, depending on the **Recorder's** capabilities and the requirements of the test.
- 2. **(Optional)** If the selected **Record Time** or **ECG Recording** sample rate is not compatible with a specific battery type, a warning will be displayed. Refer to the relevant **Recorder Manual** to check the **Record Time** parameters.

Figure 622. Record Settings - Warning Example

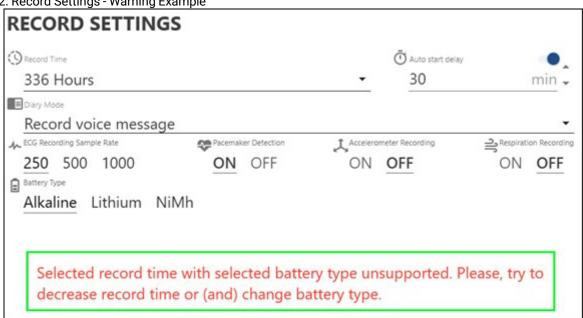


3. Set Auto start delay value:

- Toggle **Auto start delay** OFF to disable the auto start delay feature.
- Toggle **Auto start delay** ON and set the delay in minutes using the picker arrows on the right. You can increase or decrease the delay time by 5 minutes with each click.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 356 of 393

Figure 623. Record Settings - Auto start delay

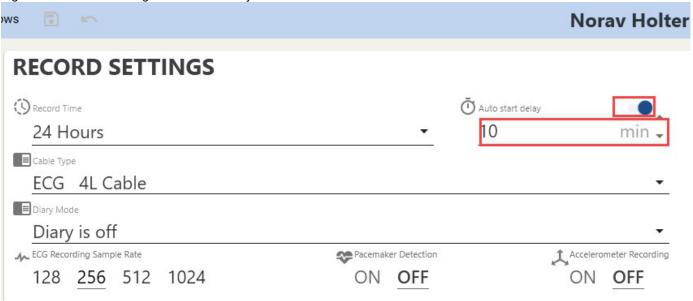

4. Select the cable type.

Figure 624. Record Settings - Selecting Cable

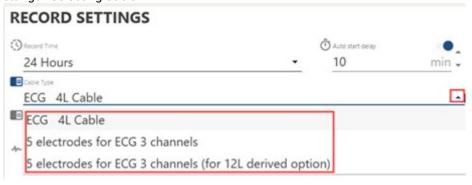


- 5. Select how the patient inputs diary events (different types of **Recorders** support different types of input, refer to the relevant manual, if needed):
 - **Diary is off:** A patient won't be able to make inputs.
 - **Press "Enter" to generate diary event:** By pressing the sole button on the **Recorder**, a patient can register an **Event**.
 - **Tapping twice on the recorder housing or pressing button will generate Event:** In addition to the button press, a patient may double-tap the device case to register an **Event**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 357 of 393

Figure 625. Record Settings - Diary Mode

RECORD SETTINGS

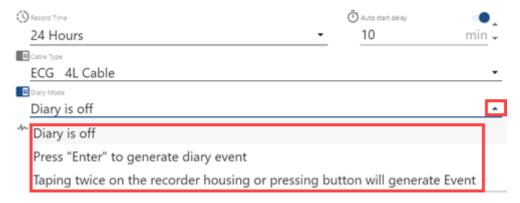

6. Click to select the **ECG Recording Sample Rate**: 128, 256, 512 or 1024 Hz. The higher the sampling rate, the more detailed the ECG signal will be. However, a higher sampling rate also requires more memory and more battery capacity. If the selected **ECG Recording Sample Rate** is not compatible with a specific battery type, a warning will be displayed.

Figure 626. Record Settings - Sample Rate

RECORD SETTINGS



7. Click to select **Pacemaker Detection**:

Figure 627. Record Settings - Pacemaker and Accelerometer






- Select **ON** to detect a pacemaker during the test and analysis. This is important for the interpretation of the ECG recordings, as the presence of a pacemaker can affect the appearance of the ECG waves.
- Select **OFF** if there is no need to detect a pacemaker for this patient.
- 8. Click to select Accelerometer Recording:
 - Select **ON** to enable the **Accelerometer Recording** during the test. Accelerometer recording can help to
 identify movement artifacts and remove them from the ECG signal, which can improve the accuracy of the
 interpretation.
 - Select **OFF** if there is no need to enable it.
- 9. **(Optional)** You may click the **Synchronize** button to synchronize the NR-314-P clock with your computer clock. It is recommended to synchronize device clock settings periodically.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 358 of 393

Figure 628. Record Settings - Synchronize

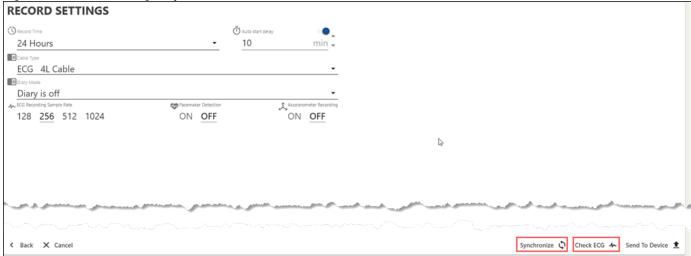

10. **(Optional)** You may click the **Check ECG** button to verify that the electrodes are properly connected and that the ECG signal is being recorded correctly. This step is crucial because if the electrodes are not properly connected, the ECG signal may be distorted or unreadable.

11. Click **Send To Device** in the bottom right corner to send the settings to the selected device. You will be briefly prompted by the **Uploading Configuration** progress bar. When it disappears, you will see a success message indicating that you have finished adjusting the **Record Settings**. If the **Send To Device** process fails, repeat the workflow from the start.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 359 of 393

Figure 630. Record Settings - Send To Device

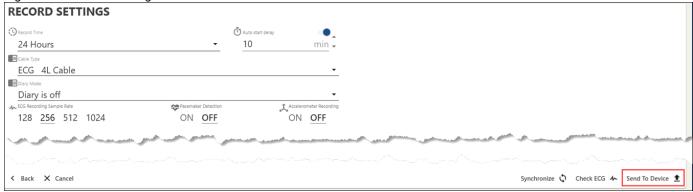


Figure 631. Record Settings - Uploading Configuration



You can also use the following buttons at every step:

- Click Cancel in the bottom left corner to skip all changes.
- Click **Back** to navigate back to the previous screen.

After completing the steps above, you may begin Initiating Recording.

5.6. Initiating Recording

After you have prepared the **Recorder**, entered the patient information, adjusted the recorder settings, and uploaded them to the **Recorder**, you are now ready to initiate the recording.



Note: If you didn't turn OFF the Auto start delay feature on the Adjusting Record Settings (on page 348) step, you may just use this feature to initiate recording automatically.

To initiate the recording, follow the steps below, depending on the method (connection type) you have used to prepare the **Recorder**:

- If you have prepared your **Recorder** using USB:
 - 1. Disconnect the **Recorder** from the USB port of the computer.
 - 2. Detach the USB cable from the **Recorder**.
 - 3. Connect the ECG cable of the prepared patient to the **Recorder**.
 - 4. Insert a **new** battery into the **Recorder**, and turn it **ON**. We recommend using new batteries of the appropriate type for every Holter test.
 - 5. Make sure the **Recorder** screen is turned **ON**: the screen should activate.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 360 of 393

Figure 632. Initiating Recording - Verifying Recorder





Note: You can see an example of a live screen from a Norav NR-1207-3 Recorder in the figure above. It may look different for other models.

- If you have prepared your **Recorder** using a **Memory Card Reader**:
 - 1. Remove the **Memory Card** from the card reader.
 - 2. Insert the **Memory Card** into the recorder of the prepared patient.
 - 3. Insert a **new** battery into the **Recorder**, and turn it **ON**. We recommend using new batteries of the appropriate type for every Holter test.
 - 4. Make sure the **Recorder** screen is turned **ON**: the screen should activate.
- If you are using the NR-314-P Recorder, use the Auto start delay feature, or:
 - 1. **(Optional)** Detach the device from the docking station, if applicable.

After completing the steps above, you are ready to proceed to Verifying Recorder Data (on page 357).

5.7. Verifying Recorder Data

After you have initiated the recording, you need to **Verify Recorder Data**. Generally, you should confirm whether the patient data and settings uploaded to the **Recorder** are accurate. For a comprehensive description of how to set up and operate a certain **Recorder** model, refer to the appropriate recorder manual. In this section, you will find the essence of this step on how to prepare for Holter recording.

To verify the recorder data, follow the steps below:

- 1. Confirm whether the data displayed on the recorder screen is correct and modify it if necessary. Review the data uploaded to the **Recorder**, such as patient information, recording duration limit, internal clock settings, and pacemaker detection, among others. Refer to the Entering Patient Information (on page 346) and Adjusting Recorder Settings (on page 347) sections to understand which parameters you have set up and uploaded to the Recorder, if needed.
- 2. Validate the quality of lead connections displayed on the recorder screen. If you see green dots near the lead pictograms on the recorder screen (as shown in the left part of the example figure below), the lead connections are

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 361 of 393

active and functional. If you see a message labeled **OFF** near a lead pictogram on the recorder screen (shown in the right part of the example figure below), inspect the lead connection and resolve any issues.


Figure 633. Verifying Recorder Data - Verifying Leads Connection





3. Assess the quality of the ECG traces displayed on the recorder screen. The waveform on the recorder screen should exhibit appropriate morphology (as shown in the example figure below). If you see a blank or wavy line on the screen, or unusual distortion, you may need to adjust the electrodes, reposition the recorder, or take other corrective actions to improve the signal quality.

Figure 634. Verifying Recorder Data - Verifying ECG Signal

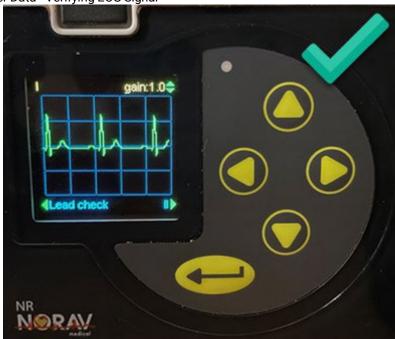

After completing the steps above, you are ready to proceed to Starting Recording (on page 359).

5.8. Verifying Recorder Data: NR-314-P

After initiating the recording, you need to **Verify Recorder Data**. You should confirm whether the electrode connections and acquired ECG signal are accurate. For a comprehensive description of how to set up and operate a specific **Recorder** model, refer to the appropriate recorder manual.

To verify the connection and ECG signal accuracy:

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 362 of 393

1. **For Bluetooth connection:** If you have connected the **Recorder** to your PC via Bluetooth, you may click the **Check ECG** button in the bottom-right corner of the screen to verify that the electrodes are properly connected and that the ECG signal will be recorded correctly.

Figure 635. Record Settings - Check ECG


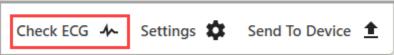


Figure 636. Record Settings - ECG Streaming

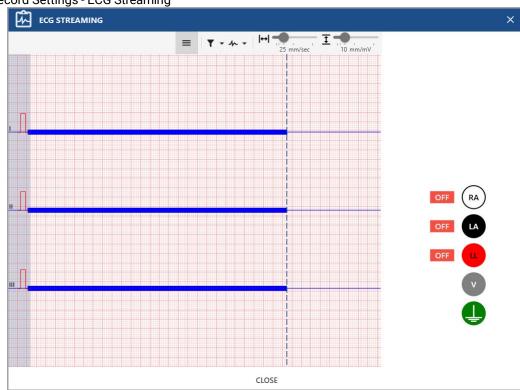

2. Assess the quality of the ECG traces displayed on the PC screen. The waveform on the recorder screen should exhibit appropriate morphology. If you see a blank or wavy line on the screen, or unusual distortion, you may need to adjust the electrodes, reposition the recorder, or take other corrective actions to improve the signal quality.

After completing the steps above, you are ready to proceed to Starting Recording (on page 359).

5.9. Starting Recording

Before this step of **Preparing for Holter Recording**, you must complete all the previous steps: prepare the **Recorder**, enter the patient information, adjust the recorder settings, upload them to the **Recorder**, initiate the recording, and verify the recorder data. Now, you are ready to start the Holter recording.

To start the Holter recording, use the Auto start delay feature, or:

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 363 of 393

1. Press the **Enter** button on the **Recorder** device. For more details specific to a particular **Recorder** model, refer to the relevant recorder manual.

Figure 637. Start Recording - "Enter" Button (example)





Note: Norav Recorder devices offer an **Auto Start Delay** *(on page 348)* function. If you forget to start the recording, it will start automatically within a set period of time if you have completed all the previous steps. For more details specific to a particular **Recorder** model, refer to the relevant recorder manual.

If you are using the NR-314-P Recorder, use the Auto start delay feature, or:

- 1. **(Optional)** Detach the device from the docking station, if applicable.
- 2. Press the sole **Enter** button for 3 seconds, until the **LED** indicator starts flashing slowly. Release the button.
- 3. Recording will start after 30 seconds, and the **LED** indicator will turn **OFF** after 60 seconds.

After you have started the recording, instruct the patient (on page 389) on how to follow the test procedure and answer any questions they may have. You may then discharge the patient. Remind the patient to return the recorder to the clinic or hospital on time. By following these instructions, you help ensure the accuracy of the Holter recording and the best possible care for the patient.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	NV-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		

6. Downloading Holter Recording Data

Before a Holter recording can be analyzed, the recorded data must be downloaded from the **Memory Card** or directly from the **Recorder** via USB. The **Record** cannot be downloaded via Bluetooth. Once the data has been downloaded, the **Memory Card** can be reused for the next recording.



Note: The **NR-314-P** device model supports recording downloads only via **USB** connection due to the large file sizes of **Holter Record** files. You need to connect the device to the docking station. Once connected, follow the general download workflow explained further to complete the **Record** file download.

When a patient returns to the physician's office after a set period of time (24-366 hours) when the Holter monitoring procedure is supposed to be finished, check if the recorder has been turned off automatically. If not, follow the procedure in the specific **Recorder** operating manual to terminate the recording and to prevent the recording of artifacts.



Note: You need to terminate the recording before downloading the data to prevent potential data loss or distortion.

To download the Holter recording data, follow these steps:


1. Double-click the **Norav Holter** icon on your desktop to launch the NH-301 analysis system.

Figure 638. Norav Holter Icon



- 2. You will see the **Record List View**. It allows you either to **import** an existing recording from the hard drive of your computer or your local network, or **download** a record from a Recorder using Memory Card or USB connection. If you need to **import** a record, refer to the Menu Bar section (on page 42) for more details. To **download** a record, proceed to the next Step.
- 3. Click the **Download from recorder** button on the top toolbar. The **Select Record** window will appear.


Figure 639. Downloading Recording Data - Download from recorder

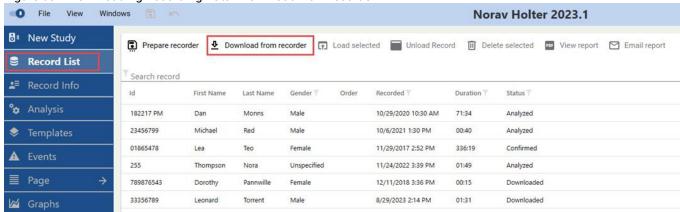

4. Select and double-click **one** record to download. Alternatively, click the **Download** button in the bottom right corner. Note that you can select **only one record** at a time.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 365 of 393

Figure 640. Downloading Recording Data - Download Record

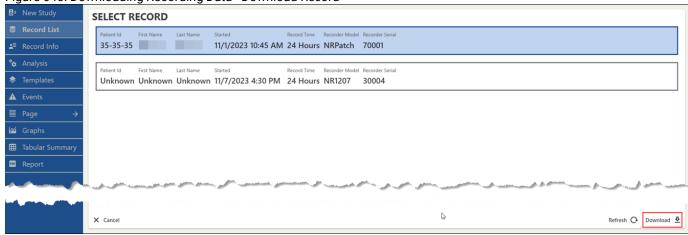

If you have connected additional **Recorders** or disconnected some of those previously plugged in, click **Refresh** in the bottom right corner to update the accessible records list.


5. The **Patient Information** window will appear. Validate and modify the patient's information if needed; refer to Step 3 in the Menu Bar section (on page 43) for instructions on modifying the patient's data.



Note: To ensure accurate software analysis of the pacemaker's performance, it's crucial to configure the pacemaker parameters to match the specific settings of the patient's implanted device.

6. Click **Proceed** in the bottom right corner to finish downloading the record. A "Downloading..." progress bar will appear. If only one record was recognized, the download is complete and the **Records List View** will be displayed. Otherwise, a list of records will be displayed for selection once more.

Figure 641. Downloading Recording Data - Click Proceed

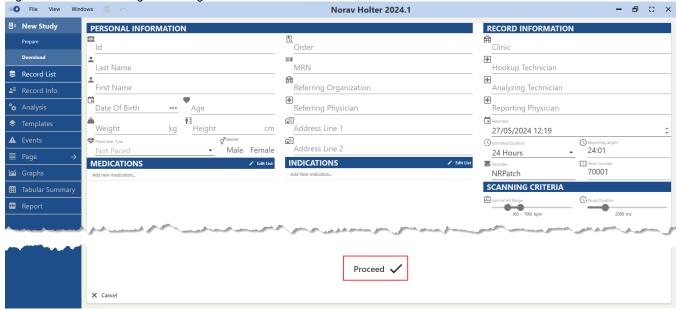

7. You will be prompted with the **Remove Record** dialog box. If you wish to remove the downloaded **Record** from the device (memory card), click **Yes**. Otherwise, click **No**.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Page 366 of 393	

Figure 642. Downloading Recorder Data - Remove Record



If the **Record** was downloaded successfully you will see a confirmation message as shown below. After that you may start analyzing the ECG Data (on page 83).

Figure 643. Downloading Recording Data - Success Confirmation



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 367 of 393

7. Recommended Workflow Options

The previous chapters have detailed the analysis process and features, and how you can tailor the software system and the analysis workflows to your requirements. In this **Chapter**, we will overview various options for editing analysis results, concluding with steps to generate a report that incorporates your personal findings.

7.1. Standard Workflow Overview


Here, we will cover the procedure for conducting the analysis, reviewing beat Templates and Events, and setting up and generating the final Report. The Norav NH-301 Holter analysis system offers a simple and streamlined workflow. To commence the Analysis and deliver a comprehensive report, follow the general top-down logic of the Views Sidebar on the left side of the screen. For more details on completing each step, please refer to the Views section (on page 53) and its respective subsections.



Step 1: Launching Analysis

Before initiating analysis, ensure a Record is loaded, define the start and end points, and adjust the necessary parameters:

1. Load your target Record in the Record List View (on page 70) by double-clicking it or using the Load Selected button.

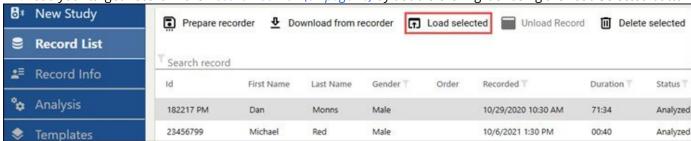


2. If needed, update patient data in the Record Info View.




3. Access the Analysis View (on page 83).



4. Assess the ECG signal on the first page. If it's abnormal, use the "Page Down" button in the Action Bar at the bottom and switch between Channels to find a clear signal.

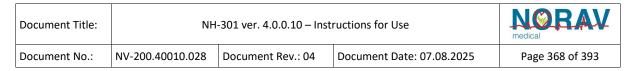

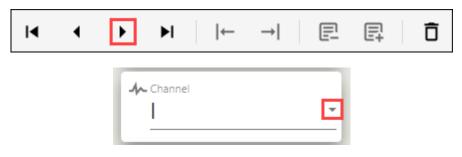

5. Right-click to access the context menu and select "Mark as start of analysis," or use the corresponding button in the Action Bar to mark the Start of Analysis.




6. Navigate to the Final Page using the "Final Page" button in the Action Bar. If noise is present, use "Page Up" and alternate Channels to locate a clear signal.



7. Mark the clear signal as the end point for the analysis.



8. (Optional) Include or Exclude Segments:

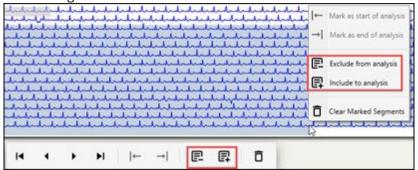

- To exclude, right-click and drag over the interval, selecting "Exclude from analysis" from the context menu or Action Bar.
- To re-include a segment, right-click within the excluded area, drag to the required position, and select "Include to analysis."
- 9. Adjust Channels, Heart Rate Range, Pause Duration, Pacemaker settings, and other parameters in the right control boxes if necessary.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 369 of 393



10. (Optional) Clear marked segments at any time using the context menu or Action Bar.



11. Click START to begin the analysis.



Step 2: Editing Templates

After the analysis, proceed to edit beat Templates:

1. Click the Templates View (on page 91) on the left to assess algorithm decisions on heartbeat shapes.



2. Select Overlay All Beats for each template to identify those requiring collective or individual edits.



3. Check annotations of the templates that can be handled as a whole and reclassify them if necessary. It's recommended to begin with V templates, proceed with F, and then S. Use the Scale and Gain sliders located at the top right to display more details of the beat morphologies.



4. Enter template beat editing mode by double-clicking the relevant template box or right-clicking and choosing Edit Template.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 370 of 393



- 5. To delete beats:
 - · Click the Delete button in the top toolbar and right-click the beats you want to delete.
 - Alternatively, use the Select All button, Ctrl+Click, or Shift+Click to select beats first, then click Delete to mark them as deleted.



6. Click Apply in the top left corner to save changes.




Step 3. Reviewing Events

After editing Templates, proceed to review Events.


1. Click the Events View (on page 165) on the left.




2. Select the Maximum RR event from the Event Type List. The first occurrence will be displayed in the RR-trend, ECG Strip, and Page pane at the bottom (depending on the View layout).

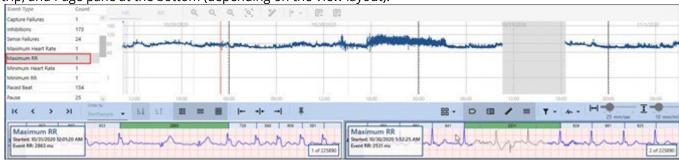


3. Drag and drop to select and exclude incorrect segments with an Event that appears incorrect in the RR-trend, ECG Strip, or Page pane.

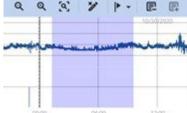

4. After deleting all incorrect Events, right-click the desired event in the middle pane and select Set as Selected to include it in the final Report.

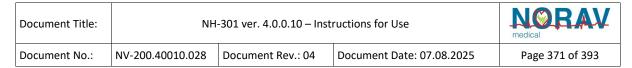

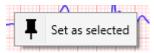


5. Select Atrial Fibrillation events from the Event Type List, if present.




- 6. Review and delete any incorrect Atrial Fibrillation events.
- 7. If remaining, navigate through these events in the middle pane:
 - Right-click on the corresponding ECG Strip or Page segment to open the context menu.
 - Choose Change Beats Annotation to access the annotation pane.
 - Reclassify using N annotation.



8. **(Optional)** To add Atrial Fibrillation events if necessary: Use Trends, ECG Strip, or Page pane - right-click, drag, and select Create User Event, then Atrial Fibrillation from the context menu.

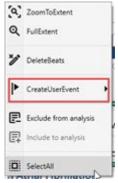

- 9. (Optional) To print an ECG segment if needed:
 - Click the start point.
 - Press and hold Shift.
 - Scroll to the desired endpoint using control buttons in the toolbar on top or a scrollbar at the bottom.
 - Click the desired endpoint.
 - Click the Print icon.

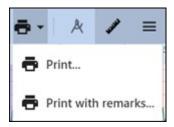

Step 4. Scanning ECG Page Data

Before generating the final Report, you may review the results in other Views: Page, Graphs, or Tabular Summary. The Page View (on page 231) displays complete ECG data, allowing you to scan the entire recording and add user Events if needed.

Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 372 of 393



1. Use navigation buttons at the top left to quickly navigate through the entire ECG recording for review.




- 2. Click to select a beat or right-click and drag to select a segment of the ECG recording.
- 3. Add user Events, delete beats, exclude noisy intervals, or print ECG samples if necessary.



Step 5. Reviewing Graphs

Graphs View (on page 272): Allows you to examine HRV analysis results and preview ST trends.

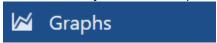

The lower panel displays a detailed ECG Strip for previewing and scanning ECG traces, event creation, beat management, and printing.

Step 6. Reviewing Tabular Summary

Tabular Summary (on page 305): Displays an Hourly Tabular Report of all detected arrhythmias, useful for contextualizing specific conditions for their appearance.



The Summary may include a Pacemaker Report. You can export it to PDF or Excel.



Step 7. Generating and Confirming Report

After completing all necessary steps, generate, review, and confirm your Report.

1. Click the Report View (on page 308).



2. Toggle ON/OFF the desired report sections using control boxes on the right. Manage sections such as Summary and Conclusion; Events; Charts; Full Disclosure, and more.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		
Document No.:	NV-200.40010.028	/-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 373 of 393



3. Click Generate Report (on page 321).

Generate Report

4. Once the Report is generated, review it and fill in the Conclusion.

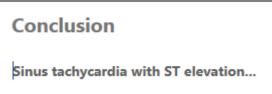

5. Finalize by clicking Confirm Report.



7.2. Perfectionist Workflow Overview

Here, we will cover the procedure for conducting the analysis, reviewing beat Templates and Events, and how to set up and generate the final Report. The Norav NH-301 Holter analysis system offers a simple and streamlined workflow. To commence the Analysis and deliver a comprehensive report, follow the general top-down logic of the Views Sidebar on the left side of the screen. For more details on how to complete each step, please refer to the Views section (on page 53) and its respective subsections.



Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use			NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 374 of 393

Step 1. Configuring Algorithm Settings

Before initiating analysis, you need to configure the algorithm sensitivity:

- 1. Navigate to File Menu > Settings > Algorithm (on page 18).
- 2. Set the Templates Sensitivity parameters:
 - 2.1. For V-type annotations = 3.
 - 2.2. For all other annotation types = 1.



Step 2. Launching Analysis

Before initiating analysis, ensure a Record is loaded, define the start and end points, and adjust the necessary parameters:

1. Load your target Record in the Record List View (on page 70) by double-clicking it or using the Load Selected button.



2. If needed, update patient data in the Record Info View.



3. Access the Analysis View (on page 83).




4. Assess the ECG signal on the first page. If it's abnormal, use the "Page Down" button in the Action Bar at the bottom and switch between Channels to find a clear signal.



5. Right-click to access the context menu and select "Mark as start of analysis," or use the corresponding button in the

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 375 of 393

Action Bar to mark the Start of Analysis.

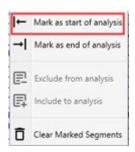


6. Navigate to the Final Page using the "Final Page" button in the Action Bar. If noise is present, use "Page Up" and alternate Channels to locate a clear signal.



7. Mark the clear signal as the end point for the analysis.



8. (Optional) Include or Exclude Segments:

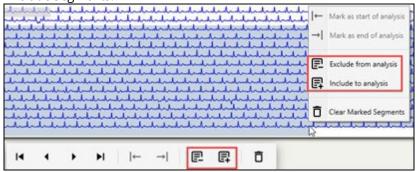

- To exclude, right-click and drag over the interval, selecting "Exclude from analysis" from the context menu or Action Bar.
- To re-include a segment, right-click within the excluded area, drag to the required position, and select "Include to analysis."
- 9. Adjust Channels, Heart Rate Range, Pause Duration, Pacemaker settings, and other parameters in the right control boxes if necessary.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 376 of 393



10. (Optional) Clear marked segments at any time using the context menu or Action Bar.



11. Click START to begin the analysis.




Step 2. Editing RR-type Events

Upon completion of the analysis, click the Events View (on page 165).



1. Select Maximum RR from the Event Type list (or use any other means to ensure the RR-trend and the ECG strip are displayed).

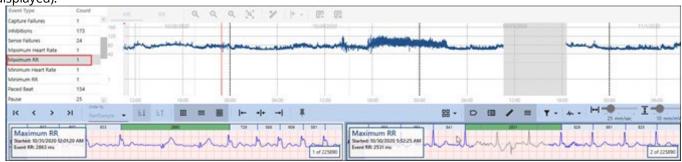

2. Using the RR-trend and the ECG strip, select regions to be excluded from analysis

Step 3. Editing Templates

Proceed to the Templates View to review the algorithm's decisions about the heartbeats' shapes but not the decisions about their prematurity i.e., reviewing whether heartbeats annotated as N and S really have normal shapes and whether heartbeats annotated as V and F really have abnormal shapes. At this stage, there is no need to review whether heartbeats annotated as S are really premature, which is done later (after AFib events are reviewed).

1. Click the Templates View (on page 91) on the left to assess algorithm decisions on heartbeat shapes.



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 377 of 393

2. Use the Scale slider at the top right of the page to set the paper speed for the template boxes to 100 mm/s, since this setting allows displaying more details of the beat morphologies.




3. Select Overlay All Beats for each template to identify those requiring collective or individual edits.



4. Check annotations of the templates that can be handled as a whole and reclassify them if necessary. It's recommended to begin with V templates, proceed with F, and then S. Use the Scale and Gain sliders located at the top right to display more details of the beat morphologies.



5. Enter template beat editing mode by double-clicking the relevant template box or right-clicking and choosing Edit Template.

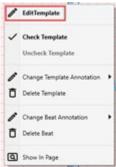

- 6. To delete beats:
 - · Click the Delete button in the top toolbar and right-click the beats you want to delete.
 - Alternatively, use the Select All button, Ctrl+Click, or Shift+Click to select beats first, then click Delete to mark them as deleted.



7. Click Apply in the top left corner to save changes.




Step 4. Reviewing Events

After editing Templates, proceed to review Events.

1. Click the Events View (on page 165) on the left.



2. Select Atrial Fibrillation events from the Event Type List, if present.

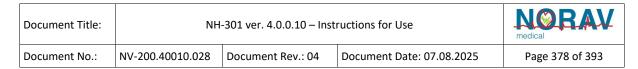



- 3. Review and delete any incorrect Atrial Fibrillation events.
- 4. If remaining, navigate through these events in the middle pane:
 - Right-click on the corresponding ECG Strip or Page segment to open the context menu.
 - Choose Change Beats Annotation to access the annotation pane.
 - Reclassify using N annotation.



5. **(Optional)** To add Atrial Fibrillation events if necessary: Use Trends, ECG Strip, or Page pane - right-click, drag, and select Create User Event, then Atrial Fibrillation from the context menu.



Step 5. Reviewing Prematurity Templates

Click the Templates View to review the algorithm's decisions about the prematurity of the heartbeats.

1. Click the Templates View (on page 91) on the left.



2. Use the Scale slider at the top right of the page to set the paper speed for the template boxes to 100 mm/s, since this setting allows displaying more details of the beat morphologies



3. Enter template beat editing mode by double-clicking the relevant template box or right-clicking and choosing Edit Template.



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 379 of 393

- 4. To delete beats:
 - · Click the Delete button in the top toolbar and right-click the beats you want to delete.
 - Alternatively, use the Select All button, Ctrl+Click, or Shift+Click to select beats first, then click Delete to mark them as deleted



5. Click Apply in the top left corner to save changes.



Step 6. Editing Events

After reviewing Templates, proceed to Events.


1. Click the Events View (on page 165) on the left.



2. Examine Example Strips for various arrhythmia types.




- 3. Review and delete incorrect Events.
- 4. If needed, mark (select) Example Strips of various arrhythmia types to include in the Report.

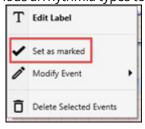


Step 7. Scanning ECG Page Data

Before generating the final Report, you may review the results in other Views: Page, Graphs, or Tabular Summary. The Page View (on page 231) displays complete ECG data, allowing you to scan the entire recording and add user Events if needed.

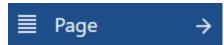

1. Use navigation buttons at the top left to quickly navigate through the entire ECG recording for review.

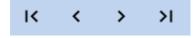

2. Click to select a beat or right-click and drag to select a segment of the ECG recording.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 380 of 393

3. Add user Events, delete beats, exclude noisy intervals, or print ECG samples if necessary.



Step 8. Reviewing Graphs

Graphs View (on page 272): Allows you to examine HRV analysis results and preview ST trends.

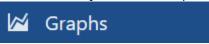

The lower panel displays a detailed ECG Strip for previewing and scanning ECG traces, event creation, beat management, and printing.

Step 9. Reviewing Tabular Summary

Tabular Summary (on page 305): Displays an Hourly Tabular Report of all detected arrhythmias, useful for contextualizing specific conditions for their appearance.



The Summary may include a Pacemaker Report. You can export it to PDF or Excel.

Step 10. Generating and Confirming Report

After completing all necessary steps, generate, review, and confirm your Report.

1. Click the Report View (on page 308).



2. Toggle ON/OFF the desired report sections using control boxes on the right. Manage sections such as Summary and Conclusion; Events; Charts; Full Disclosure, and more.



Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	028 Document Rev.: 04 Document Date: 07.08.2025		Page 381 of 393

Generate Report

4. Once the report is generated, review it and fill in the Conclusion.

Conclusion
Sinus tachycardia with ST elevation...

5. Finalize by clicking Confirm Report.



7.3. Workflow Clarifications and Explanations

In both workflows, channels are not selected for analysis. This occurs because the algorithm evaluates each channel's usefulness at a given time and responds accordingly. In most cases, this results in an additional channel being advantageous for analysis, even if its signal is of low quality.

In the Standard workflow within the Analysis View, all regions are included, and the start and end points of the analysis are not set. This approach is adopted because it is faster and easier to exclude regions from the analysis after its completion (RR-trend), and there is no benefit to doing so beforehand.

Conversely, in the Perfectionist workflow, the endpoint of the analysis is selected. This selection is made due to the algorithm slowing down as a result of combining both templates, which are enabled for annotations of type N, X, or P, along with extended periods where no channels exhibit ECG signals (due to detached electrodes).

When both these conditions occur, the algorithm attempts to find similar shapes where no such similarities can be found. Relatively short periods (up to tens of minutes) where channels lack ECG do not pose this issue. Similarly, if at least one channel has an ECG, the problem does not arise. Extended periods without ECG on any channel only occur at the end of a record. Therefore, if template sensitivity for annotations of types N, X, or P is enabled, the record's end should be examined to determine the analysis endpoint.

When SVEs are excluded from Atrial Fibrillation events, either manually or automatically, all beats previously marked as S are grouped into the same template, and information on shape similarity is lost.

Consequently, reviewing heartbeat shapes should occur before excluding any SVEs from Atrial Fibrillation, but after removing bad ECG regions from the analysis.

However, assessing heartbeat prematurity should take place after SVEs are excluded from Atrial Fibrillation. This necessity leads to multiple switches between the Events and Templates Views in the Perfectionist Workflow.


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV	
Document No.:	NV-200.40010.028	NV-200.40010.028 Document Rev.: 04 Document Date: 07.08.2025		Page 382 of 393

8. Appendix A: Overview and Utilization of RR Trend Visualizations

The RR trend comprises blue vertical lines with light-blue dots on top. Each vertical line of pixels displays information for 1 to 2 minutes of the record:

- The blue lines signify the range between the maximum and minimum RR intervals within the 1 to 2-minute intervals.
- For each RR interval within a 1-2 minute period, a light-blue dot is positioned at a specific point on the vertical axis to represent the interval's duration. Often, numerous RR intervals have similar values, leading to multiple light-blue dots converging into a single pixel (meaning a single light-blue pixel may signify several RR intervals).

Figure 644. RR Trend - RR Trend

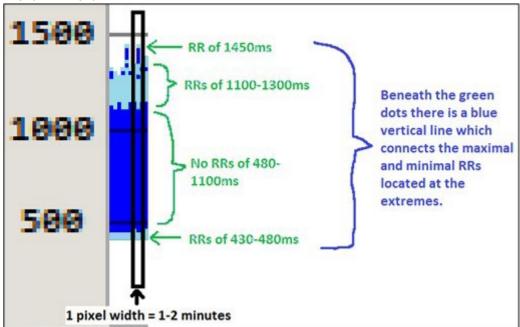

The RR trend also comprises three additional vertical lines:

- A green thick line marks the maximum RR event.
- A red thick line marks the minimum RR event.
- A red thin line represents the current ECG marker.

Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 383 of 393


Figure 645. RR Trend - ECG Markers



Hovering over the RR trend displays a vertical red line with arrow pointers at each end, following the mouse cursor. This line marks the 1-2 minute interval under the cursor, with the arrow pointers highlighting the minimum and maximum RR intervals of that period. The precise values of these RRs are shown in milliseconds below.


Document Title:	NH	NH-301 ver. 4.0.0.10 – Instructions for Use		NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 384 of 393

Figure 646. Trend - RR Values

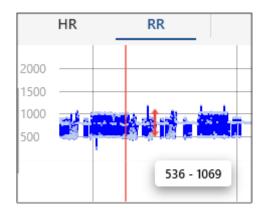

Clicking on the RR trend navigates the ECG Strip in the Event View to a specific point within the 1-2 minute interval aligned with the click location. The vertical position of the click determines the exact RR interval shown—above the blue line for the maximum RR and below for the minimum RR of that 1-2 minute period:

Figure 647. RR Trend - Min and Max RR

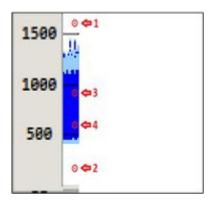


- 1. Displays the maximum RR within the 1-2 minute period (1450 ms)
- 2. Displays the minimum RR within the 1-2 minute period (420 ms)
- 3. Shows an RR of 1120 ms, the shortest RR among those longer than 1000 ms. This occurs when the mouse is clicked on a blue pixel (indicating an area without RRs) instead of a light-blue one, which leads to the display of either the longest RR below the click position (450 ms) or the shortest RR above it (1120 ms). As the click is nearer to the latter, the 1120 ms RR is displayed.
- 4. Shows an RR of 450 ms, the longest RR among those shorter than 500 ms. This happens when the mouse is clicked on a blue pixel (indicating an area without RRs) instead of a light-blue one, which leads to the display of either the longest RR below the click (450 ms) or the shortest RR above it (1120 ms). As the click is nearer to the former, the 450 ms RR is displayed.

8.1. RR Trend Examples


8.1.1. Normal Sinus Rhythm (NSR)

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 385 of 393

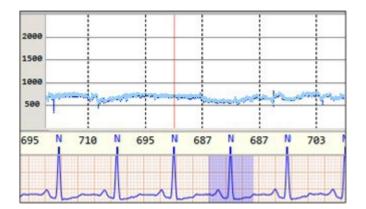


8.1.2. NSR with VPBs

- 1. Lower Line Ectopic RRs (N-V)
- 2. Upper Line Compensatory Pause RRs (V-N)
- 3. Middle Line NSR RRs (N-N)

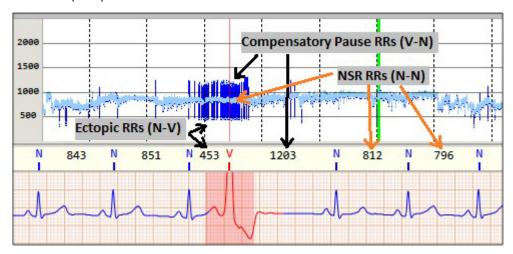

8.1.3. Sustained Supraventricular Tachycardia (SVT)


- 1. Upper light-blue line NSR RRs
- 2. Depressed light-blue line SVT RRs

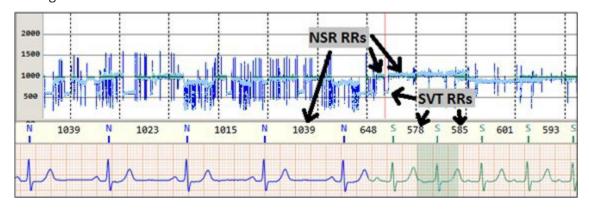

8.1.4. NSR with VPBs without Compensatory Pause


Document Title:	NH-301 ver. 4.0.0.10 – Instructions for Use		NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04 Document Date: 07.08.2025		Page 386 of 393

- 1. Upper light-blue line NSR (**N**–**N**)
- 2. Lower light-blue line Ectopic RRs (N-V)
- 3. Middle light-blue line RR after VPB (V-N)

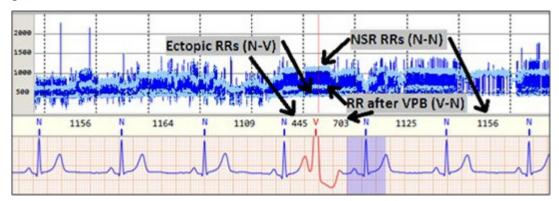

8.1.5. Ventricular Bigeminy

- 1. Lower light-blue line Ectopic RRs (N-V)
- 2. Upper light-blue line Compensatory Pause RRs (V-N)

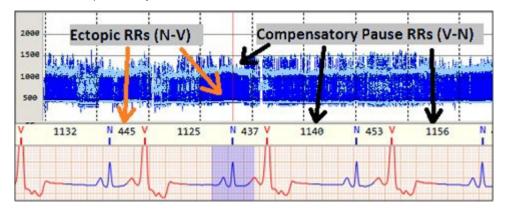


When you click on	Display
The white area below the lower light-blue line	Shortest Ectopic RR (N-V)
The lower part of the blue area	Longest Ectopic RR (N-V)
The white area above the upper light-blue line	Longest Compensatory Pause (V-N)
The upper part of the blue area	Shortest Compensatory Pause (V-N)


8.1.6. Supraventricular Bigeminy

- Lower light-blue line Ectopic RRs (N-S)
- Upper light-blue line Compensatory Pause RRs (S-N)

Document Title:	NH	-301 ver. 4.0.0.10 – Insti	NORAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 387 of 393

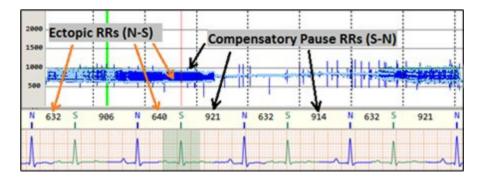


8.1.7. Pacemaker Failure to Capture

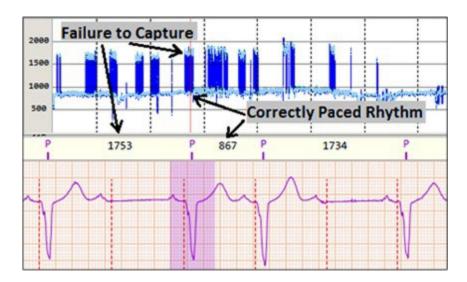

8.1.8. Atrial Fibrillation


Observe the wide light-blue ribbon indicating atrial activity.

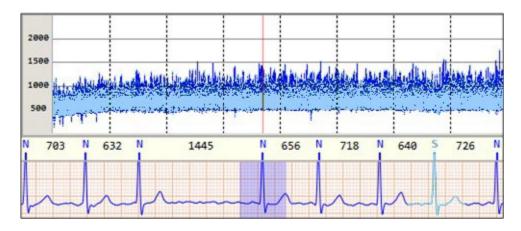

8.1.9. Wenkebach (Mobitz I)

Therefore, when the RR trend appears as a wide light-blue ribbon, it is advisable to consult both the Strip View and Page View, located below the Trends View, to distinguish between the various possibilities.


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV medical		
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 388 of 393	

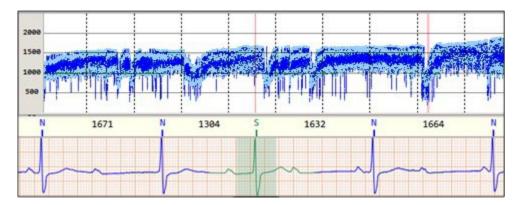



8.1.10. Sick Sinus Syndrome

When the RR trend appears as a wide light-blue ribbon and looks similar to Atrial Fibrillation or other event types, it is advisable to consult both the Strip View and Page View, located below the Trends View, to distinguish between the various possibilities.

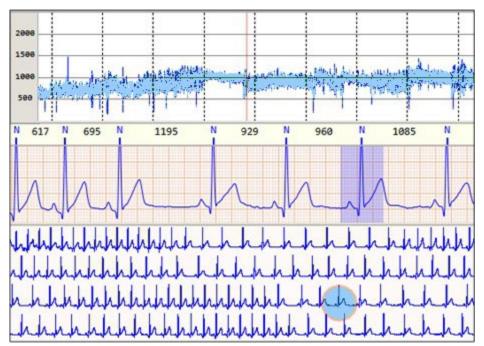


Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 389 of 393




8.1.11. Atrial Flutter


Horizontal light-blue lines of varying positions represent different P:R ratios.

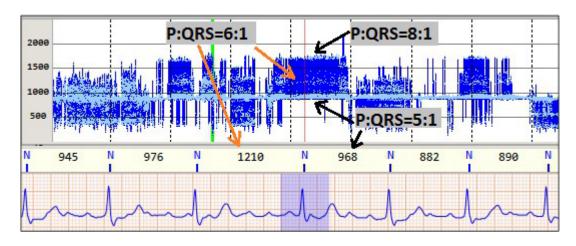

8.1.12. NSR with High HRV

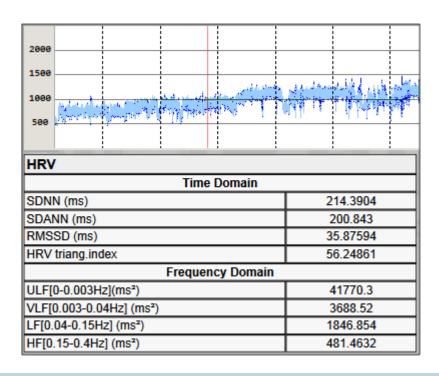

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	N@RAV medical	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 390 of 393

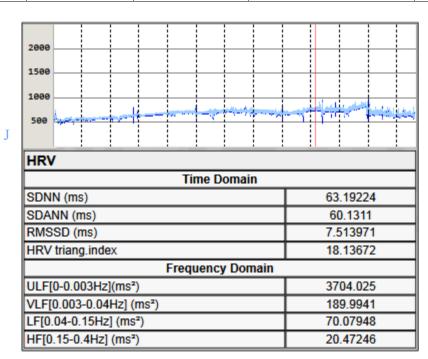

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	NO RAV	
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 391 of 393

9. Appendix B: Holter Monitoring Patient Diary Form

This is a recommended Holter Monitor Patient Diary Form for your use during Holter monitoring.

Institution:								
Referring Physician:								
Patient Name:								
Date of Birth:		Gender:		MR	N:			
Recording Period:	Hours	from:				to:		
Recorder:			Connected	by:				

Dear patient,

While undergoing Holter monitoring, it's important to keep a detailed diary of your activities and any symptoms you experience. Please record the time any symptoms start, such as chest pain, shortness of breath, irregular heartbeats, or dizziness, and what you were doing at that moment. Your diary will help compare your daily activities with the ECG data recorded by the Holter monitor.

Remember that your physician needs a complete picture of your activities. If you're unsure whether to include something, it's better to write it down.

Please use this diary form to note all your daily activities:

- Time of day Write the time for every activity or symptom that you note in the diary.
- Your symptoms Record any symptoms like chest pain, back pain, dizziness, or nausea, even if they seem minor.
- Your activities Write down all your activities, including sitting, walking, exercising, eating, sexual activity, taking medications, and so on.

Date	Time	Symptoms	Activity
			Ţ

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NORAV medical
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 392 of 393

10. Appendix C: Patient-Related Instructions



Note: It's the physician's responsibility to inform the patient of the necessary steps for a safe and effective ECG recording.

Patient Safety Cautions



CAUTION:

- If you experience skin problems, inform your physician immediately. In rare cases, even when using biocompatible electrodes, allergic reactions may occur.
- Avoid exposing the recorder to water. This includes adverse weather conditions and bathing.
- In humid or bad weather, store the recorder in its pouch and wear it under a coat to avoid damage.
- Do not expose the recorder to extreme temperatures. Stay in air-conditioned places in hot weather, and keep the recorder under your clothes in cold weather.
- Handle the ECG cable with care. Bending or wrapping it tightly around the recorder can damage it.
- Keep away from electrical devices to ensure safety.
- Avoid using electric blankets while you have the recorder on.

Patient Diary

Depending on the physician's recommendation, patients can use a Holter Monitor Patient Diary Form to log activities, symptoms, and their times during ECG recording. The diary options include (based on recorder settings):

- Pressing the recorder's Event button to mark the time, then recording symptoms/activities separately.
- Pressing the Event button to choose an event from a preset list.
- Recording a voice message about the event using the Event button.

Submitting Patient Event

As per the physician's instructions, patients can use the Event button on the recorder to mark times when they experience symptoms or engage in specific activities. Options for logging these events, based on the recorder's settings, include selecting an activity from a list or recording a voice message. To record an event, press and hold the Event button for 2-3 seconds until the recorder beeps.

Document Title:	NH	-301 ver. 4.0.0.10 – Inst	ructions for Use	NO RAV
Document No.:	NV-200.40010.028	Document Rev.: 04	Document Date: 07.08.2025	Page 393 of 393

11. Storage, Cleaning and Disposal of License Physical Key

Operation, Storage, and Transportation Conditions

Parameter	Value		
Operating temperature	10 °C to 45 °C		
Storage and transportation temperature	-20 °C to 60 °C		
Operating atmospheric pressure	68 kPa to 106 kPa (680 mb to 1060 mb)		
Atmospheric pressure for storage and transportation			
Operating humidity	10% RH to 95% RH non-condensing		
Humidity for storage and transportation			

During storage, avoid exposure to extreme temperatures, humidity, dust, or vibrations.

Do not expose the software license key to direct sunlight or any other UVA/UVB radiation.

Cleaning of License Physical Key

To clean the license key, use a cleaning kit or a soft cotton cloth.

Do not use solvents or cleaning agents to clean the license key

Disposal of License Physical Key

Dispose of the license key according to local regulations. The key contains metal, plastic, and electronic components.